


Clamp On Wheel Lock

Pull to Lock

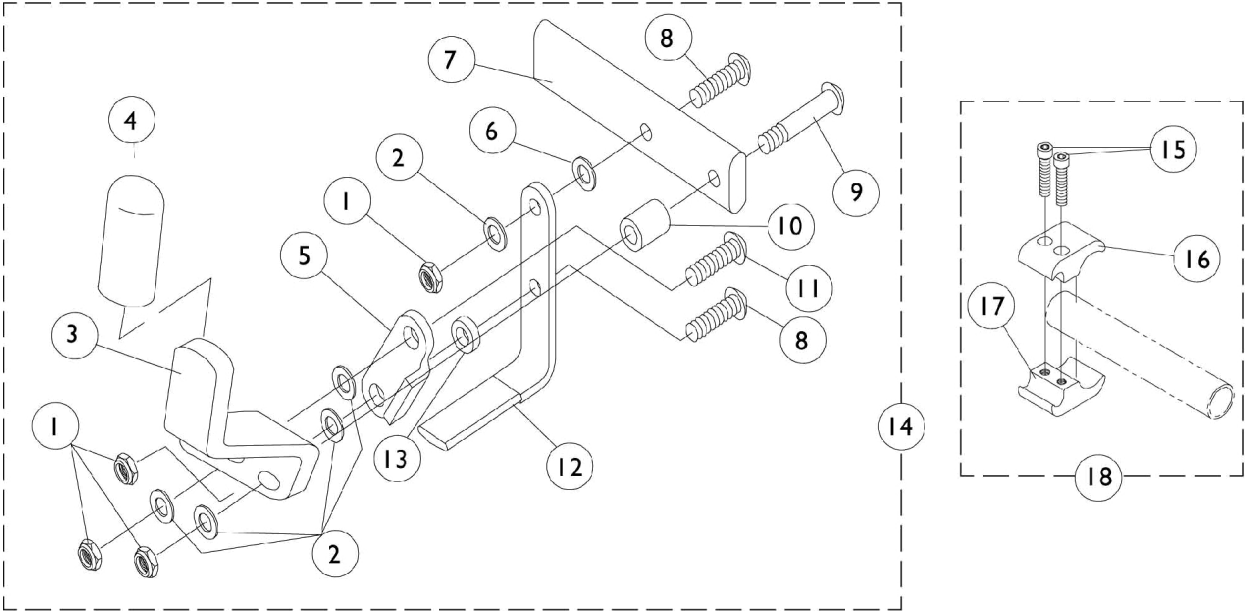

Clamp On Wheel Lock

Pull to Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930334-NLA	NLA - Locknut (1/4-28)	25	4
2		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	5
3		1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
3		1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
4		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
5		1018733-NLA	NLA - Link, Stop, Black	1	1
6		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
7		1028399-NLA	NLA - Strap, Wheel Lock	1	1
8		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
9		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
12		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
13		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
14	A	1038552-NLA	NLA - Wheel Lock, Clamp On, Pull to Lock - Right	1	1
14	A	1038553	Wheel Lock, Clamp On, Pull to Lock - Left	1	1
14	B	1049539	Wheel Lock, Clamp On, Pull to Lock - Right	1	1
14	B	1049540	Wheel Lock, Clamp On, Pull to Lock - Left	1	1
15		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
16		1038114-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	1
17		1038113	Clamp, Wheel Lock, Threaded (7/8")	1	1
18		1039343	Kit, Clamp Wheel Lock Hardware (Pair)	2	1
			NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	2
			Clamp, Wheel Lock, Threaded (7/8")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4
NOTE: A - Before 6/94. Includes items 1 - 13. B - After 5/94. Includes items 1 - 13.					