
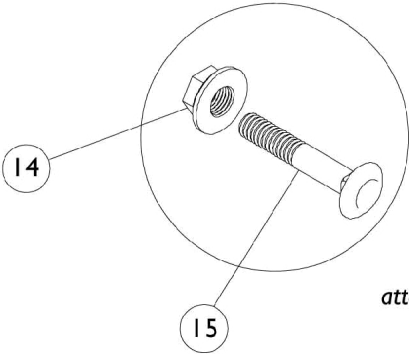


Hill Holder Wheel Locks Conventional Arms

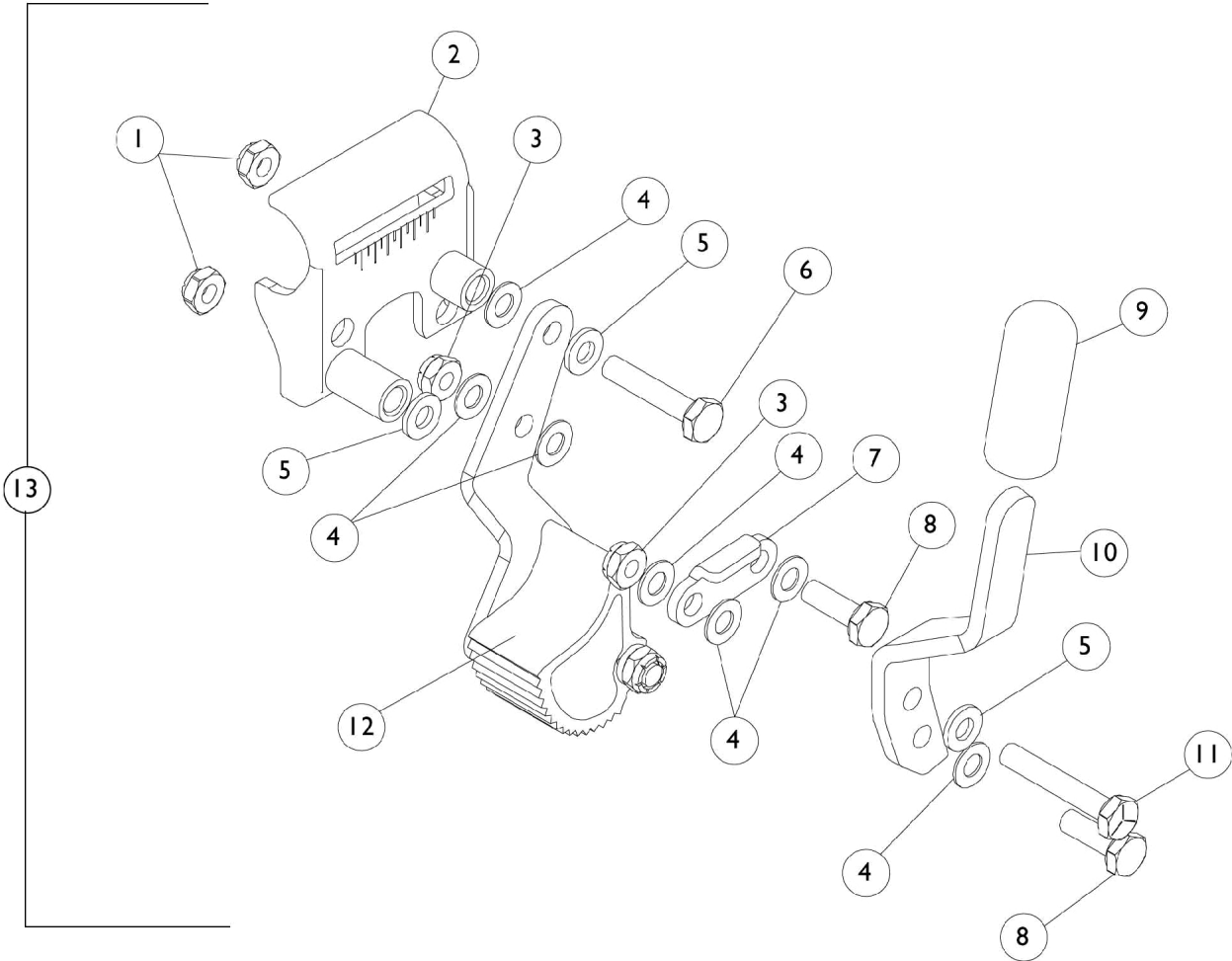

attaching hardware

Hill Holder Wheel Locks Conventional Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
2		1016037-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
2		1016038-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
3		7930367-NLA	NLA - Locknut (1/4-28)	25	2
4		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	7
5		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
6		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
7		1017861-NLA	NLA - Link, Chrome	1	1
8		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
9		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
10	B	1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
10	B	1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
10	C	1016590-NLA	NLA - Handle, Pull, Hemi, Chrome - Right	1	1
10	C	1016591-NLA	NLA - Handle, Pull, Hemi, Chrome - Left	1	1
11		7930350-NLA	NLA - Package, Hex Head Screw (1/4-20 x 1-1/2")	25	1
12	D	1016892-NLA	NLA - Hill Holder Assembly - Right	1	1
12	D	1016893-NLA	NLA - Hill Holder Assembly - Left	1	1
13	A,B	1034210	Wheel Lock Assembly, Hill Holder, Push to Lock - Right	1	1
13	A,B	1034211	Wheel Lock Assembly, Hill Holder, Push to Lock - Left	1	1
13	A,C	1025429-NLA	NLA - Wheel Lock Assembly, Hill Holder, Pull to Lock - Right	1	1
13	A,C	1025430-NLA	NLA - Wheel Lock Assembly, Hill Holder, Pull to Lock - Left	1	1
14		1021403	Nut, Hex Flang (1/4-20)	1	1
15		1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1

NOTE: A - Hill Holder Wheel Locks not for use with urethane tires.
 B - Push to Lock
 C - Pull to Lock
 D - Part #1016892 and 1016893 are no longer available as a service part individually. Order item #13 as a replacement.

Push to Lock Shown