
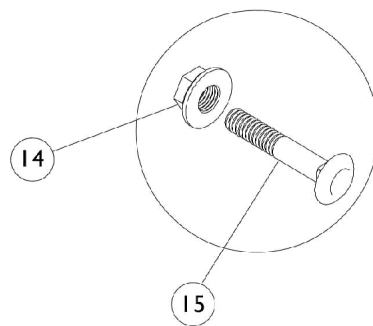


Hill Holder Wheel Locks Space Saver Arms

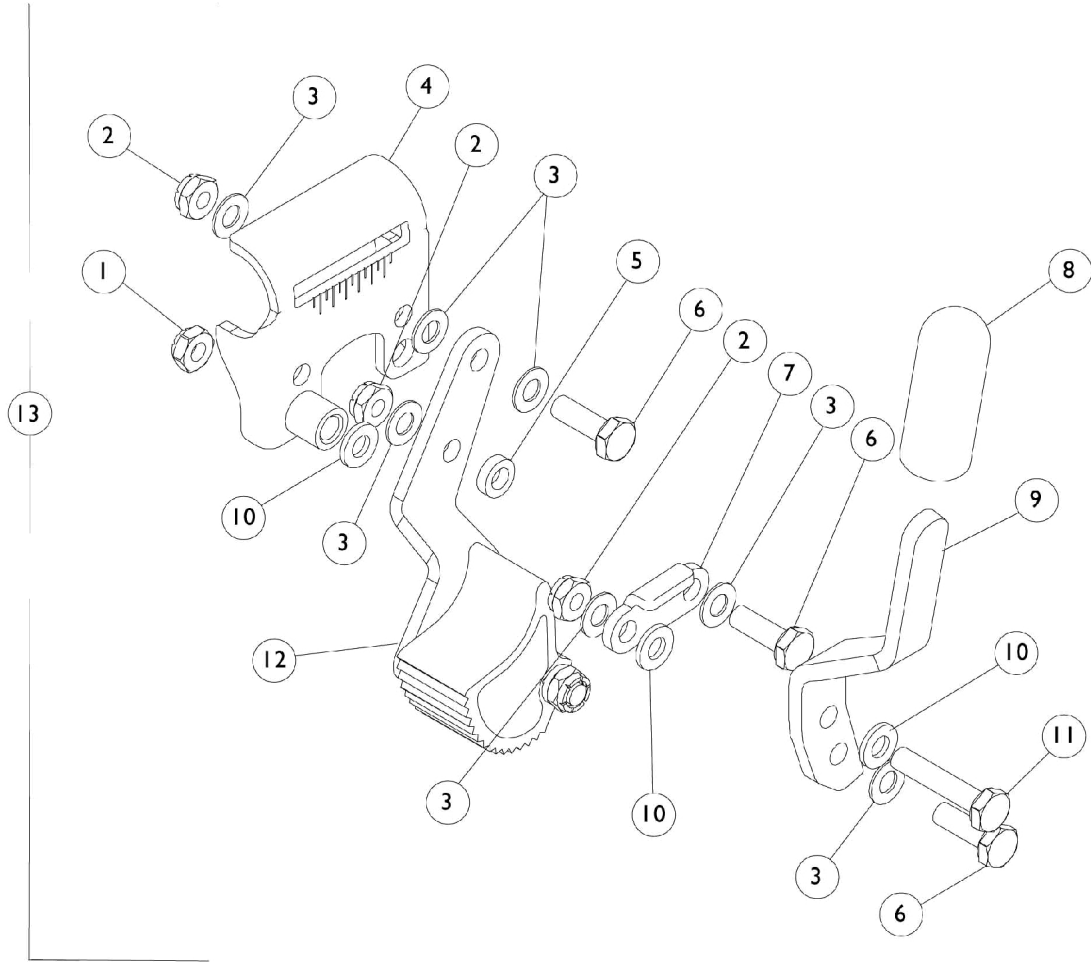

Attaching hardware

Hill Holder Wheel Locks Space Saver Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
2		7930367-NLA	NLA - Locknut (1/4-28)	25	3
3		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	7
4		1015445-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
4		1015446-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
5		40157X004	Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
6		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	3
7		1017861-NLA	NLA - Link, Chrome	1	1
8		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
9	B	1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
9	B	1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
9	C	1016590-NLA	NLA - Handle, Pull, Hemi, Chrome - Right	1	1
9	C	1016591-NLA	NLA - Handle, Pull, Hemi, Chrome - Left	1	1
10		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
11		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
12	D	1016892-NLA	NLA - Hill Holder Assembly - Right	1	1
12	D	1016893-NLA	NLA - Hill Holder Assembly - Left	1	1
13	A,B	1034212	Wheel Lock Assembly, Hill Holder, Push to Lock - Right	1	1
13	A,B	1034213	Wheel Lock Assembly, Hill Holder, Push to Lock - Left	1	1
13	A,C	1034214	Wheel Lock Assembly, Hill Holder, Pull to Lock - Right	1	1
13	A,C	1034215	Wheel Lock Assembly, Hill Holder, Pull to Lock - Left	1	1
14		1021403-NLA	Nut, Hex Flang (1/4-20)	1	1
15		1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
NOTE: A - Hill Hlder Wheel Locks not for use with urethane tires. B - Push to Lock C - Pull to Lock D - Part #1016892 and 1016893 are no longer available as a service part individually. Order item #13 as a replacement.					
Push to Lock Shown					