


Wheel Lock - Conventional Arms

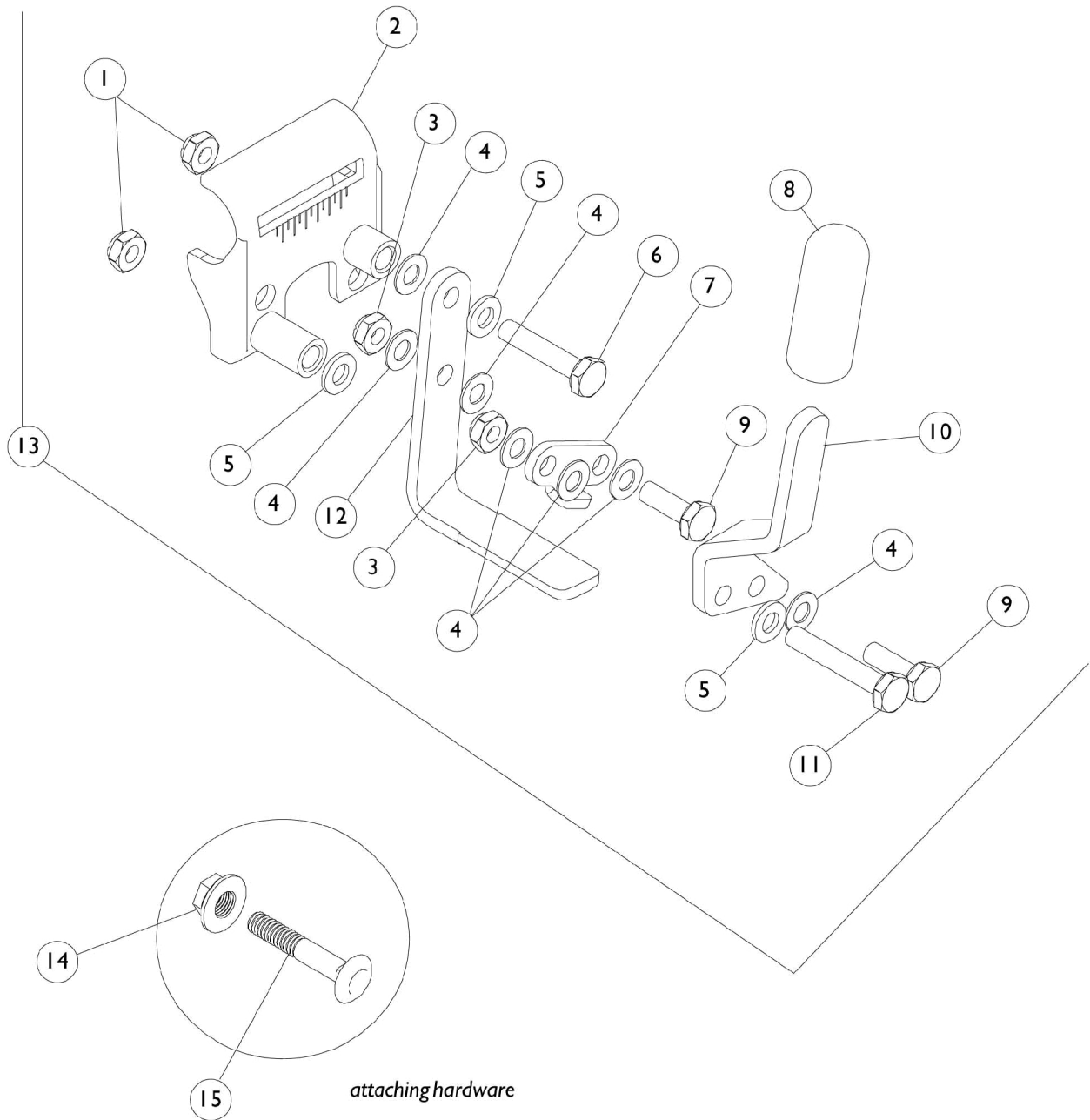

Wheel Lock - Conventional Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
2	A,B	1026049-NLA	NLA - Bracket, Mounting - Right	1	1
2	A,B	1026050-NLA	NLA - Bracket, Mounting - Left	1	1
3		7930334-NLA	NLA - Locknut (1/4-28)	25	2
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
5		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
6		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
7		1021404-NLA	NLA - Link, Black	1	1
8		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
9		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	2
10	A	1021408-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
10	A	1021409-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
10	B	1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
10	B	1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
11		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
12		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
13	J	1031782	Wheel Lock Assembly, Push to Lock, Adult - Right	1	1
13	J	1031783	Wheel Lock Assembly, Push to Lock, Adult - Left	1	1
13	J	1031796	Wheel Lock Assembly, Pull to Lock, Hemi - Right	1	1
13	J	1031797	Wheel Lock Assembly, Pull to Lock, Hemi - Left	1	1
14	C	1021403	Nut, Hex Flang (1/4-20)	1	1
15	C	1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
	D,E,G, H	7400109	Kit, Wheel Lock Shoe Spacer (Pair)	2	1
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
			Spacer, Wheel Lock Shoe (2-7/16")	1	2
	D,F,G, H	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	2	1
			Spacer, Wheel Lock Shoe (1-45/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
	D,E	7930306-NLA	NLA - Package, Screw, Set Socket Cup Point (#8-32 x 3/16")	25	2

NOTE: A - Push to Lock
 B - Pull to lock
 C - Attaching Hardware
 D - These items are not included with wheel lock, they may be needed if chair has 20 or 20" rear wheels.
 E - 20" Rear Wheels
 F - 22" Rear Wheels
 G - Assembly includes Set Screws
 H - Not Shown
 J - Includes items 1 - 12.

Pull to Lock Shown