


Wheel Lock - Space-Saver Arms

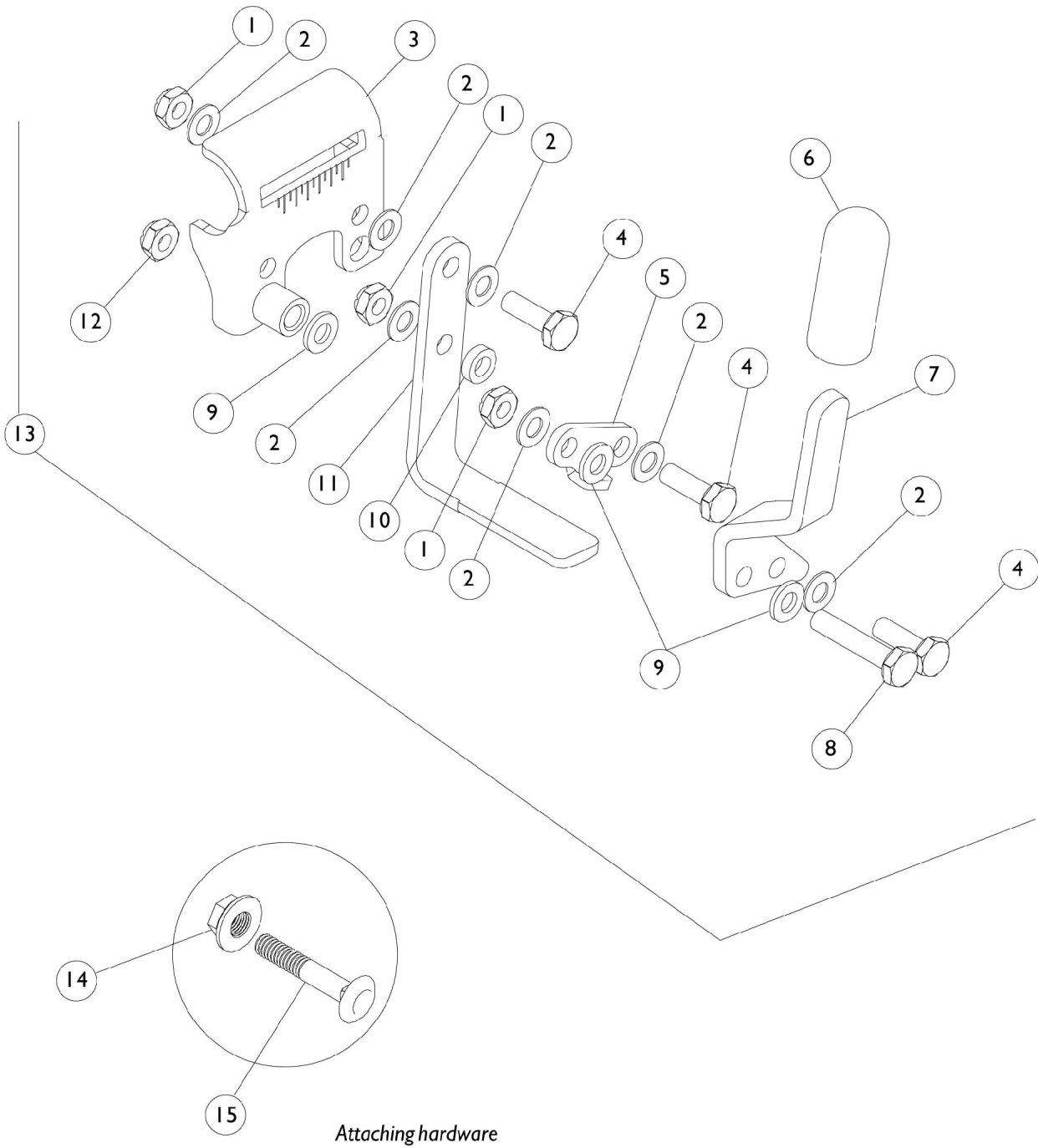

Wheel Lock - Space-Saver Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930334-NLA	NLA - Locknut (1/4-28)	25	3
2		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
3		1031787-NLA	NLA - Bracket, Mounting, Black - Right	1	1
3		1031788-NLA	NLA - Bracket, Mounting, Black - Left	1	1
4		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	3
5		1021404-NLA	NLA - Link, Black	1	1
6		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
7	A	1021408-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
7	A	1021409-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
7	B	1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
7	B	1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
8		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
9		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
10		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
11		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
12		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
13	J	1031792	Wheel Lock Assembly, Push to Lock, Adult - Right	1	1
13	J	1031793	Wheel Lock Assembly, Push to Lock, Adult - Left	1	1
13	J	1031794	Wheel Lock Assembly, Pull to Lock, Hemi - Right	1	1
13	J	1031795	Wheel Lock Assembly, Pull to Lock, Hemi - Left	1	1
14	C	1021403	Nut, Hex Flang (1/4-20)	1	1
15	C	1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
	D,E,F, H	7400109	Kit, Wheel Lock Shoe Spacer (Pair)	2	1
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
			Spacer, Wheel Lock Shoe (2-7/16")	1	2
	D,E,G, H	1037422	Kit, Wheel Lock Shoe Spacer, 22" Rear Wheels (Pair)	2	1
			Spacer, Wheel Lock Shoe (1-45/64")	1	2
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	4
	H	7930306-NLA	NLA - Package, Screw, Set Socket Cup Point (#8-32 x 3/16")	25	2

NOTE: A - Push to Lock
 B - Pull to Lock
 C - Attaching Hardware
 D - These items are not included with wheel lock, they may be needed if chair has 20" or 22" rear wheels.
 E - Assembly includes Set Screws (pair)
 F - 20" Rear Wheels
 G - 22" Rear Wheels
 H - Not Shown
 J - Includes items 1 - 12.