


# ATL4A Legrest Support Assembly

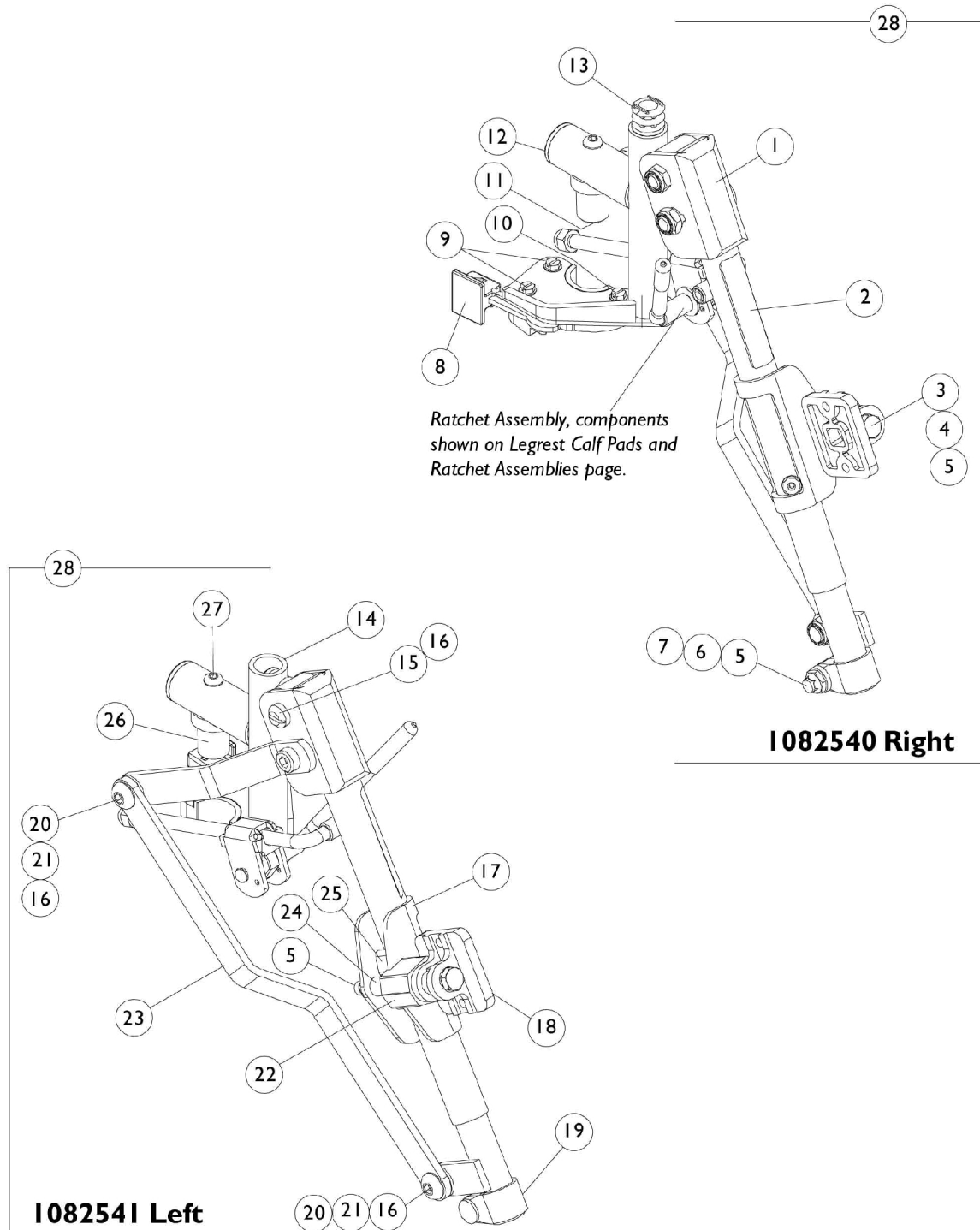

# ATL4A Legrest Support Assembly

| Item Number | Note | Part Number | Description                                                       | Quantity Per |          |
|-------------|------|-------------|-------------------------------------------------------------------|--------------|----------|
|             |      |             |                                                                   | Package      | Assembly |
| 1           |      | 1022877-NLA | NLA - Tube, Support and Drive Link, Chrome - Right                | 1            | 1        |
| 1           |      | 1022878-NLA | NLA - Tube, Support and Drive Link, Chrome - Left                 | 1            | 1        |
| 2           |      | 1024911-NLA | NLA - Decal, Smartleg                                             | 1            | 1        |
| 3           |      | 1014807     | Screw, Hex Head (5/16-24 x 3-1/4")                                | 1            | 1        |
| 4           |      | 47330X003   | Spacer (5/16 x 1/2 x 3/8")                                        | 1            | 1        |
| 5           |      | 7930363-NLA | NLA - Package, Locknut (5/16"-24)                                 | 25           | 2        |
| 6           |      | 1027377     | Washer (#20)                                                      | 1            | 1        |
| 7           |      | 11232X009   | Bolt, Lug (5/16-24 x 1-3/8")                                      | 1            | 1        |
| 8           |      | 1025454     | Housing, Latch - Right                                            | 1            | 1        |
| 8           |      | 1025455     | Housing, Latch - Left                                             | 1            | 1        |
| 9           |      | 1025451     | Screw, Slotted Hex Washer Head (#10-32 x 5/8")                    | 1            | 2        |
| 10          |      | 7930314     | Package, Screw, Slotted Hex Washer Head (#10-32 x 1/2")           | 25           | 1        |
| 11          |      | 7930333     | Locknut (1/4-20)                                                  | 25           | 1        |
| 12          |      | 1029501     | Plug Button, Black (1")                                           | 1            | 1        |
| 13          |      | 1027097     | Plug Button, Black (3/4")                                         | 1            | 1        |
| 14          |      | 1082538-NLA | NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Right | 1            | 1        |
| 14          |      | 1082539-NLA | NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Left  | 1            | 1        |
| 15          |      | 1022854-NLA | NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)         | 1            | 1        |
| 16          |      | 40363X000   | Hex Nut (3/8"-16)                                                 | 1            | 3        |
| 17          |      | 1025104     | Bracket, Legrest, Chrome                                          | 1            | 1        |
| 18          |      | 04526X019   | Plate, Legrest Pad                                                | 1            | 1        |
| 19          |      | 1022848     | Tube, Slide, Articulating, Chrome - Right                         | 1            | 1        |
| 19          |      | 1022849     | Tube, Slide, Articulating, Chrome - Left                          | 1            | 1        |
| 20          |      | 1022866-NLA | NLA - Screw, Socket Button Head (3/8-16 x 3/4")                   | 1            | 2        |
| 21          |      | 1049527     | Washer, Nylon (3/8 x 3/4 x 1/16")                                 | 1            | 2        |
| 22          |      | 1025133     | Spacer, Legrest Pad, Black                                        | 1            | 1        |
| 23          |      | 1022846-NLA | NLA - Bar, Link, Black                                            | 1            | 1        |
| 24          |      | 47328X003   | Bushing, Pivot (5/16 x 7/16 x 7/8")                               | 1            | 1        |
| 25          |      | 48823X008   | Spring, Leaf                                                      | 1            | 1        |
| 26          |      | 1017148     | Pivot, Footrest                                                   | 1            | 1        |
| 27          |      | 7930301-NLA | NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")            | 25           | 1        |
| 28          | A    | 1082540     | Articulating Legrest Support Assembly, Sector, Chrome - Right     | 1            | 1        |
| 28          | A    | 1082541     | Articulating Legrest Support Assembly, Sector, Chrome - Left      | 1            | 1        |

NOTE: A - Includes items 1-27 and Ratchet Assembly