


Power Articulating/Non-Articulating Legrest Harnesses & Hardware

POWER18
POWER20
ELRPW

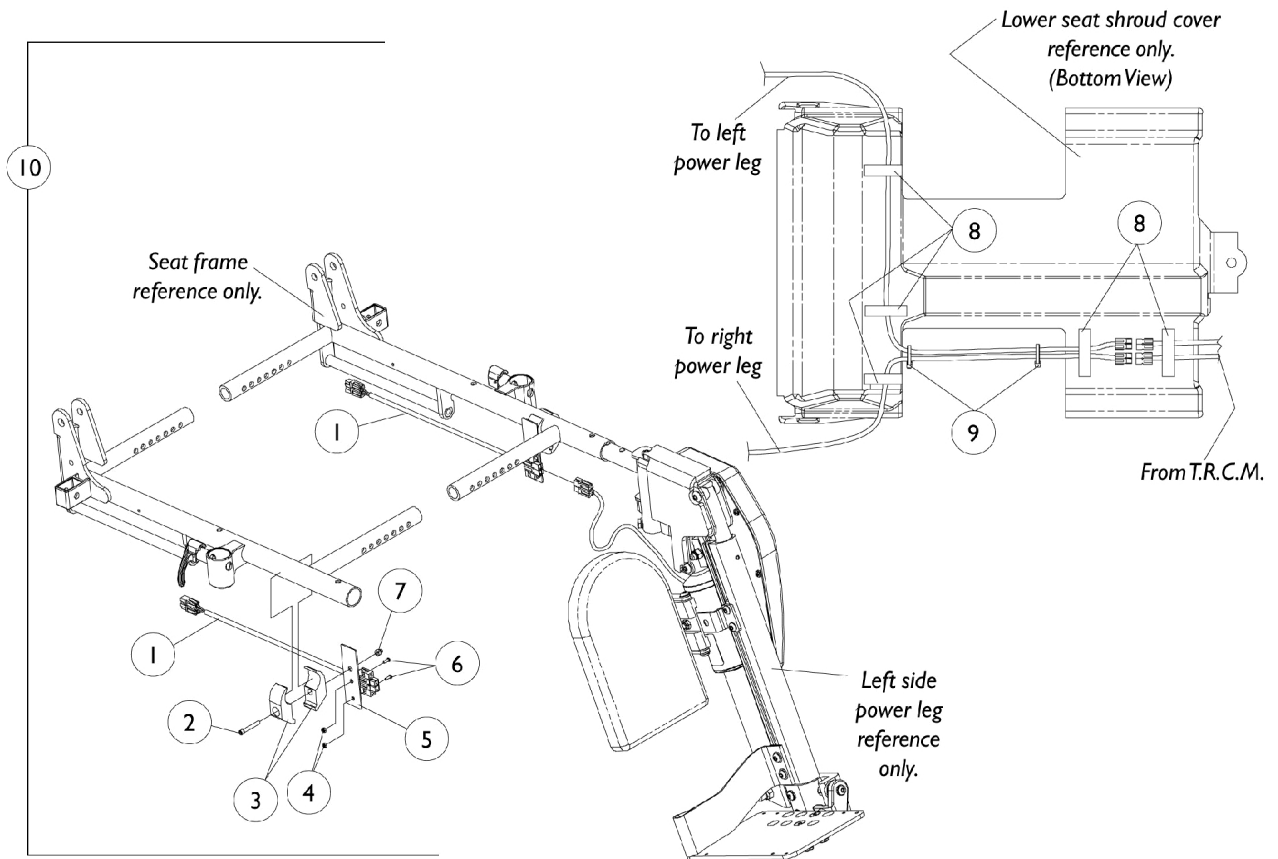

Power Articulating/Non-Articulating Legrest Harnesses & Hardware

POWER18
POWER20
ELRPW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1101400	Harness, ELRPW Power Legrest - Right	1	1
1		1101401	Harness, ELRPW Power Legrest - Left	1	1
2		1003424	Screw, Socket Head (1/4-20 x 1-3/4")	1	2
3		1018692-NLA	NLA - Clamp, Half, Non-Threaded	1	4
4		1056098	Locknut (#8-32)	1	4
5		1102022-NLA	NLA - Bracket, Harness Wire, Black	1	2
6		1091871-NLA	NLA - Screw, Socket Head (#8-32 x 5/8")	1	4
7		7930338-NLA	NLA - Nut, Acorn (1/4-20)	25	1
8		1070509-NLA	NLA - Fastener, Loop with Adhesive Back (1" W x 2" L)	1	5
9		2001682	Tie Wrap (8" L)	1	2
10	A	1101520-NLA	NLA - Kit, Power Leg Harness and Mounting Hardware	2	1

NOTE: A - Includes Items 1-9