


I.V./Oxygen Holder - 1496 SX Oxygen Holder - 1497 SX

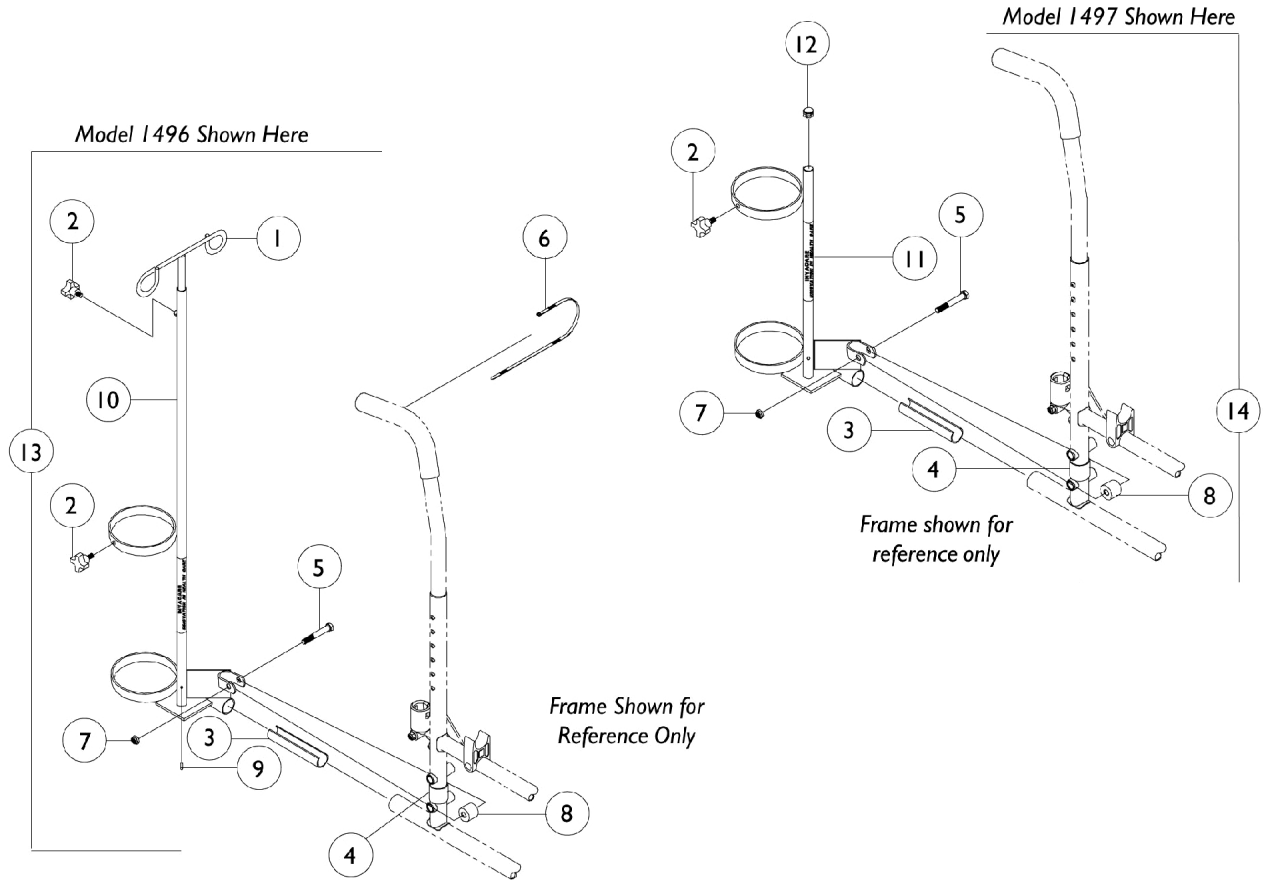

I.V./Oxygen Holder - 1496 SX Oxygen Holder - 1497 SX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027468	Rod, I.V.	1	1
2	A	1027683	Knob, Adjustment, Black	1	1
2	B	1027684	Knob, Clamping, Black	1	1
3		1023560	Insert, Guide Tube, Black (3-3/4" Long)	1	1
4		1023618	Insert, Guide Tube, Black (1" Long)	1	1
5		11239X000-NLA	NLA - Screw, Hex Head (5/16-24 x 1-3/4")	1	1
6		98007X005	Tie Wrap (11-1/2" L)	1	1
7		1107321	Nut, Low Cap (5/16"-24)	1	1
8		1102577	Spacer, Black (5/16 x 11/16 x 15/16")	1	1
9		1027789	Pin, Roll	1	1
10		1070898	Base, I.V./Oxygen	1	1
11		1070900-NLA	NLA - Base, Oxygen Holder	1	1
12		1017672	Plug Button, Grey (1/2")	1	1
13	C	1497	Holder, Oxygen, Black (D and E Tanks)	1	1
14	D	1496	Holder, I.V./Oxygen, Chrome	1	1
	E	1027931	Instruction Sheet, I.V./Oxygen Holder	1	1
	E	1027930	Instruction Sheet, Oxygen Holder	1	1
	E	1027708-NLA	NLA - Decal, Invacare	1	1

NOTE: A - Used on I.V./Oxygen Holder.
B - Used on Oxygen Holder.
C - Includes items 1-13 and an instruction sheet.
D - Includes items 2-5, 7-8, 11-12 and an instruction sheet.
E - Not shown.