
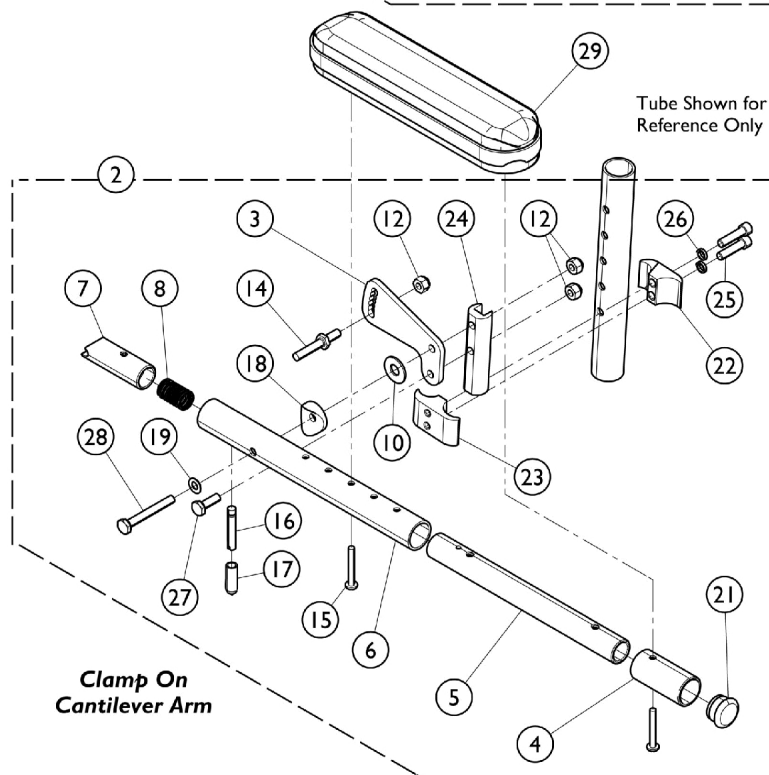


Cantilever Arms

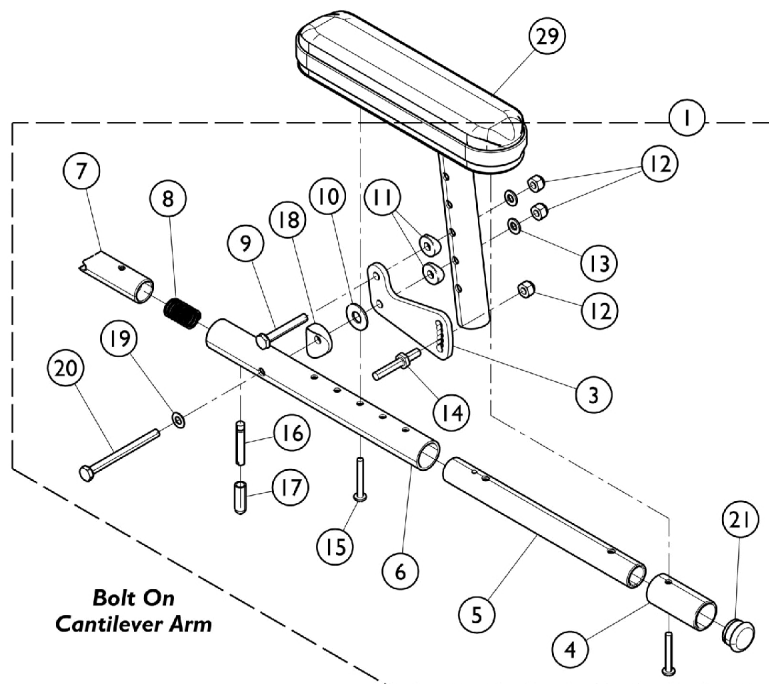

Cantilever Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1068654	Kit, Cantilever Arm, Bolt On - Right	1	1
1	A	1068655	Kit, Cantilever Arm, Bolt On - Left	1	1
2	B,D	1068656	Arm Assembly, Cantilever - Right	1	1
			Bracket, Armrest Mounting, Black	1	1
			Tube, Arm, Extension, Black	1	1
			Tube, Slide, Cantilever Arm, Black	1	1
			Tube, Arm, Black	1	1
			Plug, Latch	1	1
			Spring, Compression (5/8 x 1 x 3/64")	1	1
			Washer, Nylon (3/8 x 7/8 x 1/16")	1	1
			Locknut (1/4-28)	1	3
			NLA - Pin, Locking	1	1
			Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
			Lever, Actuator	1	1
			Tip, Plastic, Black	1	1
			Spacer, Coved (1/4 x 1 x 9/16")	1	1
			Washer, Nylon (1/4 x 1/2 x 1/32")	1	1
			Plug Button, Black (1")	1	1
			Clamp, Half, Non-Threaded	1	1
			Clamp, Half, Threaded	1	1
			Arm Support, Clamp On	1	1
			Screw, Socket Head (1/4-20 x 1")	1	2
			NLA - Lockwasher, High Collar (1/4")	1	2
			NLA - Screw, Hex Head (1/4-28 x 3/4")	1	1
			Screw, Hex Head (1/4-28 x 2")	1	1
2	B,D	1068657	Arm Assembly, Cantilever - Left	1	1
			Bracket, Armrest Mounting, Black	1	1
			Tube, Arm, Extension, Black	1	1
			Tube, Slide, Cantilever Arm, Black	1	1
			Tube, Arm, Black	1	1
			Plug, Latch	1	1
			Spring, Compression (5/8 x 1 x 3/64")	1	1
			Washer, Nylon (3/8 x 7/8 x 1/16")	1	1
			Locknut (1/4-28)	1	3
			NLA - Pin, Locking	1	1
			Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
			Lever, Actuator	1	1
			Tip, Plastic, Black	1	1
			Spacer, Coved (1/4 x 1 x 9/16")	1	1
			Washer, Nylon (1/4 x 1/2 x 1/32")	1	1
			Plug Button, Black (1")	1	1
			Clamp, Half, Non-Threaded	1	1
			Clamp, Half, Threaded	1	1
			Arm Support, Clamp On	1	1
			Screw, Socket Head (1/4-20 x 1")	1	2
			NLA - Lockwasher, High Collar (1/4")	1	2
			NLA - Screw, Hex Head (1/4-28 x 3/4")	1	1

Cantilever Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
3		1058542	Screw, Hex Head (1/4-28 x 2")	1	1
4		1042598	Bracket, Armrest Mounting, Black	1	1
5		1068600	Tube, Arm, Extension, Black	1	1
6		1068588	Tube, Slide, Cantilever Arm, Black	1	1
7		1068588	Tube, Arm, Black	1	1
8		1058232	Plug, Latch	1	1
9		1011511	Spring, Compression (5/8 x 1 x 3/64")	1	1
10		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	1
11		1012371	Washer, Nylon (3/8 x 7/8 x 1/16")	1	1
12		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
13		1065362	Locknut (1/4-28)	1	3
14		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
15		1058231-NLA	NLA - Pin, Locking	1	1
16		1026107	Screw, Phillips Pan Head (#10-32 x 1-3/8")	1	2
17		1058545	Lever, Actuator	1	1
18		1059271	Tip, Plastic, Black	1	1
19		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	1
20	C	1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	1
21		1059374	Screw, Hex Head (1/4-28 x 3")	1	1
22		1056328	Plug Button, Black (1")	1	1
23		1043711	Clamp, Half, Non-Threaded	1	1
24		1043712	Clamp, Half, Threaded	1	1
25		1062787	Arm Support, Clamp On	1	1
26		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
27		1013970-NLA	NLA - Lockwasher, High Collar (1/4")	1	2
28	D	1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	1
29	E	1055349	Screw, Hex Head (1/4-28 x 2")	1	1
29	E	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
29	E	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1

NOTE: A - Includes items 3 - 20.
 B - Includes items 3 - 8, 10, 12, 14 - 17, 19, and 21 - 28.
 C - Bolt On Arm.
 D - Clamp On Arm.
 E - Specify Upholstery Color U67 when ordering.

Cantilever Arm Hardware accommodates the Desk and Full Length Arm Pads. The hardware is adjustable.