


Caster Head Tube Mounting Hardware

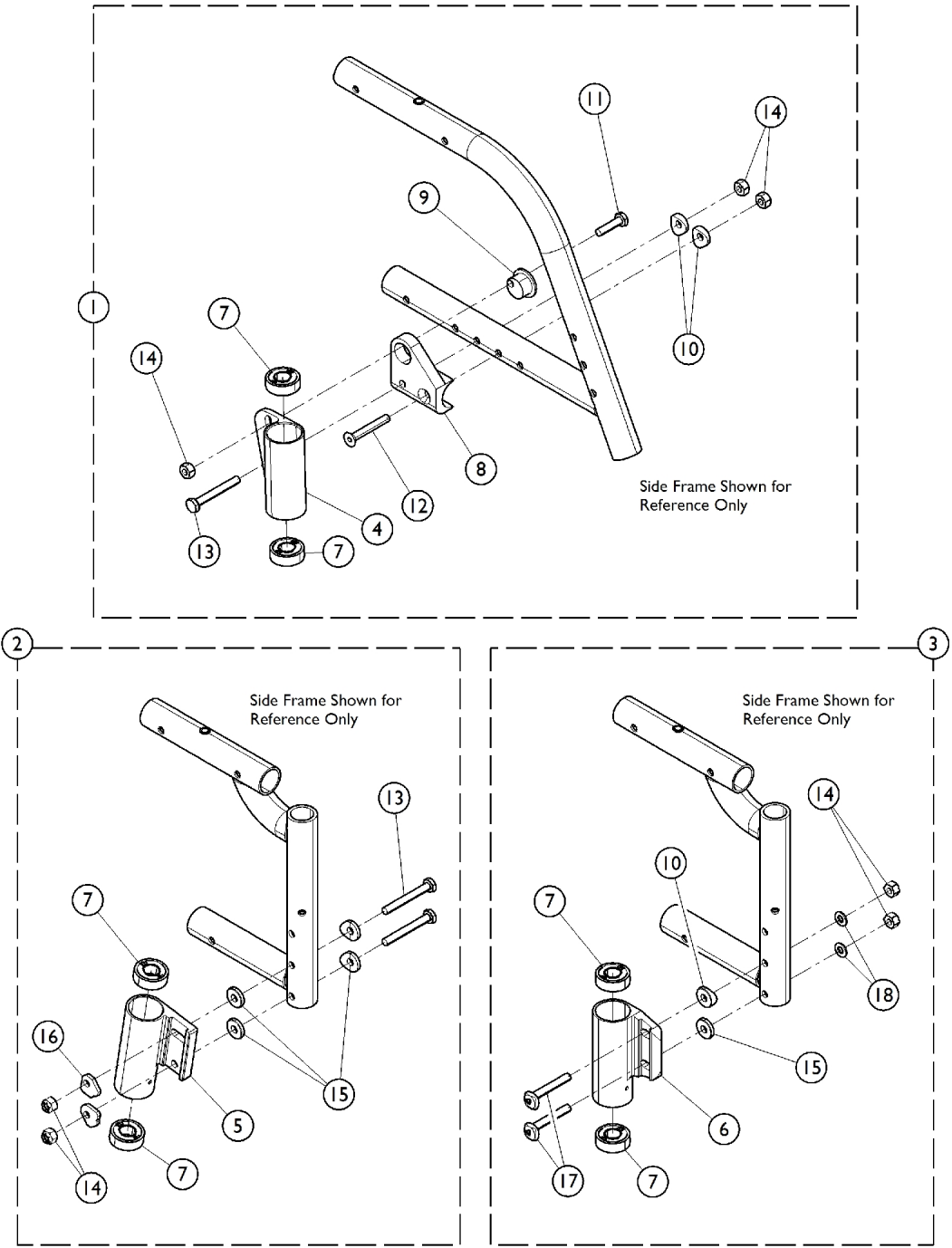

Caster Head Tube Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,D	1046147-NLA	NLA - Kit, Caster Headtubes w/ Hardware (Pair)	1	1
2	A,E	1061060-NLA	NLA - Kit, Caster Headtubes w/ Hardware (Pair)	1	1
3	B,F	1114704-NLA	NLA - Kit, Caster Headtubes w/ Hardware (Pair)	1	1
4		1076369-NLA	NLA - Headtube, Caster, Black - Right	1	1
4		1076370-NLA	NLA - Headtube, Caster, Black - Left	1	1
5	G	1066604-NLA	NLA - Headtube, Caster, Black - Right	1	1
5	G	1066605-NLA	NLA - Headtube, Caster, Black - Left	1	1
6	H	1113576	Headtube, Caster, Black - Right	1	1
6	H	1113577	Headtube, Caster, Black - Left	1	1
7		40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	2
8		1040759-NLA	NLA - Bracket, Headtube Adjustment, Black - Right	1	1
8		1040760-NLA	NLA - Bracket, Headtube Adjustment, Black - Left	1	1
9		1040766	Cam, Headtube Adjustment	1	1
10		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
11		1055296-NLA	NLA - Screw, Hex Head (1/4-28 x 1")	1	1
12		1055332-NLA	NLA - Screw, Socket Flat Head (1/4 -28 x 1-3/4")	1	1
13		1055349	Screw, Hex Head (1/4-28 x 2")	1	2
14		1055366	Locknut (1/4-28)	1	4
15		1009861	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
16		1040355	Cam, Headtube Adjustment	1	1
17		1113501	Screw, Eccentric (1/4-28 x 1-11/16")	1	4
18		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
NOTE: A - Swing Away Front Frames manufactured before (11-2002). B - Swing Away Front Frames manufactured after (11-2002). C - Fixed Front Frames. D - Includes items 4, 7, 8, and 9 thru 14. E - Includes items 5, 7, 13, 14, 15, and 16. F - Includes items 6, 7, 10, 14, 15, 17, and 16. G - For use with Caster Swivel Locks. H - Not for use with Caster Swivel Locks.					