
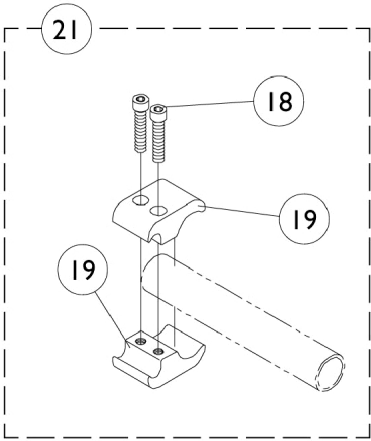


High Mount Wheel Locks

Pull to Lock

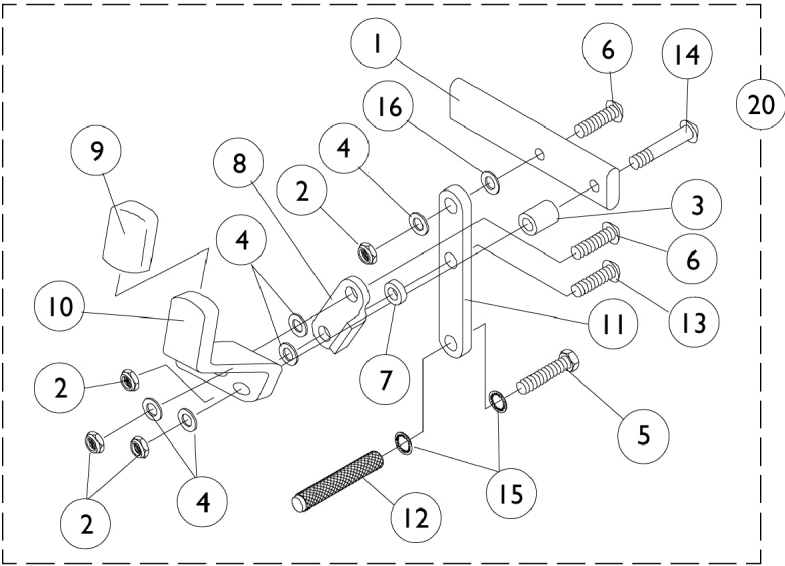

High Mount Wheel Locks

Pull to Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1028399-NLA	NLA - Strap, Wheel Lock	1	1
1	D	1028397-NLA	NLA - Strap, Wheel Lock	1	1
2		7930334-NLA	NLA - Locknut (1/4-28)	25	4
3		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	5
5		1080135-NLA	NLA - Screw, Hex Head w/ Patch (1/4-28 x 3/4")	1	1
6		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
7		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
8		1018733-NLA	NLA - Link, Stop, Black	1	1
9		1006490	Tip, Plastic, Black	1	2
10		1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
10		1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
11		1028416-NLA	NLA - Link, Wheel Lock, Black	1	1
12		1029420-NLA	NLA - Shoe, Wheel Lock	1	1
13		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
14		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
15		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	2
17	E	1026907	Washer (1/4 x 1/2 x 1/16")	1	1
18		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
19		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
19		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
20	A	1038552-NLA	NLA - Wheel Lock, Clamp On, Pull to Lock - Right	1	1
20	A	1038553	Wheel Lock, Clamp On, Pull to Lock - Left	1	1
21	B	1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
18			Screw, Allen Head (1/4-20 x 1")	1	4
19			NLA - Clamp, Wheel Lock, Threaded (1")	1	2
19			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
NOTE: A - Includes items 1 - 17. B - Includes items 18 - 19. C - Swingaway Front Frames D - Rigid Front Frames E - Not Shown.					