


Footrest Support Assembly

90 Degree Fixed

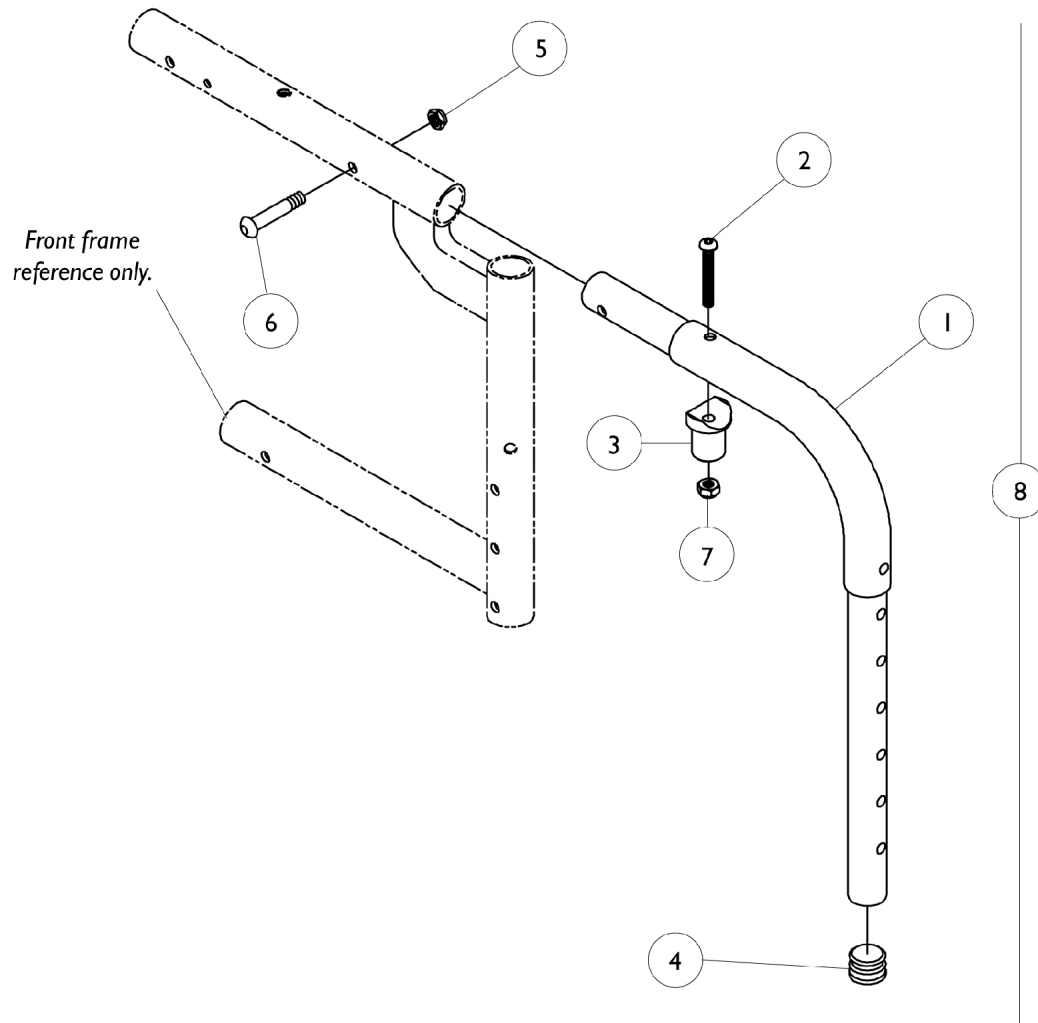

Footrest Support Assembly 90 Degree Fixed

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881029426-NL A	NLA - Tube, Support, Color	1	2
2		1055309	Screw, Socket Button Head (1/4-28 x 1-1/4")	1	2
3		1017148	Pivot, Footrest	1	2
4		1027097	Plug Button, Black (3/4")	1	2
5		1025736	Locknut (1/4-20)	1	2
6		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	2
7		1025765	Locknut (1/4-28)	1	2
8	A	8881029462-NL A	NLA - Footrest Support Assembly w/ Hardware, 90 Degree, Color - Right/Left	2	1
NOTE: A - Specify frame finish when ordering					