


Adjustable Angle Footplate Assemblies (AT5543)

Chairs Built Before 11/02

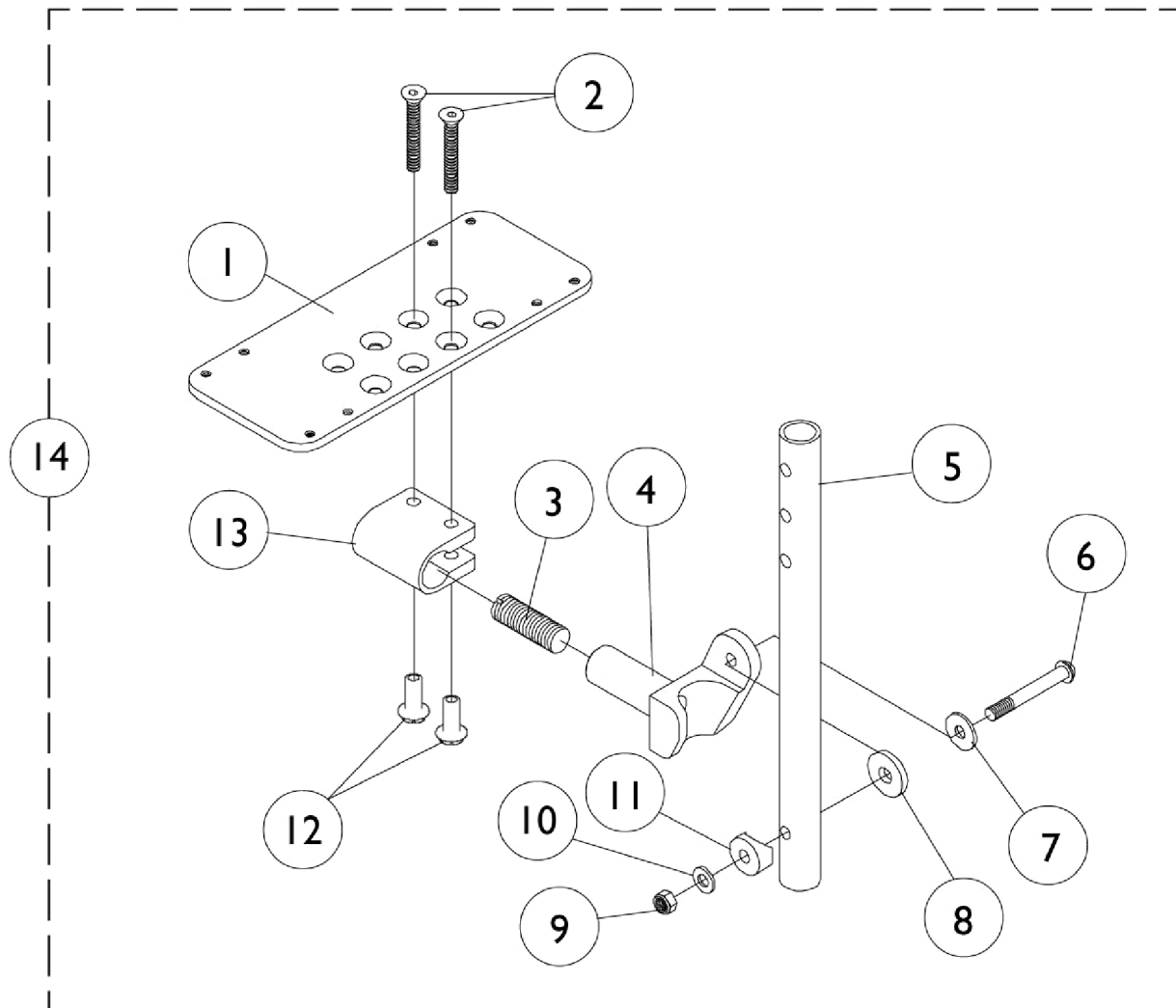

Adjustable Angle Footplate Assemblies (AT5543)

Chairs Built Before 11/02

Adjustable Angle Footplates with or without Pivot Slide Tubes - AT5543

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060425-NLA	NLA - Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
1		1060426-NLA	NLA - Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
1		1060427-NLA	NLA - Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
1		1062569-NLA	NLA - Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
1		1068617-NLA	NLA - Footplate, Adjustable Angle, XX-Large, Black (7-1/4" W x 8" L)	1	1
2		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
4		1052102	Hinge, Footplate - Right	1	1
4		1052103	Hinge, Footplate - Left	1	1
5	A	1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
5	A	1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
5	B	1091745-NLA	NLA - Tube, Pivot/Slide, Straight, Steel, Chrome (13-7/8" L)	1	1
6		1022831	Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
7		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
8		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
9		1025736	Locknut (1/4-20)	1	1
10		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
11		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
12		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
13		1058517	Clamp, Footplate	1	1
14		See Chart	#14 60 Degree and 80 Degree Fixed Front	1	1
14		See Chart	#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's	1	1
14		See Chart	#14 70 Degree Tapered	1	1
14		See Chart	#14 90 Degree and 70 Degree MFX	1	1
14		See Chart	#14 Articulating Legrests	1	1
	C	1060474	Instruction Sheet	1	1

NOTE: A - Swingaway Front Riggings
B - Articulating Legrest
C - Not Shown

#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's

Straight Slide Tube (9-7/8")	Right	Left
12-13" Wide (4-1/4" Footplate)	1060430	1060431
14-15" Wide (5-1/4" Footplate)	1060432	1060433
16-20" Wide (6-1/4" Footplate)	1062572	1062573

Adjustable Angle Footplate Assemblies (AT5543)

Chairs Built Before 11/02

#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered and ELR's

Straight Slide Tube (9-7/8")	Right	Left
18-22" Wide (7-1/4" Footplate)	1068620	1068621
ALPT Straight Slide Tube (Aluminum 11-7/8")	Right	Left
12-13" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
14-15" Wide (5-1/4" Footplate)	1060440-NLA	1060441
16-18" Wide (6-1/4" Footplate)	1062578	1062579
19-22" Wide (7-1/4" Footplate)	1068622	1068623

#14 70 Degree Tapered

Straight Slide Tube (9-7/8")	Right	Left
16" Wide (4-1/4" Footplate)	1060430	1060431
17-18" Wide (5-1/4" Footplate)	1060432	1060433
19-22" Wide (6-1/4" Footplate)	1062572	1062573
ALPT Straight Slide Tube (Aluminum 11-7/8")	Right	Left
16" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
17-18" Wide (5-1/4" Footplate)	1060440-NLA	1060441
19-22" Wide (6-1/4" Footplate)	1062578	1062579

Adjustable Angle Footplate Assemblies (AT5543)

Chairs Built Before 11/02

#14 60 Degree and 80 Degree Fixed Front

Straight Slide Tube (5-1/2")	Right	Left
12-13" Wide (3-1/4" Footplate)	1061974-NLA	1061975-NLA
14-15" Wide (4-1/4" Footplate)	1061978-NLA	1061979-NLA
16-17" Wide (5-1/4" Footplate)	1062586	1062587
18-22" Wide (7-1/4" Footplate)	1060432	1060433

#14 90 Degree and 70 Degree MFX

Adjustable Angle Footplates w/o Pivot Slide Tubes

No Slide Tube Bolts Through Support	Right	Left
12-13" Wide (4-1/4" Footplate)	1060436-NLA	1060437
14-15" Wide (5-1/4" Footplate)	1060438	1060439
16-17" Wide (6-1/4" Footplate)	1062570	1062571
18-22" Wide (7-1/4" Footplate)	1068618	1068619
Item 5 is not included. It is not required due to the fact that the footplate assembly bolts directly through the footrest support tube.		

#14 Articulating Legrests

Articulating	Right	Left
16-17" Wide (5-1/4" Footplate)	1091775-NLA	1091776-NLA
18-19" Wide (6-1/4" Footplate)	1091777-NLA	1091778-NLA
20-22" Wide (7-1/4" Footplate)	1091779-NLA	1091780-NLA