
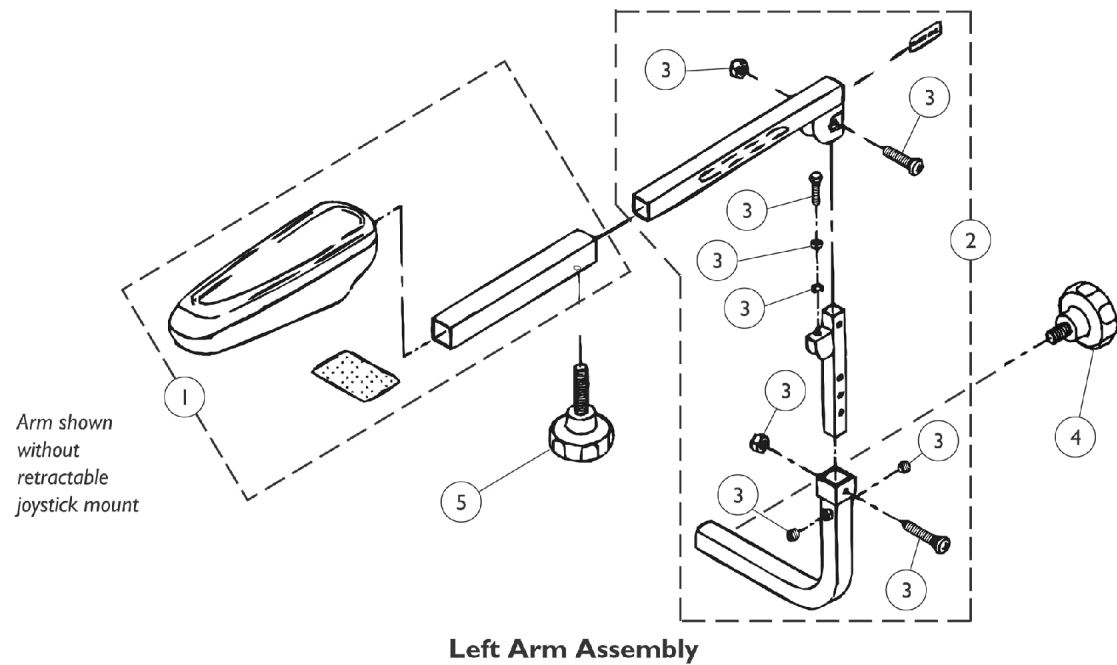


Arm Assembly

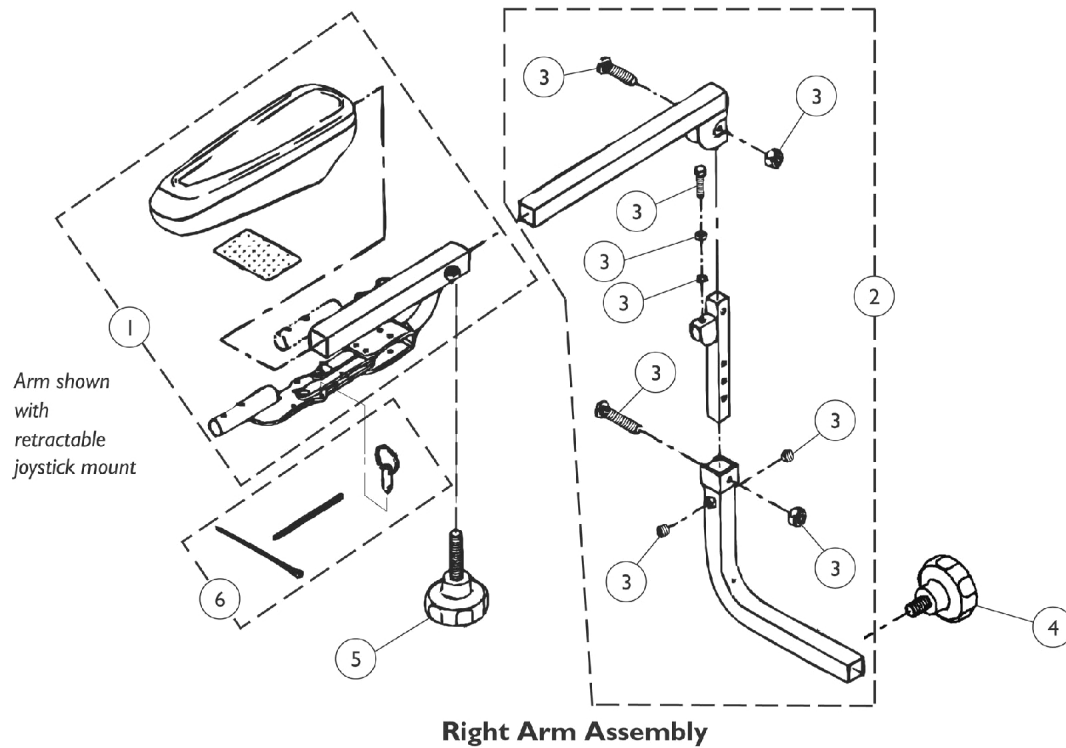

Arm Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099748-NLA	NLA - Pad, Arm with Arm Tube Sleeve and Retractable Joystick Mount - Right	1	1
1		1099749	Pad, Arm w/ Arm Tube Sleeve and Retractable Joystick Mount - Left	1	1
1		1099747-NLA	NLA - Pad, Arm w/ Arm Tube Sleeve - Right/Left	1	1
2		1099751-NLA	NLA - Arm Assembly without Arm Pad - Right	1	1
2		1099750-NLA	NLA - Arm Assembly without Arm Pad - Left	1	1
3		1099746-NLA	NLA - Arm Hardware Package (Pair) Includes: 4 ea. Screw, Nylok Socket (M8 x 1.25 x 8) 2 ea. Screw, Phillips Head (M8 x 1.25 x 40) 4 ea. Locknut (M8 x 1.25 x 13) 2 ea. Screw, Phillips Head (M8 x 1.25 x 32) 2 ea. Washer, Spring (M8 x 13) 2 ea. Nut (M8 x 1.25 x 13) 2 ea. Screw, Hex Head Cap (M8 x 1.25 x 35)	2	2
4	A	1088768-NLA	NLA - Knob, Lower Arm Tightening	1	1
5	B	1088769-NLA	NLA - Knob, Upper Arm Tightening	1	1
6		1099769-NLA	NLA - Hardware, Connecting Pin Retractable Joystick Mount Includes: 5 ea. Connecting Pin 5 ea. Chain 5 ea. Cable Tie	5	5
NOTE: A - Width adjustment. B - Depth adjustment.					