



# Link-Pivot Style Wheel Locks

Chrome Chairs - Before 9/1/05

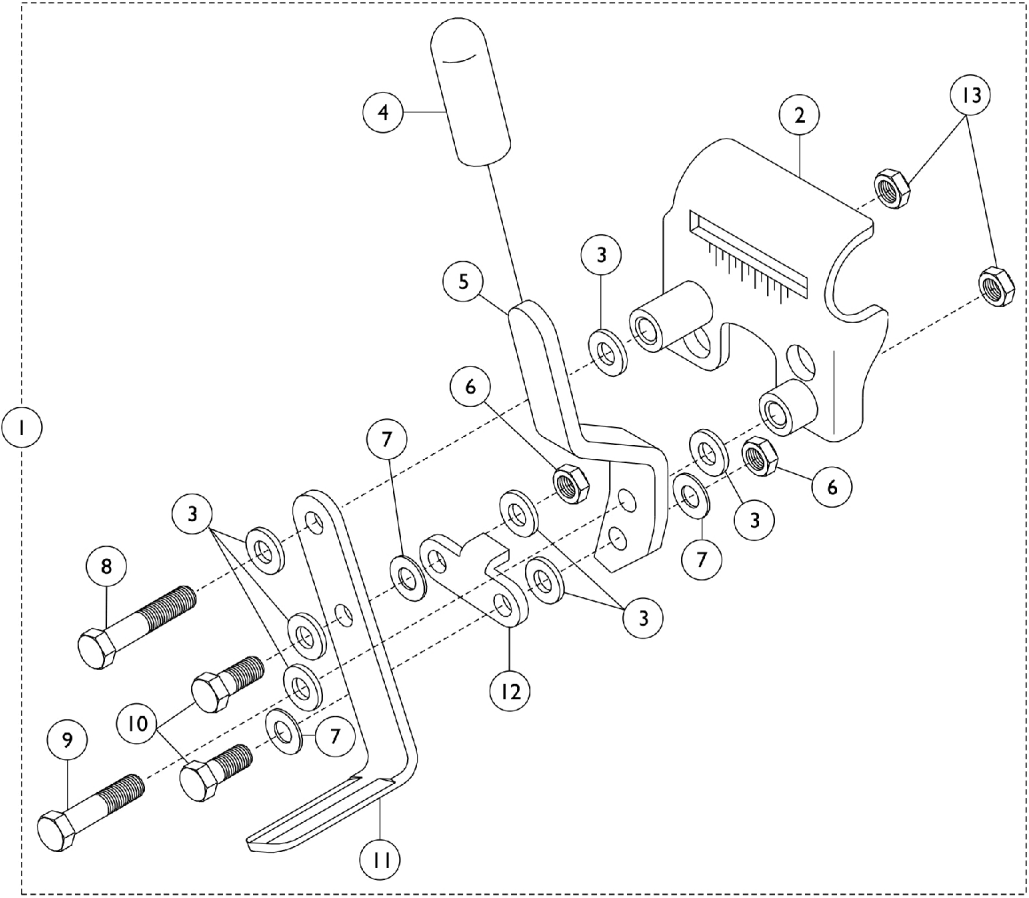

Attaching Hardware

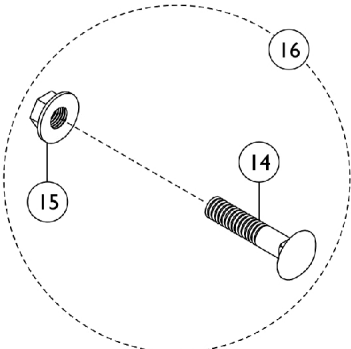

## Link-Pivot Style Wheel Locks

### Chrome Chairs - Before 9/1/05

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,E	1045853	Wheel Lock Assembly, Adult, Push To Lock, Black - Right	1	1
1	C,E,H	1045854	Wheel Lock Assembly, Adult, Push To Lock, Black - Left	1	1
1	D,F,H	1045849-NLA	NLA - Wheel Lock Assembly, Hemi, Pull to Lock, Black - Right	1	1
1	D,F	1045850-NLA	NLA - Wheel Lock Assembly, Hemi, Pull to Lock, Black - Left	1	1
1	D,F	1101454	Wheel Lock Assembly, Pull to Lock, Black - Right	1	1
1	D,F	1101455	Wheel Lock Assembly, Pull to Lock, Black - Left	1	1
2		1045847-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
2		1045848-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
3		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	7
4		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
5	E	1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
5	E	1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
5	F	1016590-NLA	NLA - Handle, Pull, Hemi, Chrome - Right	1	1
5	F	1016591-NLA	NLA - Handle, Pull, Hemi, Chrome - Left	1	1
6		60320X000	Locknut (1/4-28)	1	1
7		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	1
8		16720X004	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
9		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
10		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
11		1015443-NLA	NLA - Shoe	1	1
12		1016592-NLA	NLA - Link, Chrome	1	1
13		60275X000	Locknut (1/4-20)	1	1
14		1011479	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
15		16721X000	Locknut (1/4-20)	1	1
16	B	1034165-NLA	NLA - Hardware, Attaching Kit (Pair)	1	1
			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
			Locknut (1/4-20)	1	2
16		1017677	Kit, Hardware, Wheel Lock (Pair)	1	1
			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
			Nut, Hex Flang (1/4-20)	1	2
	A	00078X021	Instruction Sheet, Wheel Lock	1	1
	A,G	1058900	Wheel Lock Extension Handle Assembly, Black (8")	1	1

NOTE: A - Not Shown.  
 B - Includes Items 14-15. 1034165 in Zinc Finish is NLA. Use part number 1017677 in Black finish as a replacement.  
 C - Includes Items 2-13.  
 D - Includes Items 2-13  
 E - Adult.  
 F - Hemi/Super Hemi.  
 G - Includes: 1 each Extension Handle and Rubber Tip. Chrome is No Longer Available.  
 H - NLA. See part numbers 1101454 and 1101455 as replacements.

NOTE: The chrome finish is no longer available for chairs manufactured after 1/30/01.