


Crossbrace and Crossbrace Hardware

9000SL - Before 7/12/99

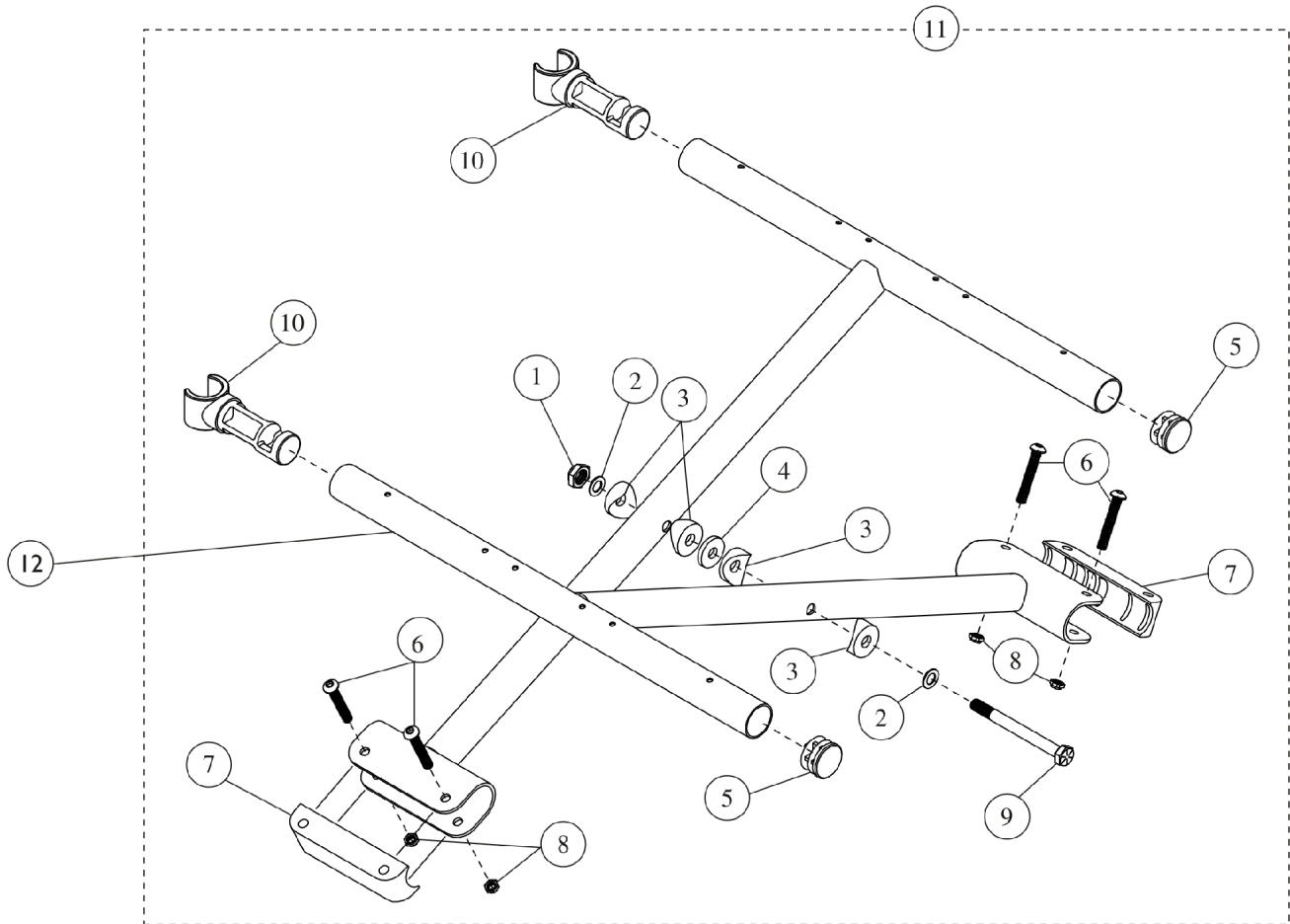

Crossbrace and Crossbrace Hardware

9000SL - Before 7/12/99

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60264X001	Locknut (5/16-24)	1	1
2		1016652	Washer (5/16 x 9/16 x 1/32")	1	2
3		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
4		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
5		1029501	Plug Button, Black (1")	1	2
6		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	4
7		1012172-NLA	NLA - Clamp, Saddle	1	2
8		60275X000	Locknut (1/4-20)	1	4
9		1016261	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
10		1055419-NLA	NLA - Guide, Seat	1	2
11	B	1054887-NLA	NLA - Kit, Crossbrace Hardware	2	1
12		1090818-NLA	NLA - Crossbrace w/o Hardware, Black - Right/ Left (16" W)	1	1
12		1090820-NLA	NLA - Crossbrace w/o Hardware, Black - Right/ Left (18" W)	1	1
12		1090822-NLA	NLA - Crossbrace w/o Hardware, Black - Right/ Left (20" W)	1	1
		1019027	Kit, Extension, Seat (2")	1	1
NOTE: A - Includes items 1-10. B - Not Shown.					
NOTE: Chairs that are 18" deep use a 2" seat extension kit. See category titled, "Seat Extension Kit."					