


Footrest Support Assembly

H903, T943A/C/S

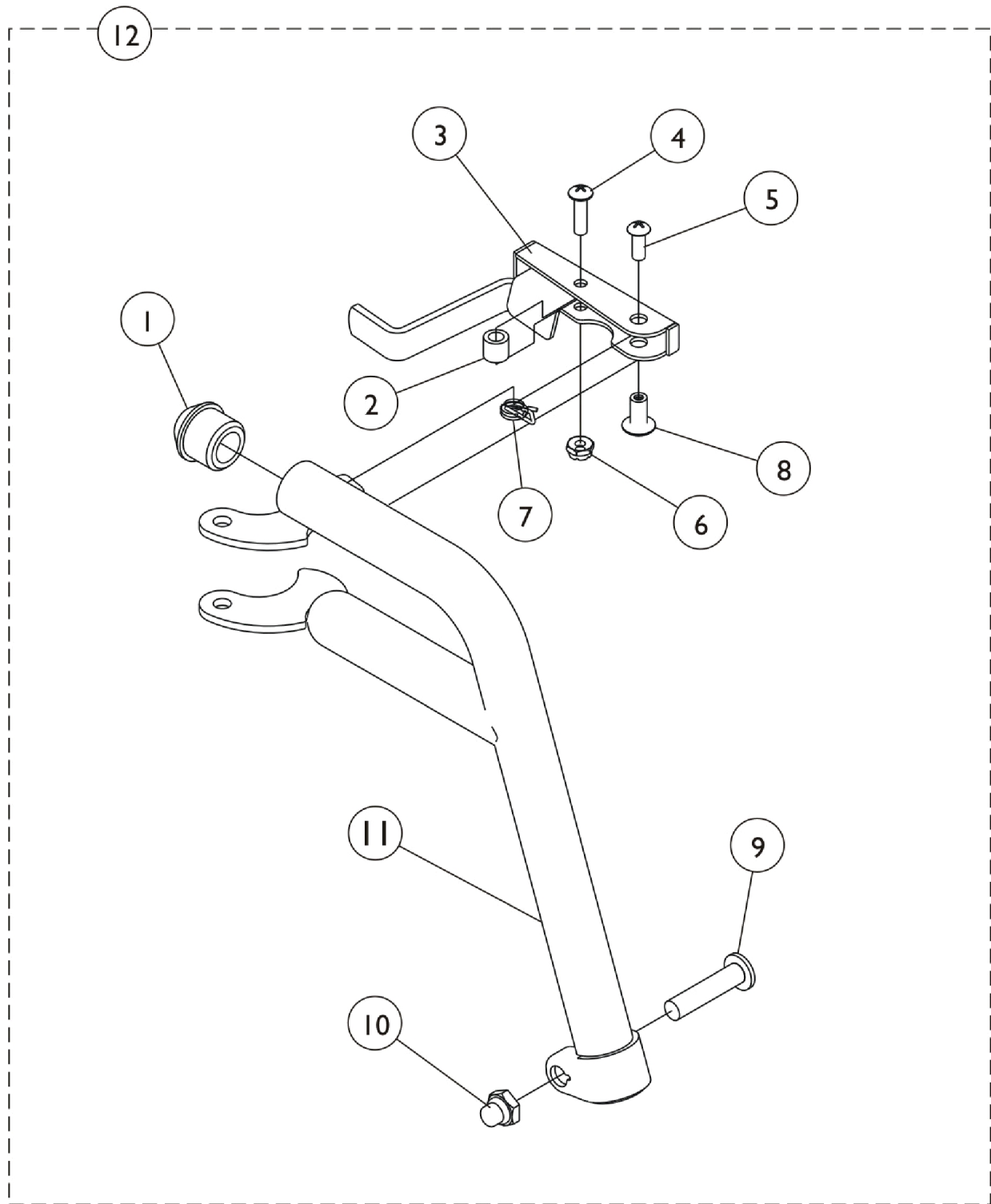

Footrest Support Assembly H903, T943A/C/S

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020719	Anti-Rattle, Black	1	1
2		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
3	A,C	1033305-NLA	NLA - Handle, Release, Chrome - Right	1	1
3	A	1033306-NLA	NLA - Handle, Release, Chrome - Left	1	1
3	A,C,D	1031469-NLA	NLA - Handle, Release, Black - Right	1	1
3	A,C,D	1031470-NLA	NLA - Handle, Release, Black - Left	1	1
	E	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
	E	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
4		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
4	D	1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
5	D	1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
6		60279X003	Locknut (#10-32)	1	1
6	D	1025735	Locknut (#10-32)	1	1
7		1008935	Spring, Release - Right	1	1
7		1008936	Spring, Release - Left	1	1
7	D	1028707	Spring, Release - Right	1	1
7	D	1028708	Spring, Release - Left	1	1
8		67005X000	Rivnut (#10-32)	1	1
8	D	1028846-NLA	NLA - Rivnut (#10-32)	1	1
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
9	D	1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		1107321	Nut, Low Cap (5/16"-24)	1	1
11		1020416-NLA	NLA - Tube, Support, Footrest, Chrome - Right	1	1
11		1020417-NLA	NLA - Tube, Support, Footrest, Chrome - Left	1	1
11	D	1020428-NLA	NLA - Tube, Support, Footrest, Black - Right	1	1
11	D	1020429-NLA	NLA - Tube, Support, Footrest, Black - Left	1	1
11	B,D	8881065636	Tube, Support, Footrest, Color - Right	1	1
11	B,D	8881065637	Tube, Support, Footrest, Color - Left	1	1
12		See Chart	Footrest Support Assembly - H903, T943A/C/S	1	1

NOTE: A - When front rigging are ordered for a 9000XT Chair with clamp-on wheel locks, the front riggings should be ordered through Invacare Specials due to a different release handle style.
B - Specify frame color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."
C - Release Handles used on 9000XT with Clamp-on Wheellock After 8/1/02.
D - For use on black or colored chairs.
E - Not shown. Order this item when ordering chair with Cyclical Lever Drive only.

NOTE: Due to design change, for all chairs built prior to 10/01/00 it is suggested that release handles be replaced in pairs.

Footrest Support Assembly - H903, T943A/C/S

Prior Model Number	Current Model Number	Chair Width	Right Side	Left Side
H903, T943 A/C Silver Vein	T93H A/C	14-22"	1030706-NLA	1030707-NLA
H903, T943 A/C Black	T93H A/C	14-22"	1034139	1034140

Footrest Support Assembly

H903, T943A/C/S

Footrest Support Assembly - H903, T943A/C/S

Prior Model Number	Current Model Number	Chair Width	Right Side	Left Side
H903	T943 A/C	14-22"	8881032955	8881032956