


Footrest Support Assembly

SH93ES, SH93E

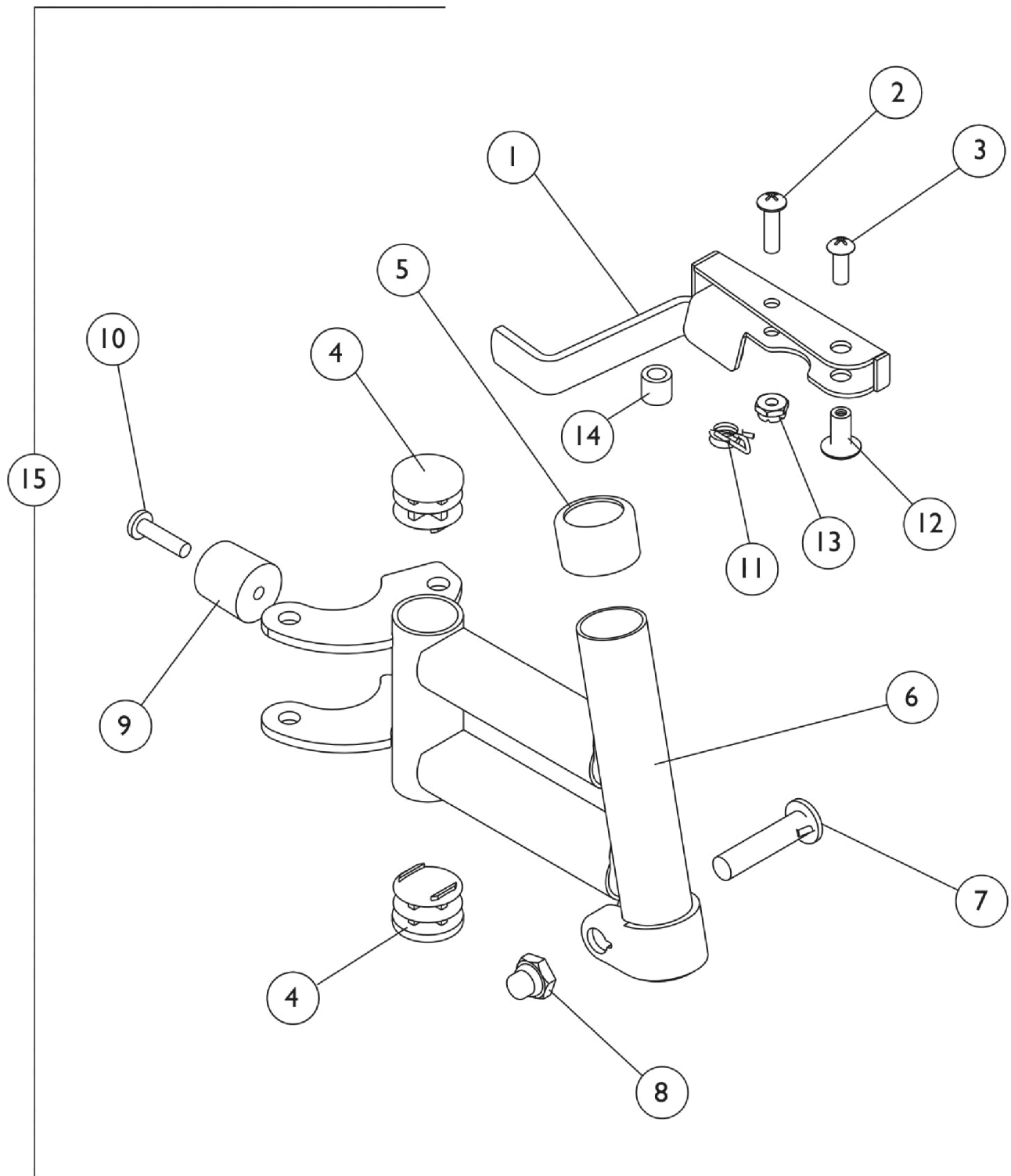

Footrest Support Assembly SH93ES, SH93E

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1124549-NLA	NLA - Handle, Release, Chrome - Right	1	1
1	A	1124550-NLA	NLA - Handle, Release, Chrome - Left	1	1
1	A	1124551	Handle, Release, Black - Right	1	1
1	A	1124552	Handle, Release, Black - Left	1	1
	F	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
	F	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
2	D	40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
2	E	1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3	D	40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
3	E	1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5		15680X012	Collar, Anti-Rattle (7/8")	1	1
6	G	1007123-NLA	NLA - Tube, Support, Footrest, Chrome - Right	1	1
6		1007124-NLA	NLA - Tube, Support, Footrest, Chrome - Left	1	1
6		1065180	Tube, Support, Footrest, Black - Right	1	1
6		1065181	Tube, Support, Footrest, Black - Left	1	1
6	B	8881065182	Tube, Support, Footrest, Color - Right	1	1
6	B	8881065183	Tube, Support, Footrest, Color - Left	1	1
7	D	11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
7	E	1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
8		1107321	Nut, Low Cap (5/16"-24)	1	1
8		7930330-NLA	NLA - Locknut (5/16-24)	25	1
9		1022874	Bumper	1	1
10		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
11	D	1008935	Spring, Release - Right	1	1
11	D	1008936	Spring, Release - Left	1	1
11	E	1028707	Spring, Release - Right	1	1
11	E	1028708	Spring, Release - Left	1	1
12	D	67005X000	Rivnut (#10-32)	1	1
12	E	1028846-NLA	NLA - Rivnut (#10-32)	1	1
13	D	60279X003	Locknut (#10-32)	1	1
13	E	1025735	Locknut (#10-32)	1	1
14	D	47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
14	E	1031884-NLA	NLA - Bushing, Black or Color Chairs (9/32 x 3/8 x 11/32")	1	1
15	C	See Chart	Footrest Support Assembly - SH93ES / SH93E	1	1
NOTE: A - Due to design change, release handles should be replaced in pairs for chairs manufactured before 10/1/00. B - Specify frame color when ordering, see category titled, "Frame Finishes and Upholstery Colors." C - Includes items 1 thru 15. D - For use on chrome chairs. E - For use on color chairs. F - Not shown. Order this item when ordering chair with Cyclical Drive only. G - Part #1007123 is no longer available individually. Order item 15 part #1124555 Footrest Support Assembly as a replacement.					

Footrest Support Assembly

SH93ES, SH93E

Footrest Support Assembly - SH93ES, SH93E

Prior Model Number	Current Model Number	Chair Width	Right Side	Left Side
SH93ES, SH93E Chrome	SH93ES, SH93E Chrome	14-22"	1124555-NLA	1124556-NLA
SH93ES, SH93E Black	SH93ES, SH93E Black	14-22"	1065218	1065219
SH93ES, SH93E Color	SH93ES, SH93E Color	14-22"	8881065220 See Note(s) A	8881065221 See Note(s) A
NOTE: A - Specify frame color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."				