


Legrest Support Assembly

H904, T944A/C

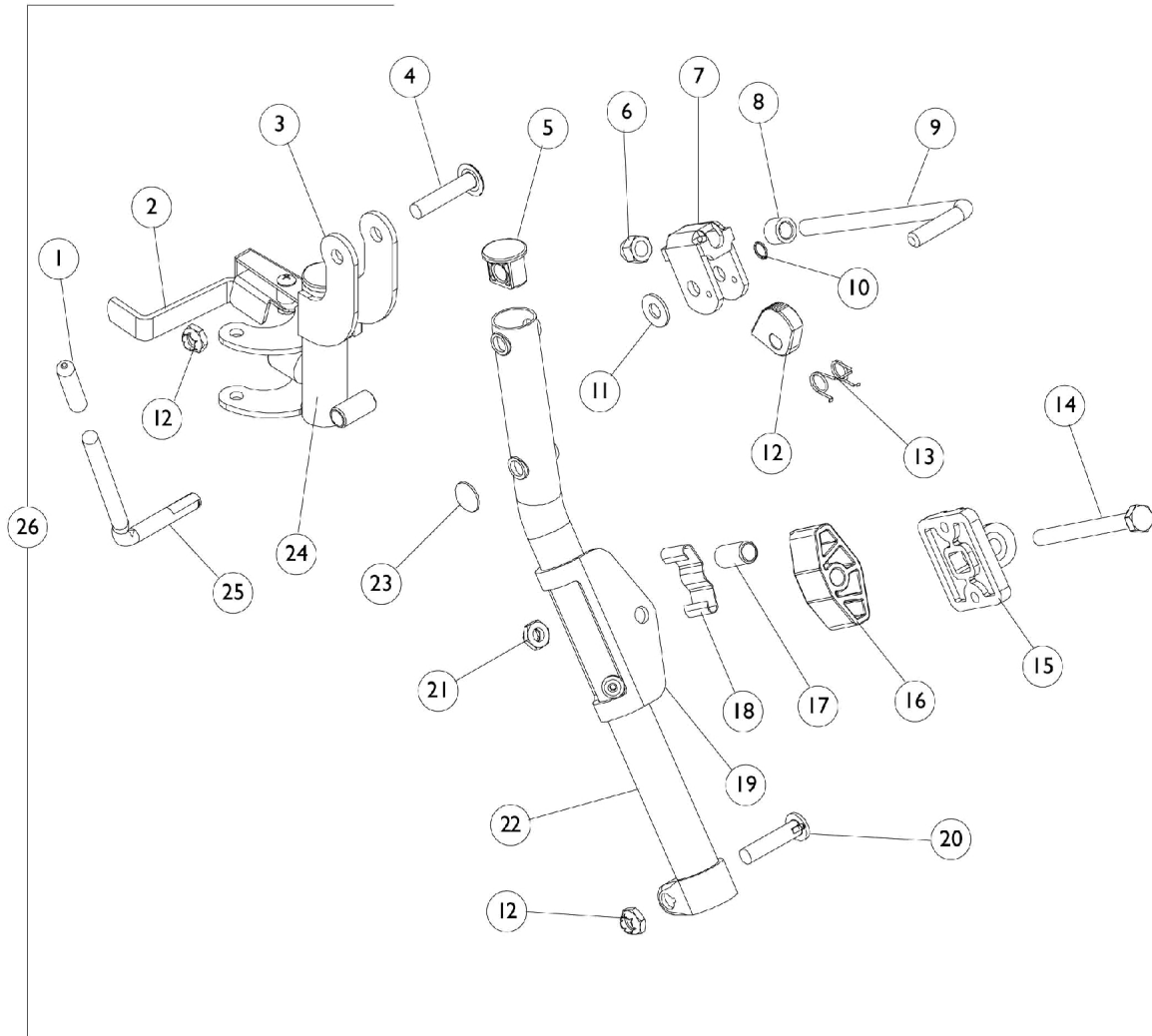

Legrest Support Assembly H904, T944A/C

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040060	Tip, Plastic, Black	1	1
2	A	1031235-NLA	NLA - Handle, Release, Chrome - Right	1	1
2	A	1031236-NLA	NLA - Handle, Release, Chrome - Left	1	1
2	A,C	1033305-NLA	NLA - Handle, Release, Chrome - Right	1	1
2	A,C	1033306-NLA	NLA - Handle, Release, Chrome - Left	1	1
	G	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
	G	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
3		1031354-NLA	NLA - Support, Elevating - Right	1	1
3		1031355-NLA	NLA - Support, Elevating - Left	1	1
4		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
5		1095433	Plug Button, Black (7/8")	1	1
6		1107321	Nut, Low Cap (5/16"-24)	1	1
7		04528X002	Bracket, Release, Black	1	1
8		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
9		42326X001	Bar, Ratchet	1	1
10		1040831	Ring, Snap	1	1
11		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
12		1037125	Cam, Legrest	1	1
13		1037485	Spring, Release	1	1
14		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
15		04526X019	Plate, Legrest Pad	1	1
16		1091220	Spacer, Legrest	1	1
17		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
18		48823X008	Spring, Leaf	1	1
19		1025104	Bracket, Legrest, Chrome	1	1
20		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
21		60264X001	Locknut (5/16-24)	1	3
22		1031358-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Right	1	1
22		1031359-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Left	1	1
23		1096773	Nut, Push Dome (5/16")	1	1
24	B	1031356-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
24	B	1031357-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
25		1037126	Handle, Release	1	1
	C,E	67005X000	Rivnut (#10-32)	1	1
	C,E	1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
	C,E	1022874	Bumper	1	1
	D,E	1046328-NLA	NLA - Kit, Ratchet	2	1
	C,E	1027095	Plug Button, Black (7/8")	1	1
	C,E	7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	2
	C,E	1008935	Spring, Release - Right	1	1
	C,E	1008936	Spring, Release - Left	1	1

Legrest Support Assembly H904, T944A/C

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
26	F	See Chart	Legrest Support Assembly - H904, T944A/C	1	1
NOTE: A - Due to design change, for all chairs built prior to 10/1/00 it is suggested that release handles be replaced in pairs. B - Includes items 2 and 3. C - Before 10/01/00 only. D - Includes Items 1, 6 thru 14, and 25 E - Not shown. F - Includes items 1 thru 25. G - Not Shown. Order this item when ordering chair with Cyclical Drive only.					

Legrest Support Assembly - H904, T944A/C

Prior Model Number	Current Model Number	Chair Width	Right Side	Left Side
H904 - Chrome	T944 A/C - Chrome	14-22"	1031360-NLA See Note(s) A	1031361-NLA See Note(s) A
NOTE: A - Calf pad is sold separately and is not included in support assemblies after 7/97.				