


Captain Seat Assembly


Before 11/01/01

NOTE: Low Back (L.B.) Captain Seat description changed to Extended Back (E.B.) Captain Seat as of 10/01/00.


NOTE: Captain Seats incorporated an assisted recline of approximately 80-114 degrees for extended backs and high back 22" wide models. High back 18" wide models are 85-170 degrees.

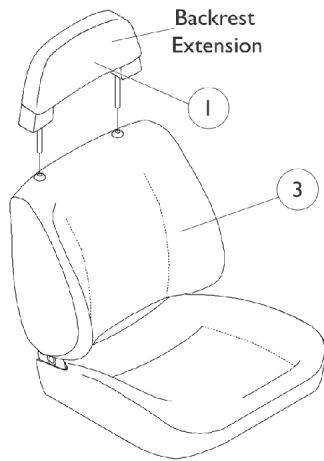

Extended Back 18" Wide

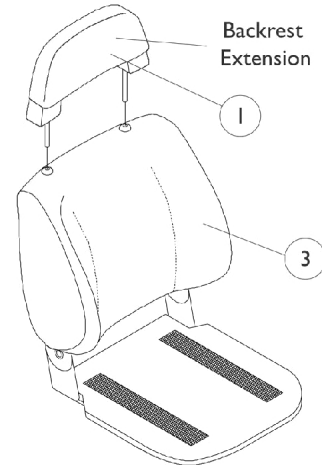


Extended Back, Forward Folding 18" Wide w/ Solid Seat Base

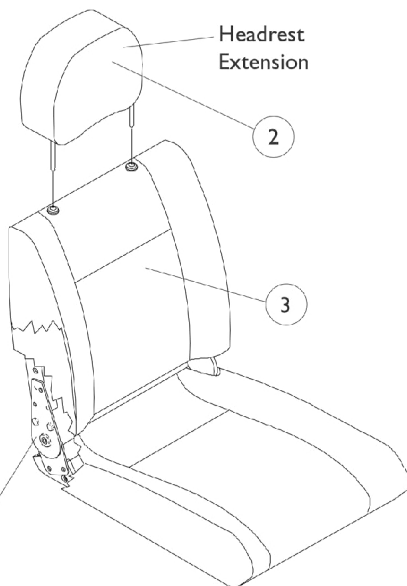

NOTE: High back 18" wide captain seat includes the actuator hardware. See seat frame and actuator hardware page.

High Back 18" Wide w/ Full Recline

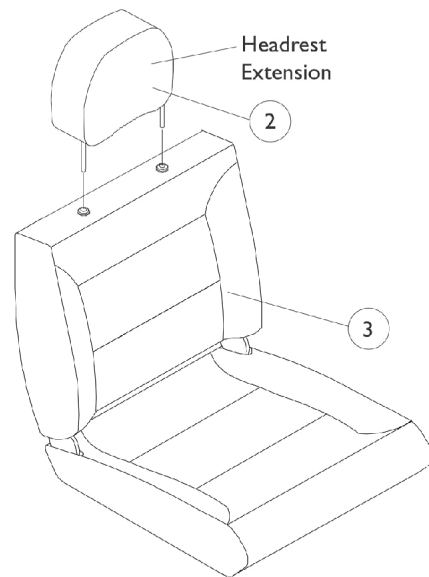

High Back 22" Wide w/ Semi Recline

Captain Seat Assembly

Before 11/01/01

NOTE: Low Back (L.B.) Captain Seat description changed to Extended Back (E.B.) Captain Seat as of 10/01/00.

NOTE: Captain Seats incorporated an assisted recline of approximately 80-114 degrees for extended backs and high back 22" wide models. High back 18" wide models are 85-170 degrees.

Item Number	Backrest/Headrest Extension and Captain Seat Assemblies	Grey Fabric/Vinyl	All Grey Vinyl	Brunswick Buff All Tan Vinyl
1.	Backrest, Extended Back	1090374-NLA See Note(s) A	1090375-NLA See Note(s) A	1090376-NLA See Note(s) A
1.	Backrest, Extended Back Forward Folding (Before 5/12/00)	1078997-NLA See Note(s) B	N/A	N/A
1.	Backrest, Extended Back Forward Folding (After 5/11/00)	1090374-NLA See Note(s) A	N/A	N/A
2.	Headrest, High Back (For 18 and 22" Wide)	1090379-NLA See Note(s) A	1090380-NLA See Note(s) A	N/A
3.	18" W Extended Back	1090371-NLA	1090372-NLA	1090373-NLA
3.	18" W Extended Back Forward Folding (Before 5/12/00)	1085104-NLA	N/A	N/A
3.	18" W Extended Back Forward Folding (After 5/11/00)	1090370-NLA	N/A	N/A
3.	18" Wide High Back with Full Recline	1091068-NLA See Note(s) C	1091069-NLA	N/A
3.	22" Wide High Back with Semi Recline	1098287-NLA	N/A	N/A

NOTE: A - 4-3/4" Backrest/Headrest Post Spacing

B - 9" Backrest Post Spacing

C - High back 18" wide captain seats includes actuator attaching hardware.

NOTE: When converting from a extended back captain seat to an 18" wide high back captain seat, a sensor cable must be ordered as well.

NOTE: For captain seats marked NLA, use captain seats dated after 10/31/01 as replacements.