


Pivot Slide Tube with Tubular Footplate

H903, T943A/C/S, T963A/C/S, TH93A/C/S Footrest

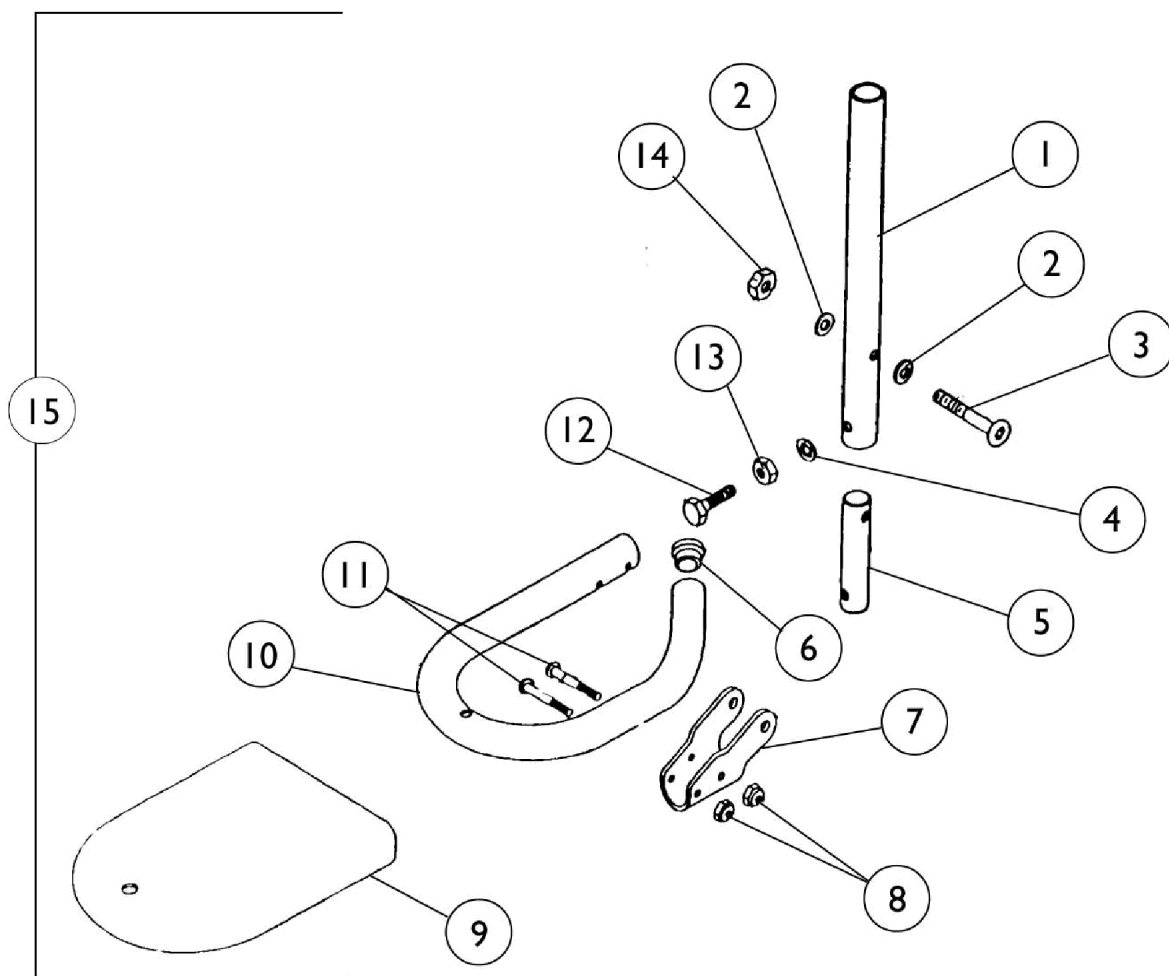

Pivot Slide Tube with Tubular Footplate H903, T943A/C/S, T963A/C/S, TH93A/C/S Footrest

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1012183-NLA	NLA - Tube, Pivot/Slide, Black	1	1
2		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	2
3		1015176-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
4		10502X001-NLA	NLA - Lockwasher (1/4")	1	1
5	D	1012158-NLA	NLA - Insert, Lower Slide Tube	1	1
6		1027097	Plug Button, Black (3/4")	1	1
7		1012182-NLA	NLA - Bracket, Pivot, Black	1	1
8		7930365-NLA	NLA - Locknut (#10-32)	25	2
9	D	1008552-NLA	NLA - Cover, Footplate - Medium	1	1
9		1008553-NLA	NLA - Cover, Footplate - Large	1	1
10	A	1016294-NLA	NLA - Tube, Footrest (Right/ Left) 14"-15" Wide Chairs	1	1
10	A	1012181-NLA	NLA - Tube, Footrest (Right/ Left) 16"-22" Wide Chairs	1	1
10	A,B	1026404-NLA	NLA - Tube, Footrest, Black - Medium	1	1
10	A,C	1026405-NLA	NLA - Tube, Footrest, Black - Large	1	1
11		1012608-NLA	NLA - Screw, Truss Head (#10-32 x 1-1/8")	1	2
12		16690X000-NLA	NLA - Screw, Hex Head (1/4-20X1)	1	1
13		15108X000	Locknut (1/4-20)	1	1
14		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
15		See Chart	Pivot Slide Tube with Footplate	1	1

NOTE: A - Right/Left
 B - For chairs 14 - 15" wide.
 C - For chairs 16 - 22" wide.
 D - Part number is no longer available as a service part individually. See item 15 and order a complete Pivot and Slide Tube with Footplate Assembly.

NOTE: Tubular Footrests are painted black and should be ordered in pairs for chairs built before 10/6/00. NOTE: Pivot Slide Tube and Footplate is attached to Footrest Support using a compression clamp welded to the Support Assembly.

Pivot Slide Tube with Footplate

Prior Model Number	Current Model Number	Chair Width	Side	Part Number
H903 Chrome/Black	T943S	14 - 15"	Right	1030752-NLA
H903 Chrome/Black	T943S	14 - 15"	Left	1030753-NLA
H903 Chrome/Black	T943C	16 - 22"	Right	1012263-NLA
H903 Chrome/Black	T943C	16 - 22"	Left	1012264-NLA