


Fold Down Canes and Hardware

Transport Chairs Only

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

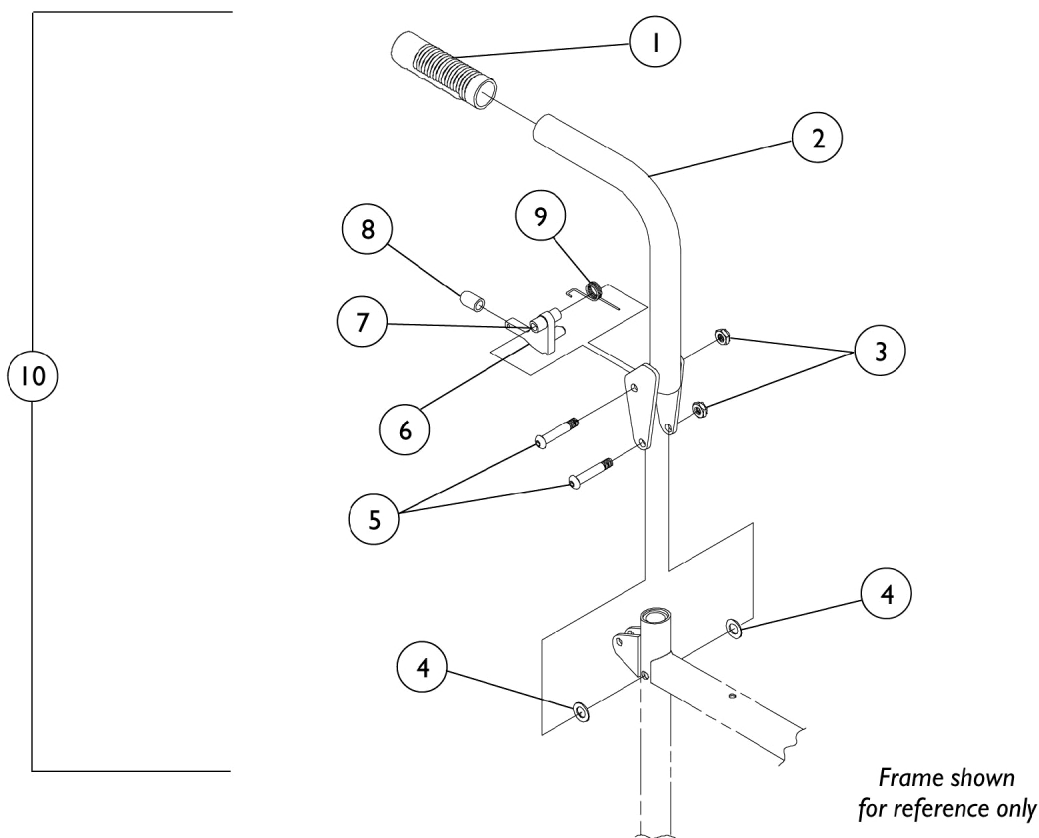

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1021261	Grip, Hand (6")	1	1
2		1061861-NLA	NLA - Cane, Back, Fold Down, Chrome - Right/Left	1	1
3		60275X000	Locknut (1/4-20)	1	2
4		40157X016-NLA	NLA - Washer (1/4 x 1/2 x 1/16")	1	2
5		1029583-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	2
6		1068905	Cam	1	1
7		1019585-NLA	NLA - Spacer (1/4 x 7/16 x 5/16")	1	1
8		47376X023-NLA	NLA - Cap, End, Black	1	1
9		1007355	Spring, Torsion	1	1
10	A	1061862-NLA	NLA - Fold Down Cane with Hardware - Right	1	1
10	A	1061863-NLA	NLA - Fold Down Cane with Hardware - Left	1	1

NOTE: A - Includes Items 1-9.