


Bolt On Push Handles

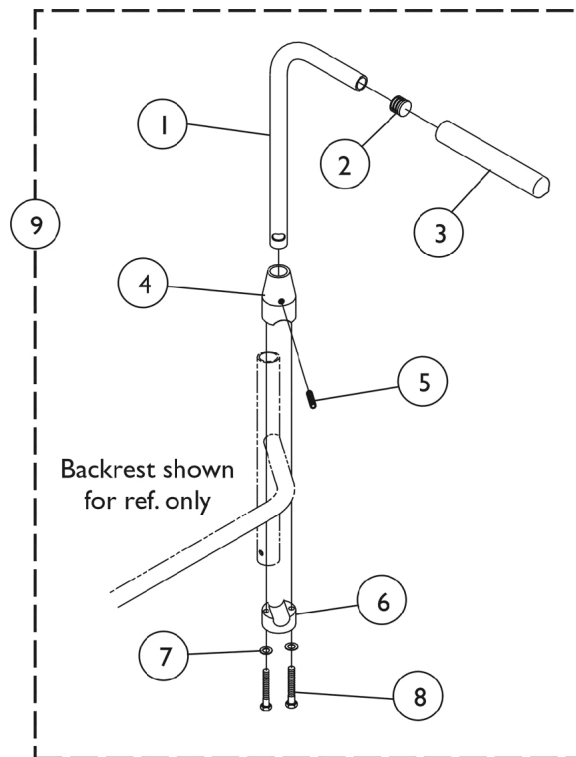

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1032209-NLA	NLA - Handle, Push, Black - Right	1	1
1		1032210-NLA	NLA - Handle, Push, Black - Left	1	1
2		1027097	Plug Button, Black (3/4")	1	2
3		1028431-NLA	NLA - Grip, Foam, Black (5")	1	2
4		1032211-NLA	NLA - Clamp, Threaded, Top	1	2
5		1032217-NLA	NLA - Screw, Set Flat Point (1/4-20 x 1")	1	2
6		1032212-NLA	NLA - Clamp, Non-Threaded, Bottom	1	2
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
8		1025713-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
9	A	1032213-NLA	NLA - Push Handle Assembly w/ Hardware, Bolt On (Pair)	2	1

NOTE: A - Includes items 1-8