


Reclining Back Hardware

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

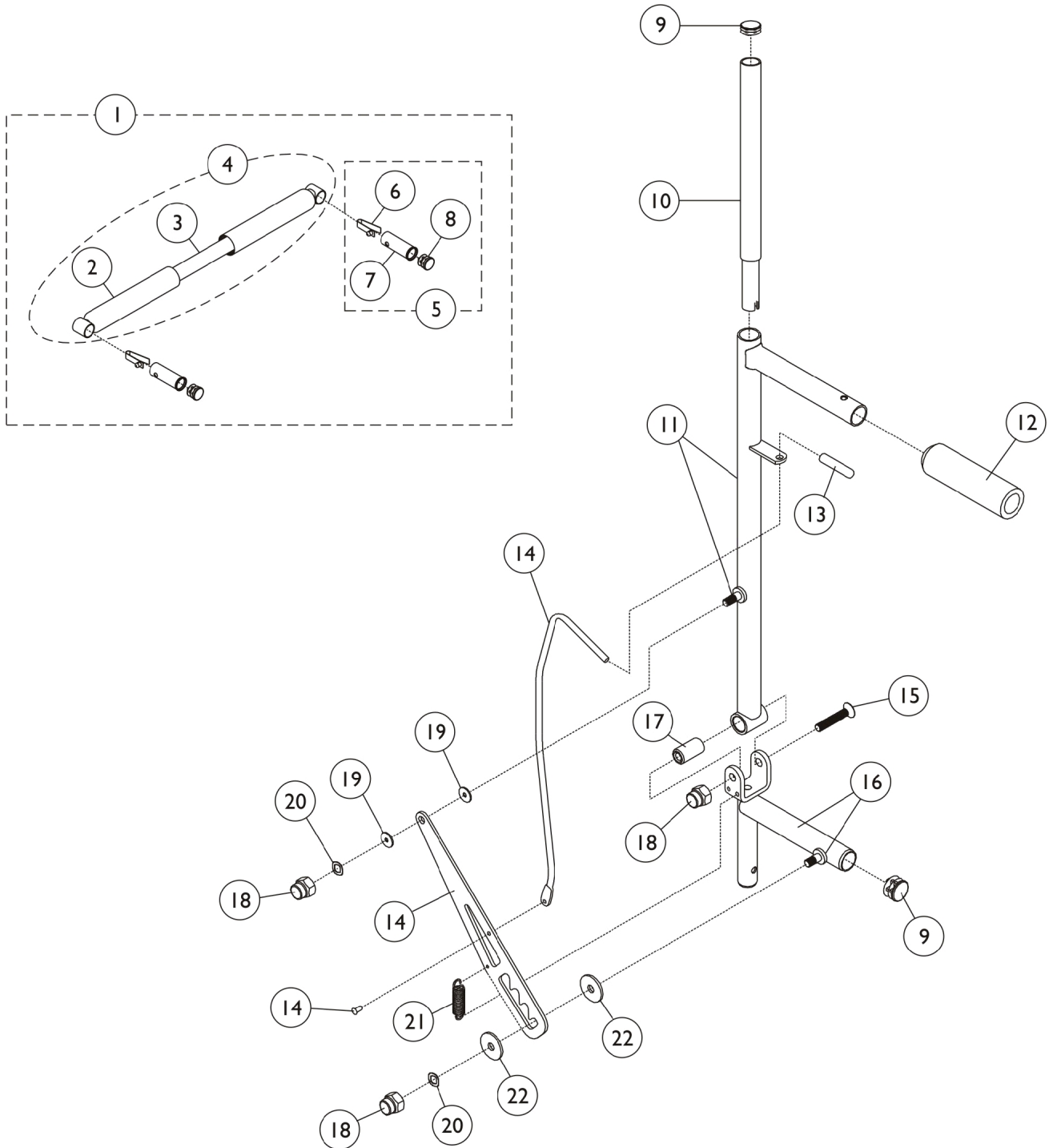

Reclining Back Hardware

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,D	1071121-NLA	NLA - Bar, Spreader with Locking Pins, Chrome (16" W)	1	1
1	A,D	1071122-NLA	NLA - Bar, Spreader with Locking Pins, Chrome (18" W)	1	1
1	A	1182041	Bar, Spreader with Locking Pins, Black (16" W)	1	1
1	A	1182042	Bar, Spreader with Locking Pins, Black (18" W)	1	1
4	B	1062638-NLA	NLA - Bar, Spreader with Grips, Chrome (16" W)	1	1
4	B,D	1062639-NLA	NLA - Bar, Spreader with Grips, Chrome (18" W)	1	1
5	C	1071120	Lock Handle	1	1
6			Button, Single Snap	1	1
7			Insert, Seat Rail	1	1
8			Plug Button, Black (3/4")	1	1
9		1027095	Plug Button, Black (7/8")	1	2
10		1062784-NLA	NLA - Headrest Tube Assembly	1	1
11		1062099-NLA	NLA - Cane, Back, Recliner, Chrome - Right	1	1
11		1062100-NLA	NLA - Cane, Back, Recliner, Chrome - Left	1	1
12		1061484	Grip, Handle, Foam, Black (5")	1	1
13		1062197-NLA	NLA - Tip, Protective, Black	1	1
14		1064413	NLA - Indexing Plate Assembly - Right	1	1
			Rod - Right	1	1
			Rivet, Oval Head	1	1
			Plate, Indexing	1	1
14		1064414	NLA - Indexing Plate Assembly - Left	1	1
			Rod - Left	1	1
			Rivet, Oval Head	1	1
			Plate, Indexing	1	1
15		1058920	Screw, Socket Flat Head (5/16-24 x 2")	1	1
16		1059274-NLA	NLA - Brace, Rear Cane - Right	1	1
16		1059275-NLA	NLA - Brace, Rear Cane - Left	1	1
17		1058938	Bushing, Pivot (5/16 x 5/8 x 1-1/8")	1	1
18		1060057	Nut, Cap (5/16-24)	1	3
19		1059655-NLA	NLA - Washer, Steel (5/16 x 5/8 x 1/16")	1	2
20		1058937	Washer, Spring (5/16 x 1/2 x 1/16")	1	2
21		1058966	Spring, Extension (1/2 x 1-21/32" L)	1	1
22		1064439-NLA	NLA - Washer, Steel (5/16 x 1-1/4 x 1/16")	1	2
NOTE: A - Includes items 2-8. B - Includes items 2-3. C - Includes items 6-8. D - Chrome is No Longer Available. Select Item #1 Spreader Bar in Black as a replacement. Specify 34P Flat Black frame finish when ordering.					