
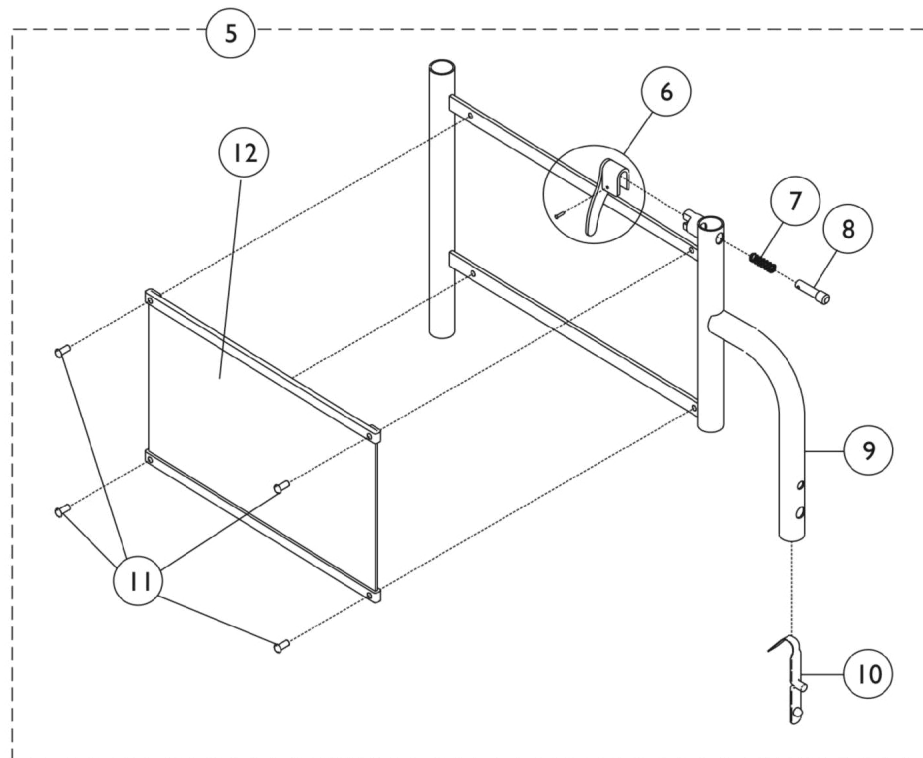
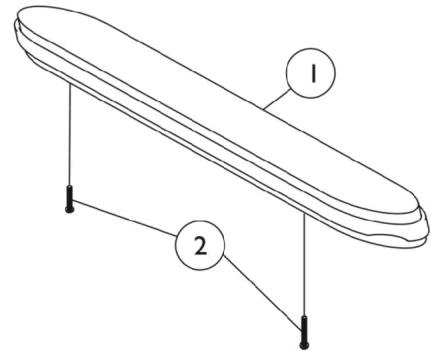


Conventional Adjustable Height

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

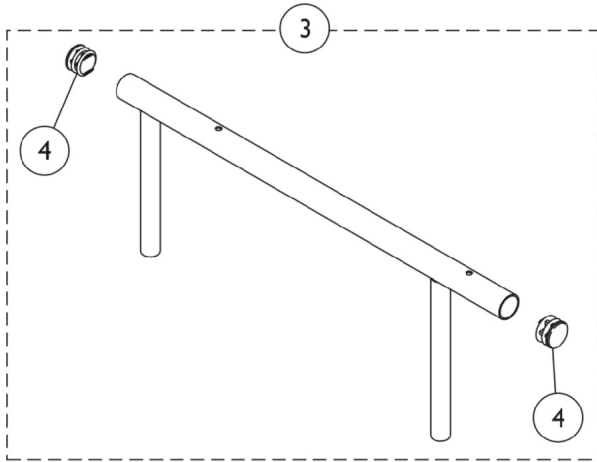

Conventional Adjustable Height

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881091684	Pad, Arm, Desk Length, Vinyl	1	1
1	A	8881091682	Pad, Arm, Full Length, Vinyl	1	1
2		40183X002	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
3	B	1057734	Tube, Arm, Upper with Plug Buttons, Black, Desk Length	1	1
3	B	1057736-NLA	NLA - Tube, Arm, Upper with Plug Buttons, Chrome, Full Length	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5	C	04702X256-NLA	NLA - Adjustable Height Arm Base Assembly, Chrome - Right	1	1
6			Handle, Lock, Black - Right	1	1
6			Rivet, Tubular	1	1
7			Spring, Arm Lock	1	1
8			Pin, Locking	1	1
9			NLA - Arm Base Assembly, Chrome - Right/Left	1	1
10			Lock, Spring and Button	1	1
11			NLA - Rivet	1	4
12			Guard, Clothing	1	1
5	C	04702X257-NLA	NLA - Adjustable Height Arm Base Assembly, Chrome - Left	1	1
6			Handle, Lock, Black - Left	1	1
6			Rivet, Tubular	1	1
7			Spring, Arm Lock	1	1
8			Pin, Locking	1	1
9			NLA - Arm Base Assembly, Chrome - Right/Left	1	1
10			Lock, Spring and Button	1	1
11			NLA - Rivet	1	4
12			Guard, Clothing	1	1
	D	1101334	Decal, Clothing Guard - Tracer IV	1	1
	D	1063187	Decal, Clothing Guard - Tracer Recliner	1	1
	D	1101333-NLA	NLA - Decal, Invacare Logo (2.75)	1	1
	D,E	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
	D,E	7100054	Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2

Conventional Adjustable Height

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

NOTE:	A - Specify upholstery color, see page titled "Upholstery Colors."
	B - Includes item 4.
	C - Includes Items 6-12. Item #7 is not listed.
	D - Not Shown.
	E - Includes items 6-8.