


Rear Wheel Hardware

5/8" & 7/16" Axle

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

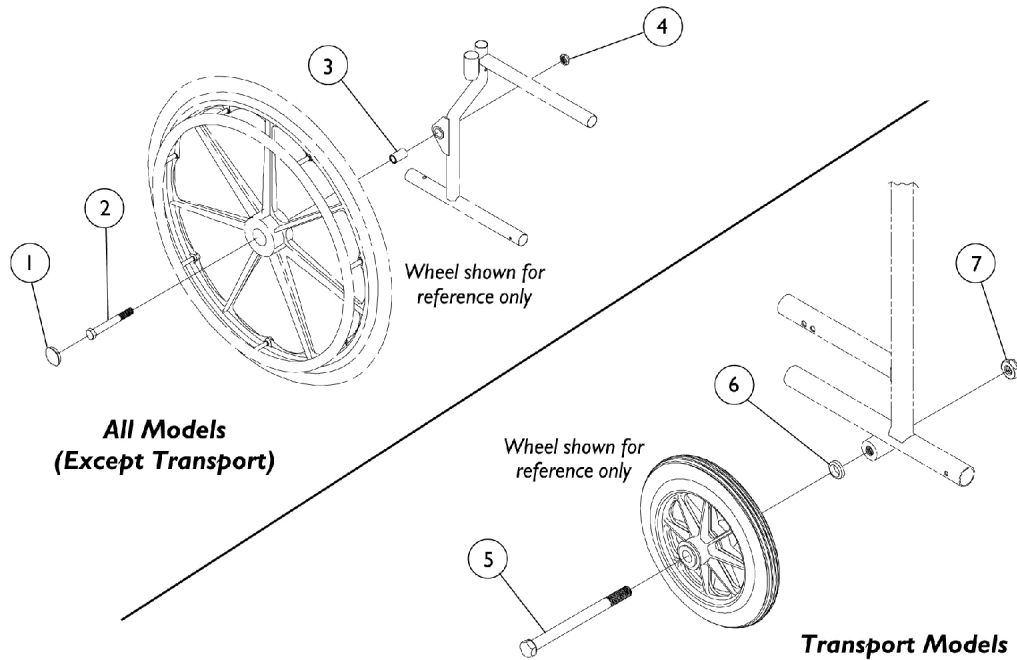

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1027600-NLA	NLA - Cap, Black	1	1
2	B	47203X007	Screw, Hex Head (5/8-18 x 4")	1	1
2	C	1018865-NLA	NLA - Screw, Hex Head (7/16-20 x 3-11/16")	1	1
2	D,E	47203X029	Screw, Hex Head (7/16-20 x 4-1/2")	1	1
2	D	47203X020	Axle (5/8-18 x 4-3/4")	1	1
2	D	47203X025	Bolt, Hex Head Cap (7/16-20 x 4-3/4")	1	1
2	D	1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
3		47183X021	Spacer (5/8 x 7/8 x 3/4")	1	1
3	F	47183X037-NLA	NLA - Spacer (1/2 x 7/8 x 7/8")	1	1
3	G	1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
4		20225X000	Locknut (7/16-20)	1	1
4		60030X000	Locknut (5/8-18)	1	1
4	G	1045608	Locknut (7/16-20)	1	1
5		47203X029	Screw, Hex Head (7/16-20 x 4-1/2")	1	1
6		1026101	Spacer (7/16 x 5/8 x 3/16")	1	1
7		20225X000	Locknut (7/16-20)	1	1

NOTE: A - For use on Tracer IV chairs manufactured before 02/07/00.
 B - For use on Fixed Front Frame chairs only.
 C - For use on Permanent Arm chairs with Swingaway Frame only.
 D - For use on Removeable Arm chairs only.
 E - Not for use on Tracer IV or LX Recliner chairs.
 F - For use on Tracer LX Recliner only.
 G - For use on side frames replaced after 10/13/99.