


Footrest Support Assembly

TR13, TLX3A/C/S, TLXH3A/C, T933A/C/S, T943A/C

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

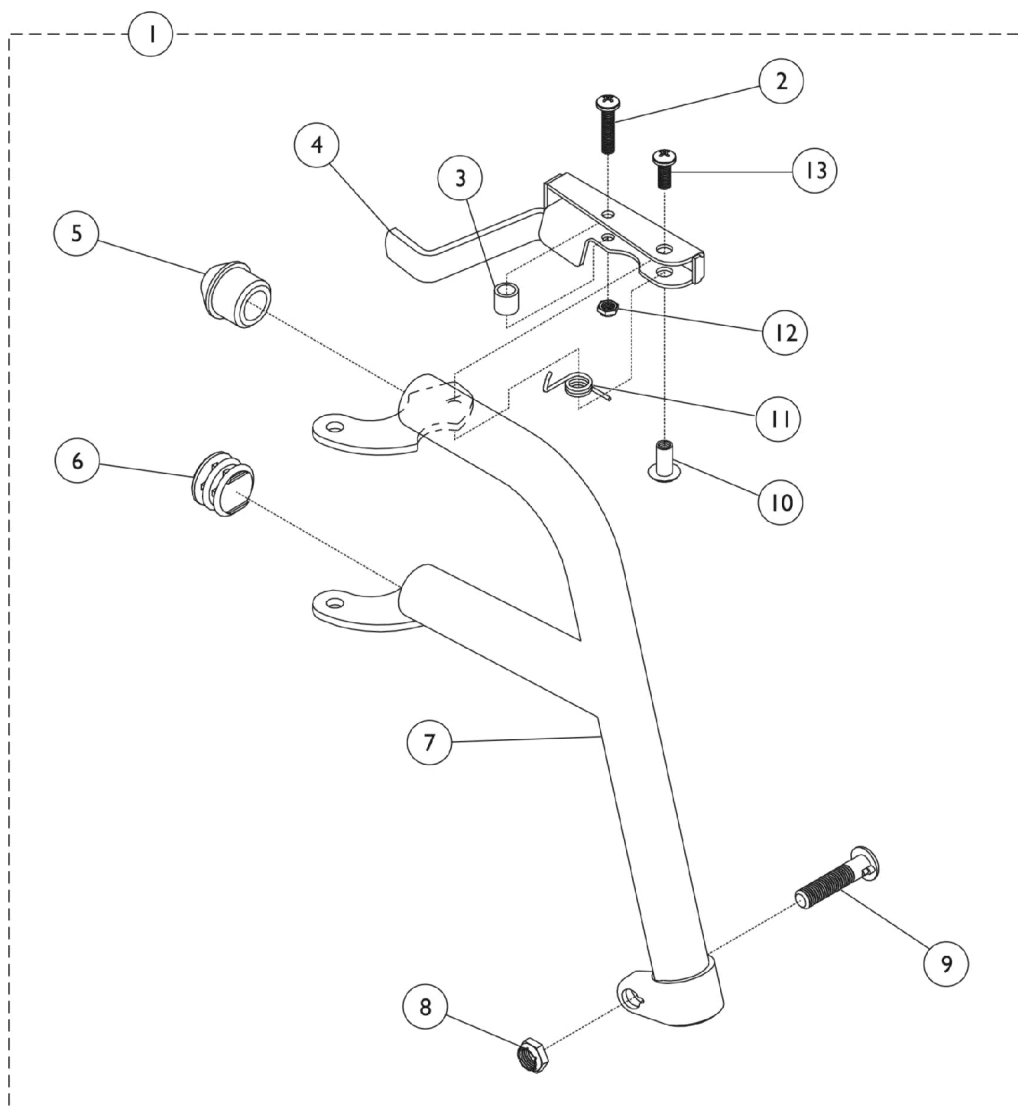

Footrest Support Assembly

TR13, TLX3A/C/S, TLXH3A/C, T933A/C/S, T943A/C

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1053174-NLA	NLA - Footrest Support Assembly, Adult, Chrome - Right	1	1
1	A,C	1053175-NLA	NLA - Footrest Support Assembly, Adult, Chrome - Left	1	1
1	B,D	1030706-NLA	NLA - Footrest Support Assembly, Silver Vein - Right	1	1
1	B,D	1030707-NLA	NLA - Footrest Support Assembly, Silver Vein - Left	1	1
2		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
4		1035539	Handle, Release, Black - Right	1	1
4		1035540	Handle, Release, Black - Left	1	1
5		1020719	Anti-Rattle, Black	1	1
6	A	1027095	Plug Button, Black (7/8")	1	1
7	A	1053170-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
7	A	1053171-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
7	B	1020416-NLA	NLA - Tube, Support, Footrest, Chrome - Right	1	1
7	B	1020417-NLA	NLA - Tube, Support, Footrest, Chrome - Left	1	1
8		1107321	Nut, Low Cap (5/16"-24)	1	1
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		67005X000	Rivnut (#10-32)	1	1
11		1008935	Spring, Release - Right	1	1
11		1008936	Spring, Release - Left	1	1
12		60279X003	Locknut (#10-32)	1	1
13		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
NOTE: A - Adult chair B - Hemi Chair C - Includes items 2-13. D - Includes items 2-5 and 7-13.					