


Legrest Support Assembly

TLXH4A/C, T944A/C

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

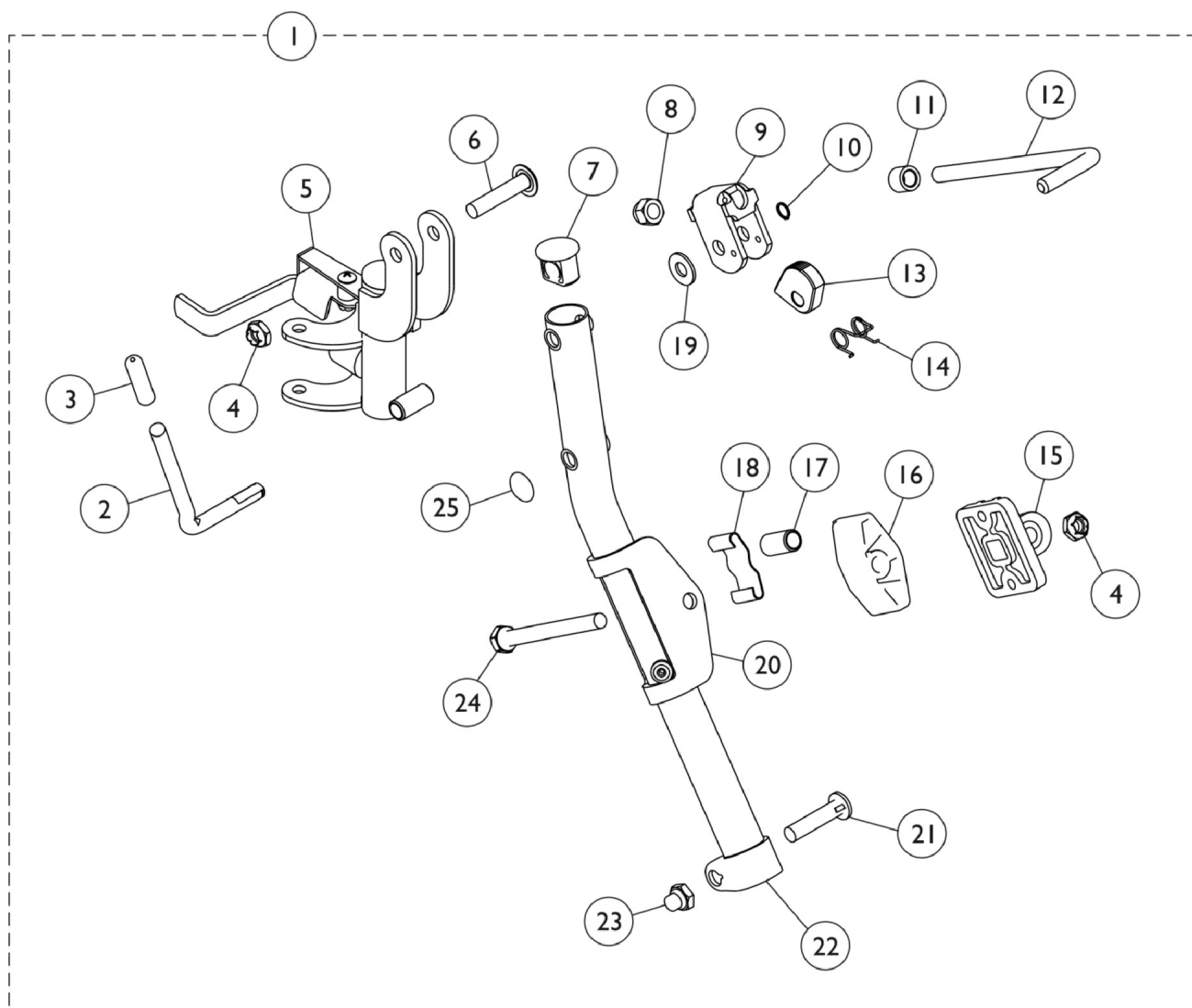

Legrest Support Assembly

TLXH4A/C, T944A/C

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1041822	Legrest Support Assembly, Hemi, Black - Right	1	1
1	A	1041823	Legrest Support Assembly, Hemi, Black - Left	1	1
2		1037126	Handle, Release	1	1
3		1040060	Tip, Plastic, Black	1	1
4		60264X001	Locknut (5/16-24)	1	2
5		1031356-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
5		1031357-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
6		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
7		1095433	Plug Button, Black (7/8")	1	1
8		10261X000	Nut, Acorn (5/16-24)	1	1
9		04528X002	Bracket, Release, Black	1	1
10		1040831	Ring, Snap	1	1
11		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
12		42326X001	Bar, Ratchet	1	1
13		1037125	Cam, Legrest	1	1
14		1037485	Spring, Release	1	1
15		04526X019	Plate, Legrest Pad	1	1
16		1091220	Spacer, Legrest	1	1
17		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
18		48823X008	Spring, Leaf	1	1
19		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
20		1025104	Bracket, Legrest, Chrome	1	1
21		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
22	B	1095434-NLA	NLA - Tube, Support, Chrome - Right	1	1
22	B	1095435-NLA	NLA - Tube, Support, Chrome - Left	1	1
23		1107321	Nut, Low Cap (5/16"-24)	1	1
24		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
25		1096773	Nut, Push Dome (5/16")	1	1
	C,D	1046328-NLA	NLA - Kit, Ratchet	1	1
NOTE: A - Includes items 2-25. B - Due to product improvement, item #7 must also be ordered. C - Not shown. D - Includes items 2, 6-10, 12, 15 and 24.					