
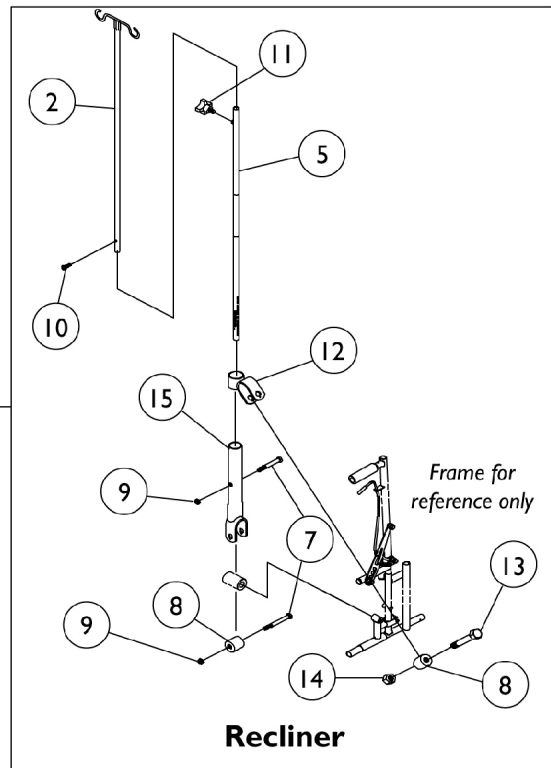


I.V. Rod - Telescoping

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

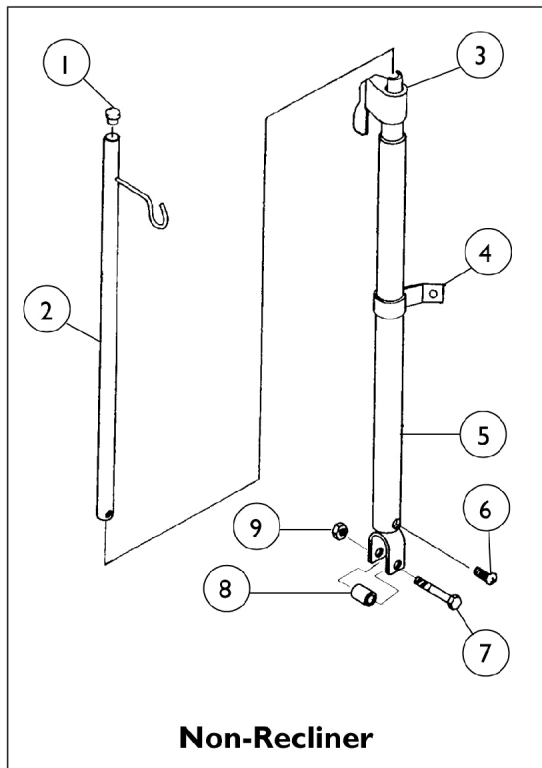

I.V. Rod - Telescoping

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	1
2		04701X028	Rod, I.V.	1	1
2		04701X041	Rod, I.V., Recliner, Black	1	1
3		12592X000-NLA	NLA - Adjustment Lock Assembly, Chrome	1	1
4		04701X032-NLA	NLA - Bracket, Mounting, Chrome	1	1
5		04701X031	Rod, I.V. Support, Non-Recliner	1	1
5		1076521	Tube, Support, I.V. Rod, Black	1	1
6		21243X000-NLA	NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	1
7		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
8		47324X011	Bushing, Pivot (5/16 x 7/8 x 3/4")	1	1
9		1107321	Nut, Low Cap (5/16"-24)	1	1
10		40121X000-NLA	NLA - Screw, Phillips Pan Head (#6-32 x 3/16")	1	1
11		1027683	Knob, Adjustment, Black	1	1
12		1063849-NLA	NLA - Bracket, Upper Attachment, Chrome	1	1
13		16690X004-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/8")	1	1
14		1008882	Locknut (1/4-20)	1	1
15		1063851-NLA	NLA - Tube, Support	1	1
16	B	1067766-NLA	NLA - Rod, I.V., Telescoping, Non-Recliner	1	1
16	C	1063880-NLA	NLA - I.V. Rod Telescoping, Recliner	1	1
	A,D	1067765-NLA	NLA - Instruction Sheet, Non-Recliner	1	1
	A	1063879-NLA	NLA - Instruction Sheet, Recliner	1	1
NOTE: A - Not Shown B - Includes items 1-9 C - Includes items 2, 5, 7-15 D - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					