


Luggage Carrier

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

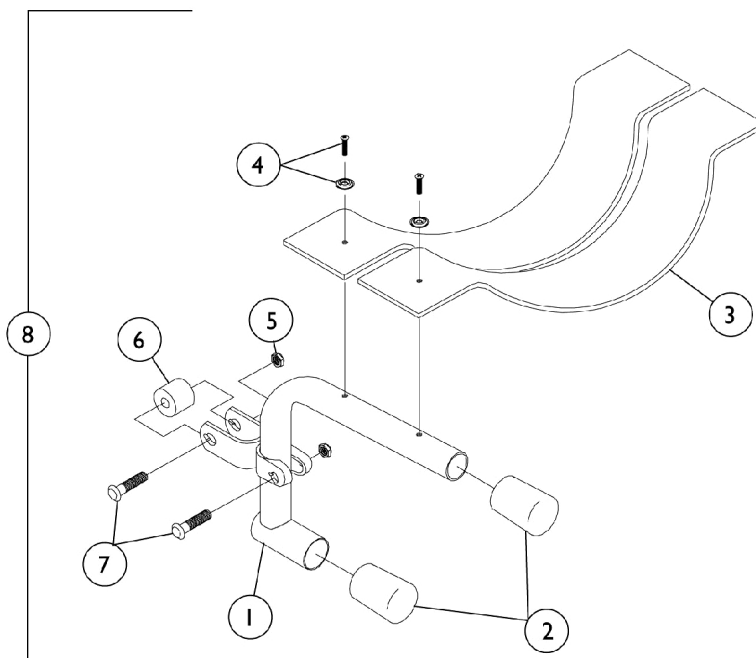

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		04609X017-NLA	NLA - Bracket, Support	1	1
2		1016346	Cap, End, Tube, Black (7/8")	1	2
3		47465X000-NLA	NLA - Webbing, Luggage Strap	1	2
4		1076283	Screw, Phillips Oval Head w/ Washer (#10-16 x 7/8")	1	2
4		1036087	Package, Screws with Washers (#10 x 7/8")	1	2
5		1107321	Nut, Low Cap (5/16"-24)	1	2
6		47324X011	Bushing, Pivot (5/16 x 7/8 x 3/4")	1	1
7		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	2
8	B	1042422-NLA	NLA - Luggage Carrier	1	1
	A	1016945-NLA	NLA - Loop, Luggage Belt	1	1
	A	1016946-NLA	NLA - Hook, Luggage Belt	1	1

NOTE: A - Not Shown
B - Includes items 1-7