


Half Arms and Hardware

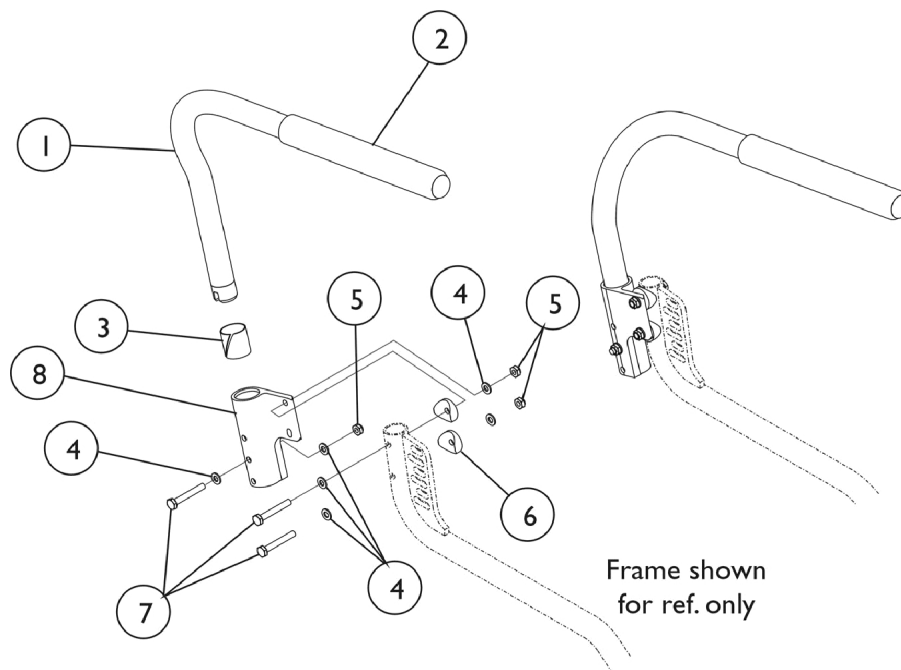

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1056644-NLA	NLA - Half Arm Tube Assembly - Right	1	1
1		1056643-NLA	NLA - Half Arm Tube Assembly - Left	1	1
2		1064064	Grip, Foam - Full Length	1	2
3		1018687	Sleeve, Bearing	1	1
4		1026907	Washer (1/4 x 1/2 x 1/16")	1	12
5		1025765	Locknut (1/4-28)	1	6
6		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
7		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	1
8		1097056-NLA	NLA - Socket, Half Arm - Right	1	1
8		1097057-NLA	NLA - Socket, Half Arm - Left	1	1
	A,B	1114144	Half Arm Mounting Hardware (Pair)	2	1
NOTE: A - Not Shown B - Includes items 3-8					