


# Shopping Basket

## Service Parts Only

### Discontinued 6/29/98

**NOTE:** Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

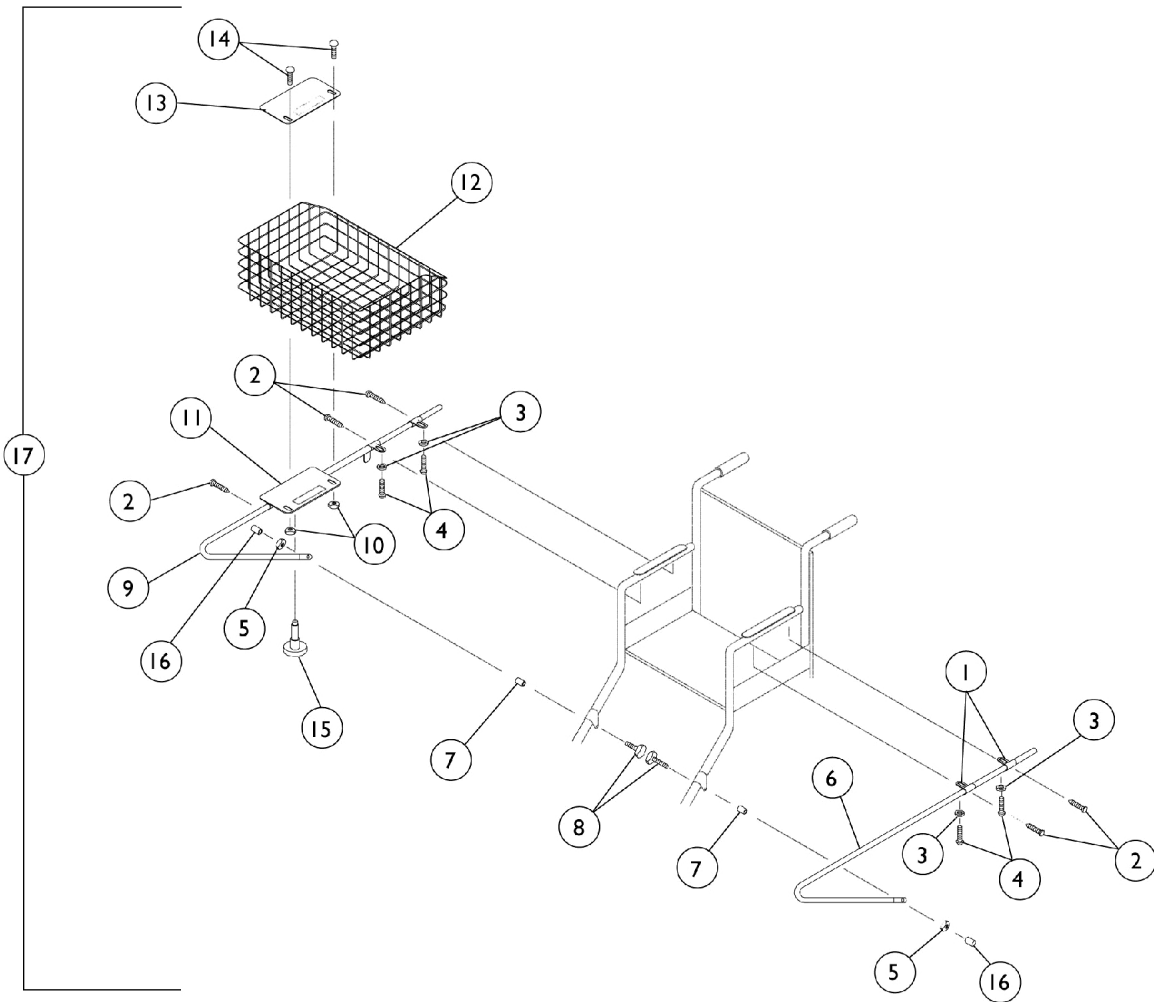

# Shopping Basket

## Service Parts Only

### Discontinued 6/29/98

**NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1021835-NLA	NLA - Bar, Slide Tube Support	1	4
2		21243X000-NLA	NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	5
3		1013218	Washer (#10 Flat)	1	4
4		11010X002-NLA	NLA - Screw, Phillips Round Head (#10-32 x 1-5/8")	1	4
5		1107321	Nut, Low Cap (5/16"-24)	1	2
6		1021764-NLA	NLA - Bracket, Support, Chrome - Left	1	1
7		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	2
8		11240X000-NLA	NLA - Screw, Hex Head (5/16-24 x 2")	1	2
9		1021765-NLA	NLA - Bracket, Support, Chrome - Right	1	2
10		16721X000	Locknut (1/4-20)	1	2
11		1021763-NLA	NLA - Plate, Pivot with Tube, Bottom	1	1
12		1037615-NLA	NLA - Basket	1	1
13		1021904-NLA	NLA - Plate, Pivot, Top	1	1
14		10123X006	Bolt, Carriage (1/4-20 x 3/4")	1	2
15		06807X002-NLA	NLA - Plunger, Pin	1	1
16		1001634-NLA	NLA - Cap, End, Protective, Black	1	2
17		1042418-NLA	NLA - Basket, Shopping w/ Hardware	1	1
	A	1021912-NLA	NLA - Instruction Sheet	1	1

NOTE: A - Not Shown