


Transport 8" Rear Wheel Kit

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

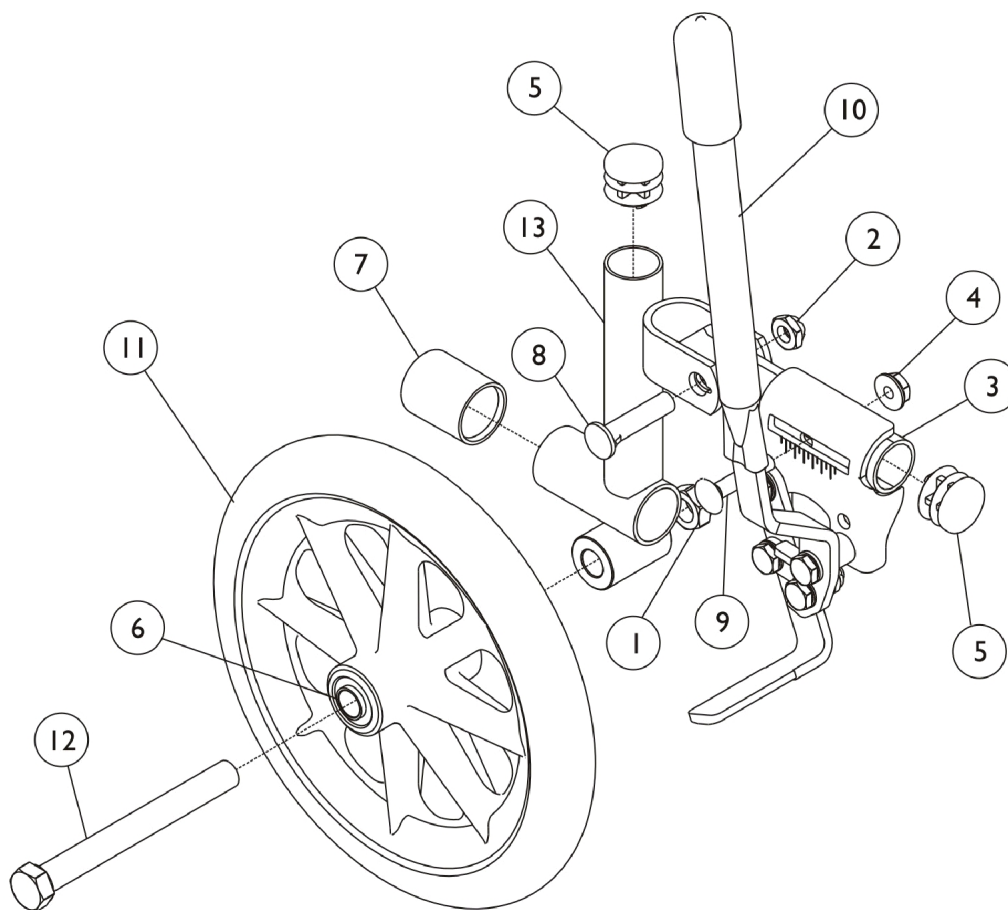

Transport 8" Rear Wheel Kit

NOTE: Due to the possibility of various manufacturing dates for Tracer IV Wheelchairs built with a serial # starting with "03H" or "03I", replacement parts will need to be ordered thru an Invacare Parts Specialist at 1-800-333-6900.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		20225X000	Locknut (7/16-20)	1	1
2		1107321	Nut, Low Cap (5/16"-24)	1	1
3		1014333-NLA	NLA - Tube, Mounting	1	1
4		16721X000	Locknut (1/4-20)	1	1
5		1027095	Plug Button, Black (7/8")	1	2
6		1040102	Bearing (7/16")	1	2
7		1016346	Cap, End, Tube, Black (7/8")	1	2
8		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
9		1011479	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
10		1014353-NLA	NLA - Wheel Lock, Push to Lock - Right	1	1
10		1014354-NLA	NLA - Wheel lock, Push to Lock - Left	1	1
11		1041590	Wheel, Black (8")	1	1
12		47203X024-NLA	NLA - Bolt, Hex Head (7/16-20 x 4")	1	1
13		04609X069-NLA	NLA - Tube, Support, Chrome - Right	1	1
13		04609X070-NLA	NLA - Tube, Support, Chrome - Left	1	1
14		1037911	Spacer, Bearing (7/16 x 5/8 x 1-5/8")	1	2