




Axles, Axle Tubes, Camber Inserts, and Suspension Hardware

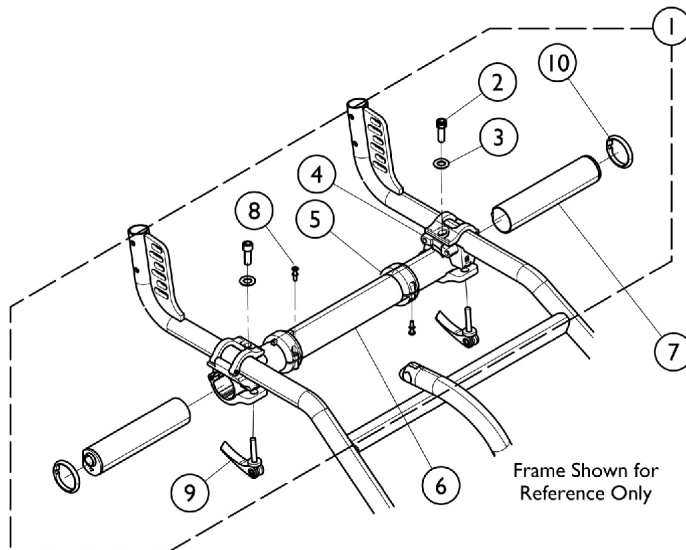


**Non-Suspension Chairs
(Quick Release Pin Camber Release)**

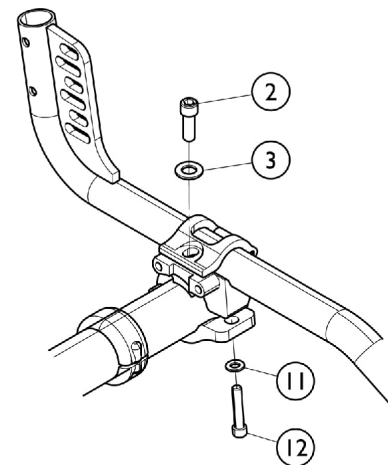

**Non-Suspension Chairs
(Nut and Bolt Camber Release)**

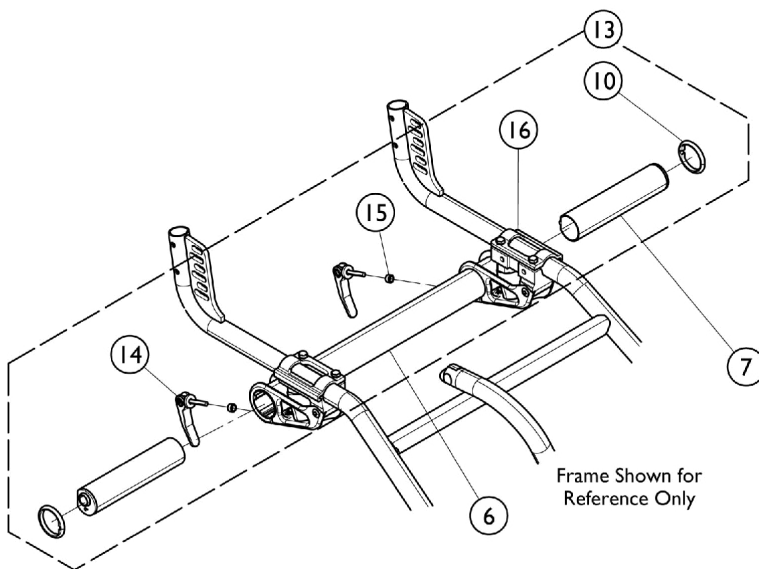

**Suspension Chairs
(Quick Release Pin Camber Release)**

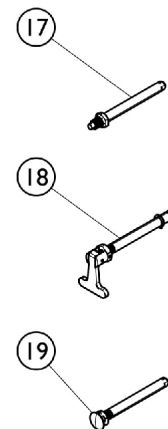

Axles, Axle Tubes, Camber Inserts, and Suspension Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1114074	Hardware, Axle Tube Mounting, Quick Release	1	1
1	B	1114075	Hardware, Axle Tube Mounting, Nut and Bolt	1	1
2		1111954-NLA	NLA - Screw, Socket Head (5/16-24 x 7/8")	1	2
3		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
4		1112017	Clamp, Camber - Right	1	1
4		1112018-NLA	NLA - Clamp, Camber - Left	1	1
5		1065886	Ring, Toe-In/Toe-Out Adjusting	1	2
6		See Chart	#6 Axle Tubes	1	1
7		See Chart	#7 Single Setting Camber Inserts	1	1
7		See Chart	#7 Dual Setting Camber Inserts	1	1
8		1066489-NLA	NLA - Screw, Thread Forming (M4 x 16mm)	1	2
9		1111939-NLA	NLA - Pin, Quick Release	1	2
10		1065887-NLA	NLA - Ring, Indexing	1	2
11		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
12		1074640	Screw, Allen Head (1/4-28 x 1-1/4")	1	2
13	D	1121373-NLA	NLA - Shock Absorbers w/ Bolt-On Hardware, Soft Setting (Pair)	2	1
13	D	1121374-NLA	NLA - Shock Absorbers w/ Bolt-On Hardware, Firm Setting (Pair)	2	1
13	E	1123918-NLA	NLA - Shock Absorbers w/ Quick Release Hardware, Soft Setting (Pair)	2	1
13	E	1123919-NLA	NLA - Shock Absorbers w/ Quick Release Hardware, Firm Setting (Pair)	2	1
14		1083913	Pin, Quick Release	1	2
	G	1123920-NLA	NLA - Kit, Quick Release Pin (Pkg. of 2)	2	1
15		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	2
16	C	1116511-NLA	NLA - Shock Absorber Assembly, Soft Setting (Pair)	2	1
16	C	1116510-NLA	NLA - Shock Absorber Assembly, Firm Setting (Pair)	2	1
17		1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	2
18		1069340	Axle, Quad Release w/ Clip	1	2
19	H	1113049	Axle, Quick Release w/ Dome Button, Black	1	1
19	J	1065303	Axle, Quick Release w/ Flat Button, Black	1	1
	G	1121375-NLA	NLA - Kit, Suspension Bolt-On, Soft Setting (Pair)	2	1
	G	1121376-NLA	NLA - Kit, Suspension Bolt-On, Firm Setting (Pair)	2	1
NOTE: A - Includes items 2 - 5, and 8 - 10. B - Includes items 2 - 5, 8, and 10 - 12. C - Includes Attaching Hardware (4 ea. 1/4-20 x 1" Screws not sold separately) D - Includes items 10 and 16. E - Includes items 10, and 14 - 16. G - Not Shown. Includes 2 ea of items 14 (Q.R. Pin) and 15 (Spacer). H - Used on all Spoke wheels except Spinerger wheels J - Spinerger wheels only					

Axles, Axle Tubes, Camber Inserts, and Suspension Hardware

#6 Axle Tubes

Chair Width	12"	13"	14"	15"	16"	17"	18"
	1065990-NLA	1065991-NLA	1065992-NLA	1065993-NLA	1065994-NLA	1065995-NLA	1065996-NLA
Chair Width	19"	20"	21"	22"			
	1065997-NLA	1065998-NLA	1083015-NLA	1083016-NLA			

#7 Single Setting Camber Inserts

Camber Angle	Right	Left
0 degrees	1112121-NLA	1112122-NLA
3 degrees	1112123-NLA	1112124-NLA
6 degrees	1112125-NLA	1112126-NLA
9 degrees	1112127-NLA	1112128-NLA
12 degrees	1112129-NLA	1112130-NLA

Axles, Axle Tubes, Camber Inserts, and Suspension Hardware

#7 Dual Setting Camber Inserts

Camber Angle Dual Settings	Right	Left
0 - 3 degrees	1066219-NLA	1066229-NLA
0 - 6 degrees	1066220-NLA	1066230-NLA
0 - 9 degrees	1066221-NLA	1066231-NLA
0 - 12 degrees	1066222-NLA	1066232-NLA
3 - 6 degrees	1066223-NLA	1066233-NLA
3 - 9 degrees	1066224-NLA	1066234-NLA
3 - 12 degrees	1066225-NLA	1066235-NLA
6 - 9 degrees	1066226-NLA	1066236-NLA
6 - 12 degrees	1066227-NLA	1066237-NLA
9 - 12 degrees	1066228-NLA	1066238-NLA