


Rehab Shower/Commode Chair

Back Upholstery, Frames and Wheel Locks

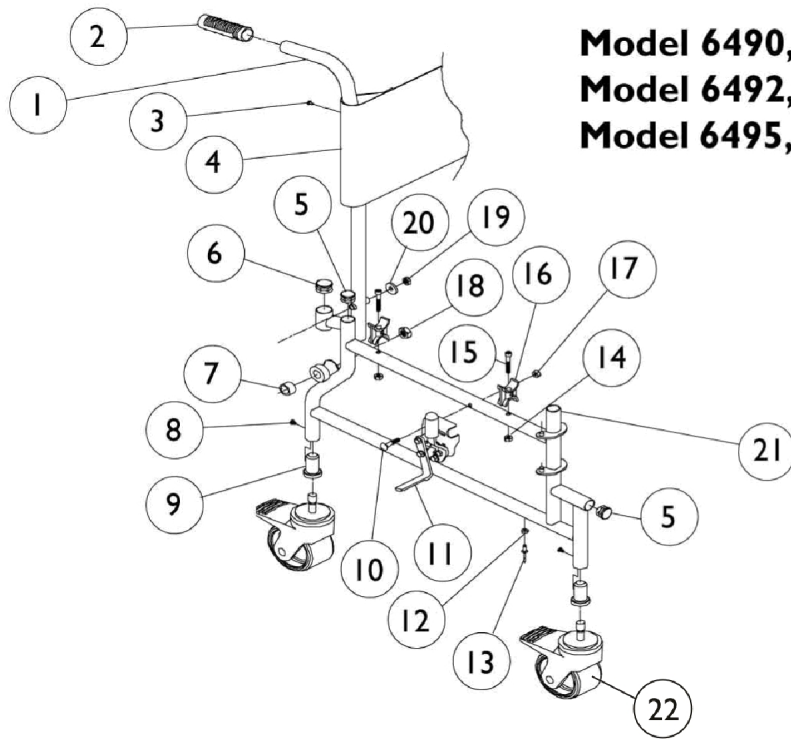

Model 6490, 6690 With Casters
Model 6492, 6692 With 24" Pneumatic
Model 6495, 6695 With 24" Urethane

Rehab Shower/Commode Chair Back Upholstery, Frames and Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1079121-NLA	NLA - Frame with Hardware, Navy Blue - Right	1	1
1	A,B	1079122-NLA	NLA - Frame with Hardware, Navy Blue - Left	1	1
1	A,C	1023623-NLA	NLA - Frame with Hardware, Navy Blue - Right	1	1
1	A,C	1023624-NLA	NLA - Frame with Hardware, Navy Blue - Left	1	1
2		40299X009-NLA	NLA - Grip, Hand, Grey	1	2
3		1017678-NLA	NLA - Fastener, Clip	1	2
3		1023632	Clip, Shroud	1	2
4		1017104	Upholstery, Back (6490 Series)	1	1
4		1028043-NLA	NLA - Upholstery, Back (6690 Series)	1	1
5		1040840-NLA	NLA - Plug Button, Grey (7/8")	1	2
6		1057287	Plug Button, Grey (1")	1	2
7	D	1028832	Spacer (5/8 x 7/8 x 1/2")	1	2
8		1021384-NLA	NLA - Screw, Phillips Pan Head (#8-32 x 1/4")	1	4
9		1016682	Bearing, Sleeve Caster	1	2
10		1021402	Bolt, Carriage (1/4-20 x 1-1/4")	1	1
11		1026243-NLA	NLA - Wheel Lock - Right	1	1
11		1026244-NLA	NLA - Wheel Lock - Left	1	1
12		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	4
13		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	4
14		1023521-NLA	NLA - Locknut, Hex (1/4-20)	1	4
15		1004170	Screw, Socket Head (1/4-20 x 1-1/2")	1	4
16		1015390	Guide, Seat	1	2
17		1021403	Nut, Hex Flang (1/4-20)	1	1
18	D	1023523-NLA	NLA - Locknut (5/8-18)	1	2
19		1021383-NLA	NLA - Locknut, Hex (5/16-18)	1	4
20		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	4
21		40186X017-NLA	NLA - Tube, Insert	1	1
22		1040673-NLA	NLA - Kit, Caster Assembly w/ Bearing Sleeve, Swivel w/o Lock (5")	1	2
22		1058160-NLA	NLA - Kit, Caster Assembly w/ Bearing Sleeve, Swivel w/o Lock (5") (Pair)	2	1
22		1040674-NLA	NLA - Caster Assembly with Lock	1	2
22		1058161-NLA	NLA - Package, Caster Assembly with Lock	4	2

NOTE: A - Includes items 2, 5, 6, 8, 9, 11, 13-16 and 21.
 B - Before 7/17/96.
 C - After 7/16/96.
 D - Wheel Hardware.