


Amputee Attachment

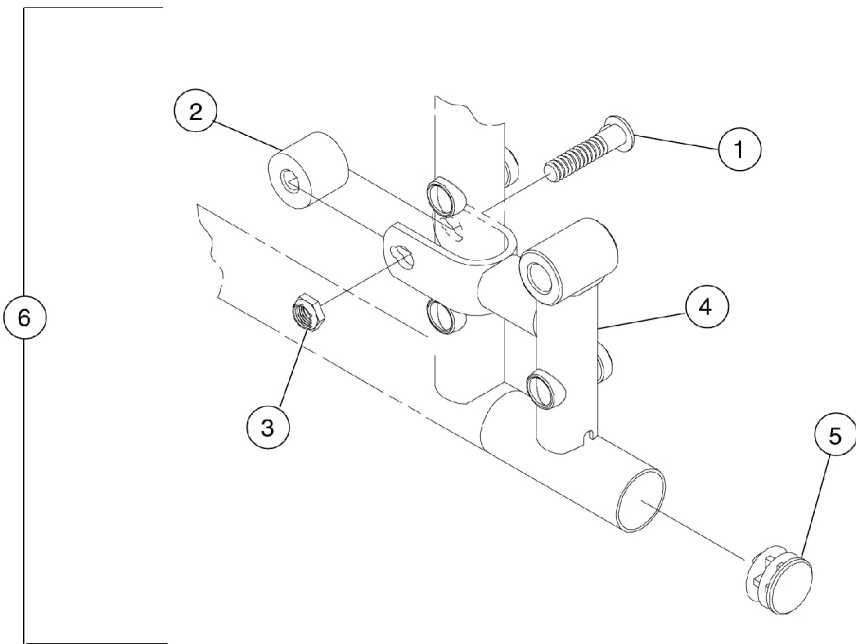

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	2
2		47324X011	Bushing, Pivot (5/16 x 7/8 x 3/4")	1	2
3		1107321	Nut, Low Cap (5/16"-24)	1	2
4		1065769	Bracket, Amputee - Right	1	1
4		1065770	Bracket, Amputee - Left	1	1
5		1027095	Plug Button, Black (7/8")	1	2
6	A	1065789-NLA	NLA - Attachment, Amputee (Pair)	2	1
NOTE: A - Includes items 1-5.					