


Seat Frame and Frame Hardware

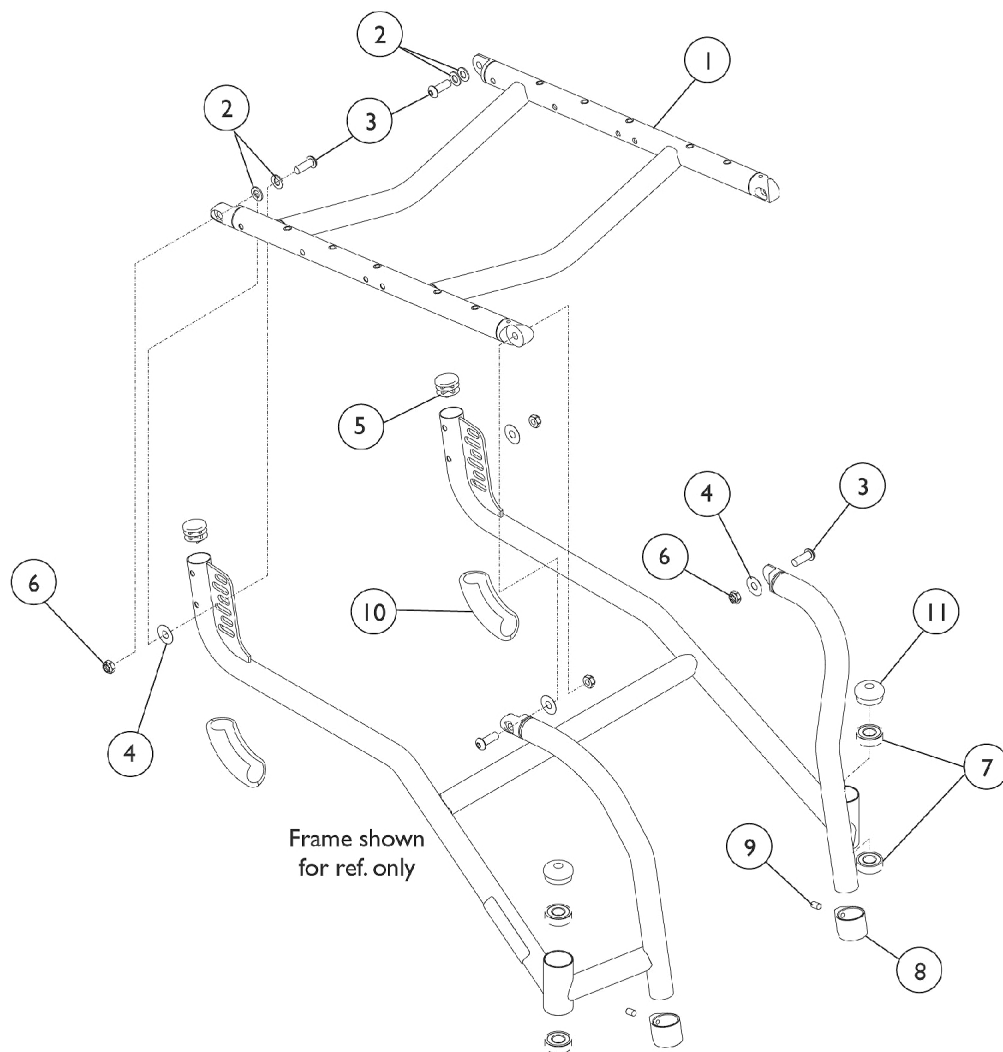

Seat Frame and Frame Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Seat Frames 14-19" D - Aluminum	1	1
1		See Chart	#1 Seat Frames 14-19" D - Titanium	1	1
1		See Chart	#1 Seat Frames 20-22" D - Aluminum	1	1
1		See Chart	#1 Seat Frames 20-22" D - Titanium	1	1
2		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
3		1046494	Screw, Socket Button Head (5/16-18 x 7/8")	1	4
4		1046497-NLA	NLA - Washer, Nylon (5/16 x 7/8 x 1/64")	1	4
5	A	1056328	Plug Button, Black (1")	1	2
5	B	1029501	Plug Button, Black (1")	1	2
6		1028769	Locknut (5/16-18)	1	4
7		40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	4
8		1111949-NLA	NLA - Clamp, Footrest	1	2
9		1026911-NLA	NLA - Screw, Socket Set Cone Point (1/4-20 x 1/2")	1	2
10		1111934	Protector, Rear Frame	1	2
11	A	1088523	Plug Button, Head Tube, Black	1	2
11	B	1116007-NLA	NLA - Plug Button, Head Tube, Black	1	2
	A,C,D	1114109-NLA	NLA - Hardware, Adjustable Frame	1	1
	B,C,D	1116322-NLA	NLA - Hardware, Adjustable Frame	1	1
NOTE: A - For Aluminum Frame B - For Titanium Frame. No Longer Available. Order part number 1088523 as a replacement. C - Not Shown D - Includes items 2-11					

#1 Aluminum Seat Frames

14-19" Deep

Chair Width	14" Deep	15" Deep	16" Deep	17" Deep	18" Deep	19" Deep
12"	8881114003-NL A	--	--	--	--	--
13"	8881114004-NL A	--	--	--	--	--
14"	8881114005-NL A	8881114010-NL A	8881114015-NL A	8881114022-NL A	8881114029-NL A	--
15"	8881114006-NL A	8881114011-NL A	8881114016-NL A	8881114023-NL A	8881114030-NL A	--
16"	8881114007-NL A	8881114012-NL A	8881114017-NL A	8881114024-NL A	8881114031-NL A	8881114038-NL A
17"	8881114008-NL A	8881114013-NL A	8881114018-NL A	8881114025-NL A	8881114032-NL A	8881114039-NL A
18"	8881114009-NL A	8881114014-NL A	8881114019-NL A	8881114026-NL A	8881114033-NL A	8881114040-NL A
19"	--	--	8881114020-NL A	8881114027-NL A	8881114034-NL A	8881114041-NL A
20"	--	--	8881114021-NL A	8881114028-NL A	8881114035-NL A	8881114042-NL A
21"	--	--	--	--	8881114036-NL A	8881114113-NL A
22"	--	--	--	--	8881114037-NL A	8881114114-NL A

Seat Frame and Frame Hardware

Specify frame color when ordering.

Seat Frame and Frame Hardware

#1 Aluminum Seat Frames

20-22" Deep

Chair Width	20" Deep	22" Deep
20"	8881114043-NLA	8881114046-NLA
21"	8881114044-NLA	8881114047-NLA
22"	8881114045-NLA	8881114048-NLA

Specify frame color when ordering.

#1 Titanium Seat Frames

14-19" Deep

Chair Width	14" Deep	15" Deep	16" Deep	17" Deep	18" Deep	19" Deep
12"	8881116273-NL A	--	--	--	--	--
13"	8881116274-NL A	--	--	--	--	--
14"	8881116275-NL A	8881116280-NL A	8881116285-NL A	8881116292-NL A	8881116299-NL A	--
15"	8881116276-NL A	8881116281-NL A	8881116286-NL A	8881116293-NL A	8881116300-NL A	--
16"	8881116277-NL A	8881116282-NL A	8881116287-NL A	8881116294-NL A	8881116301-NL A	8881116308-NL A
17"	8881116278-NL A	8881116283-NL A	8881116288-NL A	8881116295-NL A	8881116302-NL A	8881116309
18"	8881116279-NL A	8881116284-NL A	8881116289-NL A	8881116296-NL A	8881116303-NL A	8881116310-NL A
19"	--	--	8881116290-NL A	8881116297-NL A	8881116304-NL A	8881116311-NL A
20"	--	--	8881116291-NL A	8881116298-NL A	8881116305-NL A	8881116312-NL A
21"	--	--	--	--	8881116306-NL A	8881116313-NL A
22"	--	--	--	--	8881116307-NL A	8881116314-NL A

Specify frame color when ordering.

Seat Frame and Frame Hardware

#1 Titanium Seat Frames

20-22" Deep

Chair Width	20" Deep	22" Deep
20"	8881116315-NLA	8881116318-NLA
21"	8881116316-NLA	8881116319-NLA
22"	8881116317-NLA	8881116320-NLA
Specify frame color when ordering.		