
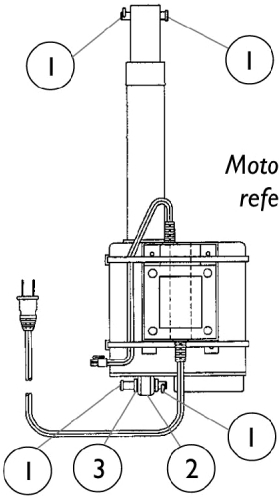
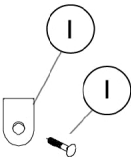


Frame and Hardware

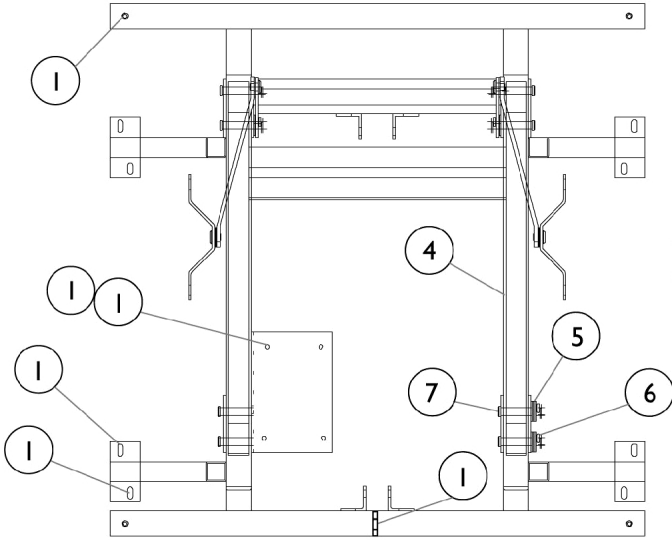

Motor shown for reference only

Frame and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1091122-NLA	NLA - Kit, Frame Hardware Includes: 6 ea. Screw, Phillips Pan Head (#10 x 5/8") 3 ea. Cable Tie 2 ea. Pin, Hitch (3/8 x 1-3/16") 2 ea. Pin, Clevis (3/8 x 2") 4 ea. Screw (#10 x 1-1/2") 4 ea. Screw, Hex Head (1/4-20 x 1-1/4") 4 ea. Screw, Pan Head (#8-32 x 1") 4 ea. Nut, Nylock (#8-32) 4 ea. End Cap (1-1/4") 4 ea. Glide, Adjustable 4 ea. Spacer, Nylon (1/4 x 3/4 x 7/16") 4 ea. End Cap (1") 1 ea. Clamp, Power Cord (1/8" Cord) 6 ea. Clamp, Cable 1 ea. Screw, Self Tapping (#8-32 x 5/16").	1	1
1	A	14222X001	Screw, Phillips Pan Head (#10 x 5/8")	1	6
1	A	51000M562-NLA	NLA - Tie Wrap	1	3
1	A	51000M743	Pin, Hitch (3/8 x 1-3/16")	1	2
1	A	51000M784-NLA	NLA - Pin, Clevis (3/8 x 2")	1	2
1	A	53000M967-NLA	NLA - Screw, Hex Head (#10 x 1-1/2")	1	4
1	A	54000M087	Screw, Hex Head (1/4-20 x 1-1/4")	1	4
1	A	54000M793-NLA	NLA - Screw, Phillips Pan Head (#8-32 x 1")	1	4
1	A	54000M794-NLA	NLA - Nut, Nylock (#8-32")	1	4
1	A	1012756	Plug Button, Black (1-1/4")	1	4
1	A	51000M598-NLA	NLA - Glide, Adjustable	1	4
1	A	1070001-NLA	NLA - Spacer, Nylon (1/4 X 3/4 X 7/16")	1	4
1	A	54000M095-NLA	NLA - Cap, End, Black (1" x 1")	1	4
1	A	1095686	Clamp, Power Cord (1/8")	1	1
1	A	1094524-NLA	NLA - Clamp, Cable	1	6
1	A	020647	Screw, Phillips Pan Head Tap (#8-32 x 5/16")	1	1
2	B	53000M015-NLA	NLA - Bearing, Flange	1	2
3	B	53000M627-NLA	NLA - Washers, Nylon (3/8 x 5/8 x 1/8")	1	2
4	B,C	7001M108-NLA	NLA - Lift Frame with Hardware	1	1
5		51000M582-NLA	NLA - Washer, Nylon (5/16 x 1 x 1/16")	1	12
6		51000M743	Pin, Hitch (3/8 x 1-3/16")	1	10
7		51000M784-NLA	NLA - Pin, Clevis (3/8 x 2")	1	10
	D	51000M764-NLA	NLA - Clamp, Cable (5/16")	1	3
	D,E	54000M743-NLA	NLA - Clamp, Cable (1/4" cord)	1	1
	D,F	8389L-NLA	NLA - Mechanism, Scissor - Left	1	1
	D,F	8389R-NLA	NLA - Mechanism, Scissor - Right	1	1
	D,G	1111901-NLA	NLA - Mechanism, Scissor, 622	2	1
	D,H	52000M354-NLA	NLA - Kit, Transformer Bracket (Includes: Linkage Plate, Washers, Screws, Bolts, Nuts, Cable Ties, and Instruction Sheet)	1	1

Frame and Hardware

NOTE: A - This is a part of the Hardware Kit.
B - Used on Units manufactured before 4/14/00 only
C - Includes Frame and items 5-7
D - Not Shown
E - Used on Units manufactured before 11/9/98
F - Used on units manufactured after 1997
G - Used on Units manufactured before 1997
H - Used on Units Manufactured before 12/15/98.