


Infinity Drop Base Hardware

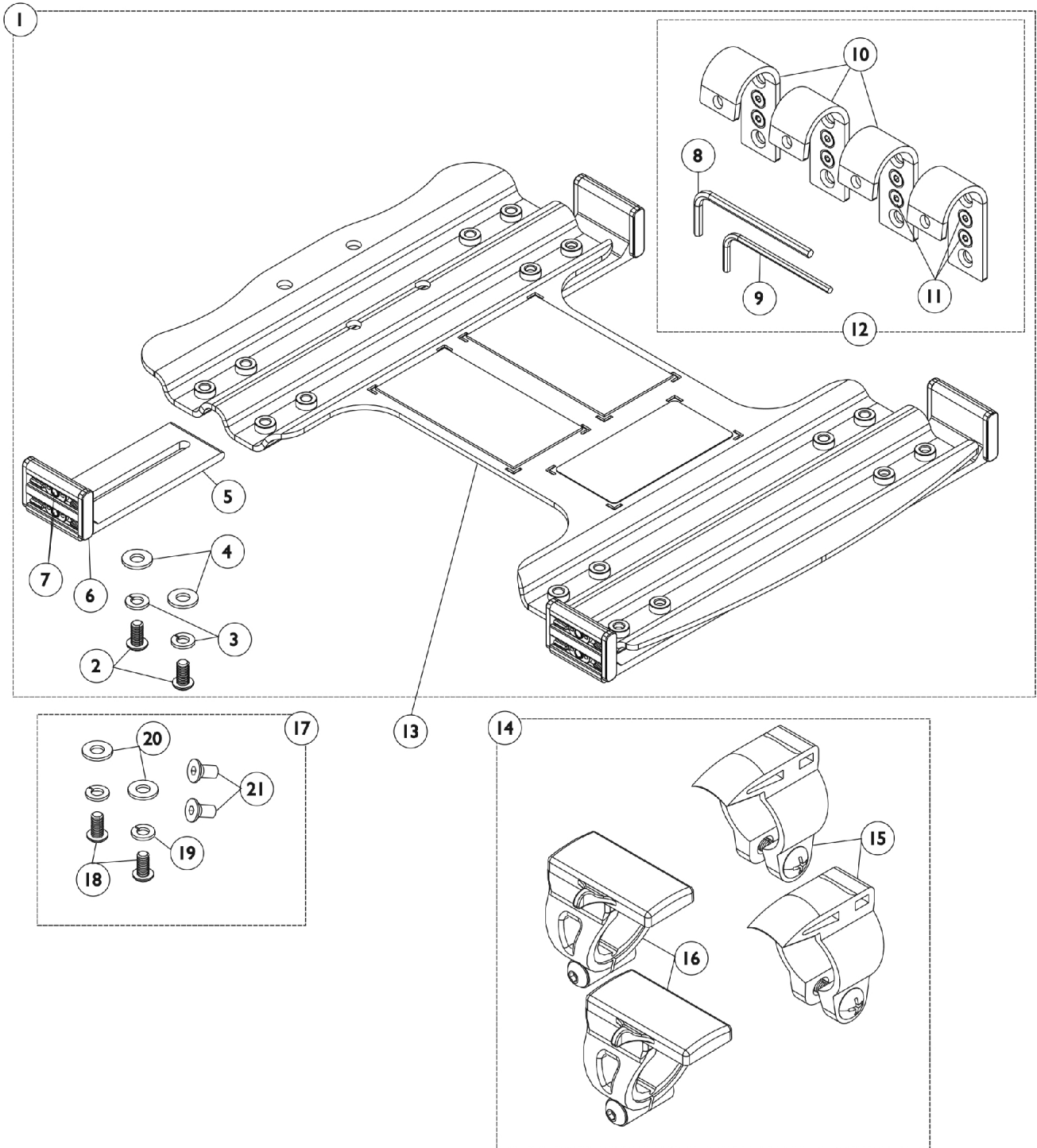

Infinity Drop Base Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1084040	Kit, Drop Base Hooks and Hardware	1	1
2			Screw, Socket Button Head (1/4-20 x 5/8")	1	8
3			Lockwasher, Split (1/4")	1	8
4			Washer, Steel (5/16 x 3/4 x 1/16")	1	8
5			Extrusion, Drop Base	1	4
6			Cap, End	1	8
7			Nut, Track	1	8
8			Key, Allen (5/32")	1	1
9			Key, Allen (1/8")	1	1
10			Hook, Drop Base	1	4
11			NLA - Screw, Socket Flat Head (#10-32 x 3/8")	1	8
12	B	1084152-NLA	NLA - Kit, Infinity Drop Hook	1	1
8			Key, Allen (5/32")	1	1
9			Key, Allen (1/8")	1	1
10			Hook, Drop Base	1	4
11			NLA - Screw, Socket Flat Head (#10-32 x 3/8")	1	8
13		See Chart	Item #13 - Drop Base Pans	1	1
14	C	1084039-NLA	NLA - Kit, Drop Base Clamp with Instruction Sheet	1	1
15			NLA - Clamps, Fixed Rail w/ Instruction Sheet (7/8" and 1" Tubing) (Pair)	2	1
16			NLA - Clamps, Slide Release Rail w/ Instruction Sheet (7/8" and 1" Tubing)	2	1
17	D	1084355-NLA	NLA - Instruction Sheet, Slide Release	1	1
18			NLA - Kit, Infinity Drop Base Hardware	1	1
19			Screw, Socket Button Head (1/4-20 x 5/8")	1	8
20			Lockwasher, Split (1/4")	1	8
21			Washer, Steel (5/16 x 3/4 x 1/16")	1	8
			NLA - Screw, Socket Flat Head (#10-32 x 3/8")	1	8

NOTE: A - Includes: Items 2 - 11
 B - Includes: Items 8 - 11
 C - Includes: Items 15, 16 and Instruction Sheet
 D - Includes: Items 18 - 21

Item #13 - Drop Base Pans

	16" Deep	18" Deep	20" Deep
14-16" Wide	1084041-NLA	1084042-NLA	1084043-NLA
17-19" Wide	1084044-NLA	1084045-NLA	1084046-NLA
20-22" Wide	1084047-NLA	1084048-NLA	1084049-NLA