


Front (Hemi) Riggings Attachment Kit

After 8/31/02

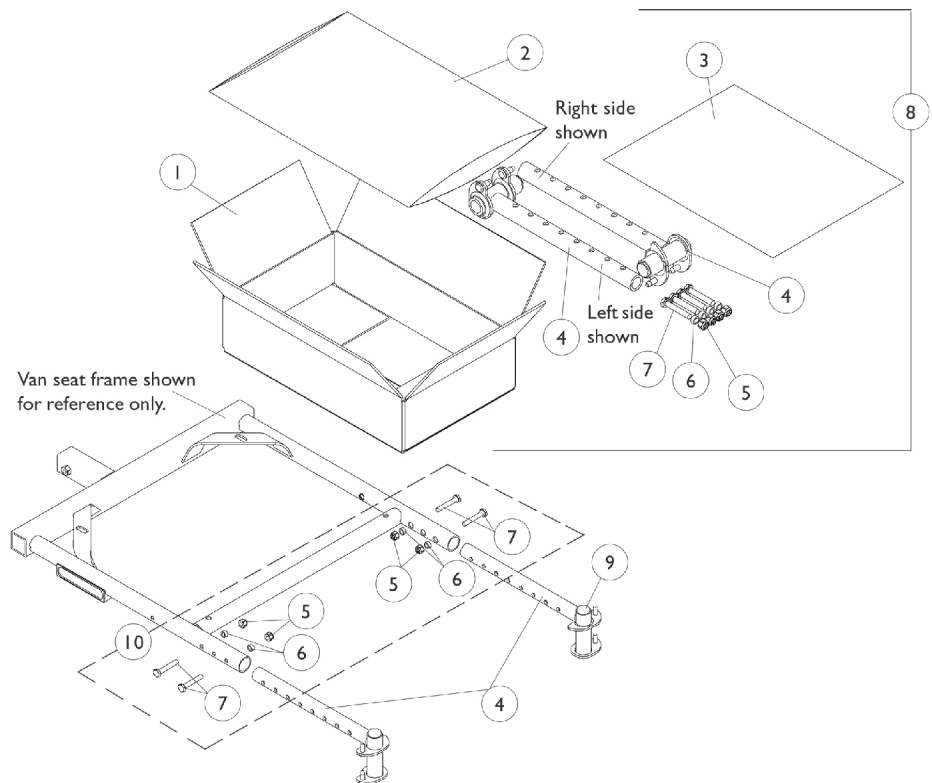

Front (Hemi) Riggings Attachment Kit After 8/31/02

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
8	A,C	1114997	Kit, Front Rigging Hanger Attachment	1	1
1			Carton, Shipper	1	1
2			NLA - Bag, Poly Zip-Lip (13" L x 9" W)	1	1
3			NLA - Instruction Sheet, Front Rigging Hanger	1	1
4			Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Right	1	1
4			Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Left	1	1
5			Locknut (1/4-20)	1	4
6			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
7			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
9			1027097	Plug Button, Black (3/4")	1
10	B	1113045	Kit, Front Rigging Hanger Hardware	1	1
5			Locknut (1/4-20)	1	4
6			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
7			Screw, Hex Head (1/4-20 x 1-1/2")	1	4

NOTE: A - Includes items 1-7
 B - Includes items 5-7
 C - Part number 1110523 is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.