
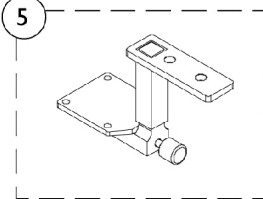
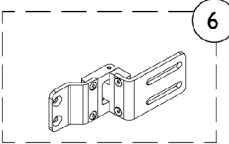
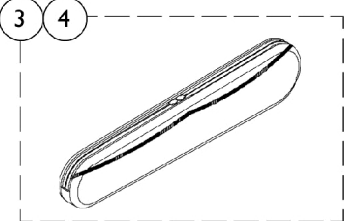
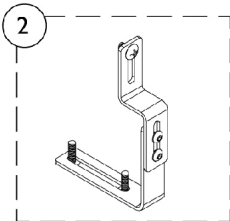


KSS Seating Accessories for Abductors, Adductors and Laterals

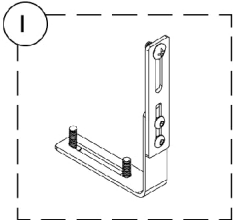

KSS Seating Accessories for Abductors, Adductors and Laterals

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1072716-NLA	NLA - Brackets, Adductor, Flush, Black	1	1
2		1073061-NLA	Brackets, Adductor Offset, Black (1")	1	1
3		1072431-NLA	NLA - Pad, Adductor - Small (10" L)	1	1
3		1072902-NLA	NLA - Pad, Adductor - Medium (12" L)	1	1
3		1072432-NLA	NLA - Pad, Adductor - Large (14" L)	1	1
4		1072433-NLA	NLA - Cover, Adductor - Small (Fits 10")	1	1
4		1072434-NLA	NLA - Cover, Adductor - Medium (Fits 12")	1	1
4		1072741-NLA	NLA - Cover, Adductor - Large (Fits 14")	1	1
5		1063831-NLA	NLA - Bracket, Abductor Flip Down - Small (Height Adjustable From 2-3/8" to 3-3/4")	1	1
5		1063830-NLA	NLA - Bracket, Abductor Flip Down - Large (Height Adjustable From 3" to 5-1/8")	1	1
	B	1063834-NLA	NLA - Pad, Abductor Flip Down - Small (3" H x 4-3/4" D x 2-1/4" W)	1	1
	B	1063833-NLA	NLA - Pad, Abductor Flip Down - Large (4" H x 6" D x 3-1/2" W)	1	1
6		1072703-NLA	NLA - Bracket, Lateral Support, Swingaway - Small	2	1
6		1072526-NLA	NLA - Bracket, Lateral Support, Swingaway - Medium/ Large	2	1
	B	1063829-NLA	NLA - Bracket, Lateral Support, Fixed - Small	1	1
	B	1063828-NLA	NLA - Bracket, Lateral Support, Fixed - Medium/ Large	1	1
	B	5000054-NLA	NLA - Pads, Lateral Support, Fixed - Small (4" H x 5" D x 1" W)	2	1
	B	5000053-NLA	NLA - Pads, Lateral Support, Fixed - Medium (4-1/2" H x 6" D x 1" W)	2	1
	B	5000052-NLA	NLA - Pads, Lateral Support, Fixed - Large (5" H x 7-1/2" D x 1" W)	2	1
	A,B	1077148-NLA	NLA - Cover, Lateral Support - Small (4" H x 5" D x 1" W)	1	1
	A,B	1077149-NLA	NLA - Cover, Lateral Support - Medium (4-1/2" H x 6" D x 1" W)	1	1
	A,B	1077150-NLA	NLA - Cover, Lateral Support - Large (5" H x 7-1/2" D x 1" W)	1	1
	B	1077157-NLA	NLA - Pad, Lateral Support, Swingaway - Small (4" H x 5" D x 1" W)	1	1
	B	1077158-NLA	NLA - Pad, Lateral Support, Swingaway - Medium (4-1/2" H x 6" D x 1" W)	1	1
	B	1077159-NLA	NLA - Pad, Lateral Support, Swingaway - Large (5" H x 7-1/2" D x 1" W)	1	1

NOTE: A - Lateral Support Covers fit both Fixed and Swingaway Support Pads.
B - Not Shown