


PinDot ContourU Adjustable Footrests

LB07 and LB08 Interfacing Hardware

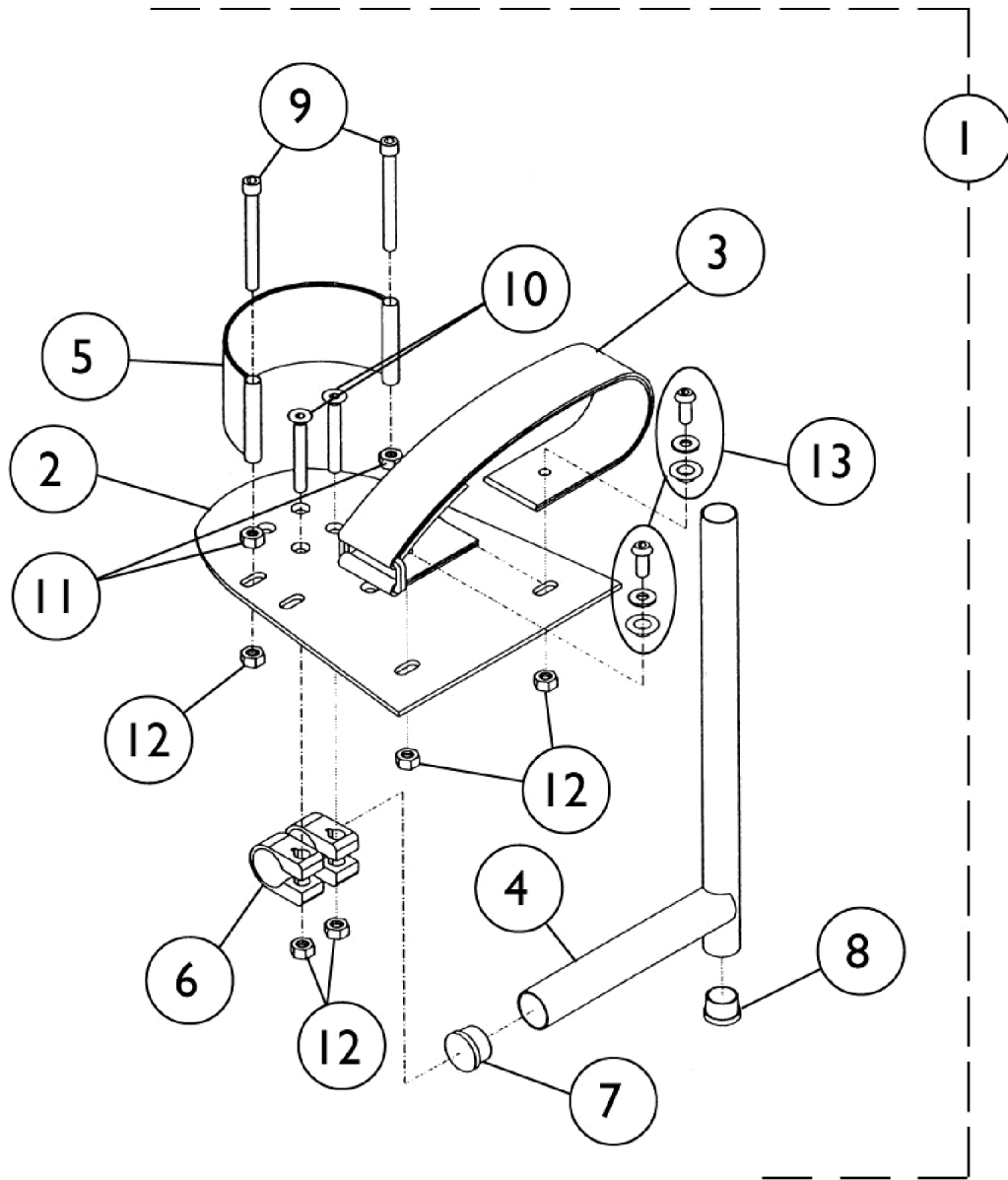

PinDot ContourU Adjustable Footrests LB07 and LB08 Interfacing Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	3001275-NLA	NLA - Adjustable Footrest Assembly - Small - LB07 (Pair)	2	1
1	B	1090100-NLA	NLA - Footrest Assembly, Adjustable, Small, Single - Right	1	1
1	B	1090101-NLA	NLA - Footrest Assembly, Adjustable, Small, Single - Left	1	1
1	B	3001264-NLA	NLA - Adjustable Footrest Assembly - Large - LB08 (Pair)	2	1
1	B	1090102-NLA	NLA - Footrest Assembly, Adjustable, Large, Single - Right	1	1
1	B	1090103-NLA	NLA - Footrest Assembly, Adjustable, Large, Single - Left	1	1
2		3000731	NLA - Plate, Foot - Small (4-25/32" W x 6-1/8" L)	1	1
2		3000732-NLA	NLA - Plate, Foot - Large (6-1/8" W x 8" L)	1	1
3		3001272-NLA	NLA - Strap, Ankle, LB07, Small	1	2
3		3001258-NLA	NLA - Strap, Ankle, LB08, Large	1	2
4		3001261-NLA	NLA - Pivot/Slide Tube Assembly with Plug Buttons, Chrome	1	2
5		3000966	Heel Loop, Small (2" W x 5-3/4" L) - LB07	1	2
5		3000967	Heel Loop - Large - Right/Left (2" W x 6-7/8" L) - LB08	1	2
6		47311X001-NLA	NLA - Clamp (7/8")	1	2
7		1200034	Plug Button, Round, Black (7/8")	1	2
8		1200036	Plug Button, Round, Black (3/4")	1	2
9		1100091-NLA	NLA - Screw, Socket Head (1/4-20 x 2-1/2")	1	2
10		1100208-NLA	NLA - Screw, Socket Flat Head (1/4-20 x 1-3/4")	1	2
11		1100030-NLA	NLA - Nut, Hex (1/4-20)	1	2
12		1100084-NLA	NLA - Locknut, Nylon (1/4-20)	1	6
13		1100209-NLA	NLA - Eyelet, Grommet and Washer	1	2
	A	1100039	Washer (1/4 x 9/16 x 1/16")	1	4

NOTE: A - Not Shown
B - Includes items 2-13