


# I.V. Stand Model 6845

Model 6845

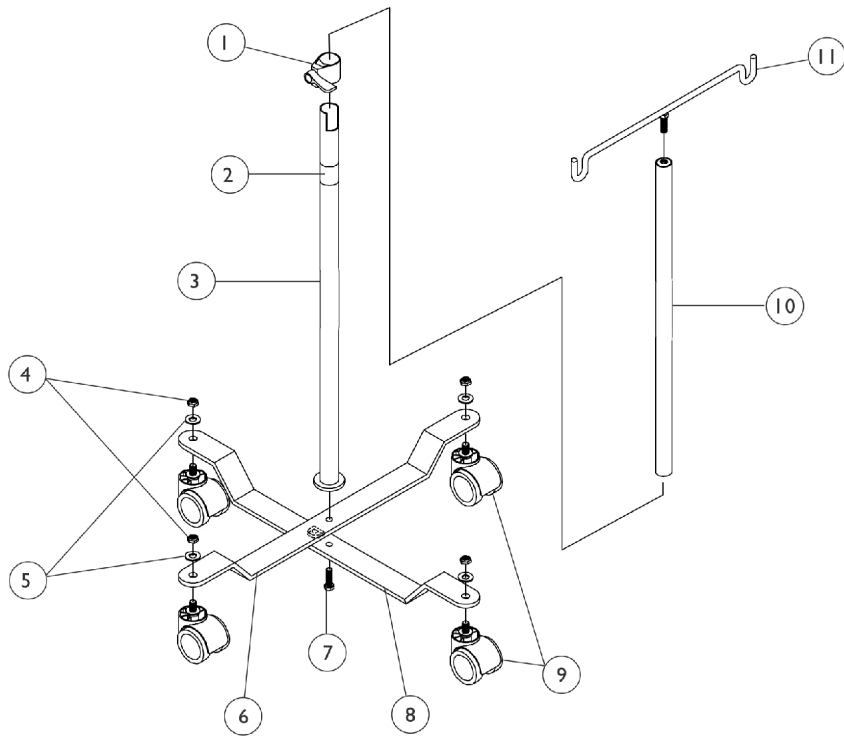

## I.V. Stand Model 6845

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		12592X000-NLA	NLA - Adjustment Lock Assembly, Chrome	1	1
2		1095497	Decal, Invacare Logo (1.6)	1	1
3	A	18051X000-NLA	NLA - Tube, Lower	1	1
4		40363X000	Hex Nut (3/8"-16)	1	4
5		16325X000-NLA	NLA - Washer (3/8 x 13/16 x 1/16")	1	4
6	A	18054X000-NLA	NLA - Base, Upper Half	1	1
7		12144X000	Screw, Hex Head (5/16-18 x 1")	1	1
8	A	18056X000-NLA	NLA - Base, Lower Half	1	1
9		1050270-NLA	NLA - Kit, Caster for I.V. Rod (2")	1	4
10		18052X000-NLA	NLA - Tube, Upper	1	1
11	A	18053X000-NLA	NLA - Hanger, I.V.	1	1

NOTE: A - For replacement parts, order 6845