

Youthmobile 9000 Wheelchair



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 89-79. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2022 Invacare Corporation. Elyria, Ohio. Invacare, Yes, you can., and the Medallion Design are registered trademarks of Invacare Corporation.

TABLE of CONTENTS

Description	Page Number
Accessories - 1" Forward Option	68
Accessories - Amputee Attachment	70
Accessories - Fabric Clothing Protectors	56
Accessories - Head Support	74
Accessories - Headrest Extension	72
Accessories - Miscellaneous	78
Accessories - Seat Extension Kits	66
Accessories - Solid Back and Solid Seat Inserts	64
Accessories - Solid Folding Seats	62
Accessories - Stroller Handle	60
Accessories - Torso Support	76
Accessories - Utility Tray	58
Arms - Non-Removable Half Arm	16
Arms - Space Saver Adjustable Height Arm - 79"/89"	20
Arms - Space Saver Fixed Height Arm - 88"	18
Canes - Back & Back Upholstery	12
Casters - Front and Mounting Hardware	25
Crossbraces & Hardware	10
Forks and Hardware	23
Frame Finishes & Upholstery Colors	4
Frames - Side & Hardware	8

TABLE of CONTENTS

Description	Page Number
Front Riggings - 90 Degree Footrest with Aluminum Footplate	46
Front Riggings - Footrest and Legrest Accessories	55
Front Riggings - Footrest Support Assembly - H903MC/H903MK	44
Front Riggings - Footrest Support Assembly - H93E	40
Front Riggings - Footrest Support Assembly - H93M	42
Front Riggings - Index and Calf Pads	38
Front Riggings - Legrest Support Assembly	48
Front Riggings - Pivot Slide Tube with Footplate - H904M Legrests	53
Front Riggings - Pivot Slide Tube with Footplate - H93E / H93M / H903MC / H903MK Footrests	51
Handrim Attaching Hardware	33
Handrims and Attaching Hardware	31
Hill Holder Wheel Lock	36
Packaging & Literature	6
Upholstery - Seat	14
Wheel Lock	34
Wheels - Rear Wheel Hardware	27
Wheels - Rear Wheels	29

Frame Finishes & Upholstery Colors

Frame Finishes

Color Number	Toupeh-up Paint Part Nnumber	Color
06P	1031454-NLA	Super Teal
08P	1031456-NLA	Deep Purple
15P	N/A	Chrome
17P	1046319-NLA	Colbalt Blue
24P	1031450-NLA	Wet Black
30P	1031448-NLA	Sunny Yellow
31P	1031453-NLA	Lollipop Pink
33P	1031455-NLA	Red
34P	1046327-NLA	Black
104P	1101316-NLA	Electric Teal

Sparkle Frame Finishes


Color Number	Color
106P	Wet Black w/Twilight Sparkle

Frame Finishes & Upholstery Colors

Upholstery Colors

Color	Description
U240	Black Nylon
U67	Black Vinyl
U75	Dove Grey Vinyl
U85	Dark Blue Vinyl
U86	Ruby Vinyl
U88	Camel Vinyl
U97	Sky Blue Vinyl


Packaging and Literature

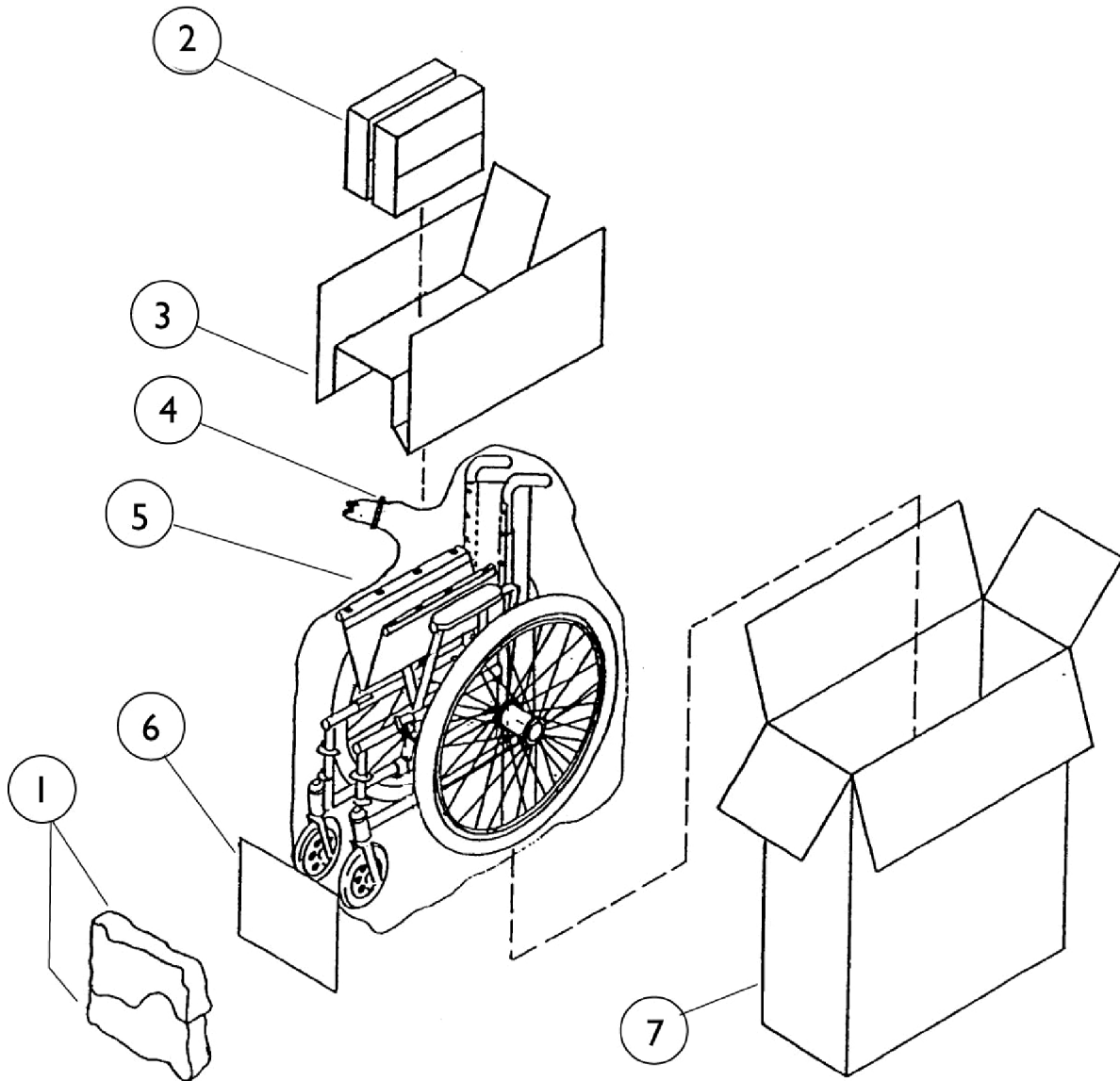

Packaging and Literature

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020106-NLA	NLA - Paper	1	--
2		40131X011-NLA	NLA - Carton, Front Riggings	1	--
3		40131X041	Insert, Top Carton	1	--
4		1016684-NLA	NLA - Tie, Ladder	1	--
5		1015592	Bag, Poly (72" L x 35" W x 19" D)	1	--
6		40131X039-NLA	NLA - Insert, Carton	1	--
7		40131X038-NLA	NLA - Carton	1	--
	A	10369X006-NLA	NLA - Wrench, Open End (3/8 - 7/16")	1	--
	A	1010292-NLA	NLA - Wrench, Allen	1	--
	A	00081X100-NLA	NLA - Owner's Manual	1	--
	A	00077X031-NLA	NLA - Warranty Card	1	--
NOTE: A - Not Shown					
When adjusting or replacing parts please refer to your Owners Manual for assistance					


Frames and Hardware

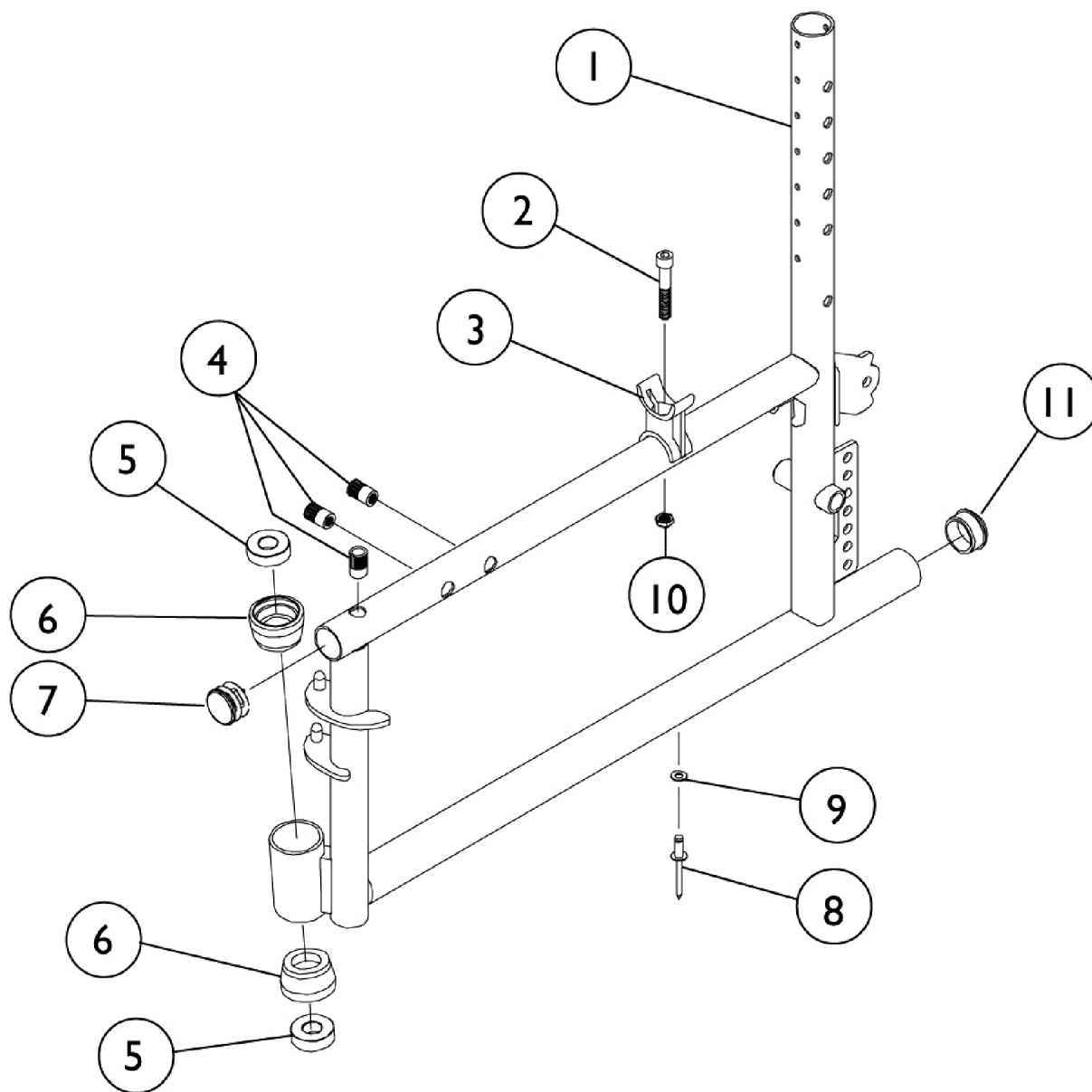

Frames and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1039365-NLA	NLA - Frame, Side with Hardware, Chrome - Right	1	1
1	B	1039366-NLA	NLA - Frame, Side with Hardware, Chrome - Left	1	1
1	B	1039363-NLA	NLA - Frame, Side with Hardware, Black - Right	1	1
1	B	1039364-NLA	NLA - Frame, Side with Hardware, Black - Left	1	1
1	A,B	8881039367-NLA A	NLA - Frame, Side with Hardware, Color - Right	1	1
1	A,B	8881039368-NLA A	NLA - Frame, Side with Hardware, Color - Left	1	1
2		1004170	Screw, Socket Head (1/4-20 x 1-1/2")	1	1
3		1015390	Guide, Seat	1	1
4	E	1019930-NLA	NLA - Insert, Threaded	1	3
5	C	1010810	Bearing, Flange (7/16")	1	2
5	D	1006049	Bearing (7/16")	1	2
6		57768X030-NLA	NLA - Sleeve, Aluminum	1	2
7		1027095	Plug Button, Black (7/8")	1	1
8		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	1
9		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
10		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
11		1029501	Plug Button, Black (1")	1	1
NOTE: A - Specify frame color, see page titled "Frame Finishes and Upholstery Colors" B - Includes items 2-11 C - Before 5/89 D - After 4/89 E - Part #1019930 is no longer available as a service part individually. See item 1 and order a Side Frame w/ Hardware as a replacement.					

Crossbraces and Hardware

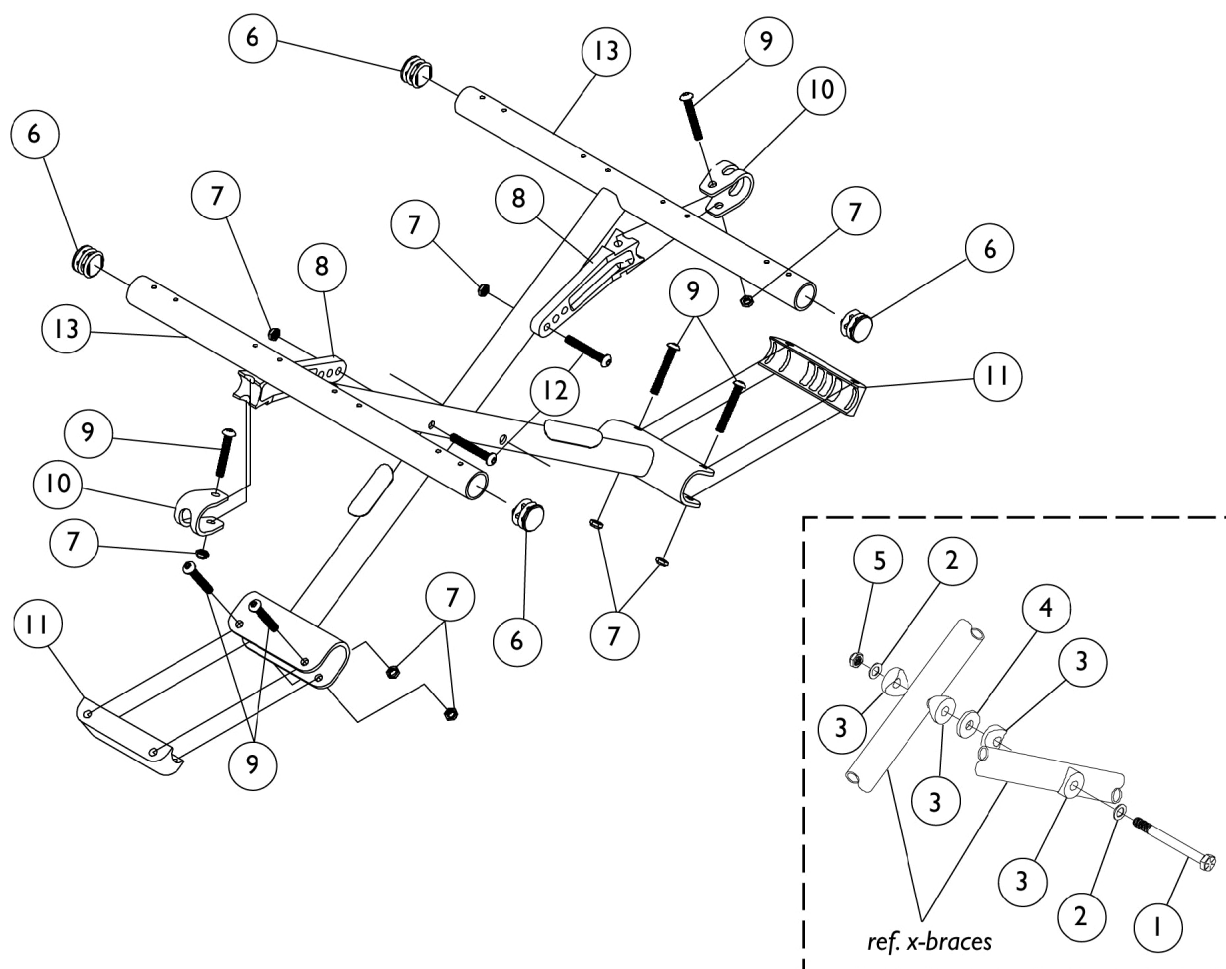


Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016261	Screw, Hex Head (5/16-24 x 3-1/4")	1	2
2		1016652	Washer (5/16 x 9/16 x 1/32")	1	1
3		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	2
4		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	4
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		1029501	Plug Button, Black (1")	1	1
7		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	8
8		1019843-NLA	NLA - Link, Pivot, Black	1	4
9		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
10		1012180	Bracket, Pivot, Black	1	6
11		1012172-NLA	NLA - Clamp, Saddle	1	2
12		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	2
13		1019848-NLA	NLA - Crossbrace (10" W)	1	1
13		1019849-NLA	NLA - Crossbrace (12" W)	1	1
13		1019850-NLA	NLA - Crossbrace (14" W)	1	1
	A	1095497	Decal, Invacare Logo (1.6)	1	1
		7930364-NLA	NLA - Package, Locknut (1/4-20)	--	--


NOTE: A - Not Shown

Back Canes & Back Upholstery

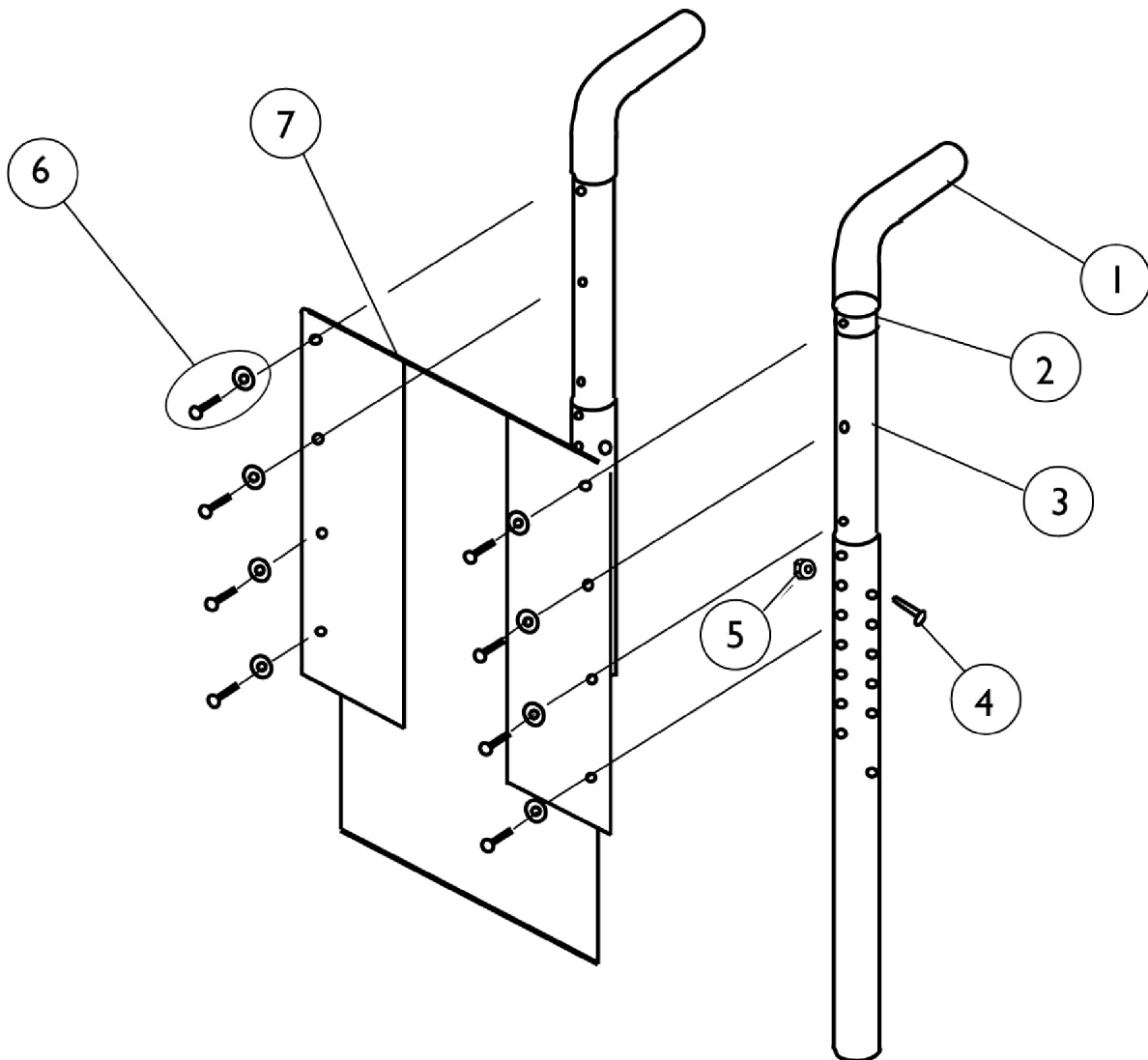

Back Canes & Back Upholstery

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1006229	Grip, Foam (8")	1	2
2		1027097	Plug Button, Black (3/4")	1	2
3		1036516-NLA	NLA - Cane, Back with Plug Button, Black	1	2
4		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
5		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
6		1076283	Screw, Phillips Oval Head w/ Washer (#10-16 x 7/8")	1	8
6		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
7		See Chart	Back Upholstery without Hardware	1	1

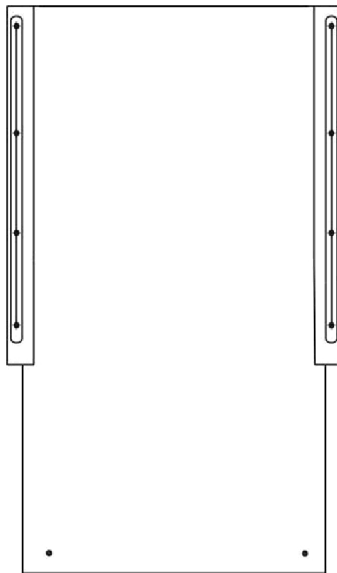


Back Upholstery
without Hardware
- Nylon

Back Upholstery without Hardware

Upholstery	10" Wide	12" Wide	14" Wide
Black Nylon	1019861-NLA	1019862-NLA	1019863-NLA
Vinyl	8881037222-NLA See Note(s) A	8881037223-NLA See Note(s) A	8881037224-NLA See Note(s) A


NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors"

Seat Upholstery

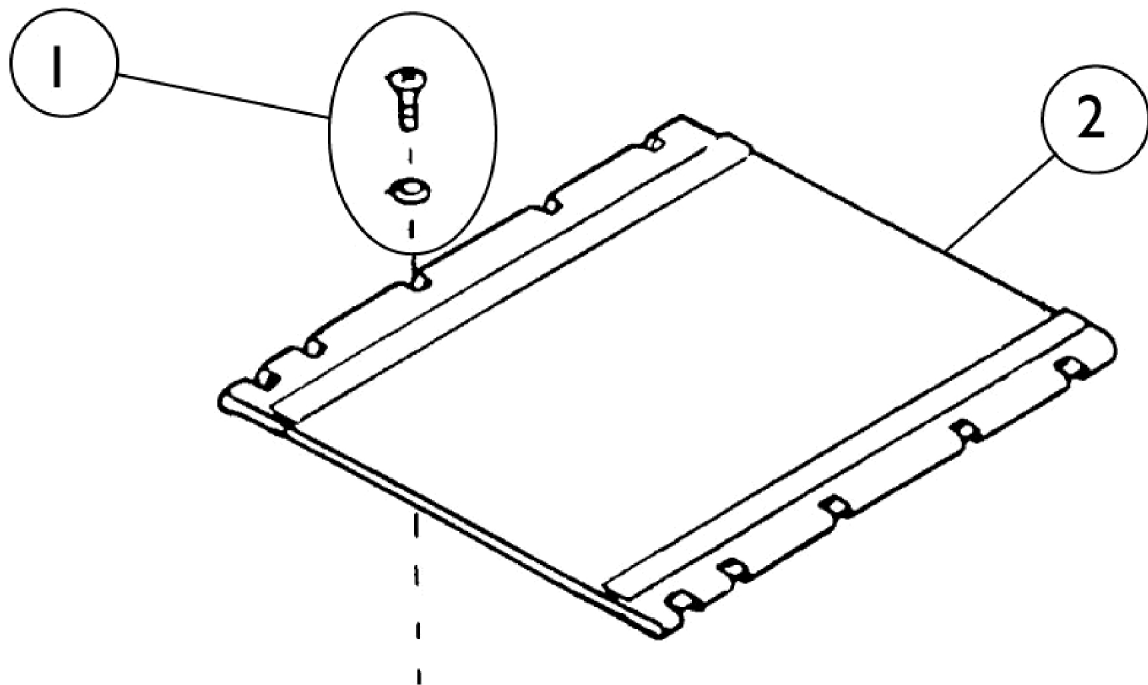

Seat Upholstery

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
2		See Chart	Seat Upholstery without Hardware	1	1

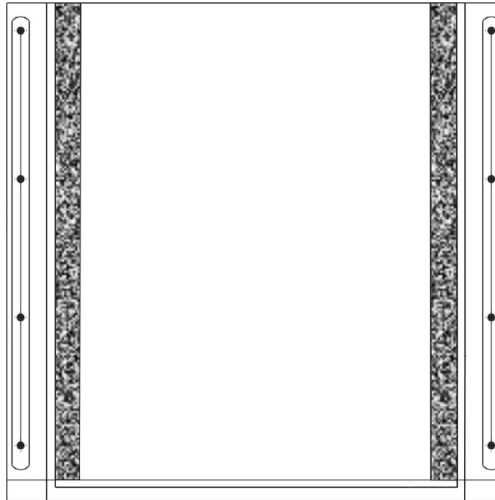


Seat Upholstery
Vinyl/Nylon

Seat Upholstery without Hardware

	Fabric	10" Depth	11" Depth	12" Depth	13" Depth	14" Depth
10" Width	Vinyl	8881037225-NL A See Note(s) A	8881037226-NL A See Note(s) A	8881037227-NL A See Note(s) A	--	--
	Nylon	1019915-NLA	1019916-NLA	1019917-NLA	--	--
12" Width	Vinyl	--	--	8881037228-NL A See Note(s) A	8881037486-NL A See Note(s) A	8881037487-NL A See Note(s) A
	Nylon	--	--	1019922-NLA	1019923-NLA	1019924-NLA
14" Width	Vinyl	--	--	8881037488-NL A See Note(s) A	8881037489-NL A See Note(s) A	8881037490-NL A See Note(s) A
	Nylon	--	--	1019927-NLA	1019928-NLA	1019929-NLA

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors"

Non-Removable Half Arm Desk Length


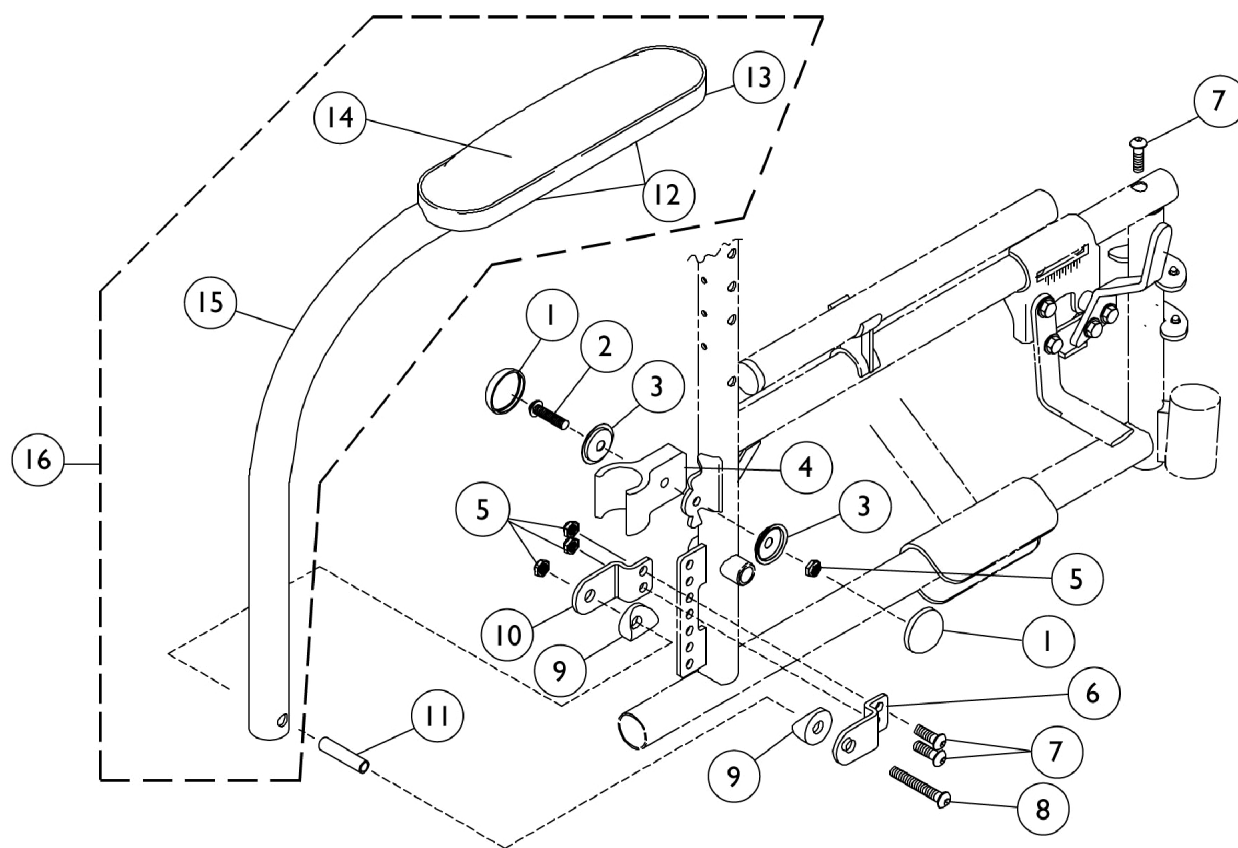


Non-Removable Half Arm Desk Length

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016291	Cap, Hub, Black	1	2
2		1009837	Screw, Socket Button Head (1/4-20 x 1")	1	1
3		1016290-NLA	NLA - Washer, Caplugs (9/32 x 1 x 3/32")	1	2
4		1019841	Lock, Arm, Black	1	1
5		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	4
6		1019957-NLA	NLA - Bracket, Outside, Black	1	1
7		1007269	Screw, Socket Button Head (1/4-20 x 5/8")	1	3
8		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	1
9		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	2
10		1019958-NLA	NLA - Bracket, Inside, Black	1	1
11		1019936	Spacer (1/4 x 11/32 x 1-7/16")	1	1
12		1036501-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-3/8")	2	2
13		1029501	Plug Button, Black (1")	1	1
14	B	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
15		1019933-NLA	NLA - Half Arm Tube - Right	1	1
15		1019934-NLA	NLA - Half Arm Tube - Left	1	1
16	B	1020033-NLA	NLA - Half Arm Tube with Arm Pad - Right	1	1
16	B	1020034-NLA	NLA - Half Arm Tube with Arm Pad - Left	1	1
16	B	1036499-NLA	NLA - Half Arm Tube with Plug Button - Right	1	1
16	B	1036500-NLA	NLA - Half Arm Tube with Plug Button - Left	1	1
	A,C	1036502-NLA	NLA - Half Arm Attaching Hardware (Pair)	2	1
NOTE: A - Not Shown B - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors" C - Includes items 1-11					

Space Saver Fixed Height Arm

Desk Length - "88"

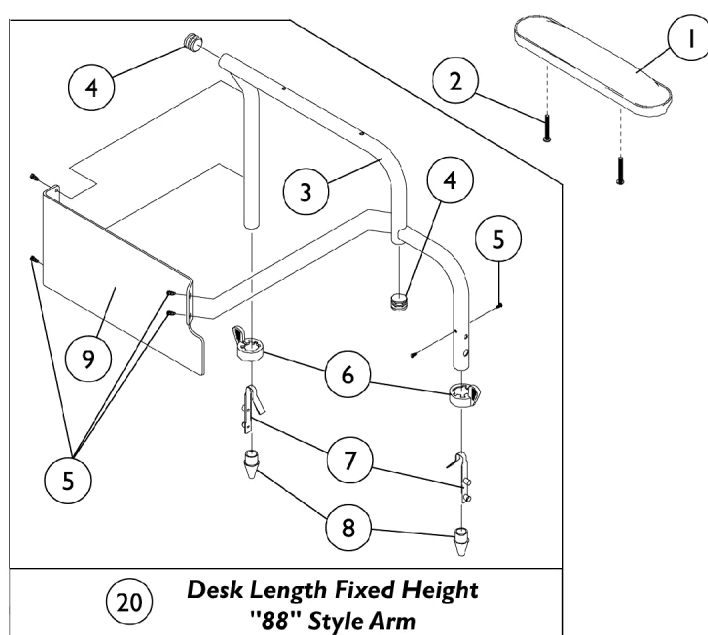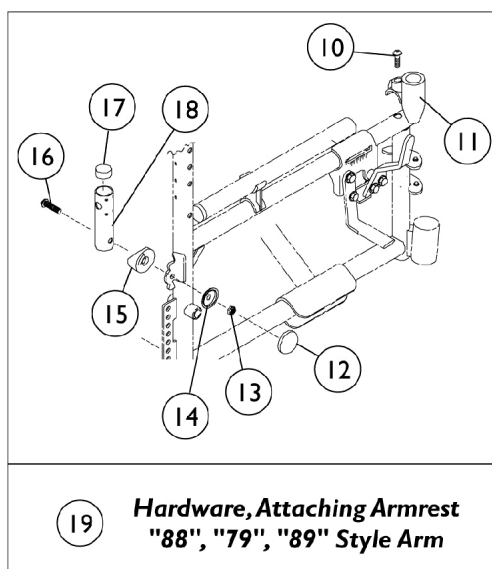



Space Saver Fixed Height Arm

Desk Length - "88"

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	2
3		1019978-NLA	NLA - Tube, Armrest, Black - Right	1	1
3		1019979-NLA	NLA - Tube, Armrest, Black - Left	1	1
4		1027097	Plug Button, Black (3/4")	1	2
5		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
5		1116900	Screw, Self Tapping (#6-20 x 1/2")	1	6
6		1015450	Handle, Quad Release	1	2
7		1018225	Spring, Torsion with Button Lock	1	2
8		1020060	Plug, Arm	1	2
9	D,E	1123305-NLA	NLA - Kit, Clothing Guard, Desk Length w/ Screws (Pair)	1	1
10		1003400	NLA - Screw, Socket Head (1/4-20 x 7/8")	1	1
11		1019842-NLA	NLA - Socket, Armrest, Front	1	1
12		1016291	Cap, Hub, Black	1	1
13		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
14		1016290-NLA	NLA - Washer, Caplugs (9/32 x 1 x 3/32")	1	1
15		1015426	Spacer, Coved Cam - Right	1	1
15		1015427	Spacer, Coved Cam - Left	1	1
16		1023968-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
17		1023975-NLA	NLA - Bushing (3/8 x 3/4 x 3/8")	1	1
18		1015856	Socket, Armrest	1	1
19	B	1036503-NLA	NLA - Hardware, Attaching, Armrest	1	1
20	C,E	1036509-NLA	NLA - Arm Assembly, Desk Length, Black - Right	1	1
20	C,E	1036510-NLA	NLA - Arm Assembly, Desk Length, Black - Left	1	1
NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors" B - Includes items 10-18 C - Includes items 3-9 D - Clothing Guards are not sold individually. Kit includes clothing guards, screws and decals. E - Due to Clothing Guard change, it is recommended (for chairs built Before 11/25/03) when only one Arm is requested the Clothing Guard Kit (p/n 1123305) be ordered also so that the arms will match.					

Space Saver Adjustable Height Arm

Desk/Full Length - "79"/"89"

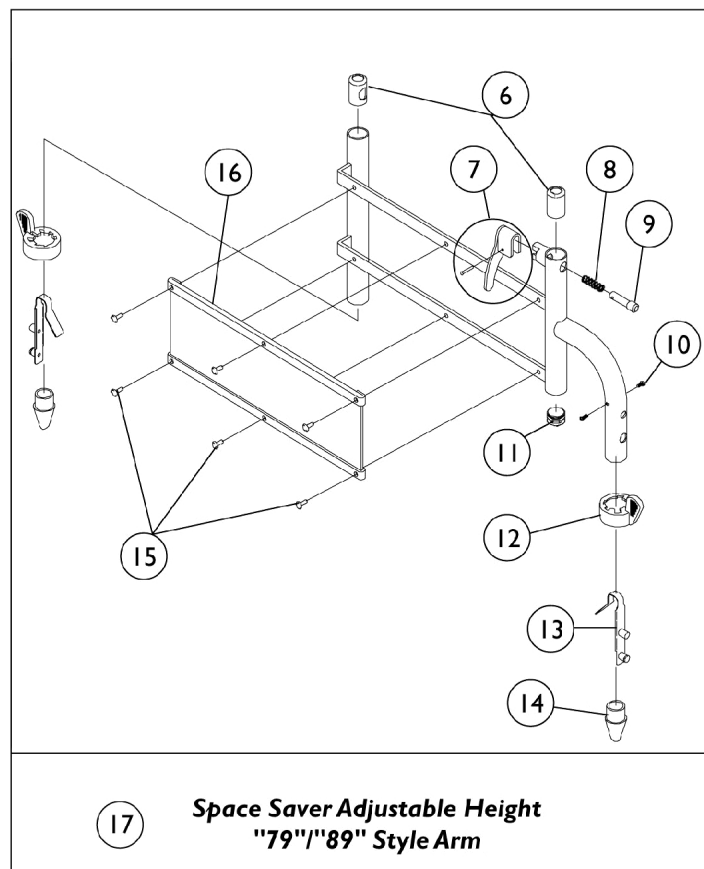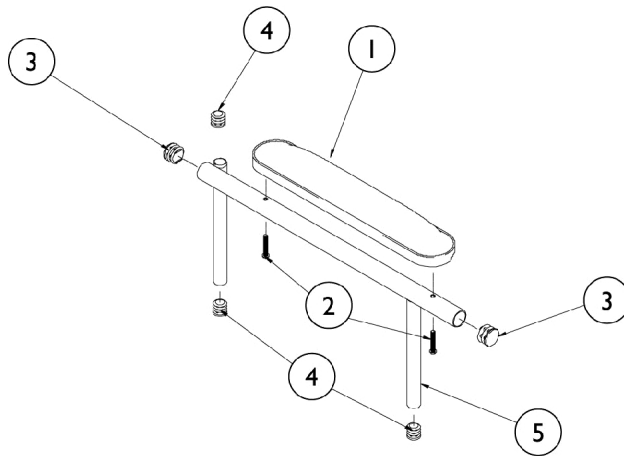

Space Saver Adjustable Height Arm


Desk/Full Length - "79"/"89"

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	2
3		1027095	Plug Button, Black (7/8")	1	2
4		1035614	Plug Button, Black (5/8")	1	3
5		1043317-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Right	1	1
5		1043318-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Chrome - Left	1	1
5		1043319-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right	1	1
5		1043320-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Left	1	1
5		1043313-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Right	1	1
5		1043314-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Chrome - Left	1	1
5		1043315-NLA	NLA - Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right	1	1
5		1043316-NLA	NLA - Arm Tube w/ Plug Buttons, Desk Length, Black - Left	1	1
6		1036150	Anti-Rattle	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		47471X074	Spring, Arm Lock	1	1
9		47471X068	Pin, Locking	1	1
10		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
10		7930315-NLA	NLA - Package, Screw, Phillips Pan Head (#6-20 x 3/8")	25	1
11		1027097	Plug Button, Black (3/4")	1	1
12		1015450	Handle, Quad Release	1	2
13		1018225	Spring, Torsion with Button Lock	1	2
14		1036484	Plug, Armrest, Black (3/4") (18GA)	1	2
15		1028645	Rivet	1	6
16		1068511	Guard, Clothing	1	1
17	B	1020027-NLA	NLA - Arm Base Assembly, Chrome - Right	1	1
17	B	1020028-NLA	NLA - Arm Base Assembly, Chrome - Right	1	1
17	B	1037634	Arm Base Assembly, Space Saver Low Adjustable Height, Black - Right	1	1
17	B	1037635	Arm Base Assembly, Space Saver Low Adjustable Height, Black - Left	1	1
17	A,B	8881037636-NLA	NLA - Arm Base Assembly, Color - Right	1	1
17	A,B	8881037637-NLA	NLA - Arm Base Assembly, Color - Left	1	1
	C,D	1036503-NLA	NLA - Hardware, Attaching, Armrest	1	1

Space Saver Adjustable Height Arm Desk/Full Length - "79"/"89"

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors"
B - Includes items 6-16
C - Not Shown
D - See page titled "Space Saver Fixed Height Arm - Desk Length - 88" item number 19 "Hardware, Attaching Armrest"


Forks and Hardware

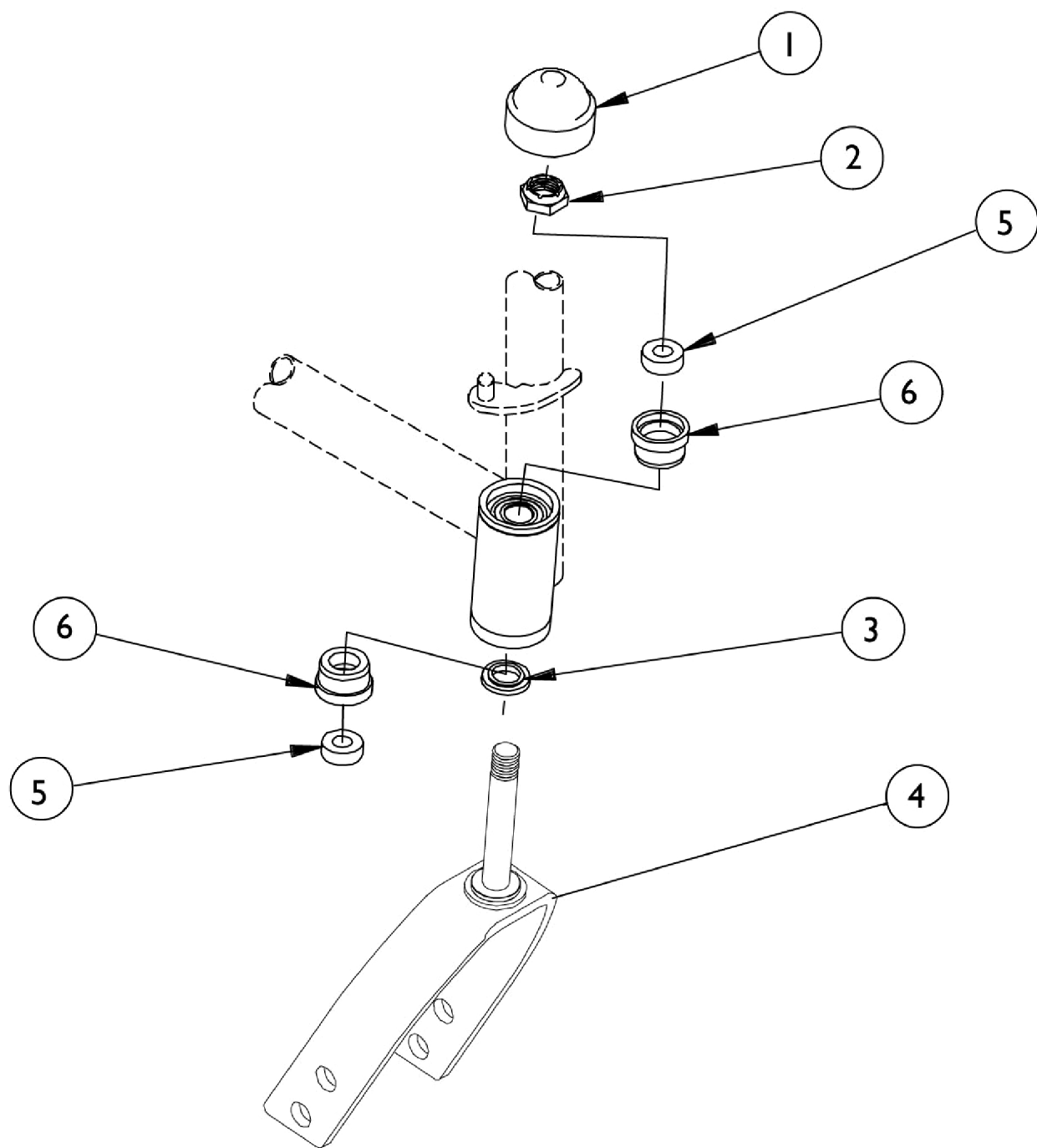


Forks and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1001843	Cap, Head Tube, Black	1	1
1		1131189	Package, Head Tube Cap	10	1
2		20225X000	Locknut (7/16-20)	1	1
3		1016300-NLA	NLA - Washer, Fork (3/4 x 7/16 x 1/8")	1	1
4	A,B	1060992-NLA	NLA - Fork and Stem (8" Caster)	2	1
4	C	1023237-NLA	NLA - Fork and Stem (8" Caster)	1	1
4		1019972	Fork, 6" Caster (7/16" Stem)	1	1
5		1006049	Bearing (7/16")	1	2
6		57768X030-NLA	NLA - Sleeve, Aluminum	1	2
NOTE: A - Kit contains 2 forks plus instruction sheet. As of 12/95 forks changed to 2 axle mounting holes. Those chairs built before 12/95 will need to replace both forks. B - Before 12/95 C - After 11/95					

Front Casters and Mounting Hardware

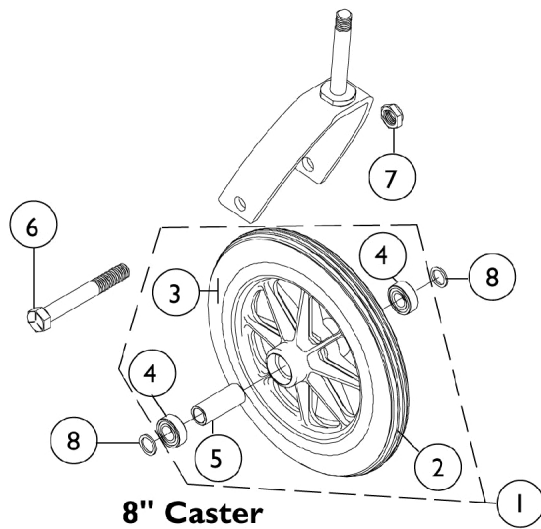


*fork shown
for reference only*

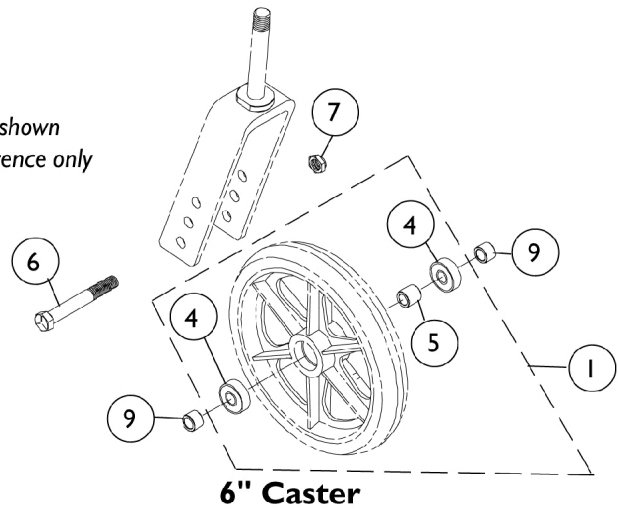

Front Casters and Mounting Hardware

Wheel Style	Composite Urethane (6 x 1)	Composite Solid Rubber (8 x 1")	Composite Pneumatic (8 x 1-1/4")	Composite Pneumatic with Flat Free Insert (8 x 1-1/4")
Single Wheel Assembly	1026956 See Note(s) A	1058137 See Note(s) A	1009481 See Note(s) A	1030773 See Note(s) A
Tire	--	1040144	40176X040	40176X040
Tube/Insert	--	--	40176X041	1010904-NLA
Bearing	40117X026	1006049	1006049	1006049
Hub Spacer	1026172	1018982	1020592	1020592
Front Axle	7930362	57667X007-NLA	57667X007-NLA	57667X007-NLA
Axle Locknut	7930363-NLA	7930352-NLA	7930352-NLA	7930352-NLA
Axle Washer	NLA See Note(s) B	1026101	7930312	7930312
Axle Spacer	1053202 See Note(s) B	--	--	--
NOTE: A - Includes tire, tube/insert, bearing and spacer. B - As of 1/95, axle spacer (1053202) replaced both washer (16308X000) and spacer (47183X047) on 6" casters.				

Rear Wheel Hardware

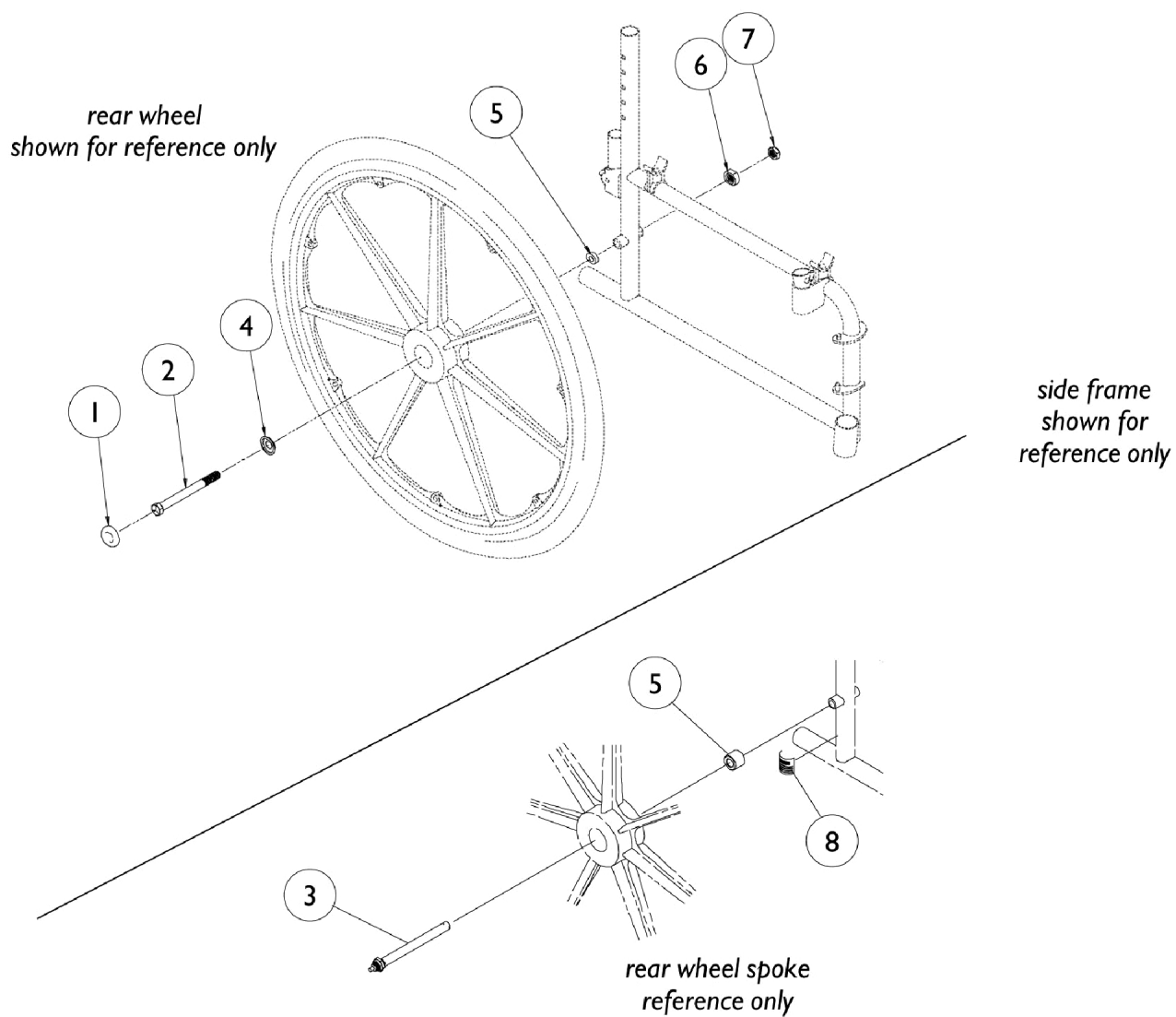

Rear Wheel Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1016291	Cap, Hub, Black	1	1
1	C	1027600-NLA	NLA - Cap, Black	1	1
2		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
3	E	56000X651	Axle, Quick Release (7/16" x 5-7/16")	1	1
4		1016289	Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
5		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
6	A	1014652-NLA	NLA - Bushing, Threaded	1	1
	D	1048319-NLA	NLA - Locknut Retrofit Kit	1	1
7		1045608	Locknut (7/16-20)	1	1
8		1001948-NLA	NLA - Label, Warning	1	1
NOTE: A - Bushing is NLA as of 7/94. Order (1 ea.) Locknut Retro-fit Kit part number 1048319. B - Pneumatic and Urethane Wheels C - DTM Wheels D - Not Shown, replaced item 6. E - For chairs with Conventional Arms					

Rear Wheels

Wheel Style	Single Wheel Assembly	Tire	Tube/Insert	Bearing	Bearing Spacer	Plugs
Composite Pneumatic (20 x 1-3/8") 1763BK	1034126 See Note(s) B, A	1017519	1017520	40117X024	1015756	N/A
Composite Pneumatic w/ Flat Free Insert (20 x 1-3/8") 4763BK	1034127 See Note(s) A, B	1017519	1023322	40117X024	1015756	1030936 See Note(s) C
Composite w/ Urethane (20 x 1-1/4") U1763BK	1038555 See Note(s) A, B	1037774	N/A	40117X024	1015756	N/A
Composite Pneumatic (22 x 1-3/8") 1764BK	1027149 See Note(s) B, A	1018544-NLA	1018545	40117X024	1015756	N/A
Composite Pneumatic w/ Flat Free Insert (22 x 1-3/8") 4764BK	1027150 See Note(s) A, B	1018544-NLA	1023323-NLA	40117X024	1015756	1030936 See Note(s) C
Composite w/ Urethane (22 x 1-1/4") U1764BK	1032115 See Note(s) B, A	1028250	N/A	40117X024	1015756	N/A
Composite DTM (24 x 7/8")	1080378 See Note(s) D, A	1080400 See Note(s) D	N/A	40117X024	1019695-NLA	N/A
Composite Corded Urethane (24 x 1") U2222C	1122726 See Note(s) A	1118209 See Note(s) E	N/A	40117X024	1019695-NLA	N/A
Composite Pneumatic (24 x 1-3/8") 2244BK	1025123 See Note(s) A	40176X060	40176X061-NL A	40117X024	1015756	N/A
Composite Pneumatic w/ Flat Free Insert (24 x 1-3/8") 4244BK See Note(s) C	1025124 See Note(s) A	40176X060	1010906	40117X024	1015756	1030936 See Note(s) C

Rear Wheels

Wheel Style	Single Wheel Assembly	Tire	Tube/Insert	Bearing	Bearing Spacer	Plugs
Composite Urethane (24 x 1-1/4") U2244BK	1032117 See Note(s) A	1026904	N/A	40117X024	1015756	N/A
NOTE: A - Includes tire, tube/insert, bearing, and spacer; Does not include handrim. B - When ordering 20 or 22" wheels, a wheel lock shoe spacer may also be required. See Wheel Locks. C - Wheels with Flat Free Inserts have a plug which caps an opening in the rim. Part number for this plug is 1030936. D - Non-Corded Tire replaced with a Corded Tire as of 11/16/09. Refer to part #1122726 for a wheel assembly or part #1118209 for a tire. It is recommended to order (2) each in quantity to match. E - Corded Tires maybe more difficult to stretch over the rim. The complete wheel assembly that has the tire and wheel rim preassembled maybe advisable.						

Handrims and Attaching Hardware

Chrome Plated - Non-Projection

Handrim Only	Quantity	Att. Points	Att. Hardware Kit	Wheel Type
1019999	1	7	1025153 See Note(s) A	A See Note(s) B
1020000	1	7	1025153 See Note(s) A	B See Note(s) B
1019836	1	7	1025153 See Note(s) A	C See Note(s) B
1027413-NLA	1	6	1025150 See Note(s) A	D See Note(s) B
NOTE: A - Kit contains enough hardware for 1 handrim. B - See Chart titled "Wheel Types".				

Flat Black - Non-Projection

Handrim Only	Quantity	Att. Points	Att. Hardware Kit	Wheel Type
1020017 See Note(s) C	1	7	1025153 See Note(s) A	A See Note(s) B
1020018 See Note(s) C	1	7	1025153 See Note(s) A	B See Note(s) B
1015406 See Note(s) C	1	7	1025153 See Note(s) A	A See Note(s) B
NOTE: A - Kit contains enough hardware for 1 handrim. B - See chart titled "Wheel Types" C - When replacing handrims for chairs built before 12/17/99 order in pairs, handrim color has changed from Pewter to Flat Black.				

Black Plastic Coated - Non-Projection

Handrim Only	Quantity	Att. Points	Att. Handrim Kit	Wheel Type
1115400 See Note(s) C	1	7	1025158 See Note(s) A	A See Note(s) B
1115401 See Note(s) C	1	7	1025158 See Note(s) A	B See Note(s) B
1115397 See Note(s) C	1	7	1025158 See Note(s) A	C See Note(s) B
NOTE: A - Kit contains enough hardware for 1 handrim. B - See Chart titled "Wheel Types". C - Grey Plastic Coated Handrims are no longer available and have been replaced with black. If only one grey handrim is requested order the black handrims in pairs.				

Handrims and Attaching Hardware

Black Plastic Coated - Projection

	Handrim Only	Quantity	Att. Points	Att. Hardware Kit	Wheel Type
8 Oblique	1115396 See Note(s) C	1	7	1062765 See Note(s) A	C See Note(s) B
8 Vertical	1115394 See Note(s) C	1	7	1062764 See Note(s) A	C See Note(s) B

NOTE: A - Kit contains enough hardware for 1 handrim.
 B - See Chart titled "Wheel Types".
 C - Grey Plastic Coated Handrims are no longer available and have been replaced with black. If only one grey handrim is requested order the black handrims in pairs.

Wheel Types

A = 20"	Composite w/Urethane or Pneumatic Tires
B = 22"	Composite w/Urethane or Pneumatic Tires
C = 24"	Composite w/Urethane or Pneumatic Tires
D = 24"	Composite w/DTM Tires

Handrim Attaching Hardware

Non-Projection Handrim Hardware

Part Number	Part Description	Hardware Kit	Hardware Kit	Hardware Kit
		1025150	1025158	1025153
		See Note(s) A	See Note(s) A	See Note(s) A
1007375 See Note(s) B	Screw, Phillips Pan Head w/ Patch (#10-32 x 1-3/4")	6	--	--
7930305-NLA	NLA - Screw, Pan Head (#10-32 x 1-3/4")	--	7	7
1031348-NLA	NLA - Screw w/ Washer (#10-32)	--	--	--
1016438	Spacer (1/4 x 11/32 x 15/16")	--	--	7

NOTE: A - Hardware Kit
B - Replaces 1026750.

The quantity in each column is the amount of that item that is in the hardware kit.

Projection Handrim Hardware


Part Number	Part Description	Hardware Kit	Hardware Kit
		1062765	1062764
		See Note(s) A	See Note(s) A
11008X003-NLA	Screw, Phillip Pan Head (#10-32 x 1-1/4")	7	--
7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	7	7
7930305-NLA	Screw, Pan Head (#10-32 x 1-3/4")	--	7
47559X008	Link, Attaching, Chrome	7	7
7930365-NLA	NLA - Locknut (#10-32)	7	7
1013218	Washer (#10 Flat)	7	7
1062516-NLA See Note(s) B	Instruction Sheet, Handrim Hardware	1	1

NOTE: A - Hardware Kit
B - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.

The quantity in each column is the amount of that item that is in the hardware kit.

Wheel Lock


without Attaching Hardware

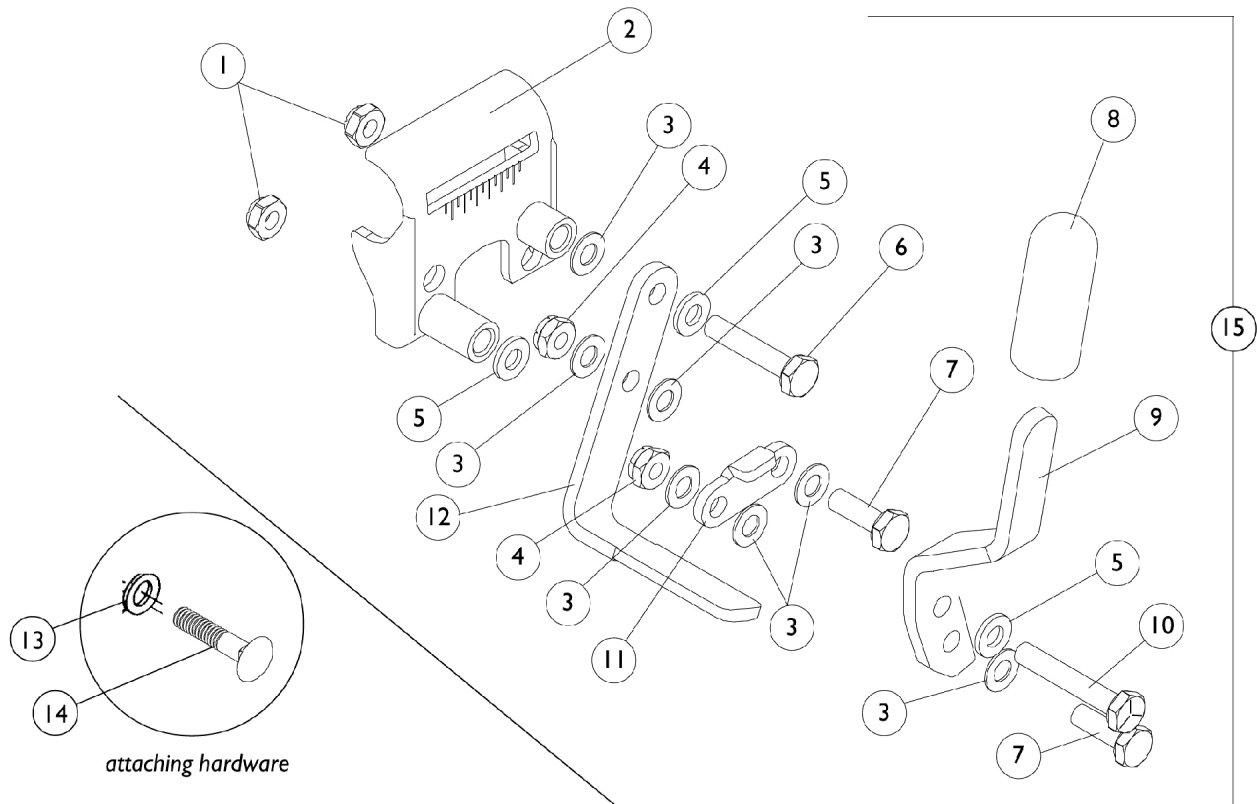

Wheel Lock without Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
2		1016037-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
2		1016038-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
3		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	7
4		7930367-NLA	NLA - Locknut (1/4-28)	25	2
5		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	3
6		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
7		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
8		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
9		1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
9		1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
10		7930350-NLA	NLA - Package, Hex Head Screw (1/4-20 x 1-1/2")	25	1
11	B	1016592-NLA	NLA - Link, Chrome	1	1
12		1015443-NLA	NLA - Shoe	1	1
13		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
14		60264X010-NLA	NLA - Screw, Hex Head (1/4-20 x 3/4")	1	1
15	A	1020001-NLA	NLA - Wheel Lock Assembly, Chrome - Right	1	1
15	A	1020002-NLA	NLA - Wheel Lock Assembly, Chrome - Left	1	1
NOTE: A - Includes items 1-12 B - All Chairs Except Super Hemi					

Hill Holder Wheel Lock without Attaching Hardware

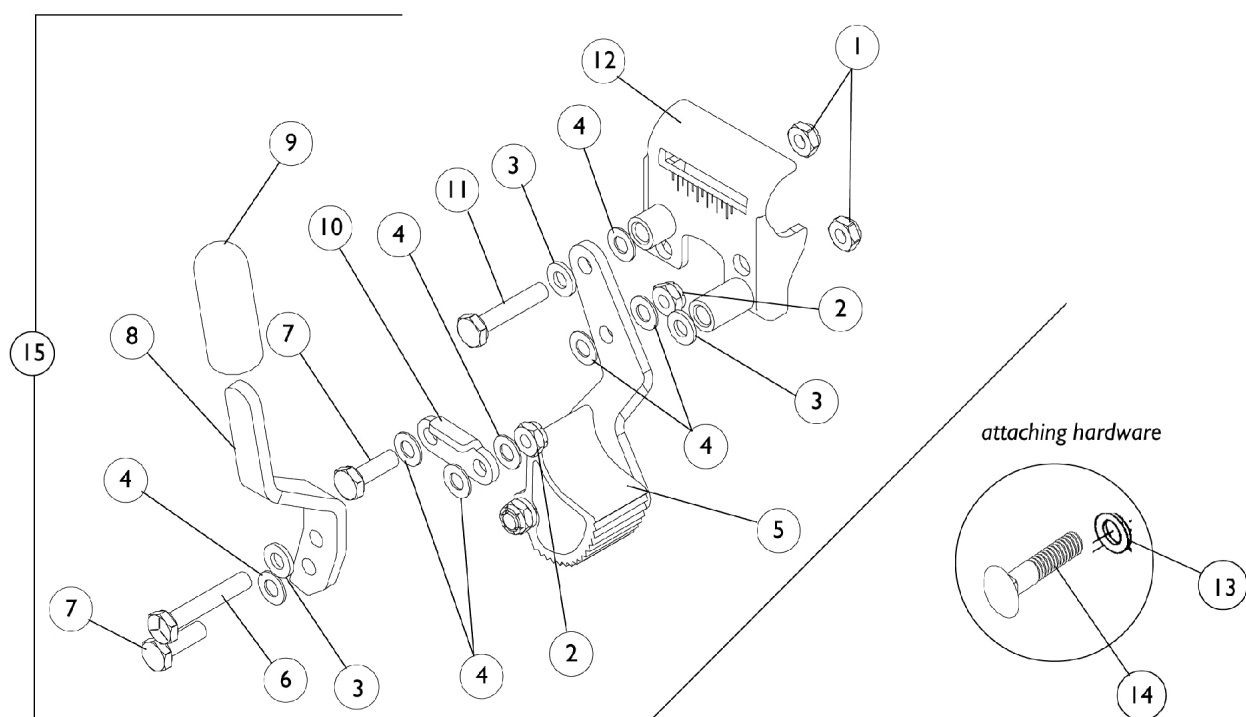

Hill Holder Wheel Lock without Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
2		7930367-NLA	NLA - Locknut (1/4-28)	25	2
3		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	7
4		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	3
5	C	1016892-NLA	NLA - Hill Holder Assembly - Right	1	1
5	C	1016893-NLA	NLA - Hill Holder Assembly - Left	1	1
6		7930350-NLA	NLA - Package, Hex Head Screw (1/4-20 x 1-1/2")	25	1
7		60274X012-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	2
8		1016588-NLA	NLA - Handle, Push, Adult, Chrome - Right	1	1
8		1016589-NLA	NLA - Handle, Push, Adult, Chrome - Left	1	1
9		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
10		1017861-NLA	NLA - Link, Chrome	1	1
11		1007017-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
12		1016037-NLA	NLA - Bracket, Mounting, Chrome - Right	1	1
12		1016038-NLA	NLA - Bracket, Mounting, Chrome - Left	1	1
13		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
14		60264X010-NLA	NLA - Screw, Hex Head (1/4-20 x 3/4")	1	1
15	A,B	1034210	Wheel Lock Assembly, Hill Holder, Push to Lock - Right	1	1
15	A,B	1034211	Wheel Lock Assembly, Hill Holder, Push to Lock - Left	1	1
NOTE: A - Hill Holder Wheel Locks not for use with Urethane Tires B - Includes Items 1-12 C - Part #1016892 and 1016893 are no longer available as a service part individually. Order item #15 as a replacement.					

Front Riggings Index and Calf Pads

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Calf Pads

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	B,H	88804530X003	Pad, Legrest, Vinyl - Right	1	1
	B,H	88804530X004	Pad, Legrest, Vinyl - Left	1	1
	A,C,H	88804529X003	Pad, Legrest, Vinyl - Right	1	1
	A,C,H	88804529X004	Pad, Legrest, Vinyl - Left	1	1
	A,D,E, H	1091767-NLA	NLA - Pad with Screws, Legrest (Pair)	2	2
	A,D,F	8881091759	Pad w/ Cover and Screws, Legrest Vinyl (Pair)	2	2
	A,D,F, H	8881091764	Pad, Legrest, Vinyl - Right/Left	1	2
	G	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
	A,D	01415X000-NLA	NLA - Cover, Legrest Pad	1	1
		See Chart	Front Riggings Index	1	1

NOTE: A - Specify upholstery color, see page titled "Frames Finishes and Upholstery Colors"
 B - 12" Calf Pads
 C - 14-15" Calf Pads
 D - 16-22"
 E - Before 9/5/00
 F - After 9/4/00
 G - Includes Screws and Instruction Sheet.
 H - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Front Riggings Index

Front Rigging	Chair Width	Support Assembly Right	Support Assembly Left	Pivot Slide Tube w/Aluminum Footplate	Pivot Slide Tube w/Aluminum Footplate
				See Note(s) A	See Note(s) A
H93E	10-12"	1040715-NLA	1040716-NLA	1007274-NLA	1007275-NLA
	14"	1040715-NLA	1040716-NLA	1033506-NLA	1033507-NLA
H93M	10-12"	1035172-NLA	1035171-NLA	1007274-NLA	1007275-NLA
	14"	1035172-NLA	1035171-NLA	1033506-NLA	1033507-NLA
H903MC	12"	1031194-NLA	1031195-NLA	1007274-NLA	1007275-NLA
	14"	1031194-NLA	1031195-NLA	1033506-NLA	1033507-NLA
H903MK	12"	8881031198-NLA	8881031199-NLA	1007274-NLA	1007275-NLA
	14"	8881031198-NLA	8881031199-NLA	1033506-NLA	1033507-NLA
H990C	10"	1020011-NLA See Note(s) B	1020012-NLA See Note(s) B	N/A	N/A
	12-14"	1020015-NLA See Note(s) B	1020016-NLA See Note(s) B	N/A	N/A


Front Riggings Index and Calf Pads

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Front Riggings Index

Front Rigging	Chair Width	Support Assembly Right	Support Assembly Left	Pivot Slide Tube w/Aluminum Footplate See Note(s) A	Pivot Slide Tube w/Aluminum Footplate See Note(s) A
H990K	10"	1020009-NLA See Note(s) B	1020010-NLA See Note(s) B	N/A	N/A
	12-14"	1020013-NLA See Note(s) B	1020014-NLA See Note(s) B	N/A	N/A
H904M	12"	8881031465-NLA	8881031466-NLA	1031471-NLA	1031472-NLA
	14"	1031467-NLA	1031468-NLA	1020234-NLA	1020235-NLA
NOTE: A - Aluminum footplates on chairs built before 2/96 had a "Ball Burnish" finish. Footplates after 1/96 now have a black finish. When replacing only 1 pivot slide tube w/footplate before 2/96, it is suggested that a second new (black) footplate be ordered. B - Pivot slide tube w/footplate is included with Support Assembly. Orders for Front Riggings in pairs must be placed through the Configurator. Please contact our Customer Service Coordinator.					


Footrest Support Assembly H93E

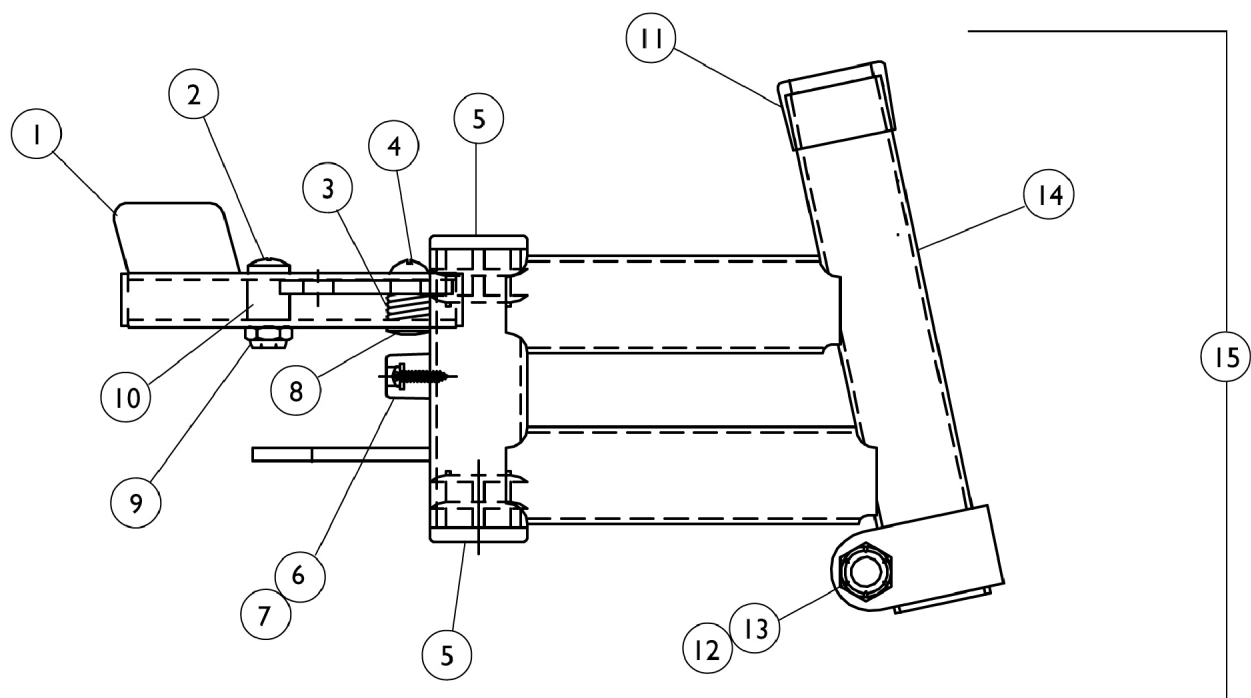

Footrest Support Assembly H93E

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1035539	Handle, Release, Black - Right	1	1
1		1035540	Handle, Release, Black - Left	1	1
2		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
3		1008935	Spring, Release - Right	1	1
3		1008936	Spring, Release - Left	1	1
4		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
5		1027095	Plug Button, Black (7/8")	1	2
6		1022874	Bumper	1	1
7		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
8		67005X000	Rivnut (#10-32)	1	1
9		7930365-NLA	NLA - Locknut (#10-32)	25	1
10		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
11		15680X012	Collar, Anti-Rattle (7/8")	1	1
12		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
13		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
14	B	1007123-NLA	NLA - Tube, Support, Footrest, Chrome - Right	1	1
14		1007124-NLA	NLA - Tube, Support, Footrest, Chrome - Left	1	1
	A	See Page	See Footnote	1	1
NOTE: A - See page titled "Front Riggings Index" for Support Assembly B - Part #1007123 is no longer available individually. See Catagory titled "Front Riggings Index and Calf Pads" and order part #1040715 Support Assembly					
Due to design change, for alll chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.					

Footrest Support Assembly H93M

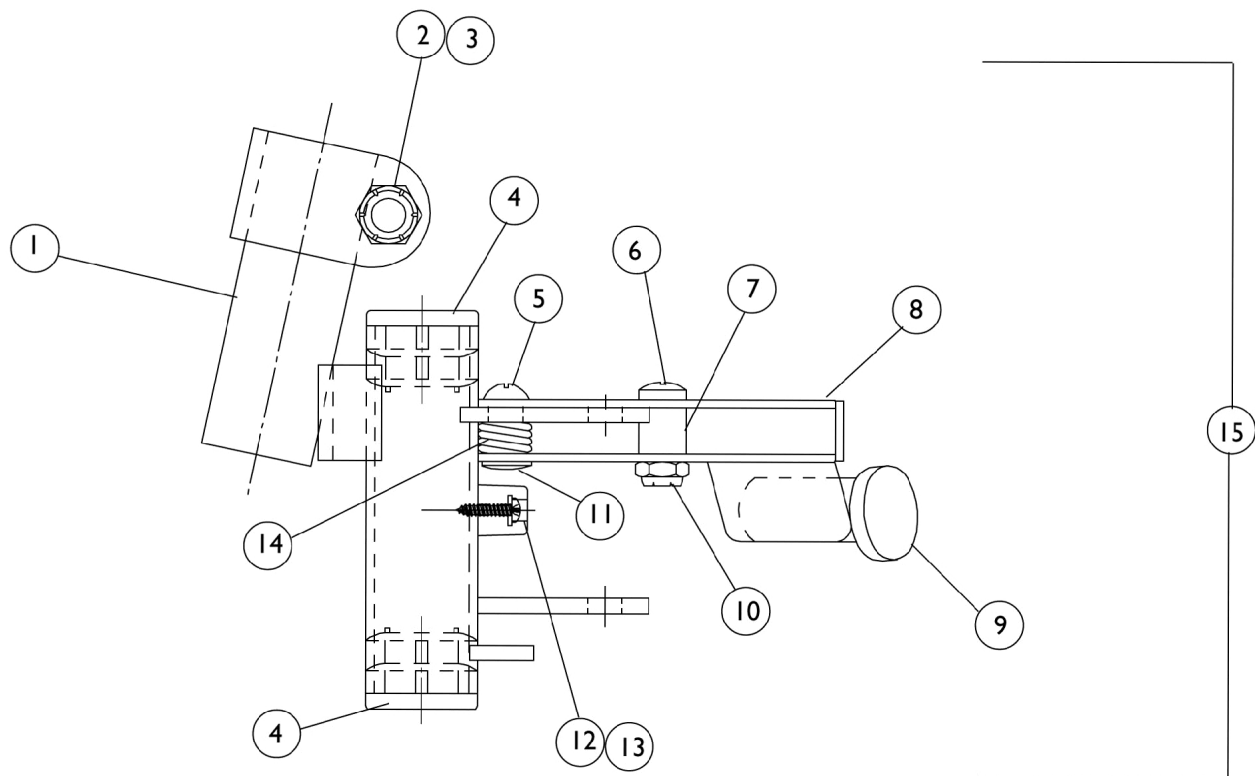


Footrest Support Assembly H93M

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1035141-NLA	NLA - Footrest Support, Chrome - Right	1	1
1		1035140-NLA	NLA - Footrest Support, Chrome - Left	1	1
2		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
3		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
6		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
7		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
8		1035539	Handle, Release, Black - Right	1	1
8		1035540	Handle, Release, Black - Left	1	1
9	A	1006490-NLA	NLA - Tip, Plastic, Black	1	1
10		7930365-NLA	NLA - Locknut (#10-32)	25	1
11		67005X000	Rivnut (#10-32)	1	1
12		1022874	Bumper	1	1
13		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
14		1008935	Spring, Release - Right	1	1
14		1008936	Spring, Release - Left	1	1
	B	See Page	See Footnote	1	1
NOTE: A - Before 10/1/00 B - See page titled "Front Riggings Index" for Support Assembly Due to design changes, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.					

Footrest Support Assembly

H903MC/H903MK

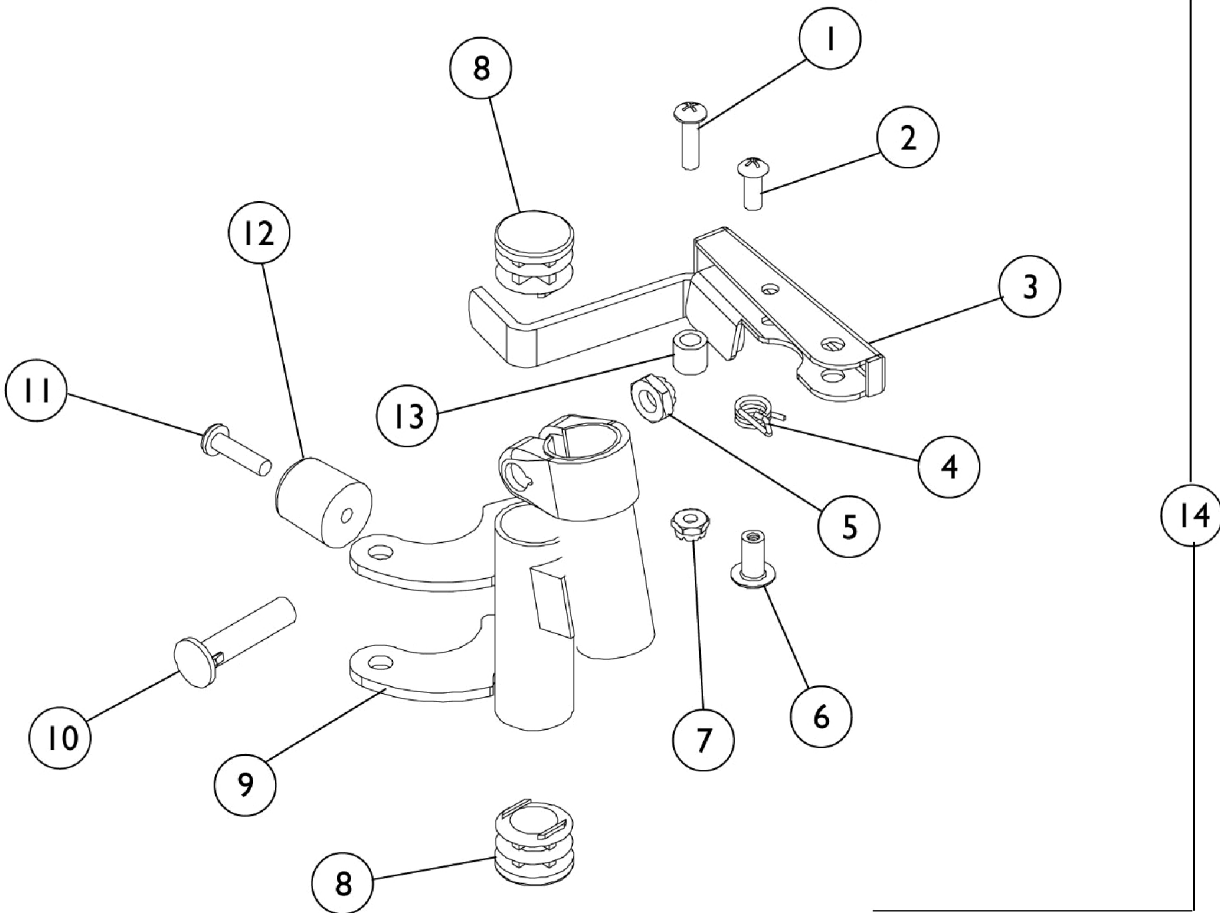

Footrest Support Assembly H903MC/H903MK


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
2		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
3		1035539	Handle, Release, Black - Right	1	1
3		1035540	Handle, Release, Black - Left	1	1
4		1008935	Spring, Release - Right	1	1
4		1008936	Spring, Release - Left	1	1
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		67005X000	Rivnut (#10-32)	1	1
7		7930365-NLA	NLA - Locknut (#10-32)	25	1
8		1027095	Plug Button, Black (7/8")	1	2
9	A	1007113-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Right	1	1
9	A	1007114-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Left	1	1
9	B,D	1021929-NLA	NLA - Tube, Footrest Support, Hemi, Black - Right	1	1
9	B,D	1021930-NLA	NLA - Tube, Footrest Support, Hemi, Black - Left	1	1
10		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
11		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
12		1022874	Bumper	1	1
13		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
14	C	See Page	See Footnote	1	1

NOTE: A - Chrome
 B - Black
 C - See page titled "Front Riggings Index" for Support Assembly
 D - Part #1021929 and 1021930 are no longer available as service parts individually. See category titled "Front Rigging Index" and order part #8881031198 or 8881031199.

Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.

90 Degree Footrest with Aluminum Footplate

H990C/H990K

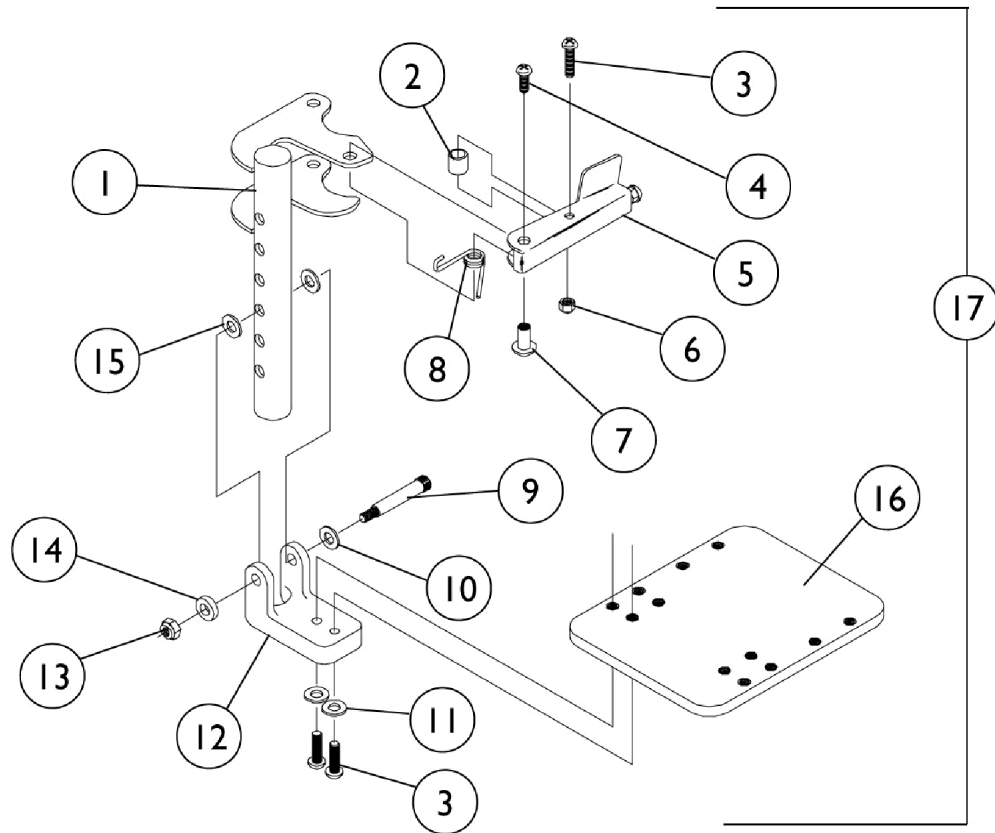


90 Degree Footrest with Aluminum Footplate H990C/H990K

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1019869-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Right	1	1
1		1019870-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Left	1	1
1		1019867-NLA	NLA - Tube, Footrest Support, Hemi, Black - Right	1	1
1		1019868-NLA	NLA - Tube, Footrest Support, Hemi, Black - Left	1	1
2		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
3		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	3
4		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
5		1031469-NLA	NLA - Handle, Release, Black - Right	1	1
5		1031470-NLA	NLA - Handle, Release, Black - Left	1	1
6		7930365-NLA	NLA - Locknut (#10-32)	25	1
7		67005X000	Rivnut (#10-32)	1	1
8		1008935	Spring, Release - Right	1	1
8		1008936	Spring, Release - Left	1	1
9		1000212	Screw, Shoulder Hex Socket (1/4 x 1-1/4", #10-24)	1	1
10		1010398	Washer, Nylon (1/4 x 1/2 x 1/16")	1	1
11		1028516	Washer, Steel (3/16 x 7/16 x 1/16")	1	2
12		1022817-NLA	NLA - Block, Hinge	1	1
13		10092X000-NLA	NLA - Locknut (1/4-2)	1	1
14		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
15		7930357-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	25	2
16	B	1044078-NLA	NLA - Footplate, Black (3-1/4" W x 5-1/4" L) (10" W)	1	1
16	B	1044082-NLA	NLA - Footplate, Black (4-1/4" W x 5-1/4" L) (12" W)	1	1
17	D	See Page	See Footnote	1	1
	A,C	1018525-NLA	NLA - Pad, Footplate (10" W)	1	1
	A,C	1018526	Pad, Footplate (12 - 14" W)	1	1

NOTE: A - Not Shown
 B - As of 4/94 this footplate now has a "wrinkle" finish versus the previous "flat black", which are NLA. When replacing 1 footplate on chairs built before 4/94, order 2 of the current footplates.
 C - Before 4/94 Only
 D - See page titled "Front Riggings Index" for Support Assembly with Footplate

Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.

Legrest Support Assembly H904M

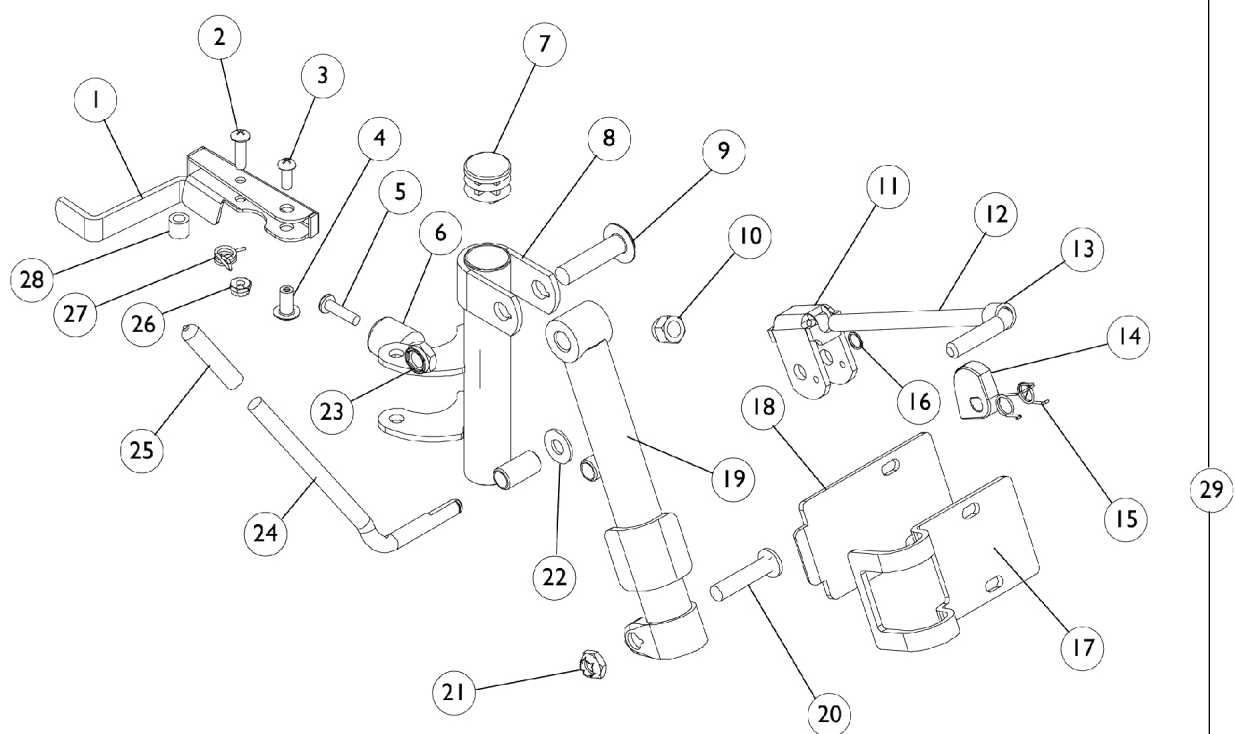

Legrest Support Assembly

H904M

H904M

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1023428-NLA	NLA - Handle, Release, Black- Right	1	1
1		1023431-NLA	NLA - Handle, Release, Black- Left	1	1
2		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
3		7930359-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
4		67005X000	Rivnut (#10-32)	1	1
5		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
6		1022874	Bumper	1	1
7		1027095	Plug Button, Black (7/8")	1	1
8		04707X069-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
8		04707X070-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
8		04707X067	Legrest Support Only, Black- Right	1	1
8		04707X068	Legrest Support Only, Black- Left	1	1
9		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
10		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
11	B	04528X002	Bracket, Release, Black	1	1
12		42326X001	Bar, Ratchet	1	1
13		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
14	B	1037125	Cam, Legrest	1	1
15	B	1037485	Spring, Release	1	1
16	C,D	1040831	Ring, Snap	1	1
16	C,E	40187X000-NLA	NLA - Pin, Roll (1/8" x 9/16")	1	1
17		47352X004	Clamp, Black	1	1
18		47352X003	Lock, Pad Plate, Legrest, Black	1	1
19		04526X006-NLA	NLA - Tube, Elevating Support, Chrome - Right	1	1
19		04526X007-NLA	NLA - Tube, Elevating Support, Chrome - Left	1	1
20		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
21		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
22		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
23		40363X000	Hex Nut (3/8"-16)	1	1
24	B	1037126	Handle, Release	1	1
25	D	1040060	Tip, Plastic, Black	1	1
25	E	1006490-NLA	NLA - Tip, Plastic, Black	1	1
26		7930365-NLA	NLA - Locknut (#10-32)	25	1
27		1008935	Spring, Release - Right	1	1
27		1008936	Spring, Release - Left	1	1
28		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
29	G	See Page	See Footnote	1	1
	A,B,F	1046328-NLA	NLA - Kit, Ratchet	2	1

Legrest Support Assembly


H904M

NOTE: A - Not shown
B - Due to product improvement - items 7, 9, 12, and 13 are NLA for chairs built before 6/94. Ratchet bar kit (item 5) must be ordered for those chairs.
C - After 5/94, the snap ring replaced the roll pin to attach the ratchet handle to the ratchet.
D - After 5/94
E - Before 6/94
F - Includes Items 10-14, 16, 22, 24, 25 & instruction sheet.
G - See page titled "Front Riggings Index" for Support Assembly

Due to design change, for all chairs built prior to 10/1/00 it is suggested that Release Handles be replaced in pairs.

Pivot Slide Tube with Footplate

H93E / H93M / H903MC / H903MK Footrests

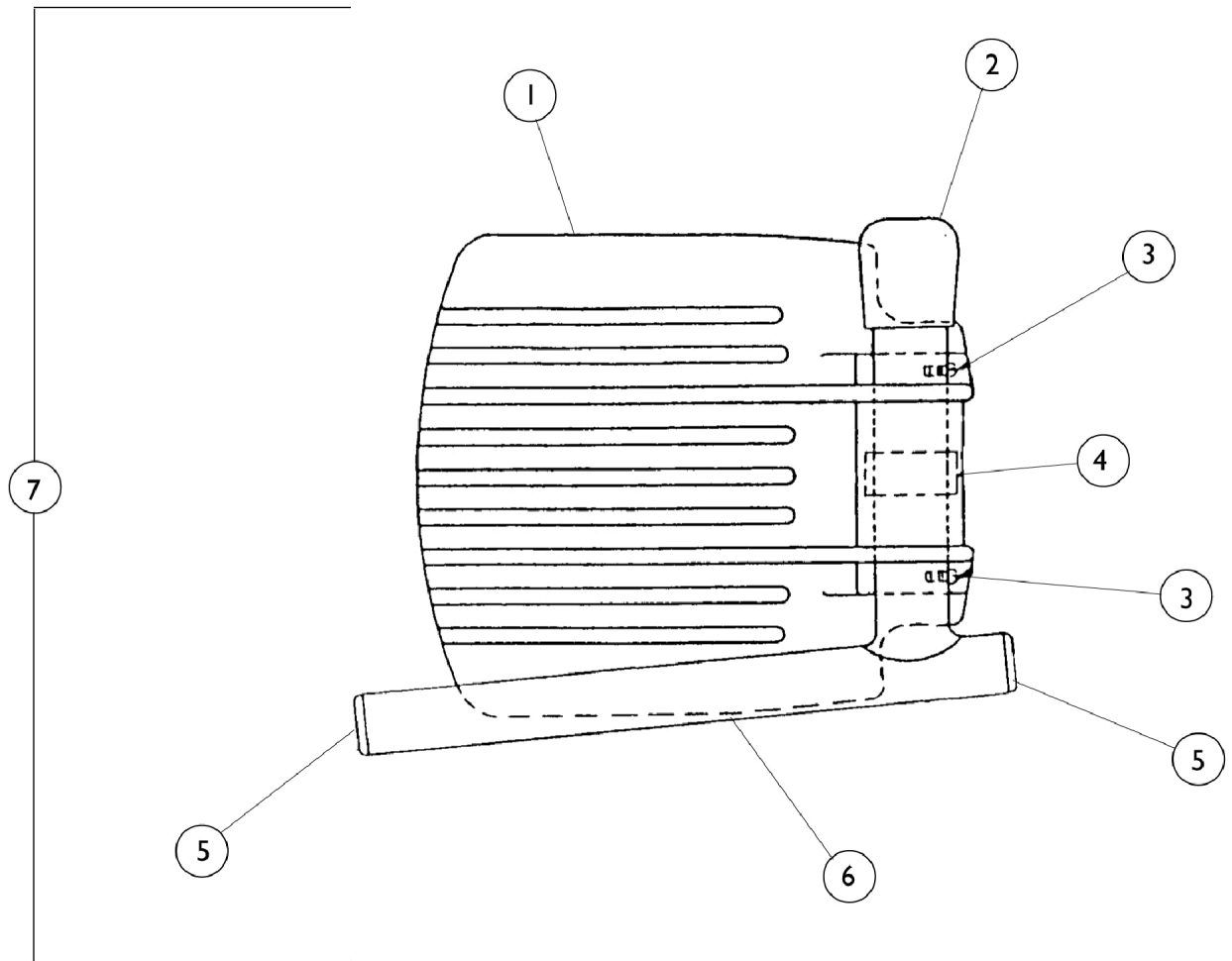


Pivot Slide Tube with Footplate

H93E / H93M / H903MC / H903MK Footrests

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	44200X032-NLA	NLA - Footplate, Aluminum, Extra Small, Black (5-3/4" W x 6" L)	1	1
1	A,C	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
2		1016346	Cap, End, Tube, Black (7/8")	1	1
3		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
4		1006981	Spring, Tension	1	1
5		1027097	Plug Button, Black (3/4")	1	2
6		1041528-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Chrome	1	1
7		See Page	See Footnote	1	1
NOTE: A - Aluminum footplates on chairs built before 2/96 had a "Ball Burnish" finish. Footplates after 1/96 now have a black finish. When replacing 1 ball burish footplate it is suggested that 2 new (black) footplates be ordered. B - 10-12" Wide Chairs C - 14" Wide Chairs					

Pivot Slide Tube with Footplate

H904M Legrests

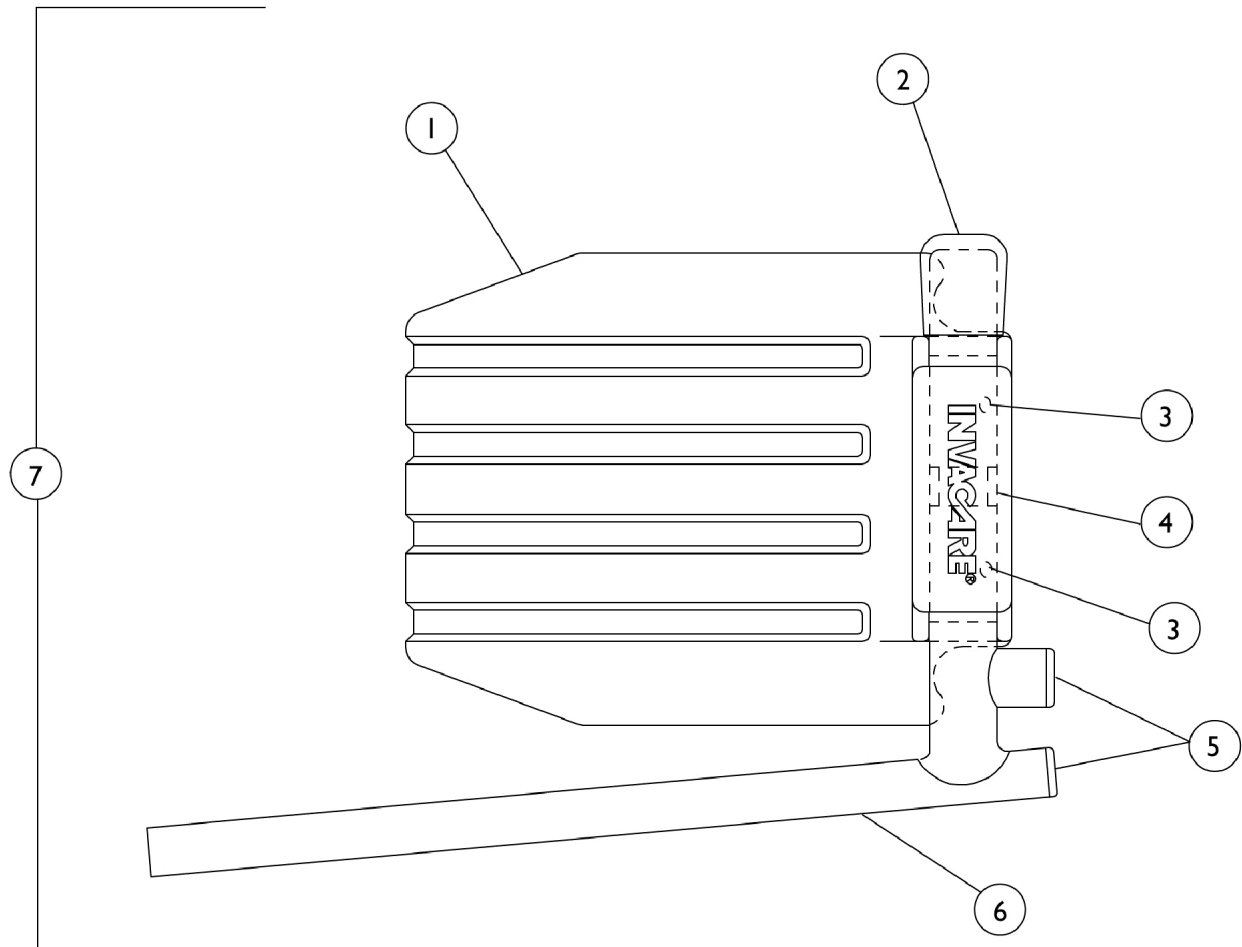


Pivot Slide Tube with Footplate H904M Legrests

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	44200X032-NLA	NLA - Footplate, Aluminum, Extra Small, Black (5-3/4" W x 6" L)	1	1
1	A,C	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
2		1016346	Cap, End, Tube, Black (7/8")	1	1
3		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
4		1006981	Spring, Tension	1	1
5		1027097	Plug Button, Black (3/4")	1	2
6		7500085-NLA	NLA - Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
7		See Page	See Footnote	1	1
NOTE: A - Aluminum footplates on chairs built before 2/96 had a "Ball Burnish finish. Footplates after 1/96 now have a black finish. When replacing 1 ball burnish footplate it is suggested that 2 new (black) footplates be ordered. B - 12" Wide Chairs C - 14" Wide Chairs					

Footrest and Legrest Accessories

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1036657-NLA	NLA - Calf Strap w/ Hook and Loop	1	--
	B	1035902	Strap, Calf w/ Hook and Loop (31" L)	1	--
	C,H	1036099-NLA	NLA - Heel Loop (Pair)	2	--
	D,H	1036100	Heel Loop (Pair)	2	--
	G	1035905	Heel Loop, Small (Pair)	2	--
		1035906	Heel Loop w/ Ankle Strap, Small (Pair)	2	--
	E	1036096	Hardware, Heel Loop (Pair)	2	--
	F	1035926	Hardware, Heel Loop, Aluminum Footplate (Pair)	2	--
		1143050	Non-Removable Device, Footrest/Legrest (1820)	1	1
			NLA - Non-Removable Device, Footrest-Legrest	1	1
			Carton	1	1
NOTE: A - 10-12" Wide Chairs B - 14" Wide Chairs C - 10" Wide Chairs D - 12-14" Wide Chairs E - H990C, H990K F - H903MC, H903MK, H93E, H904M G - H903MC and H93E (All) H - Footrests H990C, H990K					


Fabric Clothing Protectors

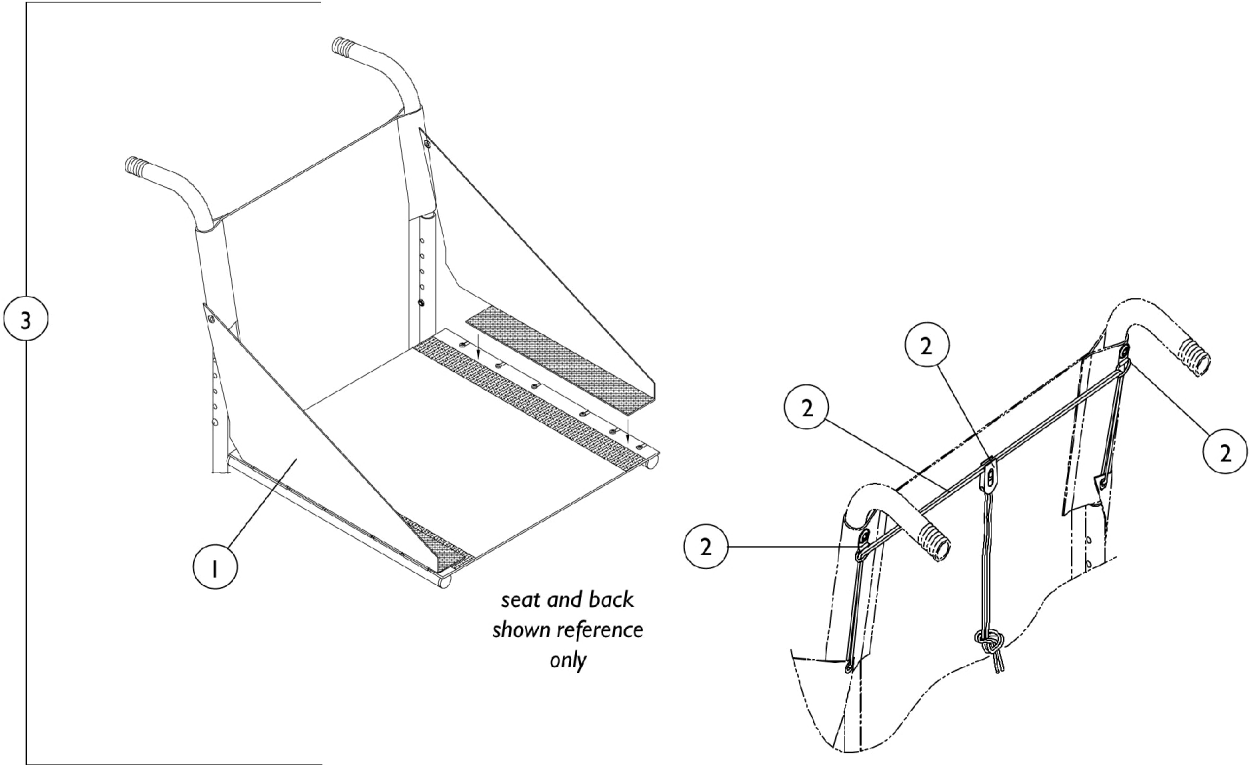

Fabric Clothing Protectors

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1036624-NLA	NLA - Protector, Fabric Clothing	1	2
1	C	1036625-NLA	Protector, Fabric Clothing	1	2
1	D	1036626-NLA	Protector, Fabric Clothing	1	2
2	A,E	1054771-NLA	NLA - Kit, Clothing Protector Hardware	1	1
3	B	1036621-NLA	NLA - Protector, Fabric Clothing with Hardware (Pair)	2	1
3	C	1036622-NLA	NLA - Protector, Fabric Clothing with Hardware (Pair)	2	1
3	D	1036623-NLA	NLA - Protector, Fabric Clothing with Hardware (Pair)	2	1
NOTE: A - Due to a design change, any clothing protector hardware needed for chairs built before 2/95 will need to order kit no. 1054771 so that the chair can be upgraded to the current design. Previous hardware is NLA. B - 10" Deep Chairs C - 12" Deep Chairs D - 14" Deep Chairs E - Includes Cord, Cord Lock, and 2 "D" Rings					


Utility Tray

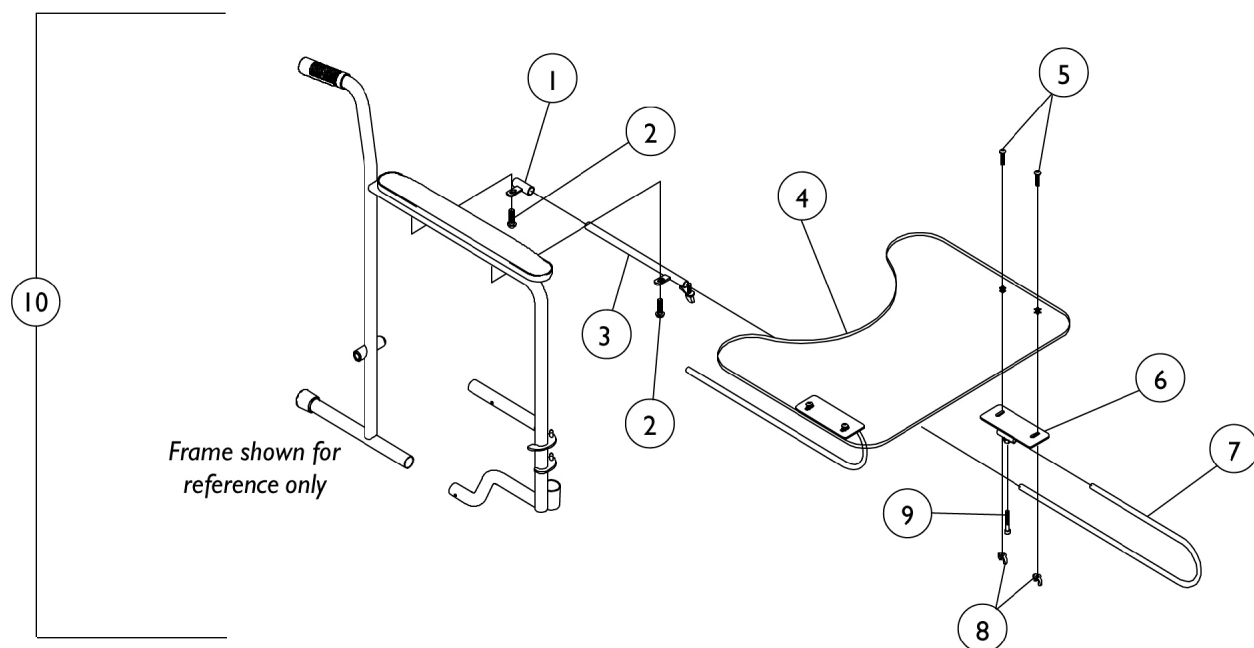

Utility Tray

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		04570X002-NLA	NLA - Tube, Support, Slide, Chrome	1	2
2		11008X011	Screw, Phillips Round Head (#10-32 x 1-3/8")	1	2
3		04570X009-NLA	NLA - Tube, Slide Lock, Full Length Arm - Right	1	1
3		04570X010-NLA	NLA - Tube, Slide Lock, Full Length Arm - Left	1	1
3		04570X011-NLA	NLA - Tube, Slide Lock, Desk Length Arm - Right	1	1
3		04570X012-NLA	NLA - Tube, Slide Lock, Desk Length Arm - Left	1	1
4		47469X006-NLA	NLA - Tray	1	1
5		10123X006	Bolt, Carriage (1/4-20 x 3/4")	1	4
6		1009341-NLA	NLA - Bracket, Tray	1	2
7		1010922-NLA	NLA - Tube, Support	1	2
8		20148X004-NLA	NLA - Nut, Wing (1/4-20)	1	4
9		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	2
10		1035965-NLA	NLA - Tray, Utility, Full Length Arm	1	1
10		1035966-NLA	NLA - Tray, Utility, Desk Length Arm	1	1
	A,B	1023704-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Not Shown B - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					


Stroller Handle

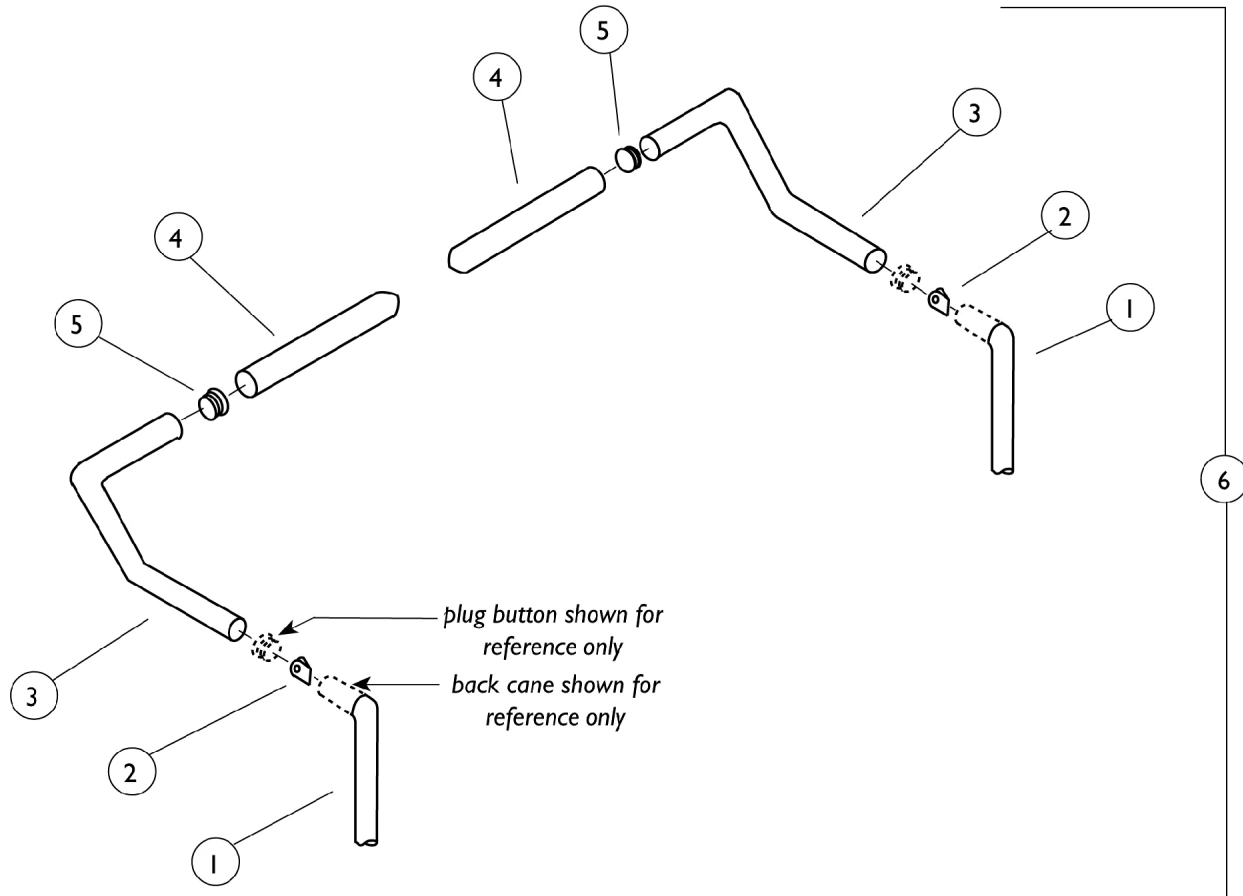

Stroller Handle

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1012832-NLA	NLA - Grip, Hand	1	2
2		40188X007-NLA	NLA - Button, Spring Double	1	2
3		1021864-NLA	NLA - Handle, Stroller - Right	1	1
3		1021865-NLA	NLA - Handle, Stroller - Left	1	1
4		1001404-NLA	NLA - Grip, Hand	1	2
5		1029501	Plug Button, Black (1")	1	2
6		1040786-NLA	NLA - Stroller Handle Assembly (Pair)	2	1

Solid Folding Seats

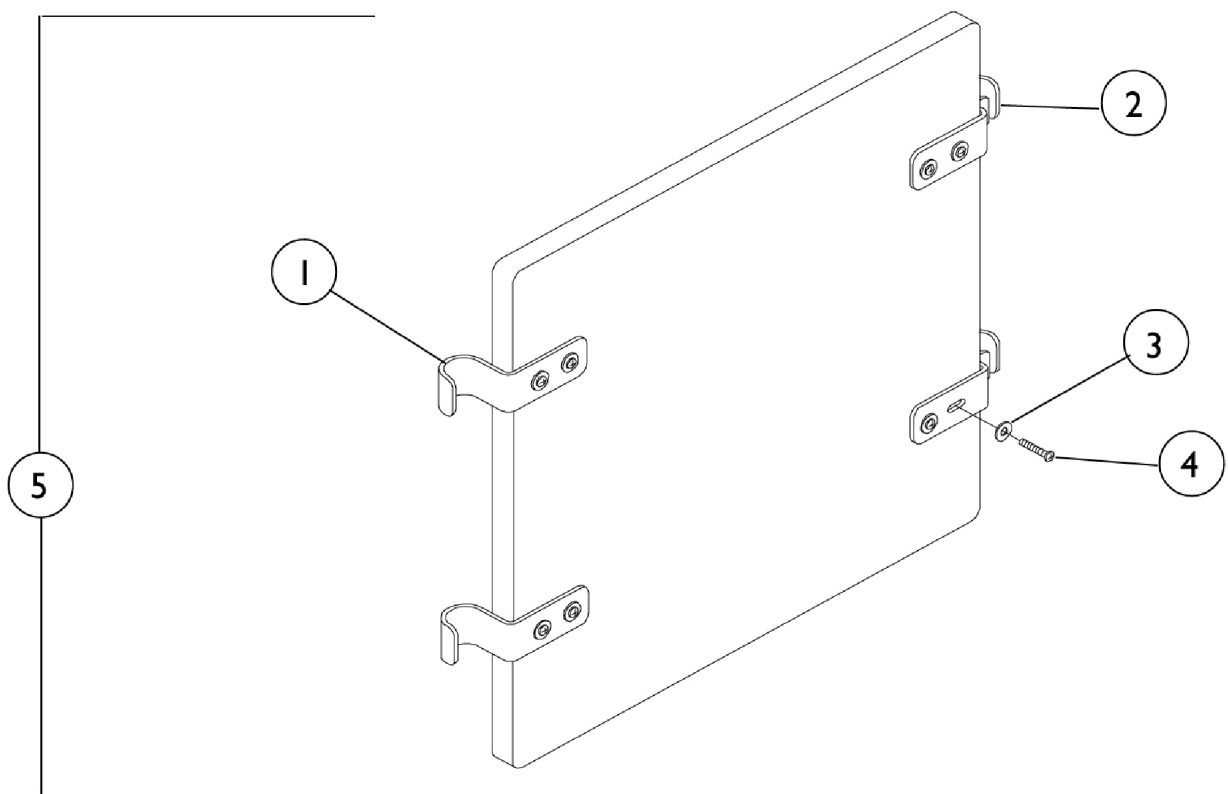

Solid Folding Seats

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1016771-NLA	NLA - Hook, Seat	1	2
2		1016770-NLA	NLA - Bracket, Seat, Chrome	1	2
3		1013218	Washer (#10 Flat)	1	8
4		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	8
5		See Chart	# 5 Solid Folding Seats	1	1

Solid Folding Seat with Hardware

Part Number	Item #5 Solid Folding Seat	Quantity
8881036531-NLA See Note(s) A	10" Wide x 10" Deep	1
8881036532-NLA See Note(s) A	10" Wide x 11" Deep	1
8881036533-NLA See Note(s) A	10" Wide x 12" Deep	1
8881036536-NLA See Note(s) A	12" Wide x 12" Deep	1
8881036537-NLA See Note(s) A	12" Wide x 13" Deep	1
8881036538-NLA See Note(s) A	12" Wide x 14" Deep	1
8881036541-NLA See Note(s) A	14" Wide x 12" Deep	1
8881036542-NLA See Note(s) A	14" Wide x 13" Deep	1
8881036543-NLA See Note(s) A	14" Wide x 14" Deep	1

NOTE: A - For upholstery color, see Upholstery Color Page.

Solid Back and Solid Seat Inserts

Solid Back Inserts


Part Number	Chair Width X Height
8881036572-NLA See Note(s) A	10" X 11"
8881036573-NLA See Note(s) A	10" X 12"
8881036574-NLA See Note(s) A	10" X 13"
8881036575 See Note(s) A	10" X 14"
8881036576-NLA See Note(s) A	10" X 15"
8881036577-NLA See Note(s) A	12" X 11"
8881036578-NLA See Note(s) A	12" X 12"
8881036579-NLA See Note(s) A	12" X 13"
8881036580 See Note(s) A	12" X 14"
8881036581-NLA See Note(s) A	12" X 15"
8881036582-NLA See Note(s) A	14" X 11"
8881036583-NLA See Note(s) A	14" X 12"
8881036584-NLA See Note(s) A	14" X 13"
8881036585 See Note(s) A	14" X 14"
8881036586-NLA See Note(s) A	14" X 15"
NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors".	

Solid Back and Solid Seat Inserts

Solid Seat Inserts

	10" Depth	11" Depth	12" Depth	13" Depth	14" Depth
10" Width	8881036544 See Note(s) A	8881036545-NLA See Note(s) A	8881036546 See Note(s) A	--	--
12" Width	--	--	8881036547 See Note(s) A	8881036548-NLA See Note(s) A	8881036549-NLA See Note(s) A
14" Width	--	--	8881036552 See Note(s) A	8881036551-NLA See Note(s) A	8881036550 See Note(s) A
NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors".					

Seat Extension Kits

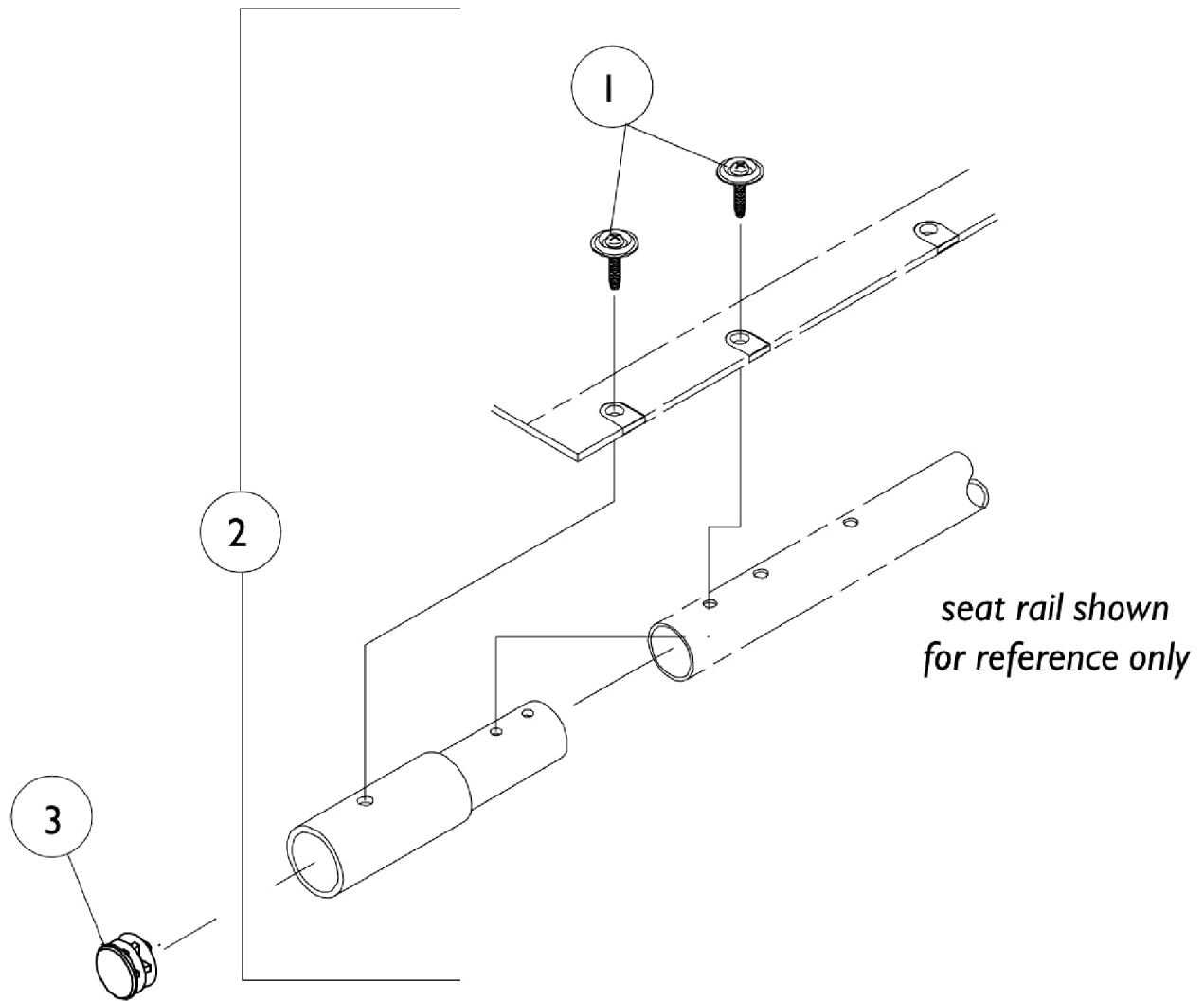


Seat Extension Kits

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
2	A	1019026-NLA	NLA - Kit, Seat Extension (1")	2	1
2	A	1019027	Kit, Extension, Seat (2")	2	1
3		1029501	Plug Button, Black (1")	1	2
NOTE: A - Seat extension kit does not include plug button (item 3).					

1" Forward Option

Available on Chairs with 20" and 22" Rear Wheels Only

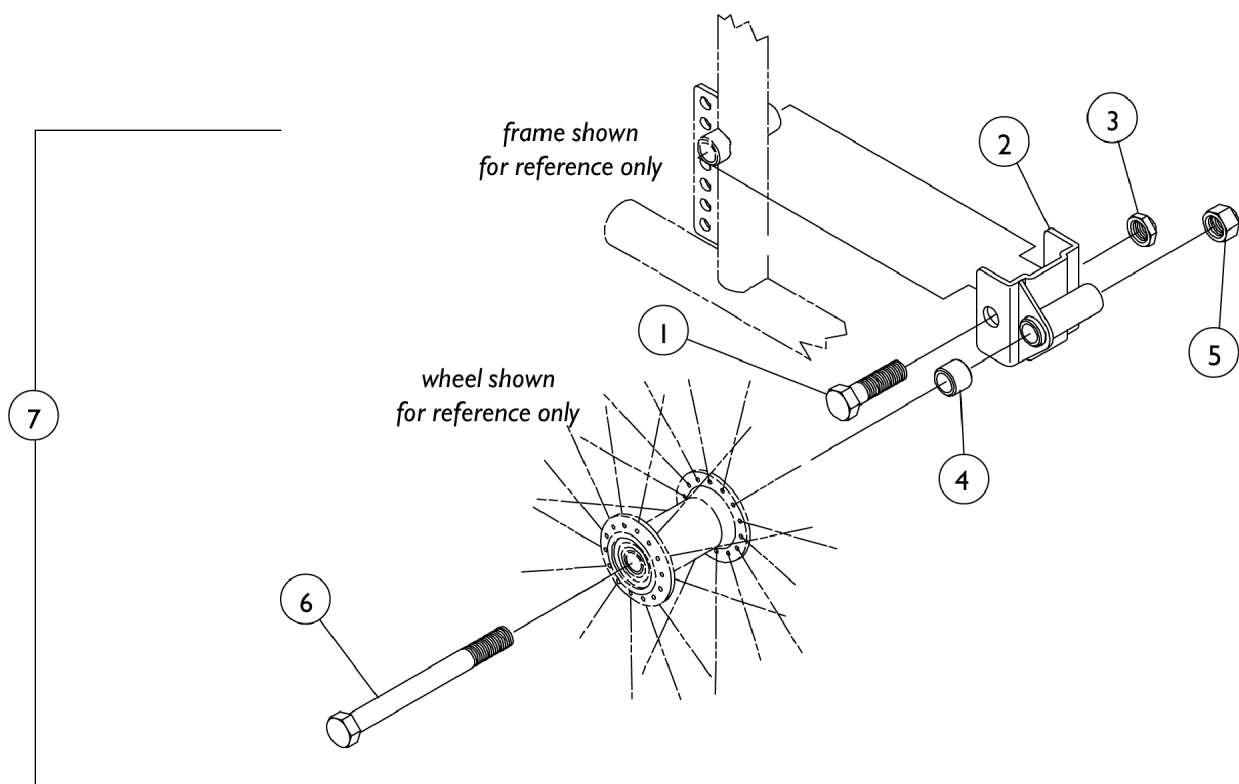


1" Forward Option

Available on Chairs with 20" and 22" Rear Wheels Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1021755-NLA	NLA - Screw, Hex Head (7/16-20 x 2-1/2")	1	1
2		1021691-NLA	NLA - Bracket, Forward, Chrome (1")	1	1
3		7930352-NLA	NLA Locknut (7/16-20)	25	1
4		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
5		1045608	Locknut (7/16-20)	1	1
6		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
7		1036619-NLA	NLA - Option, Forward (1") (Pair)	2	1
NOTE: A - Part no. 1045608 Locknut, replaced Threaded Bushing as of 7/94.					


Amputee Attachment

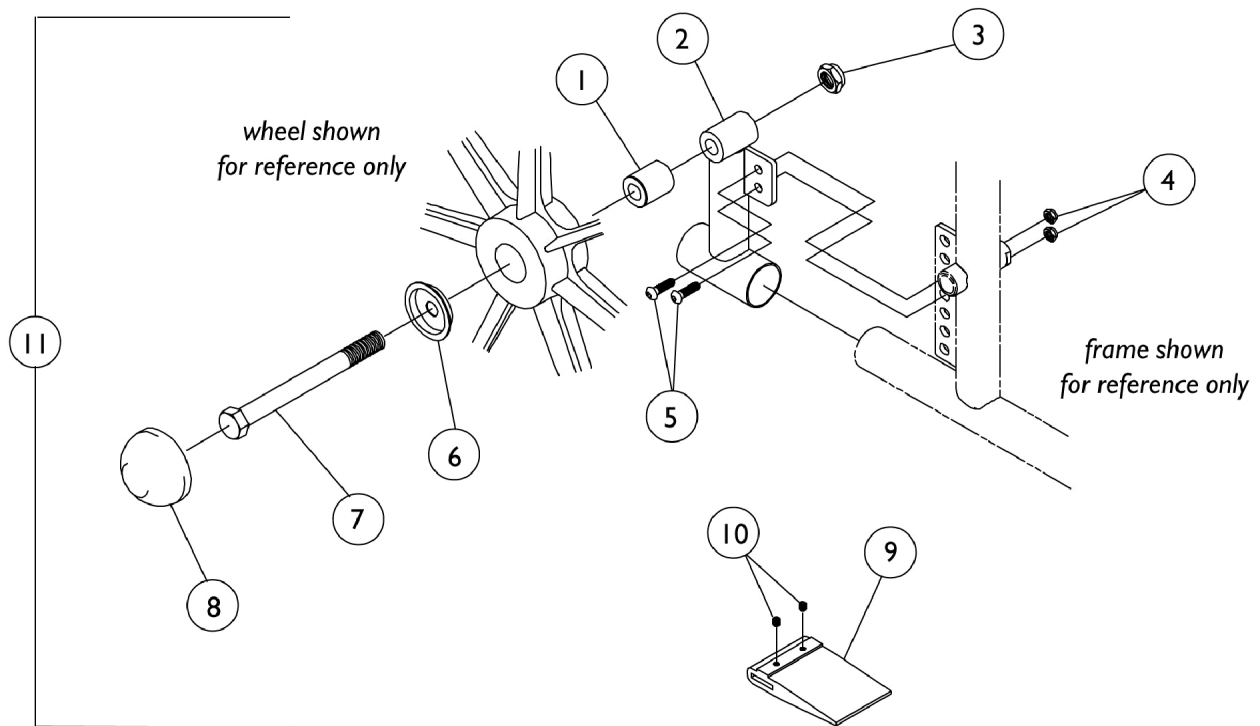

Amputee Attachment

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		47183X037-NLA	NLA - Spacer (1/2 x 7/8 x 7/8")	1	1
2		1020512-NLA	NLA - Bracket, Amputee - Right	1	1
2		1020513-NLA	NLA - Bracket, Amputee - Left	1	1
3		7930352-NLA	NLA Locknut (7/16-20)	25	1
4		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	2
5		1007269	Screw, Socket Button Head (1/4-20 x 5/8")	1	2
6		1016289	Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
7		47203X025	Bolt, Hex Head Cap (7/16-20 x 4-3/4")	1	1
8		1016291	Cap, Hub, Black	1	1
9	A	1016980	Spacer, Wheel Lock Shoe (2-63/64")	1	2
10		1008438	Screw, Set Socket Cup Point (#8-32 x 3/16")	1	2
11		1036618-NLA	NLA - Attachment, Amputee (Pair)	2	1
NOTE: A - For 20" Rear Wheel					

Headrest Extension

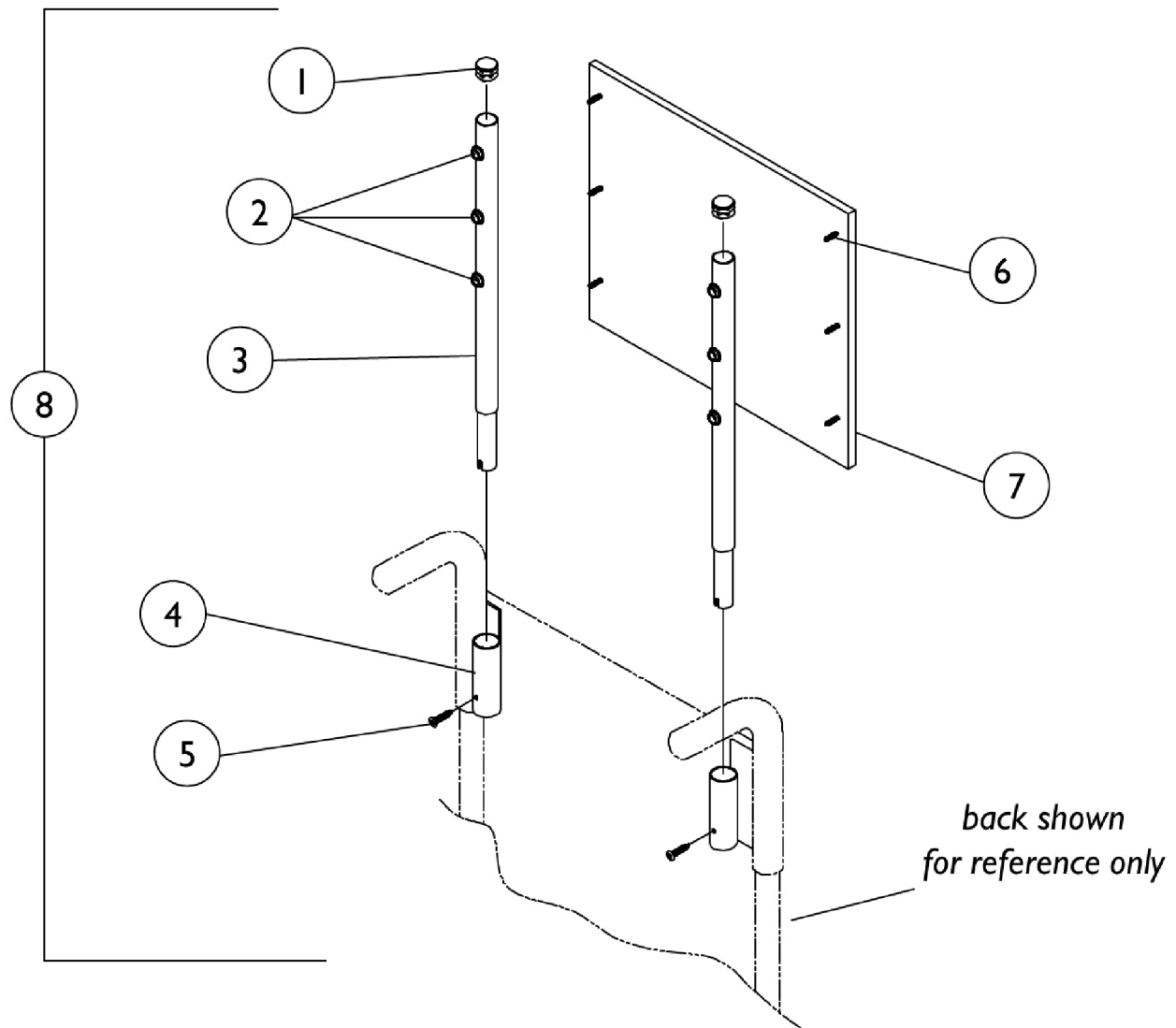

Headrest Extension

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	2
2		47447X009-NLA	NLA - Screw, Phillips (#8 x 3/8")	1	6
3		1055152-NLA	NLA - Headrest Extension Tube with Fastners	1	2
4		1053110-NLA	NLA - Bracket, Headrest, Black - Right	1	1
4		1053111-NLA	NLA - Bracket, Headrest, Black - Left	1	1
5		40163X013-NLA	NLA - Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	2
6		1041541	Package, Screw Phillips Csunk Oval Head w/ Washer (#10-16 x 7/8") (12 ea.)	12	1
	A	See Chart	Headrest Extension Assembly	1	1
	A	See Chart	Headrest Upholstery with Screws	1	1

NOTE: A - Not Shown

#7 - Headrest Upholstery with Screws

Width	10"	12"	14"
Vinyl	8881053097-NLA See Note(s) A	8881053099-NLA See Note(s) A	8881053101-NLA See Note(s) A
Nylon	1053098-NLA	1053100-NLA	1053102-NLA


NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors".

#8 - Headrest Extension Assembly

Width	10"	12"	14"
Vinyl	8881053112-NLA See Note(s) A	8881053114-NLA See Note(s) A	8881053116-NLA See Note(s) A
Nylon	1053113-NLA	1053115-NLA	1053117-NLA

NOTE: A - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors".


Head Support

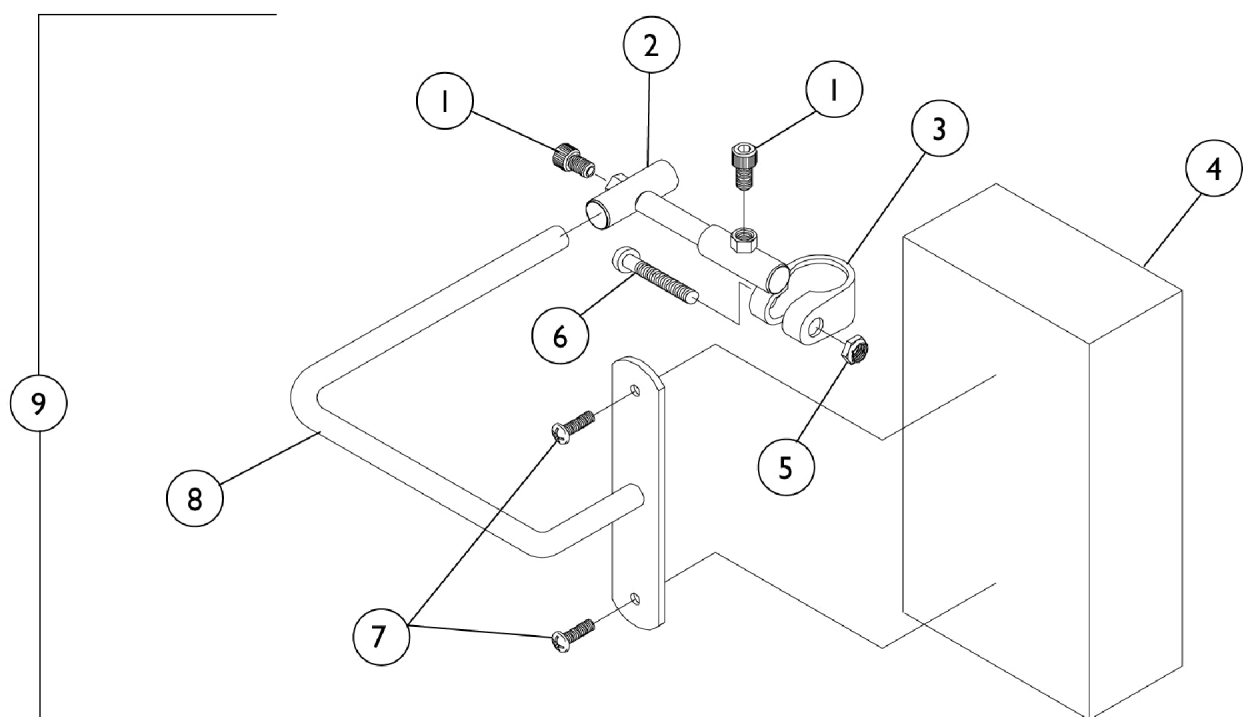

Head Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	2
2		03018X042	Wing Bracket Assembly, Chrome	1	1
3		03018X043	Bracket, Clamp, Black- Right	1	1
3		03018X046	Bracket, Clamp, Black - Left	1	1
4	B,C	88804529X040-NLA	NLA - Pad, Headrest	1	1
5		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
6		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
7		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
8	D	03018X045	Bracket, Pad Support, Chrome	1	1
9	D	8881050191-NLA	NLA - Head Support Assembly (Pair)	2	1
	A,E	1023610-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Not Shown B - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors" C - Vinyl Only D - Non-Recliner E - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					

Torso Support

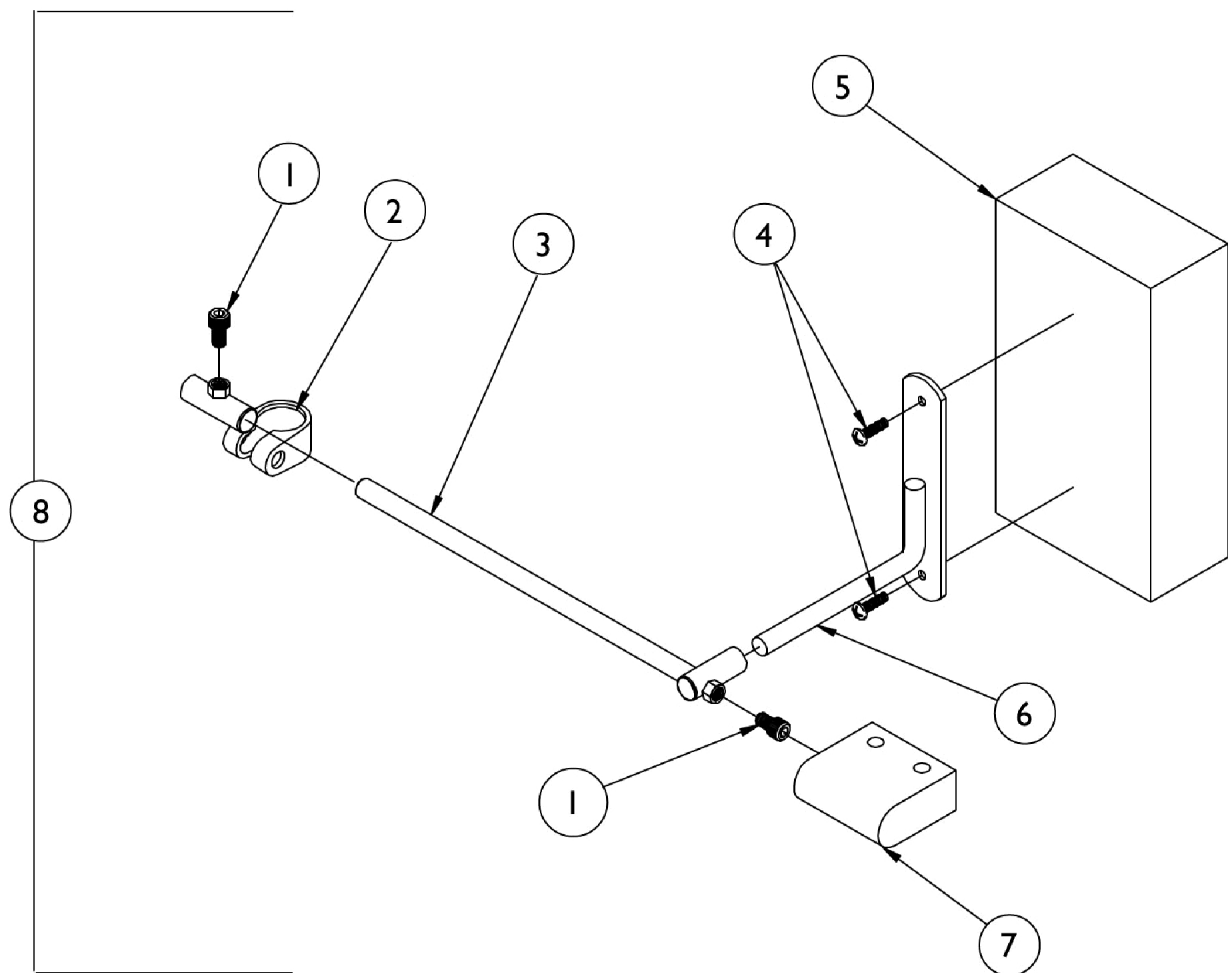

Torso Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	2
2		03018X043	Bracket, Clamp, Black- Right	1	1
2		03018X046	Bracket, Clamp, Black - Left	1	1
3		03018X012-NLA	NLA - Rod, Support w/ Guide Tube	1	1
4		7930360-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	2
5	B,C	88804529X040-NLA	NLA - Pad, Headrest	1	1
6		03018X016-NLA	NLA - Bracket, Support, Chrome	1	1
7	B	88804529X043-NLA	NLA - Cover Torso Bracket	--	--
8	B,D	8881050189-NL A	NLA - Torso Support Assembly (Pair)	2	1
	A,E	1023610-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Not Shown B - Specify upholstery color, see page titled "Frame Finishes and Upholstery Colors" C - Vinyl Only D - Non-Recliner E - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					

Miscellaneous Accessories

Part Number	Description	Package Quantity
1036616-NLA	Anti-Tippers, Rear (pair)	2
8881047954-NLA See Note(s) A	Carrying Pocket (Vinyl) See Note(s) A	1
8881047956-NLA See Note(s) A	Carrying Pocket (Nylon) See Note(s) A	1
1034224-NLA	Caster Swivel Lock (pair)	2
1034204	Crutch and Cane Carrier	1
88808017X001-NLA See Note(s) A	Lumbar Cushion (1" Wide) See Note(s) A	1
88808017X002-NLA See Note(s) A	Lumbar Cushion (2" Wide) See Note(s) A	1
1036526-NLA See Note(s) B	Seat & Chest Restraints Hook and Loop Style See Note(s) B	1
1036525-NLA	Seat & Chest Restraints Airline Buckle Style	1
1036527	Seat & Chest Restraints Auto Style	1
05600X057-NLA	Spoke Guard	1
40176X010-NLA	Tire Pump	1
1050193	6" Wheel Lock Extension Handle w/Tip (pair)	2
1143049	Kit, Wheel Lock Extension Handle, Black (8") (Pair)	2
NOTE: A - For color see page titled, "Upholstery Colors and Frame Finishes". B - Not used for chest restraints		

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Customer Service
800-333-6900

© 2022 Invacare Corporation. Elyria, Ohio.
Invacare, Yes, you can., and the Medallion
Design are registered trademarks of
Invacare Corporation.
All trademarks are owned by or licensed to
Invacare Corporation or its subsidiaries
unless otherwise noted.
(Revised 03/23/22)

Form No. 89-79