


# Seat and Ottoman - 6908A

Clinical Recliner

Model Number 6908A-XX

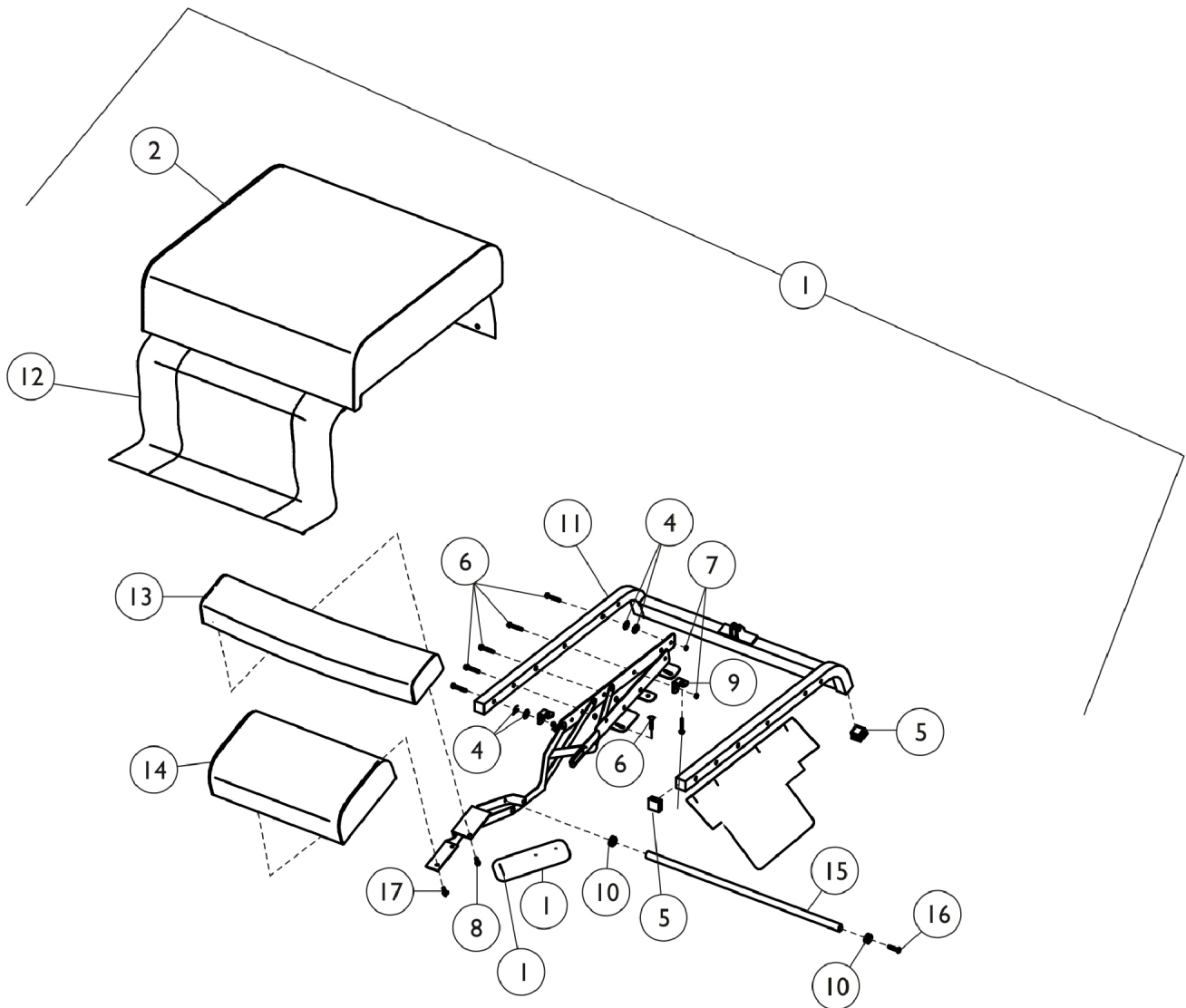

## Seat and Ottoman - 6908A

| Item Number | Note | Part Number   | Description                                                     | Quantity Per |          |
|-------------|------|---------------|-----------------------------------------------------------------|--------------|----------|
|             |      |               |                                                                 | Package      | Assembly |
| 1           | A,B  | 7001M214-NLA  | NLA - Seat Mech w/Ottoman, Brown Frame Finish, U25 Horizon Blue | 1            | 1        |
| 1           | A,B  | 7001M215-NLA  | NLA - Seat Mech w/Ottoman, Brown Frame Finish, U28 Rose Mauve   | 1            | 1        |
| 1           | A,B  | 7001M216-NLA  | NLA - Seat Mech w/Ottoman, Brown Frame Finish, U35 Dark Aqua    | 1            | 1        |
| 1           | A,B  | 7001M217-NLA  | NLA - Seat Mech w/Ottoman, Brown Frame Finish, U36 Burgundy     | 1            | 1        |
| 1           | A,B  | 7001M218-NLA  | NLA - Seat Mech w/Ottoman, Brown Frame Finish, U37 Grotto       | 1            | 1        |
| 1           | A,B  | 7001M219-NLA  | NLA - Seat Mech w/Ottoman, Brown Frame Finish, U38 Blueridge    | 1            | 1        |
| 1           | A,C  | 1109659-NLA   | NLA - Seat Mech w/Ottoman Grey Frame Finish, U25 Horizon Blue   | 1            | 1        |
| 1           | A,C  | 1109660-NLA   | NLA - Seat Mech w/Ottoman Grey Frame Finish, U28 Rose Mauve     | 1            | 1        |
| 1           | A,C  | 1109661-NLA   | NLA - Seat Mech w/Ottoman Grey Frame Finish, U35 Dark Aqua      | 1            | 1        |
| 1           | A,C  | 1109662-NLA   | NLA - Seat Mech w/Ottoman Grey Frame Finish, U36 Burgundy       | 1            | 1        |
| 1           | A,C  | 1109663-NLA   | NLA - Seat Mech w/Ottoman Grey Frame Finish, U37 Grotto         | 1            | 1        |
| 1           | A,C  | 1109664-NLA   | NLA - Seat Mech w/Ottoman Grey Frame Finish, U38 Blueridge      | 1            | 1        |
| 2           | D    | 7001M031-NLA  | NLA - Seat Cushion Assembly, U25 Horizon Blue                   | 1            | 1        |
| 2           | D    | 7001M032-NLA  | NLA - Seat Cushion Assembly, U28 Rose Mauve                     | 1            | 1        |
| 2           | D    | 7001M033-NLA  | NLA - Seat Cushion Assembly, U35 Dark Aqua                      | 1            | 1        |
| 2           | D    | 7001M034-NLA  | NLA - Seat Cushion Assembly, U36 Burgundy                       | 1            | 1        |
| 2           | D    | 7001M035-NLA  | NLA - Seat Cushion Assembly, U37 Grotto                         | 1            | 1        |
| 2           | D    | 7001M036-NLA  | NLA - Seat Cushion Assembly, U38 Blueridge                      | 1            | 1        |
| 3<br>3A     | H    | 1124312-NLA   | NLA - Kit, Mechanism Recline                                    | 1            | 1        |
|             |      |               | Reclining Mechanism - Left and Right                            | 1            | 1        |
|             |      |               | Cover, Linkage, Vinyl, Black                                    | 1            | 1        |
|             |      |               | Instruction Sheet                                               | 1            | 1        |
| 4           | F    | 40157X025     | Washer, Steel (5/16 x 3/4 x 1/16")                              | 1            | 1        |
|             | E,I  | 1124313       | NLA - Kit, Mechanism Recline Hardware                           | 1            | 1        |
| 5           |      |               | NLA - Cap, End, Black (1" x 1")                                 | 1            | 4        |
| 6           |      |               | Screw, Hex Head (1/4-20 x 1-1/2")                               | 1            | 8        |
| 7           |      |               | Locknut (1/4-20)                                                | 1            | 10       |
| 8           |      |               | Screw, Hex Washer Head (1/4-20 x 1")                            | 1            | 8        |
| 9           |      |               | Bracket, Seat Mounting, Grey                                    | 1            | 2        |
| 10          |      |               | Screw, Hex Head (1/4-20 x 1/2")                                 | 1            | 4        |
|             |      |               | Instruction Sheet                                               | 1            | 1        |
| 11          |      | 54000M101-NLA | NLA - Fastner, Thread                                           | 1            | 2        |
| 12          | B,C  | 7001M116-NLA  | NLA - Frame, Seat, Brown                                        | 1            | 1        |
| 12          | B,C  | 1109665-NLA   | NLA - Seat Frame, Grey Frame Finish                             | 1            | 1        |
| 13          | J    | 7000M777-NLA  | NLA - Panel, Leg, U25 Horizon Blue                              | 1            | 1        |
| 13          |      | 7000M778-NLA  | NLA - Panel, Leg, U28 Rose Mauve                                | 1            | 1        |
| 13          |      | 7000M779-NLA  | NLA - Panel, Leg, U35 Dark Aqua                                 | 1            | 1        |

## Seat and Ottoman - 6908A

| Item Number                                                                                                                                                                                                                                                                                                                                                                                   | Note | Part Number   | Description                                      | Quantity Per |          |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|---------------|--------------------------------------------------|--------------|----------|
|                                                                                                                                                                                                                                                                                                                                                                                               |      |               |                                                  | Package      | Assembly |
| 13                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M780-NLA  | NLA - Panel, Leg, U36 Burgundy                   | 1            | 1        |
| 13                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M781      | Panel, Leg, U37 Grotto                           | 1            | 1        |
| 13                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M867-NLA  | NLA - Panel, Leg, U38 Blueridge                  | 1            | 1        |
| 14                                                                                                                                                                                                                                                                                                                                                                                            |      | 7001M049-NLA  | NLA - Center Ottoman, U25 Horizon Blue           | 1            | 1        |
| 14                                                                                                                                                                                                                                                                                                                                                                                            |      | 7001M050-NLA  | NLA - Center Ottoman, U28 Rose Mauve             | 1            | 1        |
| 14                                                                                                                                                                                                                                                                                                                                                                                            |      | 7001M051-NLA  | NLA - Center Ottoman, U35 Dark Aqua              | 1            | 1        |
| 14                                                                                                                                                                                                                                                                                                                                                                                            |      | 7001M052-NLA  | NLA - Center Ottoman, U36 Burgundy               | 1            | 1        |
| 14                                                                                                                                                                                                                                                                                                                                                                                            |      | 7001M053-NLA  | NLA - Center Ottoman, U37 Grotto                 | 1            | 1        |
| 14                                                                                                                                                                                                                                                                                                                                                                                            |      | 7001M054-NLA  | NLA - Center Ottoman, U38 Blueridge              | 1            | 1        |
| 15                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M747-NLA  | NLA - Ottoman, End, U25 Horizon Blue             | 1            | 1        |
| 15                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M748-NLA  | NLA - Ottoman, End, U28 Rose Mauve               | 1            | 1        |
| 15                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M749-NLA  | NLA - Ottoman, End, U35 Dark Aqua                | 1            | 1        |
| 15                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M750-NLA  | NLA - Ottoman, End, U36 Burgundy                 | 1            | 1        |
| 15                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M751-NLA  | NLA - Ottoman, End, U37 Grotto                   | 1            | 1        |
| 15                                                                                                                                                                                                                                                                                                                                                                                            |      | 7000M861-NLA  | NLA - Ottoman, End, U38 Blueridge                | 1            | 1        |
| 16                                                                                                                                                                                                                                                                                                                                                                                            | B    | 54000M604-NLA | NLA - Tube, Cross, Mechanism, Brown Frame Finish | 1            | 2        |
| 16                                                                                                                                                                                                                                                                                                                                                                                            | C    | 1109666-NLA   | NLA - Tube, Cross, Mechanism, Grey Frame Finish  | 1            | 2        |
| 17                                                                                                                                                                                                                                                                                                                                                                                            |      | 51000M789     | Screw, Phillips Pan Head (1/4-20 x 3/4")         | 1            | 2        |
| NOTE: A - Includes items 2-17.<br>B - Before 1/18/02.<br>C - After 1/17/02.<br>D - Includes item 13.<br>E - Kit Includes Items 5-10 and Instruction Sheet.<br>F - Washer is not needed after 3/94 because this hardware is part of the frame.<br>H - Includes Items 3-3A and Instruction Sheet.<br>I - Not Shown.<br>J - No Longer Available. See item 2 Seat Cushion Assembly part #7001M031 |      |               |                                                  |              |          |