


Boom Assembly

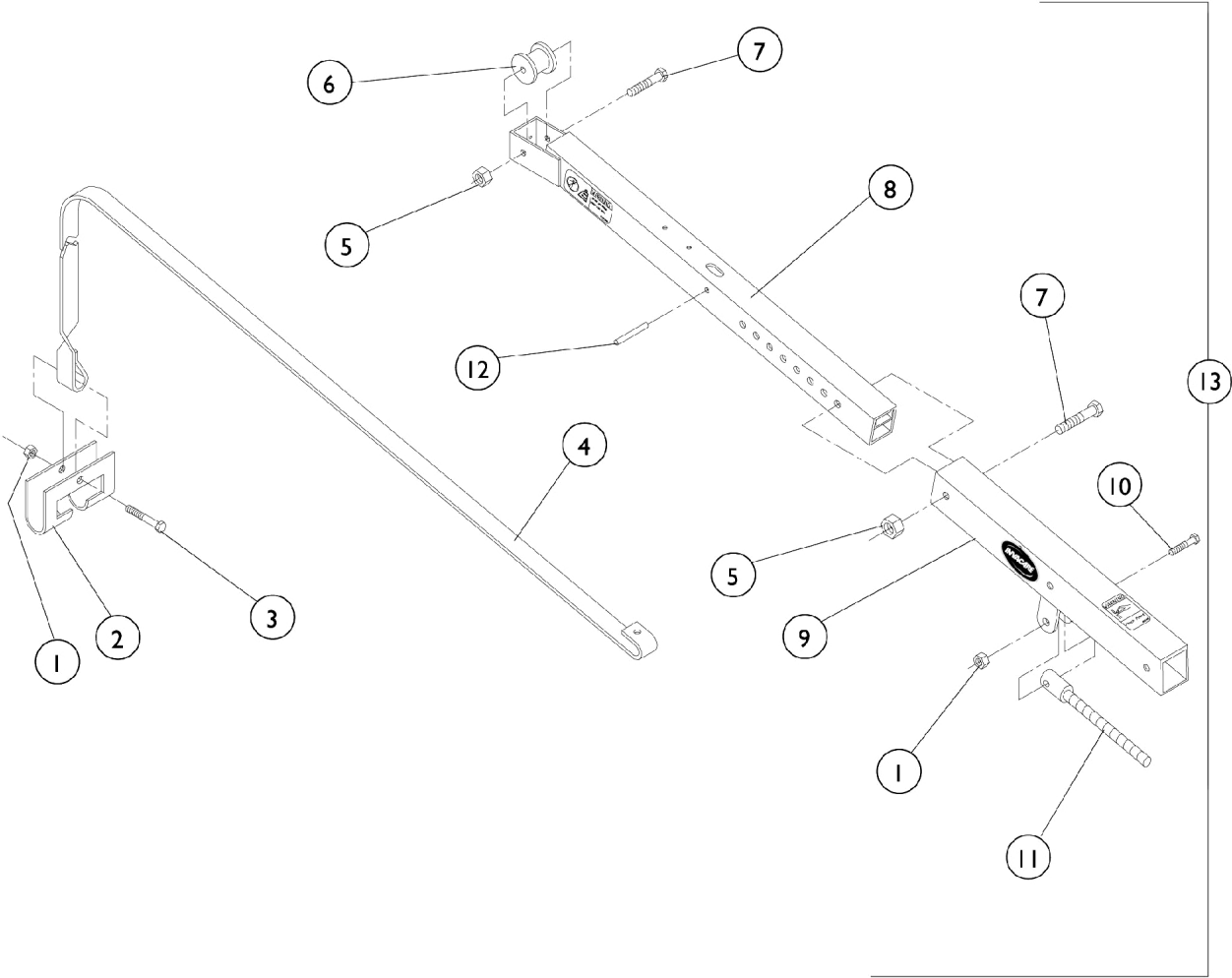

Boom Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		AP100562-NLA	NLA - Locknut (3/8-16)	1	2
2		AP072304-NLA	NLA - Hook, Pickup Claw Lift	1	1
3		AP100561-NLA	NLA - Screw, Hex Head (3/8-16 x 1-3/4")	1	1
4		AP072329-NLA	NLA - Lift, Strap	1	1
5		AP100627-NLA	NLA - Locknut (5/16-18 x 2-1/4")	1	2
6		AP072306-NLA	NLA - Spool, Guide	1	1
7		AP100879-NLA	NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	2
8		AP072305-NLA	NLA - Boom, Inner	1	1
9		AP072373	NLA - Boom, Outer	1	1
10		AP100883-NLA	NLA - Screw, Hex Head (3/8-16 x 1-1/2")	1	1
11		AP072330-NLA	NLA - Rod, Height Adjustment	1	1
12		AP101775-NLA	NLA - Pin, Roll, Slotted 1/4" Long	1	1
13	A	AP072751-NLA	NLA - Boom Assembly w/ Decals, Less Controller	1	1

NOTE: A - Includes items 1-12