


Base Assembly with Adjustable Legs

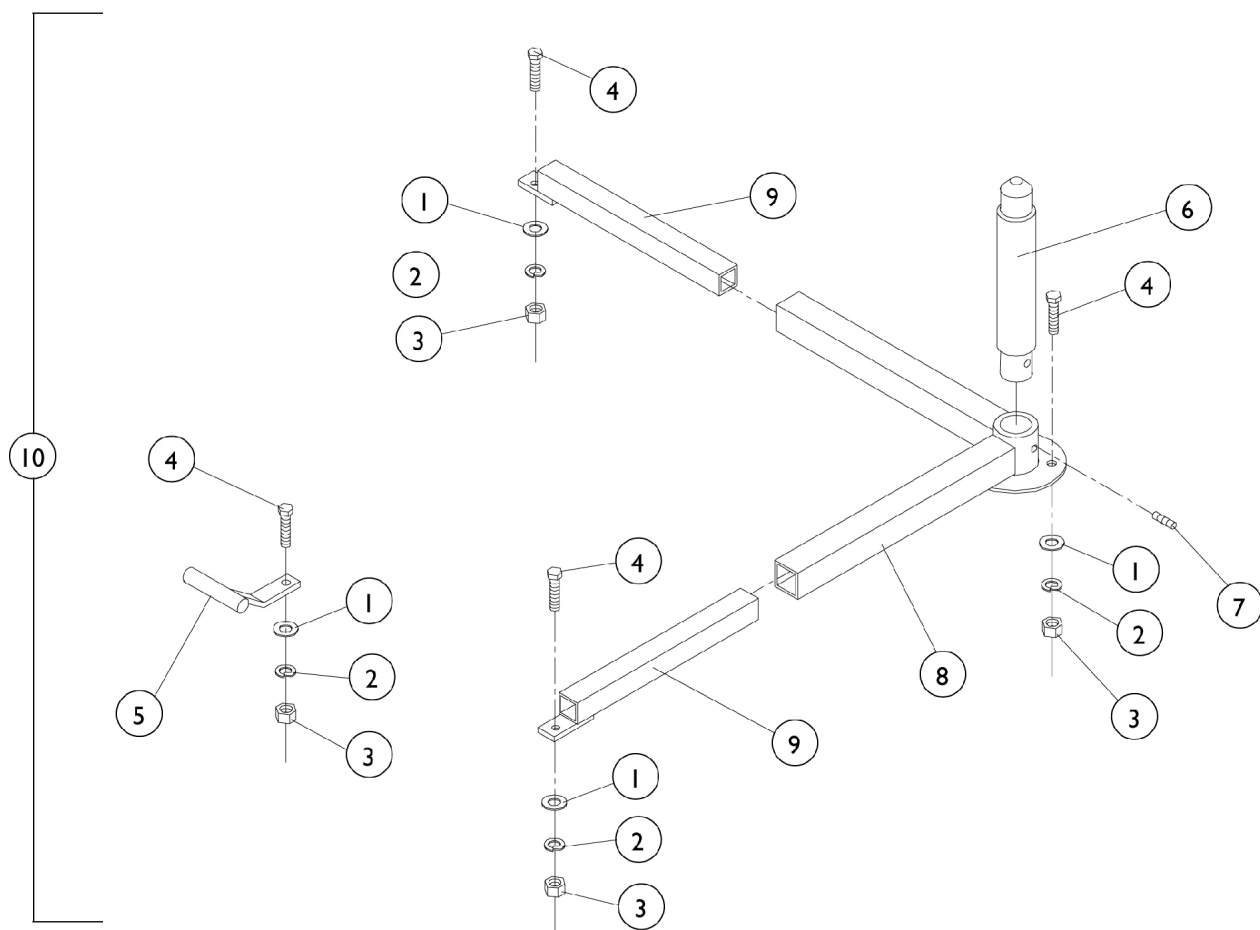

Base Assembly with Adjustable Legs

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		AP100632-NLA	NLA - Washer, Flat (5/16 x 7/8 x 5/64")	1	4
2		AP100838-NLA	NLA - Lockwasher (5/16 x 19/32 x 3/32")	1	4
3		AP100619-NLA	NLA - Hex Nut (5/16-18)	1	4
4	A	AP100581-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	4
4	A	AP100879-NLA	NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	4
4	A	AP100584-NLA	NLA - Screw, Hex Head (5/16-18 x 3")	1	4
5		AP072355-NLA	NLA - Anchor, Hold Down	1	1
6		AP072752-NLA	NLA - Bearing Rod Assembly, Standard (6")	1	1
6		AP072477-NLA	NLA - Bearing Rod Assembly, Opt (9")	1	1
7		AP100682-NLA	NLA - Screw, Socket Set (5/16-18 x 3/8")	1	1
8		AP072333-NLA	NLA - Leg, Stand Van	1	1
9		AP072309-NLA	NLA - Leg, Inner	1	2
10	B	AP072375-NLA	NLA - Base Assembly w/ Adjustable Legs	1	1
NOTE: A - The installer will choose which length screws are needed depending on the thickness of the vehicle floor the lift is mounted to. B - Includes items 1-9.					