


# C-Arm and Docking Device Assembly

## Invacare Lynx and Panther Series Scooters

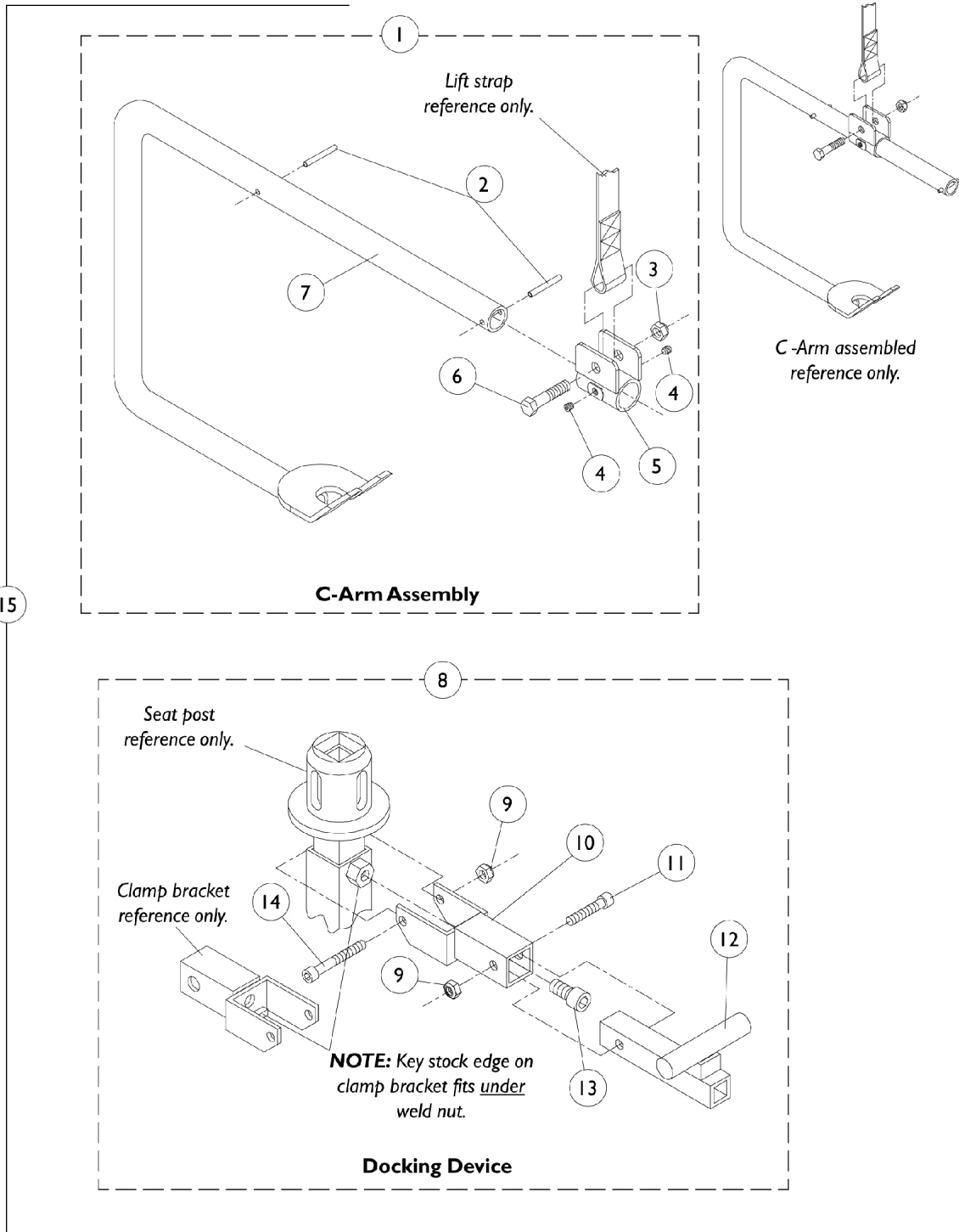

## C-Arm and Docking Device Assembly Invacare Lynx and Panther Series Scooters

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	AP072798-NLA	NLA - C-Arm Assembly	1	1
2		AP100708-NLA	NLA - Pin, Roll (3/16 x 1-1/2")	1	2
3		AP100562-NLA	NLA - Locknut (3/8-16)	1	1
4		AP101395-NLA	NLA - Screw, Socket Set (1/4-20 x 1/4")	1	2
5		AP072461-NLA	NLA - Bracket, Lift	1	1
6		AP100561-NLA	NLA - Screw, Hex Head (3/8-16 x 1-3/4")	1	1
7		AP072797-NLA	NLA - C-Arm - Scooter	1	1
8	C	AP072795-NLA	NLA - Docking Device Assembly	1	1
9		AP100626-NLA	NLA - Locknut (1/4-20")	1	2
10		AP072793-NLA	NLA - Clamp, Bracket	1	1
11		AP100902-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/4")	1	1
12		AP072794-NLA	NLA - Lift Arm - Scooters	1	1
13		AP101836-NLA	NLA - Screw, Socket Head (M10 x 1.25 x 20mm)	1	1
14		AP101839-NLA	NLA - Screw, Socket Head (1/4-20 x 1-7/8")	1	1
15	B	AP072799-NLA	NLA - Kit, C-Arm+ Docking Device Assembly	1	1
	A	APOM-MV3650-NLA	NLA - Instruction Sheet, C-Arm+ Docking Device	1	1

NOTE: A - Not Shown.  
B - Includes items 1, 8 and Instruction Sheet.  
C - Includes items 9-14.  
D - Includes items 2-7.