

Personal Care Bath Safety



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 92-43. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2022 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are registered trademarks of the Invacare Corporation. Careguard and I-Fit are trademarks of Invacare Corporation. Information is subject to change without notification.

TABLE of CONTENTS

Description	Page Number
3" Suction Cup Option - Model 6314 & 6316	63
Adjustable Raised Toilet Seat - Model 1384	22
Bath Board - Model 9664	29
Grab Bar Hardware- Model 2712-3, 2742-3, 512-3, 532-3, 312-3, 332-3 & 1910-1913	65
H605 Rio	71
Hand Shower - Model 826-4 & 828-4	31
Model Number Cross Reference Chart	9
Packaging and Instruction Sheet	4
Raised Toilet Seat - Model 1391AP & 1391P	24
Shower Chair - Bariatric - Model 9785-2	51
Shower Chair - Compact Blow Molded - Model 50-1,3 & 60-1,3	32
Shower Chair - Compact Blow Molded - Model 75-1,3 & 85-1,3	34
Shower Chair - Folding - Model 9980 & 9981	43
Shower Chair - Hygienic - Model 1362 & 1363	36
Shower Chair - IVC I-Fit & CareGuard - Model 90-2, 95-2, 9780, 9781	47
Shower Chair - Large Blow Molded - Model 9680, 9680P, 9681 & 9681P	41
Shower Chair - Mobile - Model 6358	38
Shower Chair - Tooless - Models 91-2 & 96-2	49
Shower Chair - Vinyl Padding - Model 9872 & 9872C	45
Toilet Safety Frame - Model 1392 & 635-3	26
Toilet Safety Frame - Model 1392KD	28

TABLE of CONTENTS

Description	Page Number
Transfer Bench - Bariatric - Model 9675-2	61
Transfer Bench - Compact Blow Molded - Model 98070 & 990-1	57
Transfer Bench - Large Blow Molded - Model 9670, 9670U & 9670C	53
Transfer Bench - Model 9871, 9871-1, 9873, 9871C, 9871C-1 & 9873C	55
Transfer Bench - Tooless - Model 98071-2	59
Tub Bars - Model 100-3, 110-3, 1816, 725-3, 1200-3, 1210-3 & 705-4	67
Warranty Replacement	13

Packaging and Instruction Sheets

Bath/Assists

Model	Cartons	Instruction Sheets	Inserts
1392	1086816-NLA	1087193-NLA	N/A
1364P-NLA	1062076-NLA	1052374	N/A
1365P-NLA	1062076-NLA	1062078-NLA	N/A
6312-NLA	40131X044	1070889-NLA	N/A
6313	40131X044	1070889-NLA	N/A
6316	40131X044	1070889-NLA	N/A
635-1	1089759-NLA	1089644-NLA	N/A
635-3	1089758-NLA	1089644-NLA	N/A
6318	40131X044	1070889-NLA	N/A
9664	1089751-NLA	1089629-NLA	N/A

NOTE: Refer to your Owner's Manual for Assistance, When Adjusting or Replacing Parts.

Packaging and Instruction Sheets

Shower Chair

Model	Cartons	Instruction Sheets	Inserts
50's	1085416-NLA	1089640-NLA	N/A
60's	1089187-NLA	1089640-NLA	N/A
75's	1089187-NLA	1089640-NLA	N/A
85's	1089187-NLA	1089640-NLA	N/A
1362, 1363	1086813-NLA	1073902-NLA	N/A
6358	1021705-NLA	1020707-NLA	N/A
6262-NLA, 6463-NLA, 6482-NLA, 6483-NLA	1053180	1023078-NLA	1078340-NLA
9680, 9681	1062803-NLA	1087196-NLA	N/A
9680P-NLA, 9690P-NLA	1062803-NLA	1087196-NLA	N/A
9681P, 9691P-NLA	1062803-NLA	1087196-NLA	N/A
9872, 9872C	1053180	1075941	1078340-NLA
9980	1093519-NLA	N/A	N/A
9981	1093106	1145752	N/A
90-2, 95-2, 9780, 9781	N/A	1118360	N/A

NOTE: Refer to your Owner's Manual for Assistance, When Adjusting or Replacing Parts.

Hand Showers

Model	Cartons	Instruction Sheets	Inserts
826's	1089200-NLA	1087199	N/A
828-4	1086812-NLA	1087199	N/A

NOTE: Refer to your Owner's Manual for Assistance, When Adjusting or Replacing Parts.

Packaging and Instruction Sheets

Transfer Bench

Model	Cartons	Instruction Sheets	Inserts
6541-NLA, 6461-NLA, 6481-NLA, 6484-NLA	1053180	1023078-NLA	N/A
90997-NLA	1089400	1089642-NLA	N/A
9670, 9670C	1061421	1148079	N/A
9670U	1065225-NLA	1068523	N/A
97070-NLA	1065467-NLA	1069728-NLA	N/A
9871's, 9873	1053180	1148079	1078340-NLA
9871C, 9873C	1053180	1063930	1078340-NLA
98070	1053180	1148079	1078340-NLA
NOTE: Refer to your Owner's Manual for Assistance, When Adjusting or Replacing Parts.			

Packaging and Instruction Sheets

Grip/Grab Bars

Model	Cartons	Instruction Sheets	Inserts
100-3	1073520-NLA	1089638	N/A
110-1	1080873-NLA	1089638	N/A
110-3	1073520-NLA	1089638	N/A
725-3	1089768-NLA	1089638	N/A
1200-3-NLA	1089769-NLA	1089638	N/A
1210-3	1089769-NLA	1089638	N/A
312-1	1080874-NLA	1089632-NLA	N/A
312-3	1071310-NLA	1148122-NLA	N/A
316-1	1080875-NLA	1089632-NLA	N/A
316-3	1071311-NLA	1148122-NLA	N/A
318-3	1071312-NLA	1148122-NLA	N/A
324-3	1071313-NLA	1148122-NLA	N/A
332-3	1071315-NLA	1148122-NLA	N/A
512-3	1071310-NLA	1148122-NLA	N/A
516-3	1071311-NLA	1148122-NLA	N/A
518-3	1071312-NLA	1148122-NLA	N/A
524-3	1071313-NLA	1148122-NLA	N/A
532-3	1071315-NLA	1148122-NLA	N/A
705-1	1063935-NLA	1087201-NLA	N/A
705-4	1086815-NLA	1087201-NLA	N/A
1700's	N/A	1089632-NLA	N/A
1900's	1055396-NLA	1048870-NLA	N/A
2712-3	1089406	1089632-NLA	N/A
2716-3	1089407-NLA	1089632-NLA	N/A
2718-3	1089408-NLA	1089632-NLA	N/A
2724-3	1089409-NLA	1089632-NLA	N/A
2732-3	1089410-NLA	1089632-NLA	N/A
2736-3	1089411-NLA	1089632-NLA	N/A
2742-3	1089412-NLA	1089632-NLA	N/A

NOTE: Refer to your Owner's Manual for Assistance, When Adjusting or Replacing Parts.

Packaging and Instruction Sheets

Low Height Safety Bar

Model	Cartons	Instruction Sheets	Inserts
1376-NLA, 1377-NLA	1054888-NLA	00075X083-NLA	N/A
NOTE: Refer to your Owner's Manual for Assistance, When Adjusting or Replacing Parts.			

Model Number Cross Reference Chart

Grab Bars

Discontinued Models Before 10/01/99	Discontinued Models After 9/30/99	Current Models	Description
1700CH	512-3	512-3	Grab Bar, Knurled Chrome 12"
1701CH	516-3	516-3	Grab Bar, Knurled Chrome 16"
1702CH	518-3	518-3	Grab Bar, Knurled Chrome 18"
1703CH	524-3	524-3	Grab Bar, Knurled Chrome 24"
1704CH	532-3	532-3	Grab Bar, Knurled Chrome 32"
1740	312-3	312-3	Grab Bar, Residential, White Vinyl Coated 12"
1741	316-3	316-3	Grab Bar, Residential, White Vinyl Coated 16"
1742	318-3	318-3	Grab Bar, Residential, White Vinyl Coated 18"
1743	324-3	324-3	Grab Bar, Residential, White Vinyl Coated 24"
1744	332-3	332-3	Grab Bar, Residential, White Vinyl Coated 32"
1760	2712-3	2712-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 12"
1761	2716-3	2716-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 16"
1762	2718-3	2718-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 18"
1763	2724-3	2724-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 24"
1764	2732-3	2732-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 32"
1765	2736-3	2736-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 36"
1766	2742-3	2742-3	Grab Bar, ADA, Stainless Steel 1-1/2" x 42"
1800	100	100	Bath Security Grip
1801	110	110	Bath Security Suregrip Rail
1810	1200	1200	Knurled Trigrip Rail
1811	1210	1210	Suregrip Trigrip Rail
1819	725	725	Suregrip Bathtub Grip

Model Number Cross Reference Chart

Hand Showers

Discontinued Models Before 6/29/98	Discontinued Models After 9/30/99	Current Models	Description
HH101W	826	826	Hand Shower Kit, White
HH201W	828	828	Hand Shower Kit, White

Transfer Benches

Discontinued Models Before 11/19/96	Discontinued Models After 11/18/96	Current Models	Description
6431	9670	9670	Transfer Bench
6431	9670U	9670U	Transfer Bench - Unassembled
6431	9670C	9670C	Transfer Bench with Commode
Before 4/20/98	After 4/19/98		
670	9871C	9871C	Bathing Bench with Arm and Commode Seat - Padded
6451	99871C	99871C	Transfer Bench with Commode
6461's	9871's	9871's	Transfer Bench
6481	9873	9873	Transfer Bench
6484	9873C	9873C	Transfer Bench with Commode
97070	98070	98070	Transfer Bench

Model Number Cross Reference Chart

Shower Chairs

Discontinued Models Before 6/02/97	Discontinued Models After 6/01/97	Current Models 10/01/99	Description
1368	97080	75-1	Shower Chair without Back, Adjustable Height
1369	97081	85-1	Shower Chair with Back, Adjustable Height
1390	97090	50-1	Shower Chair without Back, Fixed Height
1390B	97091	60-1	Shower Chair with Back, Fixed Height

Shower Chairs

Discontinued Model Before 4/20/96	Discontinued Models After 4/19/96	Current Models 10/01/99	Description
6462	9872C	9872C	Shower Chair with Commode
6463	9872	9872	Shower Chair without Commode
6482	9872C	9872C	Shower Chair with Commode
6483	9872	9872	Shower Chair without Commode

Shower Chairs

Discontinued Models Before 10/01/99	Discontinued Models After 9/30/99	Current Models 10/01/99	Description
90075	9680P	9680P	Bathing Seat, Adjustable, Contoured
90085	9681P	9681P	Bathing Seat with Back, Contoured, Adjustable
90994	9670U	9670U	Bathing Bench, Contoured, Adjustable

Model Number Cross Reference Chart

Accessories - Bath Mat

Discontinued Models Before 7/17/98	Discontinued Models After 7/16/98	Current Models 10/01/99	Description
1711	1711	815	Bath Mat

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Bath/Shower Assist

Reference Model Number	Warranty Number	Description	Warranty Package
HH201C-NLA	1038080-NLA	NLA - WARR RPLMT - Hand Shower Kit - Chrome	1
DV401C	1038084-NLA	NLA - WARR RPLMT - Valve, Diverter	1
TS301C	1088463-NLA	NLA - WARR RPLMT - Tub Spout, Brass Bath Mat	1
815-6	1061914-NLA	WARR RPLMT - Bath Mat	1
826-4	1097313	WARR RPLMT - Kit, Hand Shower, White	1
828-4	1088461	WARR RPLMT - Kit, Hand Shower, White	1
1364P-NLA	1061879-NLA	NLA - WARR RPLMT - Bath Board	1
1365P-NLA	1061880-NLA	WARR RPLMT - Seat, Bath	1
1384	1061913-NLA	WARR RPLMT - Seat, Toilet, Raised, Adjustable	1
1390P	1143631-NLA See Note(s) A	WARR RPLMT - Seat, Toilet, Raised w/o Arms	1
1390AP	1143632-NLA See Note(s) B	WARR RPLMT - Seat, Toilet, Raised w/ Arms	1
1301RTS	1061881	WARR RPLMT - Toilet Seat, Raised, Clamp-On w/o Arms	1
1302RTS	1061882	WARR RPLMT - Toilet Seat, Raised, Clamp-On w/ Arms	1
1391P	1061881 See Note(s) C	WARR RPLMT - Toilet Seat, Raised, Clamp-On	1
1391AP	1061882 See Note(s) D	WARR RPLMT - Toilet Seat, Raised, Clamp-On, with Arms	1
1392	1055572-NLA	WARR RPLMT - Frame, Toilet Safety	1
1393P-NLA	685-1-NLA	WARR RPLMT - Seat, Toilet, Raised	1
685-4	685-1-NLA	WARR RPLMT - Seat, Toilet, Raised	1
1392KD	1128091-NLA	WARR RPLMT - Frame, Toilet Safety	1

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

NOTE:	A - Item no longer available, refer to part #1061881 for replacement.
	B - Item no longer available, refer to part #1061882 for replacement.
	C - Due to design change, 1391P will be replaced with 1301RTS.
	D - Due to design change, 1391AP will be replaced with 1302RTS.

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Shower Chair

Reference Model Number	Warranty Number	Description	Warranty Package
50-3	1097314-NLA	NLA - WARR RPLMT - Shower Chair without Back, Fixed Height	1
60-3 - NLA	1097315-NLA	NLA - WARR RPLMT - Shower Chair with Back, Fixed Height	1
75-3-NLA	1097316-NLA	NLA - WARR RPLMT - Shower Chair without Back, Adjustable Height	1
85-3	85-1-NLA	WARR RPLMT - Shower Chair with Back, Adjustable Height	1
90-2	1128807-NLA	WARR RPLMT - Shower Chair without Back, Aluminum Legs	1
91-2	1144552-NLA	WARR RPLMT - Shower Chair without Back, Tooless	1
95-2	1149806-NLA See Note(s) A	WARR RPLMT - Shower Chair with Back, Aluminum Legs	1
96-2	1144551	WARR RPLMT - Shower Chair with Back, Tooless	1
1362	1061911-NLA	WARR RPLMT - Shower Chair without Back, Hygienic Adjustable	1
1363	1061912-NLA	WARR RPLMT - Shower Chair with Back, Hygienic, Adjustable	1
1366-NLA	1061918-NLA	NLA - WARR RPLMT - Shower Chair without Back, Adjustable Height	1
1367-NLA	1022716-NLA	NLA - WARR RPLMT - Shower Chair Assembled with Back	1
1368-NLA	1097316-NLA	NLA - WARR RPLMT - Shower Chair without Back, Adjustable Height	1
1369-NLA	1097317-NLA	WARR RPLMT - Shower Chair, with Back	1
1390-NLA	1097314-NLA	NLA - WARR RPLMT - Shower Chair without Back, Fixed Height	1
1390B-NLA	1097315-NLA	NLA - WARR RPLMT - Shower Chair with Back, Fixed Height	1

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Shower Chair

Reference Model Number	Warranty Number	Description	Warranty Package
6358	1038067-NLA	WARR RPLMT - Shower Chair, Commode - Molded	1
6462-NLA	1064819-NLA	WARR RPLMT - Shower Chair, with Commode, Padded Vinyl	1
6463-NLA	1064818	WARR RPLMT - Shower Chair, Padded Vinyl	1
6482-NLA	1064819-NLA	WARR RPLMT - Shower Chair with Commode, Padded Vinyl	1
6483-NLA	1064818	WARR RPLMT - Shower Chair, Padded Vinyl	1
9680	1061916-NLA	WARR RPLMT - Shower Chair without Back, Adjustable Height	1
9681	1061917	WARR RPLMT - Shower Chair with Back, Adjustable Height	1
9680P	1061916-NLA	WARR RPLMT - Shower Chair with Back, Adjustable Height	1
9681P	1061917	WARR RPLMT - Shower Chair with Back, Adjustable Height	1
9690P-NLA	1061920-NLA	NLA - WARR RPLMT - Shower Chair without Back, Fixed Height	1
9691P-NLA	1064788-NLA	NLA - WARR RPLMT - Shower Chair with Back, Fixed Height	1
9785-2	1141828	WARR RPLMT - Bariatric Shower Chair with Back	1
9872	1064818	WARR RPLMT - Shower Chair, Padded Vinyl	1
9872C	1064819-NLA	NLA - WARR RPLMT - Shower Chair with Commode, Padded Vinyl	1
9980	1097318-NLA	NLA - WARR RPLMT - Shower Chair without Back, Folding	1
9981	1097319	WARR RPLMT - Shower Chair with Back, Folding	1

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Shower Chair

Reference Model Number	Warranty Number	Description	Warranty Package
9781	9781-1	WARR RPLMT - Shower Chair with Back, Composite Legs	1
9780	1128808	WARR RPLMT - Shower Chair without Back, Composite Legs	1
NOTE: A - Item no longer available, use 95-2 for replacement.			

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Transfer Bench

Reference Model Number	Warranty Number	Description	Warranty Package
6431-NLA	1061897	WARR RPLMT - Bench, Transfer, Molded Economy	1
6451-NLA	1108569-NLA	NLA - WARR RPLMT - Transfer Bench with Commode - Padded Vinyl	1
6461-NLA	9871-1-NLA	WARR RPLMT - Bench, Transfer, Padded, Vinyl	1
6464-NLA	1038076-NLA	NLA - WARR RPLMT - Bench, Transfer, Economy Bath	1
6481-NLA	1064816-NLA	NLA - WARR RPLMT - Transfer Bench - Padded Ensolite	1
6484-NLA	1064817-NLA	NLA - WARR RPLMT - Transfer Bench/Commode - Padded Ensolite	1
9670	1061897	WARR RPLMT - Bench, Transfer, Molded Economy	1
9670C	1061898	WARR RPLMT - Bench, Transfer with Commode	1
9675-2	1141827	WARR RPLMT - Bariatric Transfer Bench	1
97070-NLA	1064820-NLA	WARR RPLMT - Bench, Transfer	1
98070	1064820-NLA	WARR RPLMT - Bench, Transfer	1
98071	1144553-NLA	WARR RPLMT - Bench, Transfer, Tooless	1
9871's	9871-1-NLA NLA - WARR RPLMT - Bench, Transfer, Padded, Vinyl	WARR RPLMT - Bench, Transfer, Padded, Vinyl	1
9871C	1108569	NLA - WARR RPLMT - Transfer Bench with Commode - Padded Vinyl	1
9873	1064816	NLA - WARR RPLMT - Transfer Bench - Padded Ensolite	1
9873C-NLA	1064817-NLA	NLA - WARR RPLMT - Transfer Bench/Commode - Padded Ensolite	1

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Grab Bar/Rail

Reference Model Number	Warranty Number	Description	Warranty Package
100-3 - NLA	1061883-NLA	WARR RPLMT - Bath Security Grip	1
110-3	1097320	WARR RPLMT - Rail, Bath Security Suregrip	1
312-3	1097321-NLA	WARR RPLMT - Grab Bar, White Vinyl (12")	1
316-3	1097322-NLA	WARR RPLMT - Grab Bar, White Vinyl (16")	1
318-3	1061894-NLA	WARR RPLMT - Grab Bar, White Vinyl (18")	1
324-3	1061895-NLA	WARR RPLMT - Grab Bar, White Vinyl (24")	1
332-3	1061896-NLA	NLA - WARR RPLMT - Grab Bar, White Vinyl (32")	1
512-3	1090452-NLA	WARR RPLMT - Grab Bar, Chrome (12")	1
516-3	1090453-NLA	WARR RPLMT - Grab Bar, Chrome (16")	1
518-3	1090454-NLA	WARR RPLMT - Grab Bar, Chrome (18")	1
524-3	1090455	WARR RPLMT - Grab Bar, Chrome (24")	1
532-3	1090456-NLA	WARR RPLMT - Grab Bar, Chrome (32")	1
705-4	1097323-NLA	NLA - WARR RPLMT - Bar, Molded tub	1
710-4	1143633-NLA	WARR RPLMT - Tub Bar Assembly, White	1
725-3	1061889-NLA	WARR RPLMT - Rail, Suregrip Bathtub	1
1200-3 - NLA	1061890-NLA	NLA - WARR RPLMT - Rail, Knurled Trigrip	1
1210-3	1061891-NLA	NLA - WARR RPLMT - Rail, Suregrip Trigrip	1
1376-NLA	1055579-NLA	WARR RPLMT - Low Height Safety Bar	1
1377-NLA	1055580-NLA	NLA - WARR RPLMT - Low Height Safety Bar	1
1700W-NLA	1022686-NLA	NLA - WARR RPLMT - Grab Bar, Chrome (12")	1
1701W-NLA	1022690-NLA	NLA - WARR RPLMT - Grab Bar, White (16")	1
1702W-NLA	1022694-NLA	NLA - WARR RPLMT - Grab Bar, White (18")	1

Warranty Replacement

NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Grab Bar/Rail

Reference Model Number	Warranty Number	Description	Warranty Package
1703W-NLA	1022698-NLA	NLA - WARR RPLMT - Grab Bar, White (24")	1
1704W-NLA	1704W-NLA	NLA - WARR RPLMT - Grab Bar, White (32")	1
1705CH	1705C-NLA	NLA - WARR RPLMT - Grab Bar, Chrome, Right Angle	1
1705W-NLA	1705W-NLA	NLA - WARR RPLMT - Grab Bar, White, Right Angle	1
1706CH	1706C-NLA	NLA - WARR RPLMT - Grab Bar, Chrome, Left Angle	1
1706W	1706W-NLA	WARR RPLMT - Grab Bar, White, Left Angle	1
1707W-NLA	1097323-NLA	WARR RPLMT - Tub Bar	1
1744	1061896-NLA	WARR RPLMT - Residential Wall Grab Bar, White Vinyl 32"	1
1745-NLA	1745-NLA	NLA - WARR RPLMT - Grab Bar, White, Right Angle	1
1750-NLA	1061899-NLA	NLA - WARR RPLMT - Grab Bar, ADA, Exposed (1 1/4, 12")	1
1751-NLA	1061901-NLA	NLA - WARR RPLMT - Grab Bar, ADA, Exposed (1 1/4, 16")	1
1752-NLA	1061903-NLA	NLA - WARR RPLMT - Grab Bar, ADA, Exposed (1 1/4, 18")	1
1753-NLA	1061905-NLA	NLA - WARR RPLMT - Grab Bar, ADA, Exposed (1 1/4, 24")	1
1754	1061907-NLA	WARR RPLMT - Grab Bar, ADA, Exposed (1 1/4, 32")	1
1802NLA	1061885-NLA	WARR RPLMT - Bathtub Security Rail	1
1803-NLA	1061886-NLA	WARR RPLMT - Rail, Bathtub Security, Knurled	1
1818	1061887-NLA	NLA - WARR RPLMT - Bathtub rail, Old Style Tub	1

Warranty Replacement


NOTE: The part numbers within this category are used for Warranty Replacement Purposes only and will allow you to order (1) of each product for warranty replacement of multi-packaged items (i.e., 6431 Transfer Bench - 2 per carton).

Grab Bar/Rail

Reference Model Number	Warranty Number	Description	Warranty Package
1816	1061888-NLA	NLA - WARR RPLMT - Rail, Narrow Wall Tub	1
1900-NLA	1055629-NLA	NLA - WARR RPLMT - Grab Bar, White (16")	1
1901-NLA	1055631-NLA	NLA - WARR RPLMT - Grab Bar, White (18")	1
1902-NLA	1055633-NLA	NLA - WARR RPLMT - Grab Bar, White (24")	1
1903-NLA	1055635-NLA	NLA - WARR RPLMT - Grab Bar, White (32")	1
1910-NLA	1055637-NLA	NLA - WARR RPLMT - Grab Bar, Chrome (16")	1
1911	1055639-NLA	WARR RPLMT - Grab Bar, Chrome (16")	1
1912	1055641-NLA	WARR RPLMT - Grab Bar, Chrome (24")	1
1913	1055643-NLA	NLA - WARR RPLMT - Grab Bar, Chrome (32")	1
2712-3	1090457-NLA	NLA - WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 12")	1
2716-3	1090458-NLA	WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 16")	1
2718-3	1090459-NLA	WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 18")	1
2724-3	1090460-NLA	WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 24")	1
2732-3	1090461-NLA	WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 32")	1
2736-3	1090462-NLA	WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 36")	1
2742-3	1090463-NLA	WARR RPLMT - Grab Bar, Stainless Steel (1 1/2 x 42")	1

Adjustable Raised Toilet Seat

Model 1384

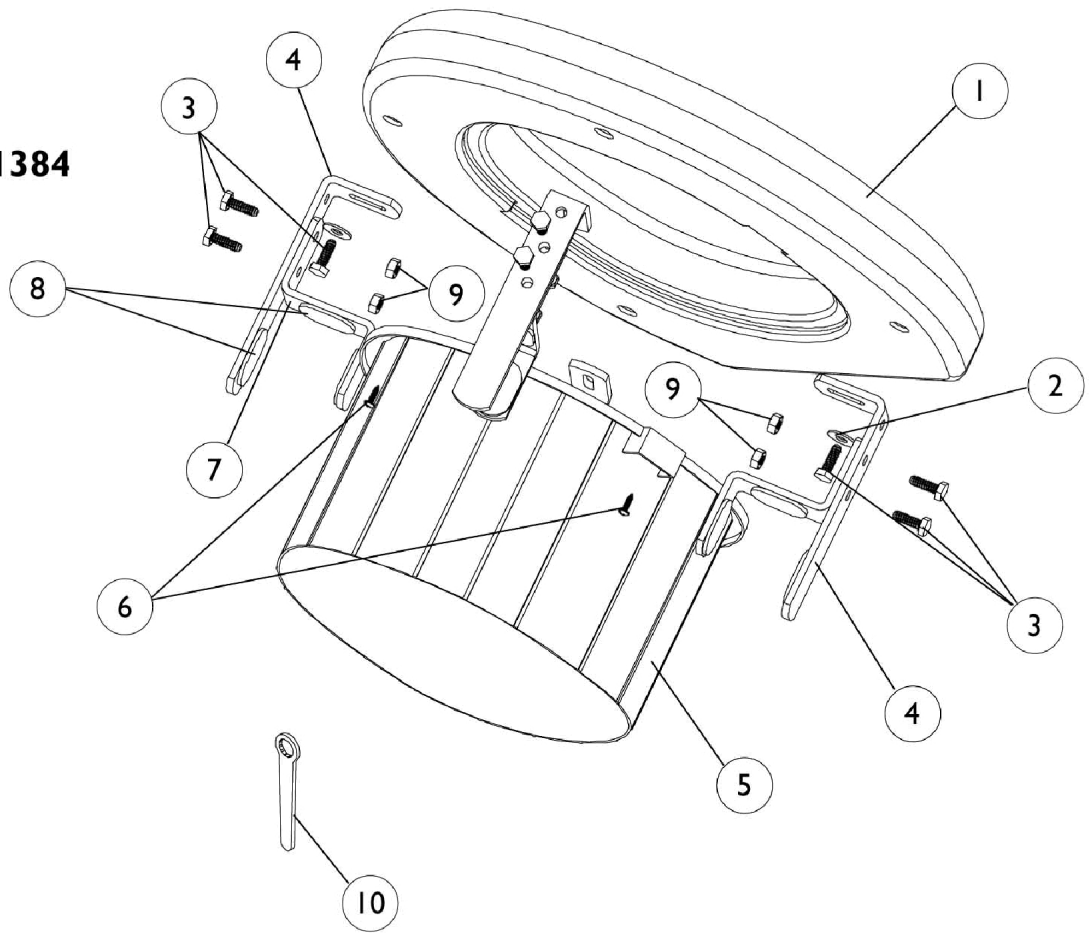


Adjustable Raised Toilet Seat

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1073843-NLA	NLA - Seat, Toilet	1	1
2		1040292-NLA	NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	4
3		60264X010-NLA	NLA - Screw, Hex Head (1/4-20 x 3/4")	1	12
4		1071093-NLA	NLA - Bracket, Small "L", Black	1	4
5		1073844-NLA	NLA - Shield, Splash	1	1
6		1073906-NLA	NLA - Screw, Self Tap (#6-20 x 1/2")	1	4
7		1071094-NLA	NLA - Bracket, Small "Z", Black	1	4
8		1071092-NLA	NLA - Pad, Non-Skid	1	12
9		15108X000	Locknut (1/4-20)	1	8
10		1073204-NLA	NLA - Wrench, Mounting Hex Nut (1/2")	1	1


Raised Toilet Seat

Models 1391AP and 1391P

**Model 1391P
Without Arms**

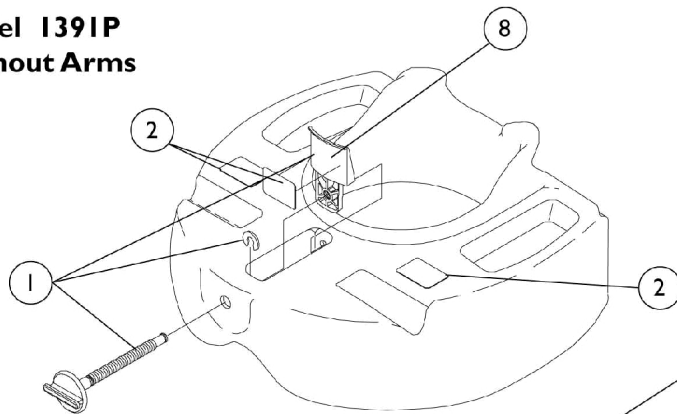

*Underside Shown for
Clarity.*

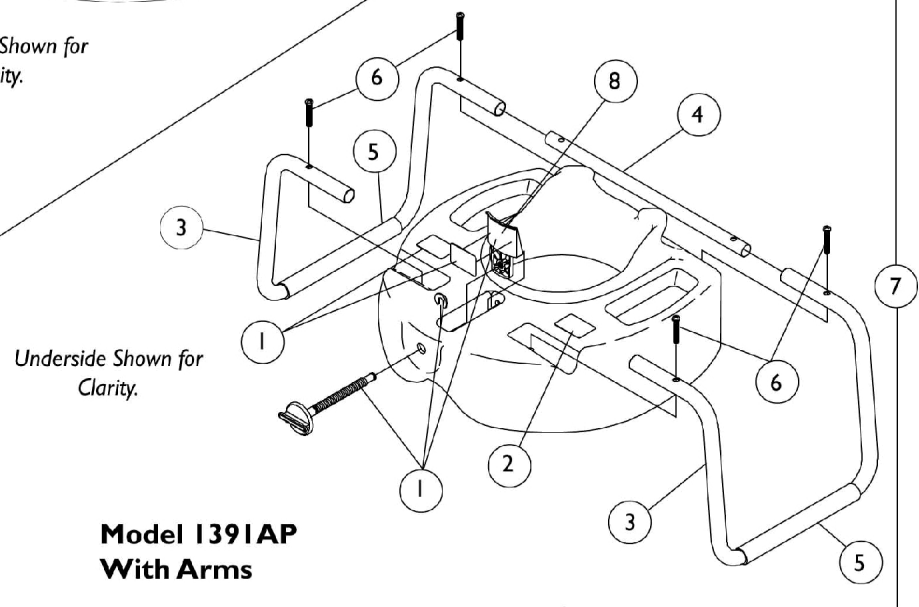


*Underside Shown for
Clarity.*

**Model 1391AP
With Arms**

Raised Toilet Seat Models 1391AP and 1391P

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1048671-NLA	NLA - Kit, Replacement Hardware	1	1
			Bracket, Adjustment, Gray	1	1
			Clip, Retaining	1	1
			Knob, Adjustment, Grey	1	1
2		1048672-NLA	NLA - Pads, Non-Skid	1	3
3		1036153-NLA	NLA - Tube, Arm, Raised Toilet Seat	1	2
4		1036231-NLA	NLA - Tube, Insert	1	1
5		1036155-NLA	NLA - Pad, Arm, Foam, Grey	1	2
6		1008665-NLA	NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	4
7	B	1391KIT-NLA	NLA - Arm Assembly, Complete	1	1
8	C	1073914-NLA	NLA - Adjustment Bracket Assembly, White	1	1
NOTE: A - Kit Includes (1) each Adjustment Bracket, Retaining Clip and Adjustment Knob. B - Includes Items 3-6. C - Includes Adjustment Bracket and Non-Skid Pad.					

Toilet Safety Frame

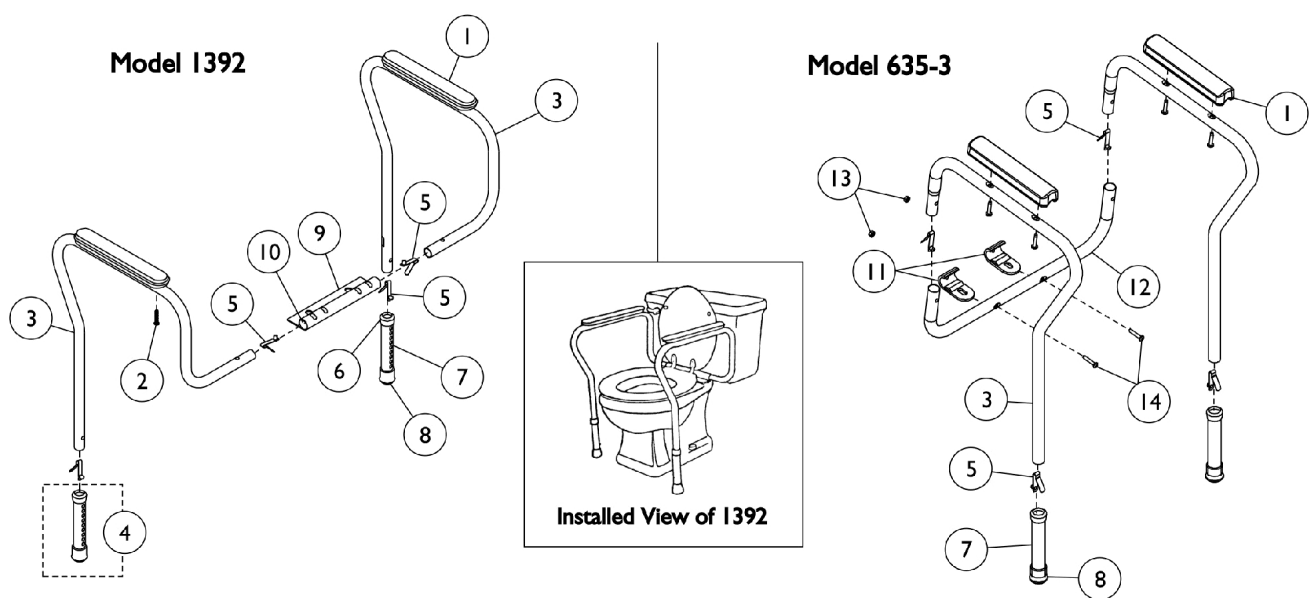


Toilet Safety Frame

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	01000X001-NLA	NLA - Armrest, Full Length, Beige	1	2
1	A,C	1163889	Armrests (Pair)	1	1
1	D,E	1073833-NLA	NLA - Armrest, Full Length, White	1	2
1	D,F,N	1059660-NLA	NLA - Armrest, Full Length, Grey	1	2
	L,M	1132261-NLA	NLA - Kit, Armrest with Hardware, Full Length - Grey	1	1
1			NLA - Armrest, Full Length, Grey	1	2
2			NLA - Screw, Phillips Pan Head (1/4 x 1-1/4")	1	4
2	A	48127X001-NLA	NLA - Screw, Phillips Round Head (#10-32 x 1/2")	1	4
2	D	1061431-NLA	NLA - Screw, Phillips Pan Head (1/4 x 1-1/4")	1	4
3	A,K	48016X000-NLA	NLA - Frame, Side - Right	1	1
3	A,K	48016X001-NLA	NLA - Frame, Side - Left	1	1
3	D	1082559-NLA	NLA - Frame, Side Rail, Chrome - Right/Left	1	2
4	A,G	1058806-NLA	NLA - Leg Extension Assembly	1	2
4	A,G,N	1058807-NLA	NLA - Kit, Leg Extension Sub-Assembly	1	1
			Anti-Rattle, Grey (1-1/8")	1	2
			Tube, Leg Extension	1	2
			Tip, Rubber, Light Grey (1-1/8")	1	2
5		1094945	Button, Snap	1	4
6		1063124	Anti-Rattle, Grey (1-1/8")	1	2
7	D	1073450-NLA	NLA - Leg Extension Assembly	1	2
8		1057797	Tip, Rubber, Light Grey (1-1/8")	1	2
9	A	1018602-NLA	NLA - Bracket, Aluminum	1	1
10		1057287	Plug Button, Grey (1")	1	2
11		1073237-NLA	NLA - Clamp, Toilet Rail	1	2
12	D	1073452-NLA	NLA - Brace	1	1
	D,J	1074444-NLA	NLA - Kit, Hardware	1	1
13			Locknut (1/4-20)	1	2
14			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	2

NOTE: A - To be used with Model 1392.
 B - Before 5/17/99.
 C - After 5/14/09.
 D - To be used on Model 635-3.
 E - Before 3/18/99.
 F - After 3/19/99.
 G - Kit Includes: Leg Tube, Anti-Rattle and Rubber Tip.
 J - Kit Includes Items 13 and 14.
 K - Item no longer available, suggested replacement is a complete unit 1392KD.
 L - Kit includes (2) Armrests Part #1059660 and (4) Screws Part #1061431.
 M - Not Shown.
 N - Item no longer available, refer to Kit Part #1132261.


Toilet Safety Frame (Model 1392KD)

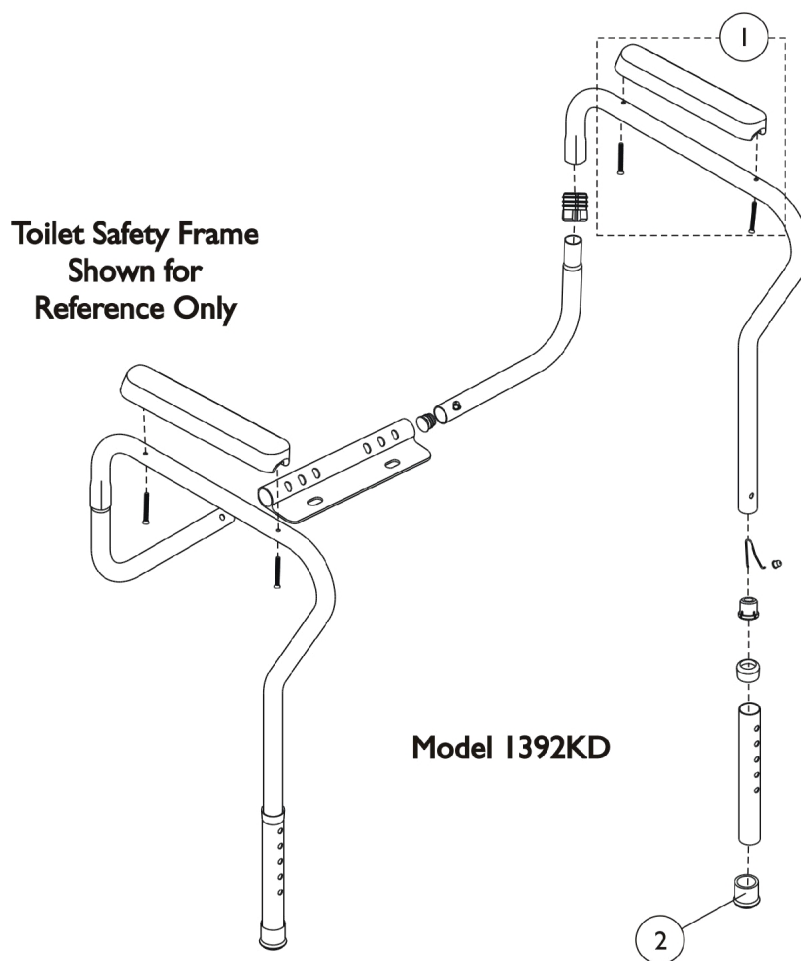

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1128393-NLA	NLA - Kit, Pad Armrest Full with Hardware	1	1
2		40186X030	Tip, Rubber, Grey (1")	4	1
NOTE: A - Kit includes (2) Armrest Pads and (4) Screws.					

Bath Board (Model 9664 - Discontinued 6/07)


Model 9664
Bath Board - Discontinued

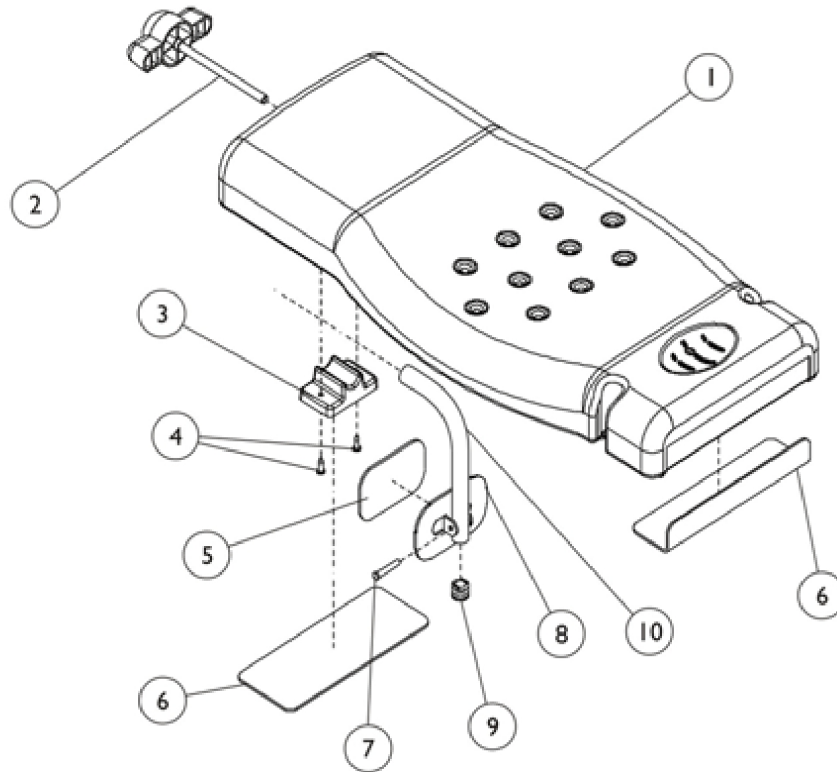

Bath Board (Model 9664 - Discontinued 6/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060311-NLA	NLA - Board, Bath	1	1
2		1052984-NLA	NLA - Knob with Threaded Rod, White	1	1
3		1060378-NLA	NLA - Bracket, Support, White	1	1
4		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	2
5		1052960-NLA	NLA - Pad, Non-Skid without Holes	1	1
6		1060380-NLA	NLA - Pad, Non-Skid without Holes	1	2
7		1061430-NLA	NLA - Pin, Self Locking Clevis	1	1
8		1052981-NLA	NLA - Bracket, Adjustable, White	1	1
9		1040840-NLA	NLA - Plug Button, Grey (7/8")	1	1
10		1061428-NLA	NLA - Tube, Bracket, White	1	1
NOTE: Replacement parts for 1364P, 1365P are No Longer Available. Please contact Customer Service for replacement parts.					

Hand Shower

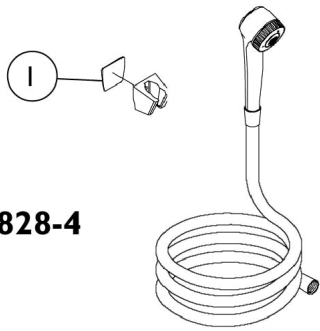


Model 826-4 and 828-4


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1074174-NLA	NLA - Tape, Double Sided	1	1

Shower Chair - Compact Blow Molded Fixed Height

**Model 50-1, 3
Without Back**

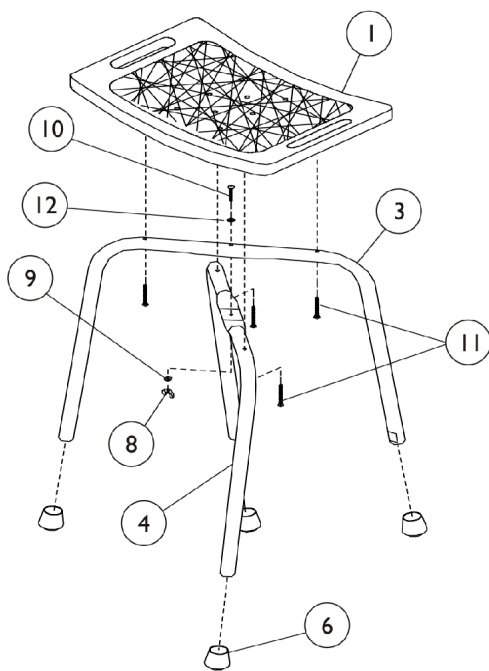

**Model 60-1, 3
With Back**

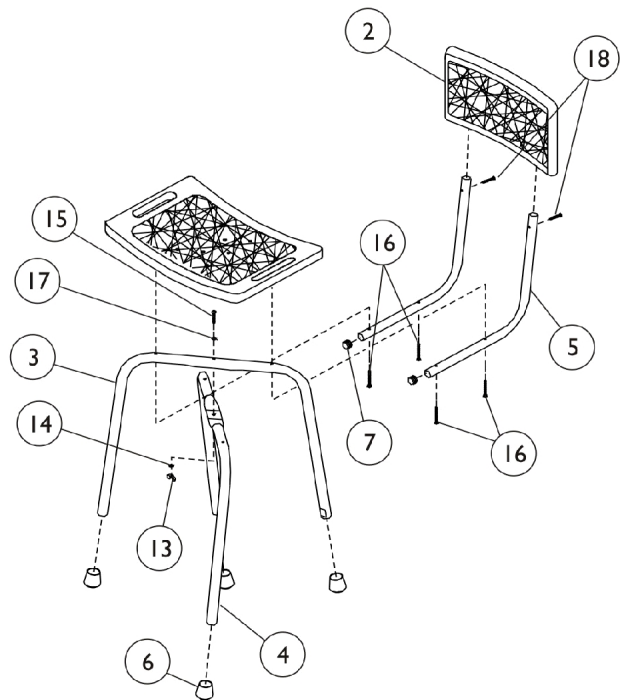

Shower Chair - Compact Blow Molded Fixed Height


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1079341-NLA	NLA - Seat	1	1
2	C	1079340-NLA	NLA - Back	1	1
3		1079313-NLA	NLA - Leg, Top	1	1
4		1079312-NLA	NLA - Leg, Bottom	1	1
5	C	1083245-NLA	NLA - Tube, Backrest Support	1	2
6		40186X030	Tip, Rubber, Grey (1")	1	4
7		1027096	Plug Button, Grey (7/8")	1	2
	A,B,D	1074249-NLA	NLA - Kit, Hardware	1	1
8			NLA - Nut, Wing (1/4-20)	1	1
9			NLA - Washer, External Tooth Lock (1-1/4")	1	1
10			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
11			NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
12			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
	B,C,E	1079912-NLA	NLA - Kit, Hardware	1	1
13			NLA - Nut, Wing (1/4-20)	1	1
14			NLA - Washer, External Tooth Lock (1-1/4")	1	1
15			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
16			NLA - Screw, Phillips Pan Head (1/4-14 x 2-1/4")	1	4
17			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
18			NLA - Screw, Phillips Pan Head Tap (#8-18 x 1/2")	1	2
NOTE: A - To be used with Model 50-1,3. B - Not Shown. C - To be used with Model 60-1,3. D - Kit Includes Items 8-12. E - Kit Includes Items 13-18.					

Shower Chair - Compact Blow Molded Adjustable Height

Model 75-1

Model 75-3 Discontinued 12/1/04


Without Back

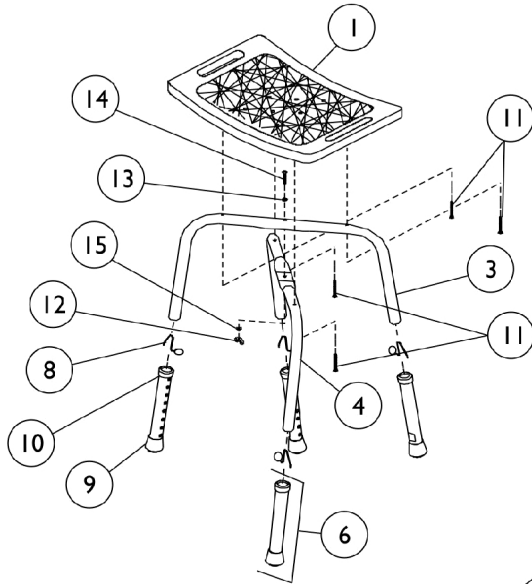

Model 85-1

Model 85-3 Discontinued 12/1/04

With Back

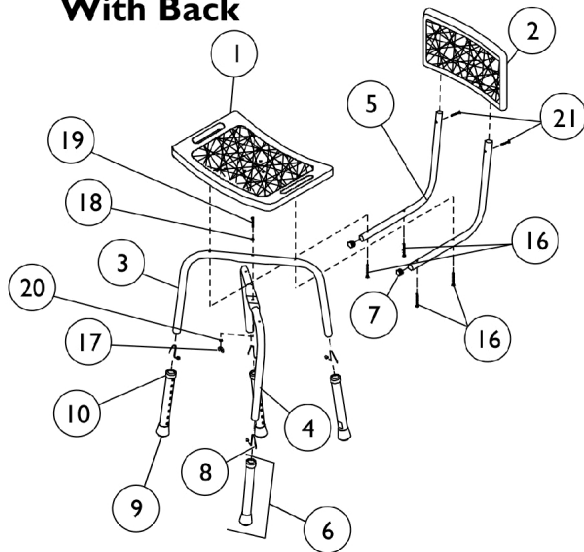


Shower Chair - Compact Blow Molded Adjustable Height


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1079341-NLA	NLA - Seat	1	1
2		1079340-NLA	NLA - Back	1	1
3		1079314-NLA	NLA - Leg, Top	1	1
4		1079315-NLA	NLA - Leg, Bottom	1	1
5	C	1083245-NLA	NLA - Tube, Backrest Support	1	2
6	A,B	1065346-NLA	NLA - Leg Extension Assembly	1	4
7		1027096	Plug Button, Grey (7/8")	1	2
8		1094945	Button, Snap	1	4
9		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
10		1063124	Anti-Rattle, Grey (1-1/8")	1	4
	D,E,H	1074249-NLA	NLA - Kit, Hardware	1	1
11			NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
12			NLA - Nut, Wing (1/4-20)	1	1
13			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
14			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
15			NLA - Washer, External Tooth Lock (1-1/4")	1	1
	F,G,H	1079912-NLA	NLA - Kit, Hardware	1	1
16			NLA - Screw, Phillips Pan Head (1/4-14 x 2-1/4")	1	4
17			NLA - Nut, Wing (1/4-20)	1	1
18			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
19			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
20			NLA - Washer, External Tooth Lock (1-1/4")	1	1
21			NLA - Screw, Phillips Pan Head Tap (#8-18 x 1/2")	1	2
NOTE: A - Due to style changes, when replacing Leg Extension after 3-2-98, order (4) each, and also replace Item 8 (Snap Button). B - Includes Items 9 and 10. C - Includes Plug Buttons. D - To be used with Model 75-1, 3. E - Kit Includes Items 11-15. F - To be used with Model 85-1, 3. G - Kit Includes Items 16-21. H - Not Shown.					

Shower Chair - Hygienic

Model I362 Without Back

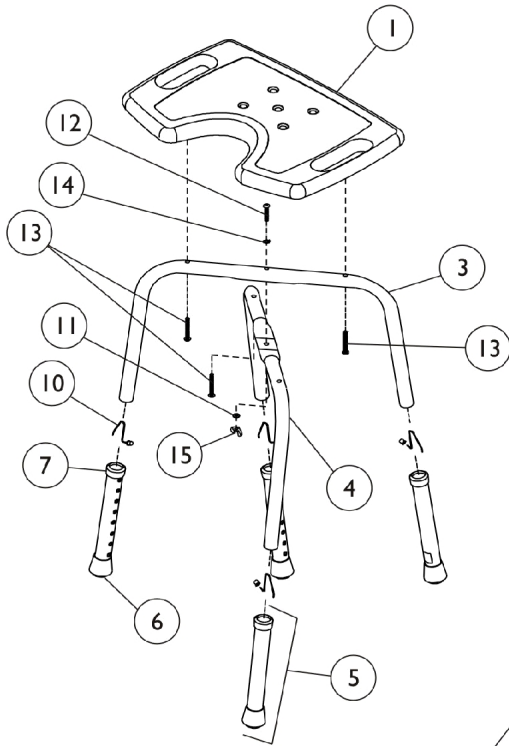

Model I363 With Back

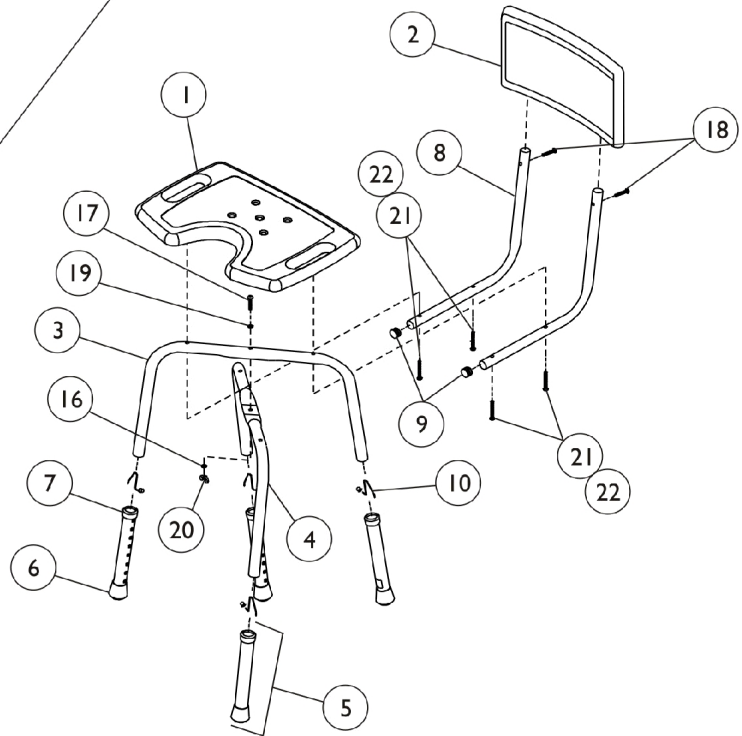


Shower Chair - Hygienic

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1070021-NLA	NLA - Seat, Hygienic	1	1
1	B	1079342-NLA	NLA - Seat, Hygienic	1	1
2	A	1068680-NLA	NLA - Back	1	1
2	B	1079340-NLA	NLA - Back	1	1
3	A	1068668-NLA	NLA - Leg, Top	1	1
3	B	1079314-NLA	NLA - Leg, Top	1	1
4	A	1068670-NLA	NLA - Leg, Bottom	1	1
4	B	1079315-NLA	NLA - Leg, Bottom	1	1
5		1065346-NLA	NLA - Leg Extension Assembly	1	4
6		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
7		1063124	Anti-Rattle, Grey (1-1/8")	1	4
8	A	1068679-NLA	NLA - Tube, Backrest, Support without Plug Buttons	1	2
8	B	1083245-NLA	NLA - Tube, Backrest Support	1	2
9		1027096	Plug Button, Grey (7/8")	1	2
10		1094945	Button, Snap	1	4
	A,D	1068666-NLA	NLA - Kit, Hardware	1	1
	A,F,G	1068678-NLA	NLA - Kit, Hardware	1	1
	B,C,D,E	1074249-NLA	NLA - Kit, Hardware	1	1
11			NLA - Washer, External Tooth Lock (1-1/4")	1	1
12			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
13			NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
14			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
15			NLA - Nut, Wing (1/4-20)	1	1
	B,C,F,G	1079912-NLA	NLA - Kit, Hardware	1	1
16			NLA - Washer, External Tooth Lock (1-1/4")	1	1
17			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
18			NLA - Screw, Phillips Pan Head Tap (#8-18 x 1/2")	1	2
19			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
20			NLA - Nut, Wing (1/4-20)	1	1
21			NLA - Screw, Phillips Pan Head (1/4-14 x 2-1/4")	1	4
22	A	18860X007	Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	4
NOTE: A - Before 11/2/98. B - After 11/1/98. C - Not Shown. D - To be used with Model 1362. E - Kit Includes Items 11-15. F - To be used with Model 1363. G - Kit Includes Items 16-21.					

Shower Chair - Mobile

Model 6358

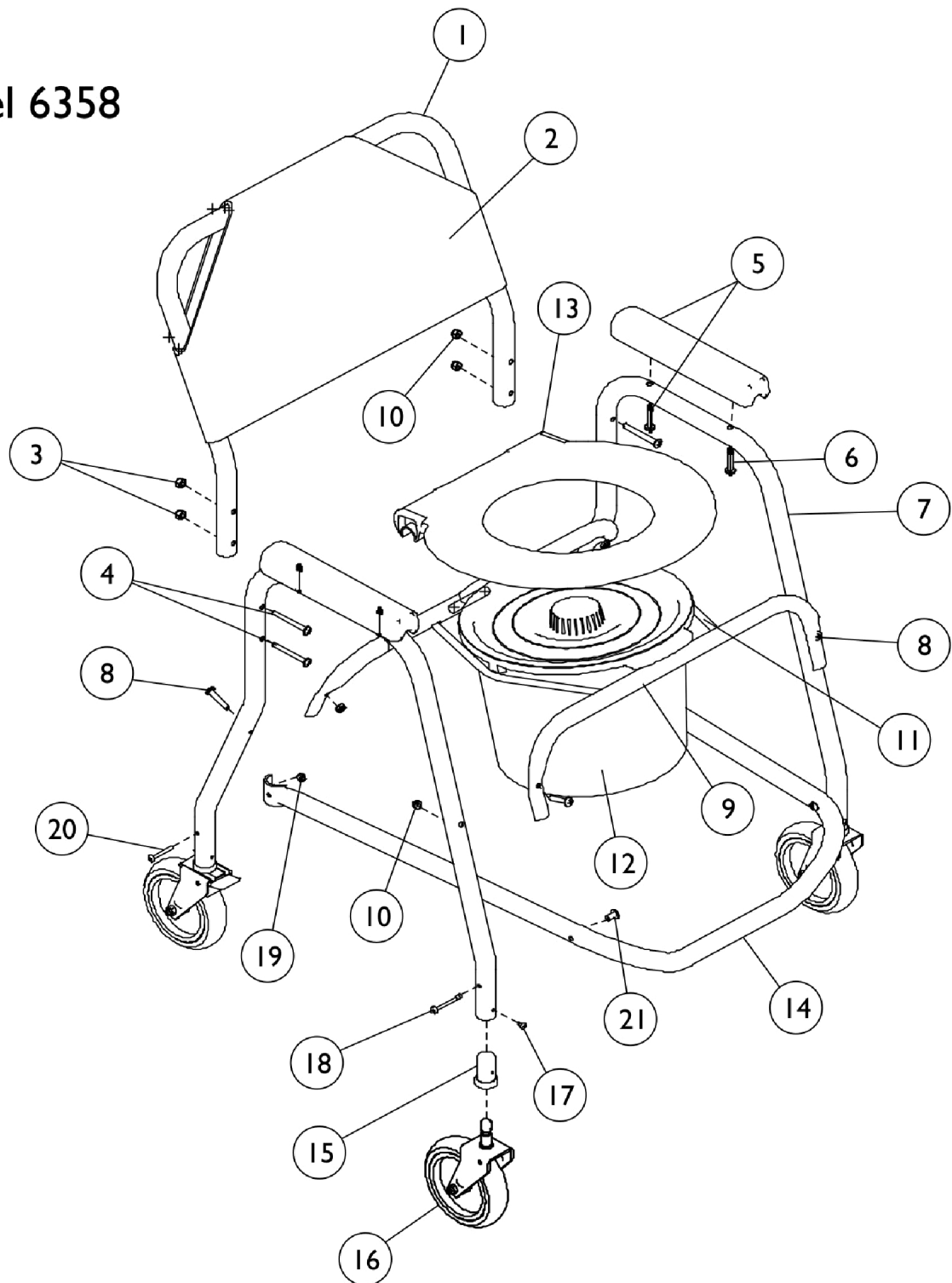

Shower Chair - Mobile

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D,J,V	1132265-NLA	NLA - Kit, Back Frame w/Upholstery and Hardware	1	1
2			NLA - Tube, Back Frame	1	1
3			NLA - Upholstery, Nylon Back, Grey	1	1
4			Locknut (1/4-20)	1	4
4			Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	4
1			NLA - Tube, Back Frame	1	1
2			NLA - Upholstery, Nylon Back, Grey	1	1
2			NLA - Upholstery, Nylon, Back, Bone	1	1
3	A	13181X000-NLA	Locknut (1/4-20)	1	4
4			Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	4
			NLA - Kit, Armrest with Hardware, Full Length - Grey	1	1
5			NLA - Armrest, Full Length, Grey	1	2
6	B,C,Q	1059660-NLA	NLA - Screw, Phillips Pan Head (1/4 x 1-1/4")	1	4
5			NLA - Armrest, Full Length, Grey	1	1
5			NLA - Armrest, Desk Length	1	2
5			NLA - Kit, Arm Pad w/ Hardware (Pair)	1	1
6	A	48127X001-NLA	NLA - Screw, Phillips Round Head (#10-32 x 1/2")	1	4
6			NLA - Screw, Phillips Pan Head (1/4 x 1-1/4")	1	4
7			NLA - Frame, Side	1	2
7			Frame, Side	1	2
8	B	1086654	NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	4
			NLA - Kit, Seat Rails with Hardware	1	1
8			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	4
9			NLA - Rail, Seat, Aluminum, Front	1	1
9	D,K	1132266-NLA	NLA - Rail, Seat, Aluminum, Rear	1	1
10			Locknut (1/4-20)	1	4
9			NLA - Rail, Seat, Aluminum, Front	1	1
9			NLA - Rail, Seat, Aluminum, Rear	1	1
10		60275X000	Locknut (1/4-20)	1	4
			NLA - Kit, Seat Ring with Pail and Holder	1	1
11			NLA - Holder, Pan/Pail	1	2
12			NLA - Pail with Lid and Handle, Grey	1	1
13	D,S,U	1152023-NLA	NLA - Seat, Ring	1	1
10			NLA - Kit, Seat Ring with Pail and Holder	1	1
11			NLA - Holder, Pan/Pail	1	2
12			NLA - Pail with Lid, Almond	1	1
12	B,R	1080083	NLA - Pail with Lid and Handle, Grey	1	1
12			Pail w/ Lid and Handle, Grey	1	1
13			NLA - Seat, Ring	1	1
13			NLA - Seat, Ring	1	1
14	A	1061998-NLA	NLA - Tube, Foot Support	1	1
			NLA - Kit, Casters 5" without Wheel Locks	1	1
15			Caster Assembly, Swivel (5")	1	2
16			Sleeve, Caster	1	2


Shower Chair - Mobile

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
17	D,E,N	1132264-NLA	Screw, Phillips Pan Head Tap (#8-32 x 5/16")	1	2
			NLA - Kit, Caster 5" with Wheel Lock	1	1
15			NLA - Caster Assembly, Swivel w/ Brake (5")	1	2
16			Sleeve, Caster	1	2
17	G L E F E D,O D	1016683	Screw, Phillips Pan Head Tap (#8-32 x 5/16")	1	2
15			Sleeve, Caster	1	4
16			NLA - Casters without Wheel Lock, 1" Tube (5")	1	2
16			NLA - Caster without Wheel Lock (5")	1	2
16		1123547	Caster Assembly, Swivel (5")	1	2
16		01007X003-NLA	NLA - Casters with Wheel Lock, 1" Tube, (5")	1	2
16		1123546-NLA	NLA - Caster Assembly, Swivel w/ Brake (5")	1	2
17		020647	Screw, Phillips Pan Head Tap (#8-32 x 5/16")	1	4
		1132267-NLA	NLA - Kit, Frame Hardware	1	1
18			Screw, Phillips Pan Head w/ Patch (#10-32 x 1-3/4")	1	2
19			Locknut (#10-32)	1	2
20			Screw, Phillips Round Head (#10-32 x 1-3/8")	1	2
21			Rivnut (#10-32)	1	2
18		1007375	Screw, Phillips Pan Head w/ Patch (#10-32 x 1-3/4")	1	2
19		60279X003	Locknut (#10-32)	1	2
20		11008X011	Screw, Phillips Round Head (#10-32 x 1-3/8")	1	2
21		67005X000	Rivnut (#10-32)	1	2
		1027708-NLA	NLA - Decal, Invacare	1	1


NOTE: A - Before 5/15/99.
 B - After 5/14/99.
 C - Due to style changes, it is recommended to order Armrests in pairs (Part Number - 1059660) after 9/3/99.
 D - Not Shown.
 E - Due to style change, it is recommended that you replace the Casters for the entire Shower Chair. You may also need to order Items 15 and 17 along with the Casters.
 F - Item no longer available, use Part Number 1123546 as its replacement.
 G - Item no longer available, use Part Number 1016936 as its replacement.
 H - Kit includes (2) Armrests and (4) Screws.
 I - Kit includes items 11-13.
 J - Kit includes items 1-4.
 K - Kit includes items 8, 9 (Front and Rear) and 10.
 L - Item no longer available, use Part Number 1123547 as its replacement.
 M - Kit includes (2) Casters without Locks, (2) Sleeve Bearings and (2) Screws.
 N - Kit includes (2) Casters with Locks, (2) Sleeve Bearings and (2) Screws.
 O - Kit includes items 18 - 21.
 Q - Item no longer available, use Part Number 1132261 as its replacement.
 R - Item no longer available, use Part Number 1152022 as its replacement.
 S - Kit includes Pail, Seat Ring, and Holder.
 T - Before 8/16/07.
 U - After 8/15/07.
 V - No longer offered by the manufacturer.

Shower Chair - Large Blow Molded Adjustable Height

Model 9680, 9681P
w/o Back
(Discontinued 12/1/04)

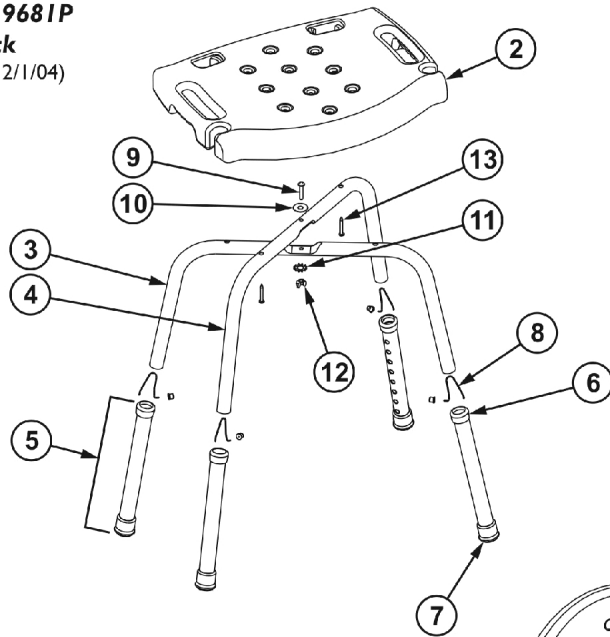

Model 9680P, 9681
w/ Back
(Discontinued 12/1/04)

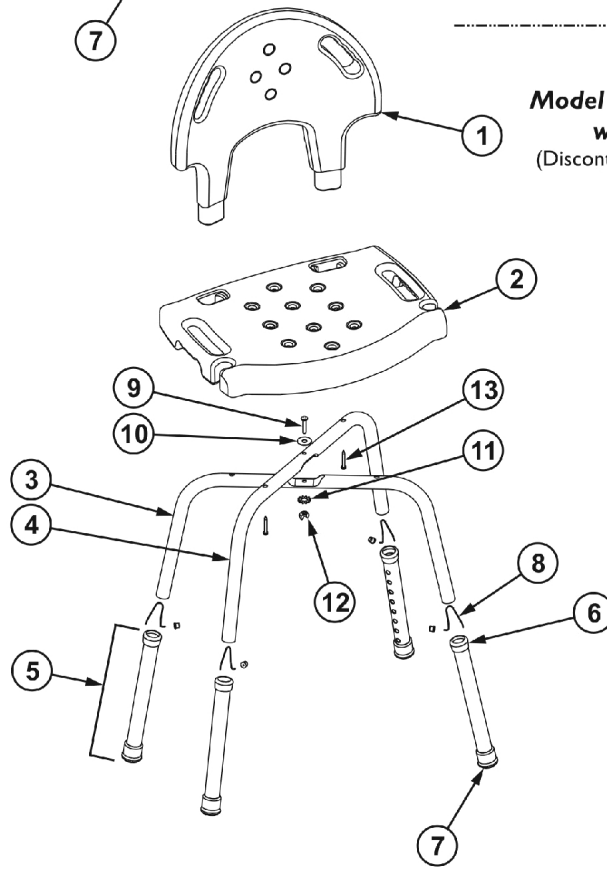


Shower Chair - Large Blow Molded Adjustable Height


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1060375-NLA	NLA - Back	1	1
2		1060376-NLA	NLA - Seat	1	1
3	F	1061419-NLA	NLA - Leg, Bottom	1	1
3	G	1079315-NLA	NLA - Leg, Bottom	1	1
4	F	1061418-NLA	NLA - Leg, Top	1	1
4	G	1079314-NLA	NLA - Leg, Top	1	1
5	E	1065346-NLA	NLA - Leg Extension Assembly	1	4
6		1063124	Anti-Rattle, Grey (1-1/8")	1	4
7		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
8		1094945	Button, Snap	1	4
9	A,D,J	1044142	Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
10		1008882	Locknut (1/4-20)	1	1
13		1021944-NLA	NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
		1074248-NLA	NLA - Kit, Hardware	1	1
9	B,C,K	1074249-NLA	Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
12			Locknut (1/4-20)	1	1
13			NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
			NLA - Kit, Hardware	1	1
9	H	1068492-NLA	Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
10			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
11			NLA - Washer, External Tooth Lock (1-1/4")	1	1
12			NLA - Nut, Wing (1/4-20)	1	1
13	H,I	1064507-NLA	NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
			NLA - Back With Hardware	1	1
			NLA - Plug, End	1	2
NOTE: A - To be used with Model 9681. B - To be used with Model 9681P. C - To be used with Model 9680P. D - To be used with Model 9680. E - Includes Items 6 and 7. F - Before 11/2/98. G - After 11/1/98. H - Not Shown. I - Used with Back before 5/15/97. J - Kit Includes Items 9, 12 and 13. K - Kit Includes Items 9, 10, 11, 12 and 13.					

Shower Chair - Folding

**Model 9980
Without Back**

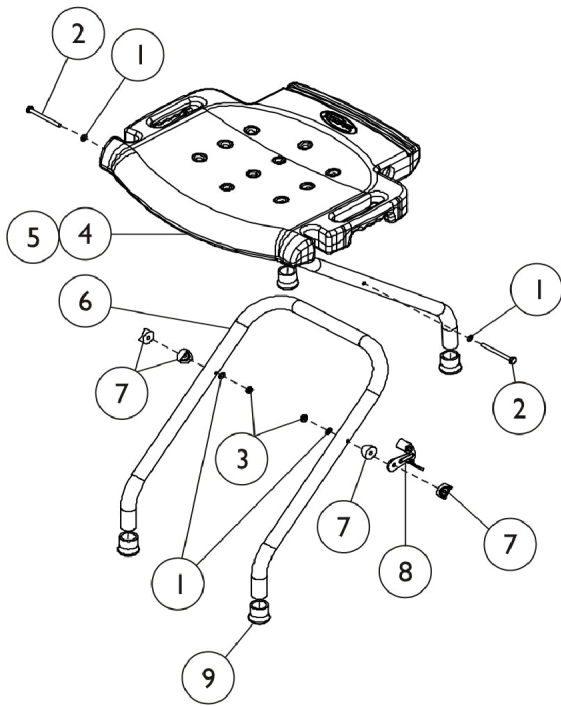

**Model 9981
With Back**

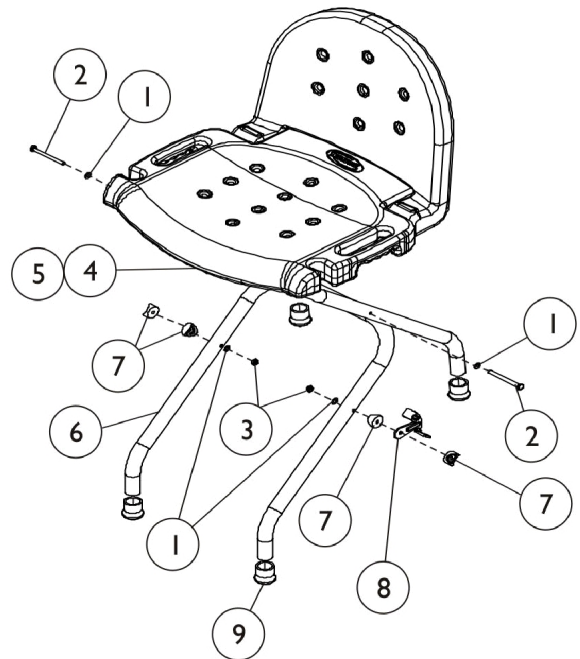


Shower Chair - Folding

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1090450-NLA	NLA - Kit, Hardware	1	1
2			NLA - Washer, Flat (1/4 x 1/2 x 1/16")	1	2
3			Screw, Hex Head (1/4-20 x 2-3/4")	1	1
4			Locknut (1/4-20)	1	1
5		1093112	Screw, Phillips Pan Head (1/4-1/8 x 3/4")	1	1
6		1026104-NLA	NLA - Spacer	1	1
7		1092405	Leg, Inside	1	1
8		1093609	Washer, Coved (1/4 x 1 x 1/2")	1	4
9		1092403	Bracket, Pivot Stop	1	1
		40186X030	Tip, Rubber, Grey (1")	1	4
NOTE: A - Not Shown. B - Kit Includes 1-3.					


Shower Chair - Vinyl Padding

NOTE: Model 9872C was discontinued on 3/12/03.

Model 9872

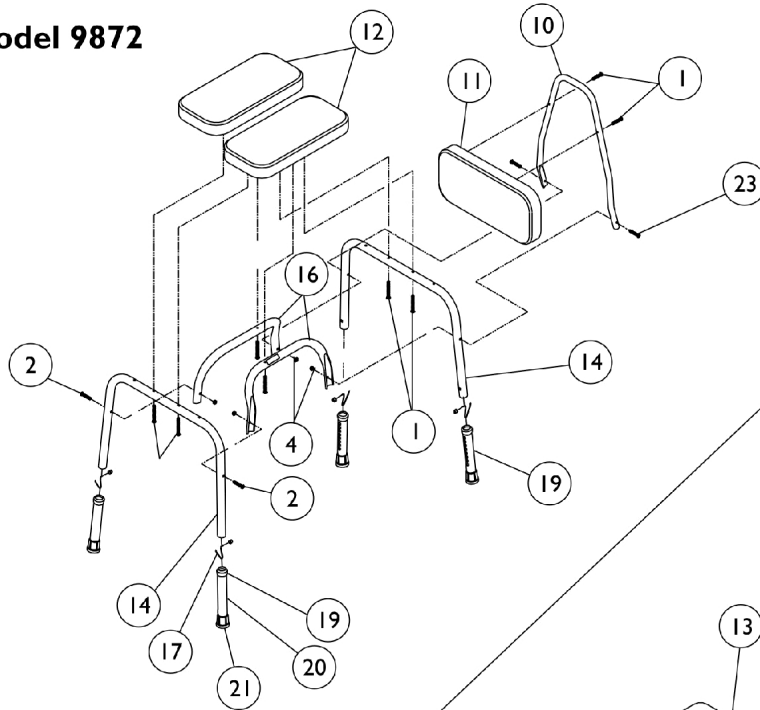

**Model 9872C
With Commode**

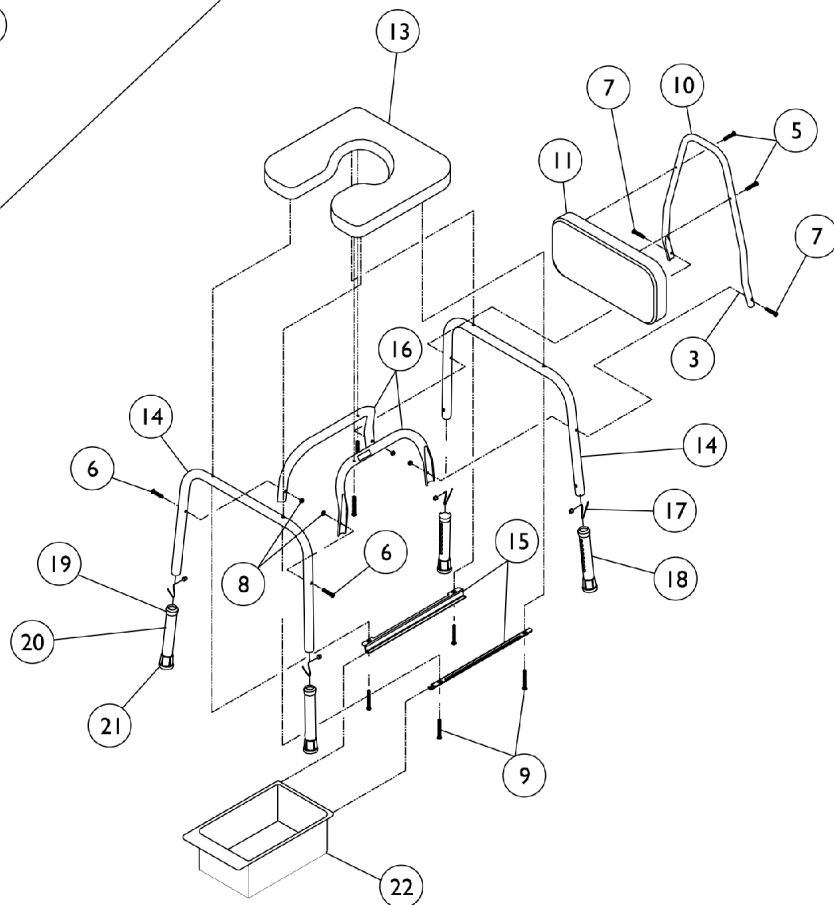


Shower Chair - Vinyl Padding

NOTE: Model 9872C was discontinued on 3/12/03.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,D,E	1075930-NLA	NLA - Kit, Hardware	1	1
			Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	6
2			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	2
3			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/2")	1	2
4	B,D,F	1075931-NLA	Locknut (1/4-20)	1	4
			NLA - Kit, Hardware	1	1
5			Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	2
6			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	2
7			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/2")	1	2
8			Locknut (1/4-20)	1	4
9			Screw, Phillips Pan Head (#10-24 x 1-1/2")	1	4
10		1022921	Tube, Back	1	1
11		1075184-NLA	NLA - Pad, Back, Vinyl	1	1
12	A	1073264-NLA	NLA - Pad, Seat, Vinyl, Grey	1	2
13	B	1075656-NLA	NLA - Pad, Commode, Vinyl, Grey	1	1
14		1080491	Leg, Chair	1	2
15		1075045	Holder, Pan	1	2
15		1044142	Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	2
16		1080492	Rail, End	1	2
17		1094945	Button, Snap	1	4
18		1058806-NLA	NLA - Leg Extension Assembly	1	4
19	C	1063124	Anti-Rattle, Grey (1-1/8")	1	4
20		1022925	Tube, Leg Extension	1	4
21		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
22		1074676	Pan	1	1
23	A	1115001	Screw, Phillips Pan Head (1/4-20 X 1-5/8")	1	2
NOTE: A - To be used with Model 9872. B - To be used with Model 9872C. C - Includes Items 19-21. D - Not Shown. E - Kit Includes Items 1, 2 and 4, Item 3 used only on the 9872C. F - Kit Includes Items 5-9.					

IVC™ I-Fit™ & CareGuard™ Shower Chairs

Model 90-2, 95-2, 9780 and 9781

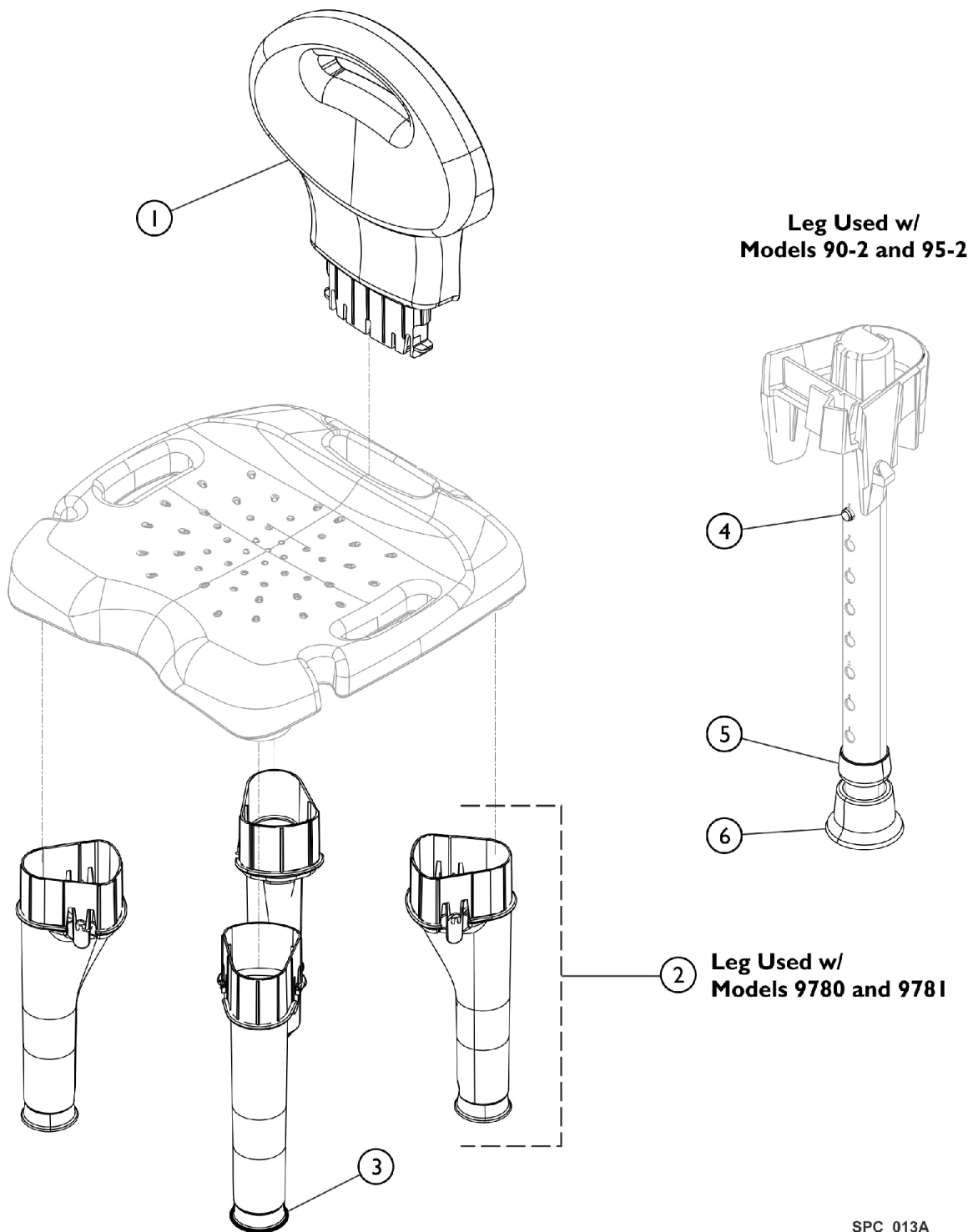

IVC™ I-Fit™ & CareGuard™ Shower Chairs

Model 90-2, 95-2, 9780 and 9781

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1116880	Backrest	1	1
2	B	1116872	Leg Assembly, Composite	1	4
3	B	1121530-NLA	NLA - Tip, Rubber, Grey	1	4
4	C	1094945	Button, Snap	1	4
5	C	1063124	Anti-Rattle, Grey (1-1/8")	1	4
6	C	1110289-NLA	NLA - Tip, Rubber, Grey	1	4
NOTE: A - For use with 96-2 (Aluminum) and 9781 (Composite) models only with Backrests. B - For use with Shower Chairs with Composite Legs (9780 and 9781). C - For use with Shower Chairs with Aluminum Legs (90-2 and 95-2).					

Shower Chair - Tooless (Models 91-2 and 96-2)

Model 96-2 - Tooless
Shower Chair w/Back



Model 91-2 - Tooless
Shower Chair w/o Back



Shower Chair - Tooless (Models 91-2 and 96-2)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1145082-NLA	NLA - Package, Tip	10	1
2	A	1145083	NLA - Package, Leg with Tip - Standard	2	1
NOTE: A - Use with Models 91-2 and 96-2.					

Shower Chair - Bariatric (Model 9785-2)




Shower Chair - Bariatric (Model 9785-2)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1141732	Kit, Back and Seat w/Hardware	1	1
2		1141734-NLA	NLA - Leg Extension Assembly	1	4


Transfer Bench - Large Blow Molded

Model 9670 - Without Commode

Model 9670U - Unassembled

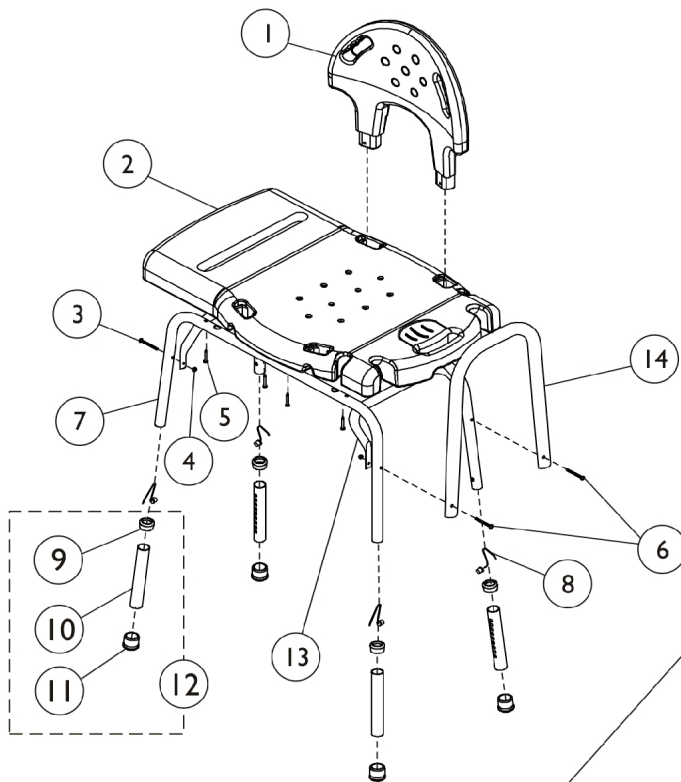

Model 9670C
With Commode

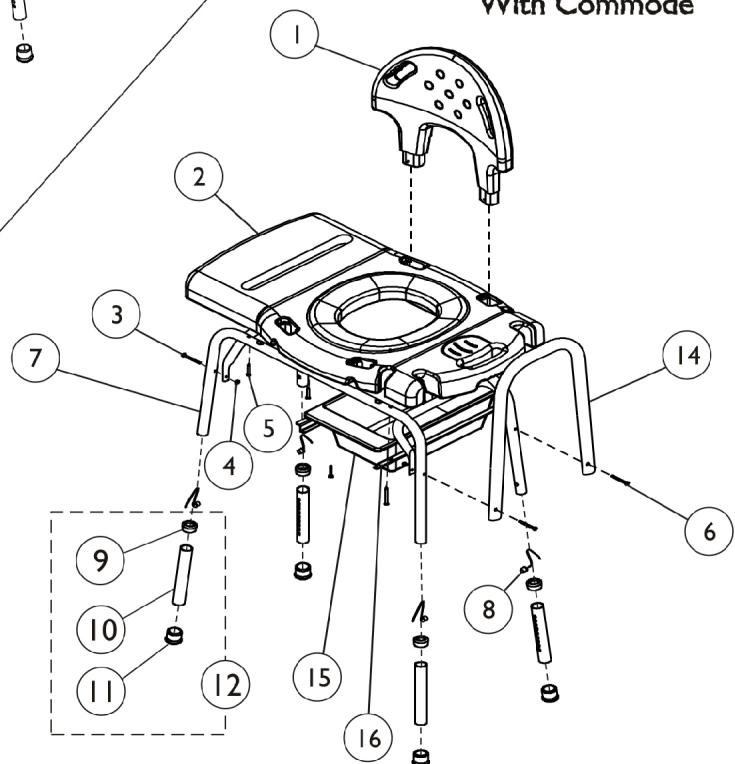

Transfer Bench - Large Blow Molded


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060375-NLA	NLA - Back	1	1
2	A	1058356	Bench, Transfer with Commode	1	1
2	B	1060372	Bench, Transfer	1	1
	G,H	1069168	Kit, Hardware	1	1
3			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	2
4			Locknut (1/4-20)	1	4
5			Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	8
6			Screw, Phillips Pan Head (1/4-20 X 1-5/8")	1	2
7		1048897	Leg, Bench	1	2
8	C	1094945	Button, Snap	1	4
9		1063124	Anti-Rattle, Grey (1-1/8")	1	4
10	C	1022925	Tube, Leg Extension	1	4
11		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
12	C,D	1058806-NLA	NLA - Leg Extension Assembly	1	1
13		1062820	Rail, End	1	2
14		1060379	Tube, Seat Arm	1	1
15	E	13245X000	Pail, Commode, White	1	1
15	F	1074676	Pan	1	1
16	E	1058357-NLA	NLA - Holder, Pan	1	2
16	F	1075045	Holder, Pan	1	2
	G,J	1068492-NLA	NLA - Back With Hardware	1	1
	G,I	1064507-NLA	NLA - Plug, End	1	2
NOTE: A - To be used with Model 9670C. B - To be used with Model 9670. C - It is recommended when replacing Leg Extension after 3/20/98, also replace Item 8 (Snap button). D - Includes Items 9-11. E - Before 4/20/98. F - After 4/19/98. G - Not Shown. H - Kit Includes Items 3-6. I - Before 5/15/97. J - Includes Items 1, 8, and 9.					

Transfer Bench


NOTE: Model 9871C was discontinued on 3/12/03.

Model 987I, 987I-I - Without Commode (Vinyl)

Model 9873 - Without Commode (Ensolite)

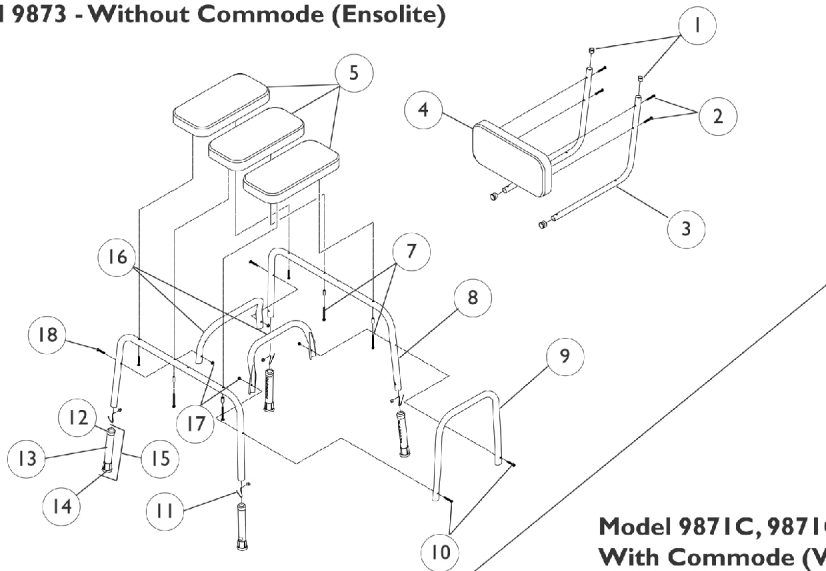

Model 9871C, 9871C-I
With Commode (Vinyl)
Model 9873C
With Commode (Ensolite)

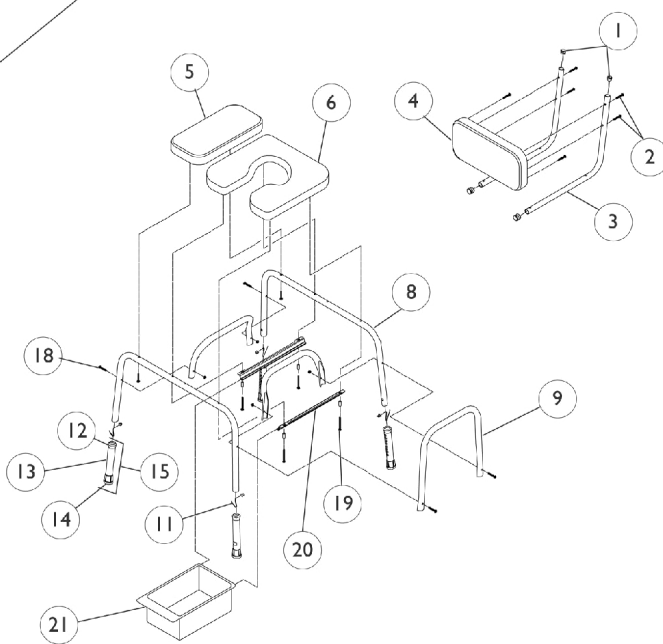

Transfer Bench


NOTE: Model 9871C was discontinued on 3/12/03.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027096	Plug Button, Grey (7/8")	1	4
2		1075176	Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	6
3		1075036	Support, Back	1	2
4		1075183	Pad, Vinyl, Grey	1	1
4	I	1073265-NLA	NLA - Pad, Seat, Ensolite, Grey	1	1
5	A,B	1073264-NLA	NLA - Pad, Seat, Vinyl, Grey	1	3
5	C	1073264-NLA	NLA - Pad, Seat, Vinyl, Grey	1	1
5	D,E	1127869-NLA	NLA - Pad, Seat, Ensolite, Grey	1	1
6	C	1075656-NLA	NLA - Pad, Commode, Vinyl, Grey	1	1
6	E	1075655-NLA	NLA - Pad, Commode, Ensolite, Grey	1	1
7		1102582	Screw, Slot Hex Washer Head Tap (#10-16 x 2-3/8")	1	4
8		1048897	Leg, Bench	1	2
9		1060379	Tube, Seat Arm	1	1
10		1115001	Screw, Phillips Pan Head (1/4-20 X 1-5/8")	1	2
11		1094945	Button, Snap	1	4
12		1063124	Anti-Rattle, Grey (1-1/8")	1	4
13		1022925	Tube, Leg Extension	1	4
14		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
15	F	1058806-NLA	NLA - Leg Extension Assembly	1	4
16		1062820	Rail, End	1	2
17		60275X000	Locknut (1/4-20)	1	4
18		1044142	Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	2
19		1022930	Screw, Phillips Pan Head (#10-24 x 2-1/2")	1	4
20		1075045	Holder, Pan	1	2
21	C,E	1074676	Pan	1	1
	D,G	1075932-NLA	NLA - Kit, Hardware	1	1
			Screw, Phillips Pan Head (#10-24 x 1-1/2")	1	6
			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	2
			Locknut (1/4-20)	1	2
			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/2")	1	2
			Screw, Phillips Pan Head (#10-24 x 2-1/2")	1	4
	A,G,H	1075038-NLA	NLA - Kit, Hardware	1	1


NOTE: A - To be used with Model 9871.
 B - To be used with Model 9871-1.
 C - To be used with Model 9871C.
 D - To be used with Model 9873.
 E - To be used with Model 9873C.
 F - Includes Items 12, 13 and 14.
 G - Not Shown.
 H - Includes Items 2, 7, 10, 17 and 18.
 I - Item No Longer Available. Refer to part number 1127869 for Replacement.

Transfer Bench - Compact Blow Molded (Models 98070 and 990-1)

Model 98070

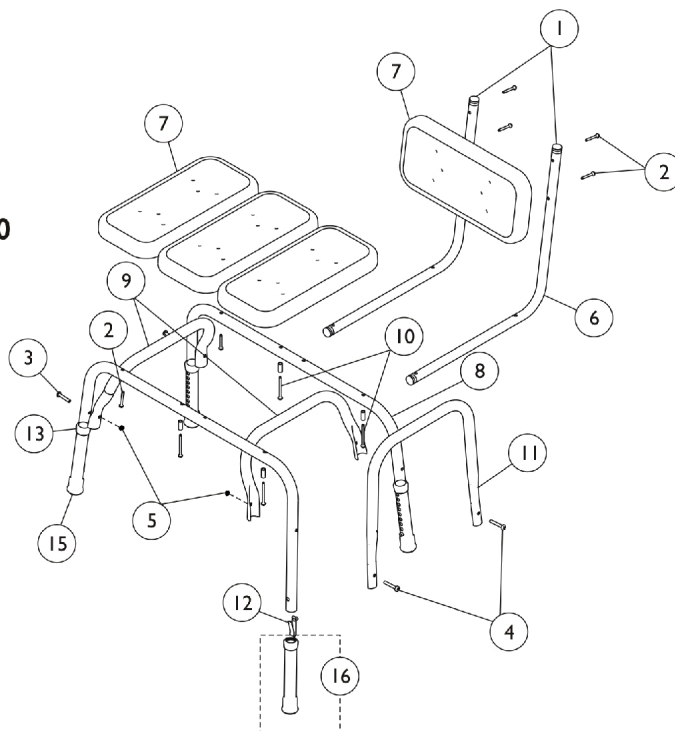

Model 990-1

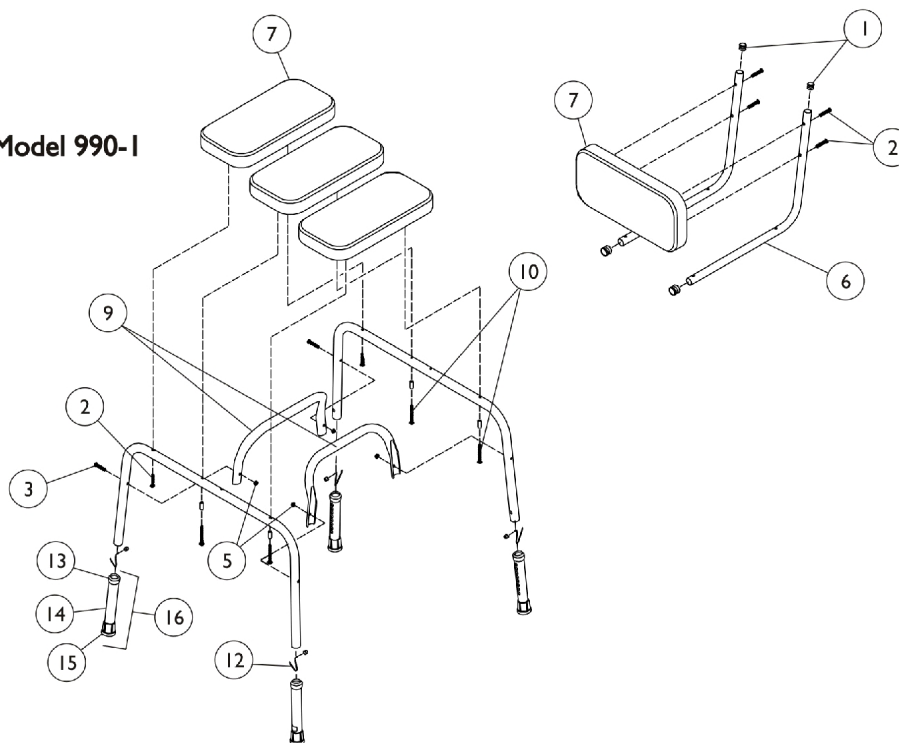

Transfer Bench - Compact Blow Molded (Models 98070 and 990-1)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D B,C,D, F	1027096	Plug Button, Grey (7/8")	1	4
		1075930	NLA - Kit, Hardware	1	1
2			Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	6
3			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	2
4			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/2")	1	2
5			Locknut (1/4-20)	1	4
6	D	1075036	Support, Back	1	2
7	D	1074987-NLA	NLA - Pad, Composite	1	4
7		1075183	Pad, Vinyl, Grey	1	4
8	D	1048897	Leg, Bench	1	2
9	D	1062820	Rail, End	1	2
10	D	1102582	Screw, Slot Hex Washer Head Tap (#10-16 x 2-3/8")	1	4
11		1060379	Tube, Seat Arm	1	1
12	D	1094945	Button, Snap	1	4
13	D	1063124	Anti-Rattle, Grey (1-1/8")	1	4
14	D	1022925	Tube, Leg Extension	1	4
15	D	1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
16	A,D	1058806-NLA	NLA - Leg Extension Assembly	1	4
NOTE: A - Includes Items 12-14. B - Not Shown. C - Kit Includes Items 2-5. D - Used with 98070 and 990-1. F - Item 4 is not needed on Model 990-1.					

Transfer Bench - Toolless (Model 98071-2)



Transfer Bench - Tooless (Model 98071-2)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1145079-NLA	NLA - Package, Leg with Suction Cup	2	1
2		1145080-NLA	NLA - Package, Leg with Tip - Standard	2	1
3		1145081-NLA	NLA - Armrest	1	1

Transfer Bench - Bariatric (Model 9675-2)



Transfer Bench - Bariatric (Model 9675-2)


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1141732	Kit, Back and Seat w/Hardware	1	1
2		1141733-NLA	NLA - Kit, Side Seat w/ Hardware	1	1
3		1141734-NLA	NLA - Leg Extension Assembly	1	6
4		1141735-NLA	NLA - Kit, Tip and Anti-Rattle	1	1

3" Suction Cup Option

Model 6314

FITS ON MODELS


6451, 6451S, 6461
6461S, 6431, 6481
6484, 9670's, 9871's
9872's, 9873's, 98070

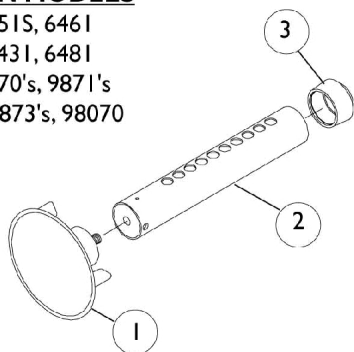

Model 6316

FITS ON MODELS

1362, 1363, 9680's,
9681's, 9690P, 9691P,
97080, 97081, 97090,
97091, 50's, 60's, 75's,
85's

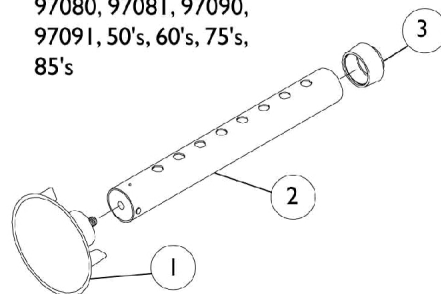

3" Suction Cup Option

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	12711X001-NLA	NLA - Cup, Suction	1	2
2	A,C	1022927-NLA	NLA - Tube, Leg Extension	1	2
2	B,D	1064509-NLA	NLA - Tube, Leg Extension	1	2
3	A,B	1063124	Anti-Rattle, Grey (1-1/8")	1	2
NOTE: A - To be used with Model 6314. B - To be used with Model 6316. C - Anti-Rattle Included. D - Anti-Rattle Not Included. NOTE: It is Recommended when replacing the Suction Cup Option for Models 6541, 6451S, 6461, 6461S, 6481, and 6484, after 3/2/98, also replace the Snap Buttons and Release Springs. Model 6314 FITS ON MODELS: 6451, 6451S, 6461, 6461S, 6431, 6481, 6484, 9670's, 9871's, 9872's, 9873's and 98070. Model 6316 FITS ON MODELS: 1362, 1363, 9680's, 9681's, 9690P, 9691P, 97080, 97081, 97090, 97091, 50's, 60's, 75's, and 85's.					


Grab Bar Hardware

Hardware for:

Models 2712-3 - 2742-3, ADA Compliant

Models 512-3 - 532-3, Knurled

Models 312-3 - 332-3, Residential

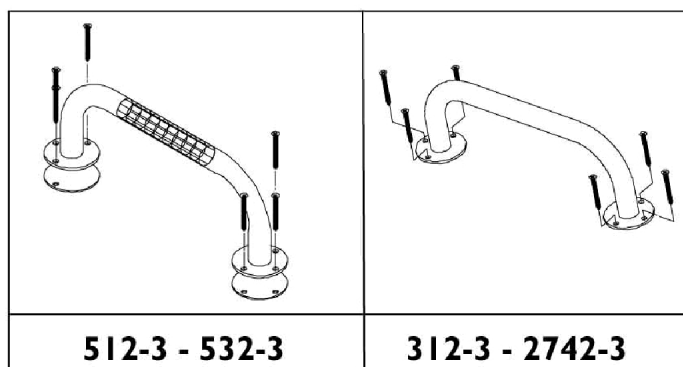

Hardware Package for Grab Bars


Includes: 6 Each, Screws - Phillips Pan Head (10-11 x 2)

Grab Bar Hardware


Model Numbers	Hardware Package	Quantity Per Package
312 - 332	1074401-NLA See Note(s) A	1
512 - 532	1074401-NLA See Note(s) A	1
1706CH	1074401-NLA See Note(s) A	1
1754	1074401-NLA See Note(s) A	1
1766	N/A	1
1910 - 1913	1074401-NLA See Note(s) A	1
2712 - 2736	1074401-NLA See Note(s) A	1
NOTE: A - Kit includes (6) screws.		

Tub Bars


Model 100-3 - Chrome
Model 110-3 - Vinyl (Shown)
Model 1816 - Narrow Wall, Chrome

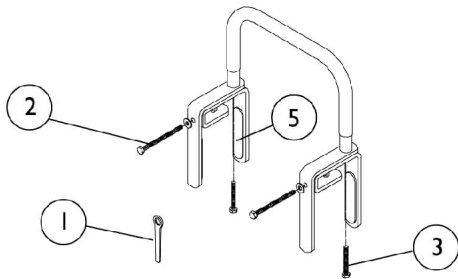

Model 725-3 - Vinyl

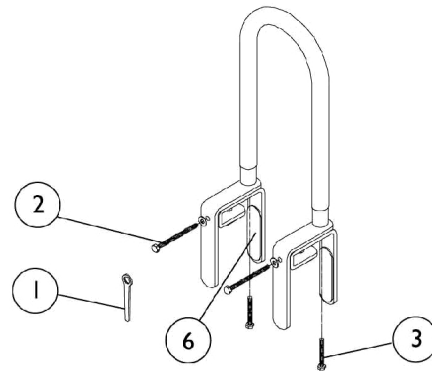


Model 1200-3 - Chrome
Model 1210-3 - Vinyl

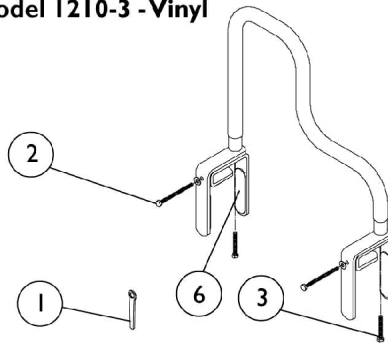

Tub Bars

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1073204-NLA	NLA - Wrench, Mounting Hex Nut (1/2")	1	1
2		1073208-NLA	NLA - Bolt, Hex Cap (1/4-20 x 3")	1	2
3		1073517	Bolt, Hex (1/4-20 x 1-3/4")	1	2
4		16720X001-NLA	NLA - Bolt, Hex Cap (1/4-20 x 2")	1	2
5		1072003-NLA	NLA - Pad, Non-Skid Adhesive Back	1	4
6		1072004	Pad, Non-Skid Adhesive Back	1	4


Tub Bars

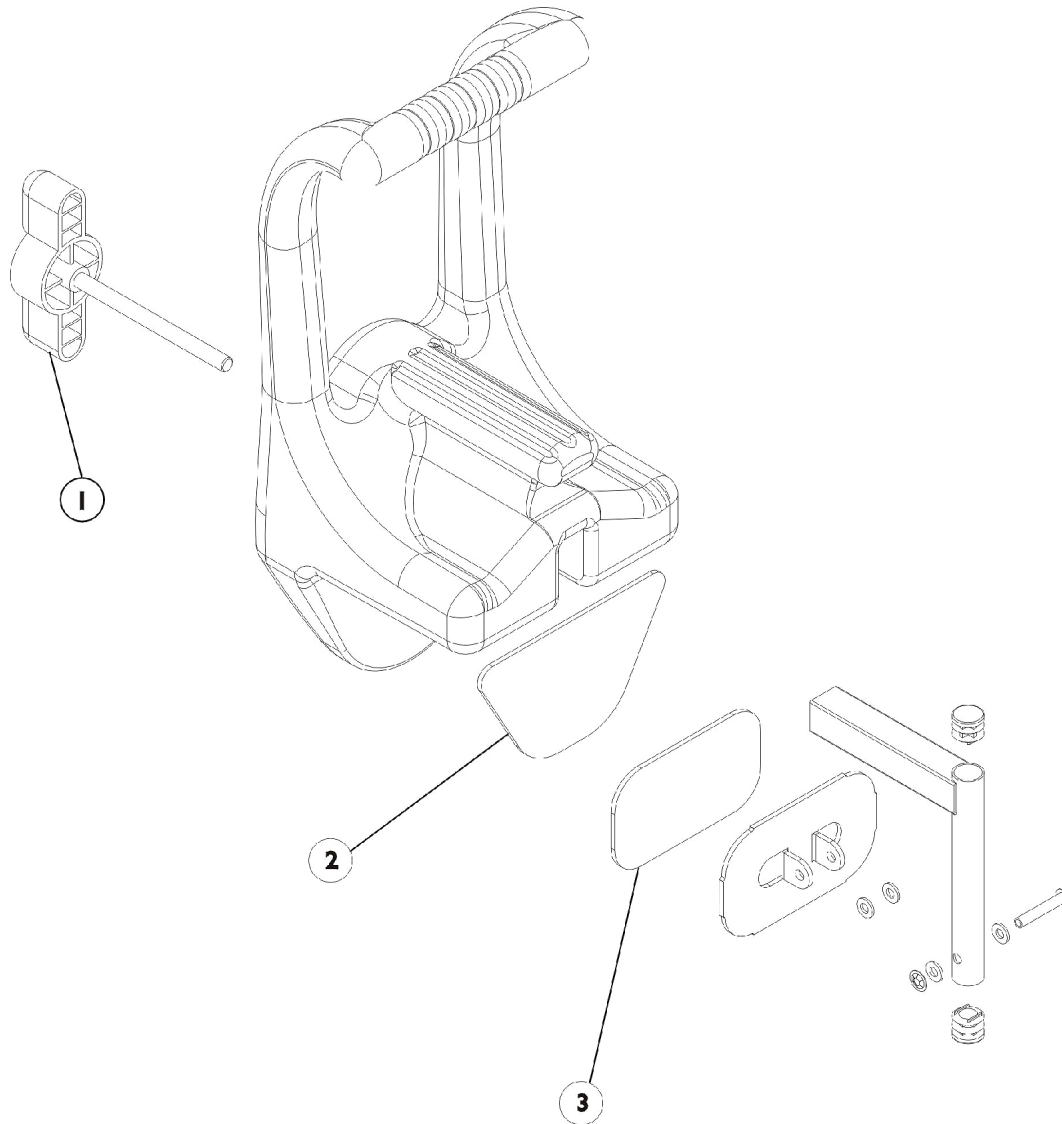

Tub Bars

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1052984-NLA	NLA - Knob with Threaded Rod, White	1	1
2		1052959-NLA	NLA - Pad, Non-Skid without Holes	1	1
3		1052960-NLA	NLA - Pad, Non-Skid without Holes	1	1

H605 Rio

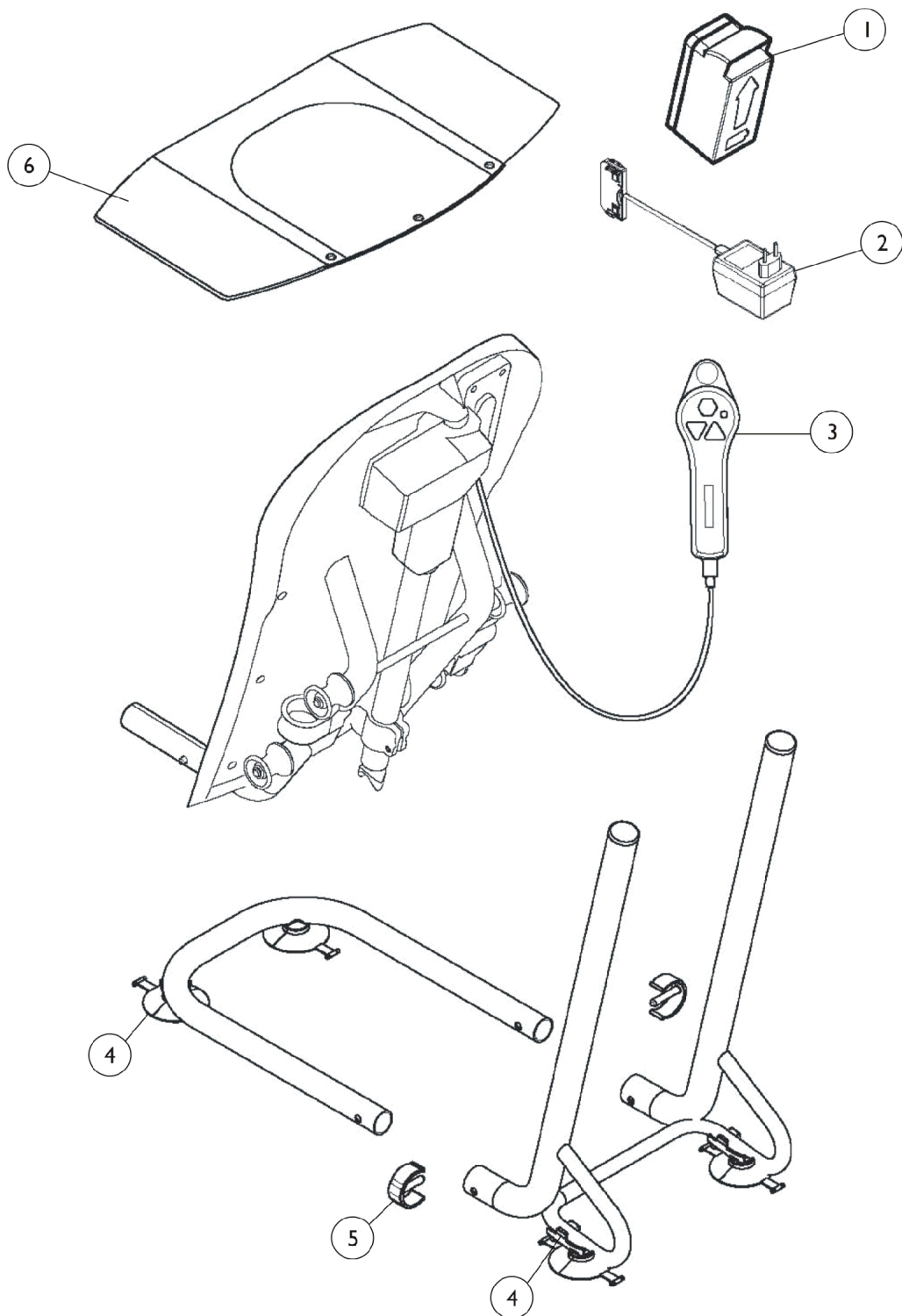

H605 Rio

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1534495-NLA	NLA - Battery w/ Cover	1	1
2		1534726-NLA	NLA - Charger	1	1
3		1534461-NLA	NLA - Pendant, Control	1	1
4		1172001	Kit, Suction Cup w/ Hardware	1	1
			Bracket, Mounting, Suction Cup	1	2
			Cup, Suction	1	2
			Holder, Suction Cup	1	2
5		1534455	Clip (Pair)	1	1
6		1534494	Sideflap	1	1

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Florida Technical Support
407-321-5630

Customer Service
800-333-6900

© 2022 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
(Revised 06/21/22)

Form No. 92-43