



Handrim

20" & 24" Wheel Only

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

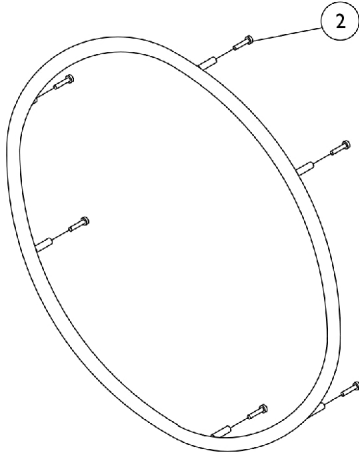

For use with 24" Handrim with Welded Studs.

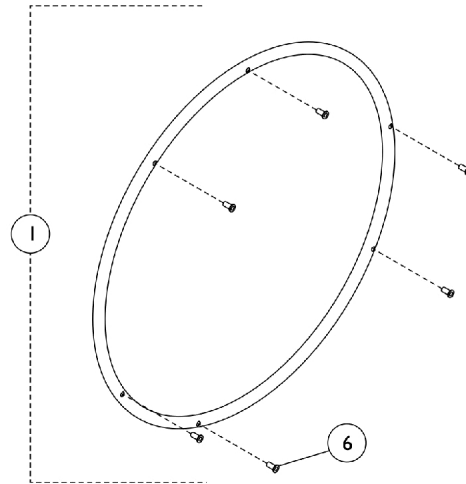

For use with 24" Handrim without Welded Studs.



For use with 20" Handrim.

Handrim

20" & 24" Wheel Only

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	H,I	1027413-NLA	NLA - Handrim, Chrome, 6 PT (24" Wheel)	1	1
1		1125209-NLA	NLA - Handrim, Steel, 6 PT, Grey (24" Wheel)	1	1
2	D,J	1026750-NLA	NLA - Screw w/ Washer (#10-32 x 7/8")	1	6
2		1007375	Screw, Phillips Pan Head w/ Patch (#10-32 x 1-3/4")	1	6
3	A,B,F	430056	Handrims with Hardware, Chrome, 12 PT (24" Wheel)	2	1
3	G	1125211-NLA	NLA - Handrims with Hardware, Steel, 6 PT, Grey (24" Wheel)	2	1
4	E,F	430051-NLA	NLA - Handrims with Hardware, Chrome, 7 PT (20" Wheel)	2	1
4	E,G	1125213-NLA	Handrims with Hardware, Steel, 7 PT, Grey (20" Wheel)	2	1
5	F	1019999	Handrim, Chrome, 7 PT (20" Wheel)	1	1
5	G	1125212-NLA	Handrim, Steel, 7 PT, Grey (20" Wheel)	1	1
6	C	1007375	Screw, Phillips Pan Head w/ Patch (#10-32 x 1-3/4")	1	7
7	A,C	1016438	Spacer (1/4 x 11/32 x 15/16")	1	7

NOTE: A - Not Shown.
 B - Includes (2) Chrome Handrims, (12) Screws item #6 and (12) Spacers item #7.
 C - 20" Wheel Only.
 D - For use with 24" Wheel with Welded Stud Only.
 E - Includes items 5 and 6.
 F - Before 1/24/05. For use on chairs with "Chrome Frame Finish."
 G - After 1/23/05. For use on chairs with "Grey Frame Finish."
 H - Includes (1) Chrome Handrim and (6) Rivet Nuts.
 I - Use with 24" Handrim without Welded Stud.
 J - No Longer Available, Replace with 1007375.